


# SaeboStep Barefoot Accessory Strap



SaeboStep sold separately.


## PARTS OF THE DEVICE

### TOP VIEW

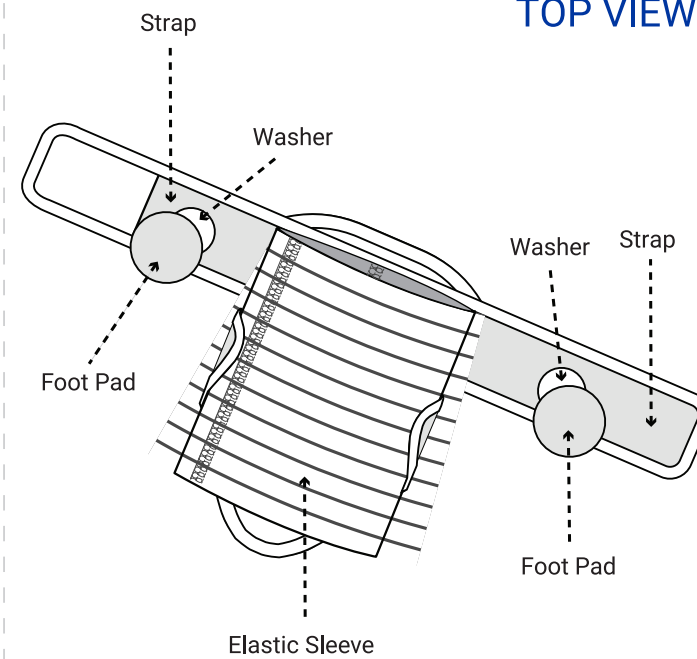

## PARTS OF THE DEVICE


### BOTTOM VIEW

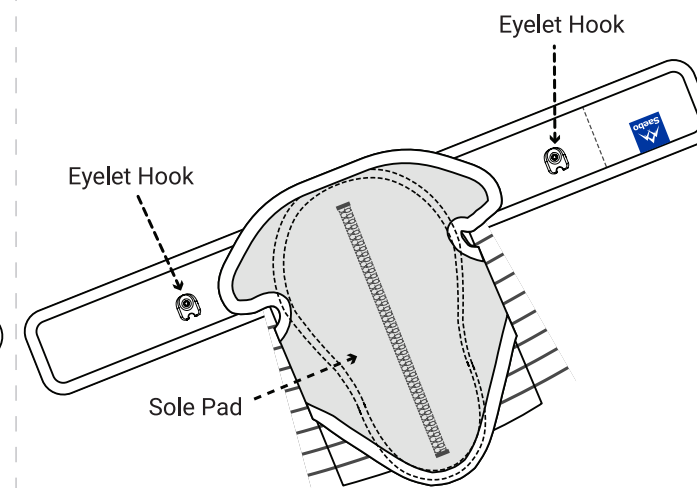

1

## APPLY SAEBO STEP

Apply Saebo Step to ankle per instructions.

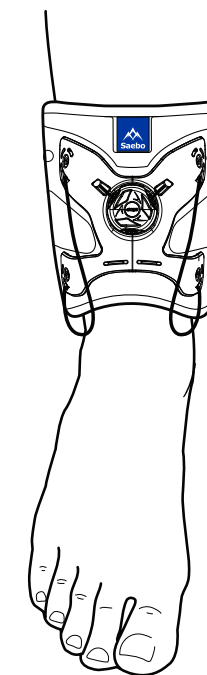


2

## APPLY SAEBO BAREFOOT


Slide Barefoot Accessory Strap to just past toes as shown.

**Tip:** If your toes curl or flex during walking, consider positioning the strap down to block toes from curling.

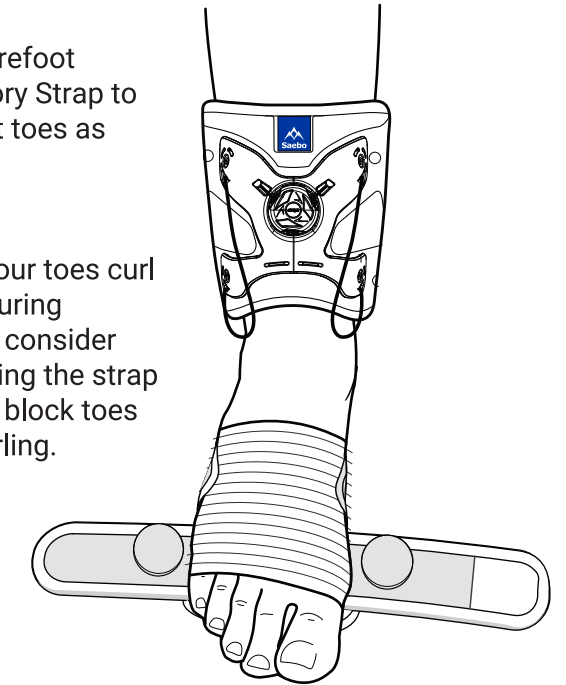

3


## SECURE STRAPS

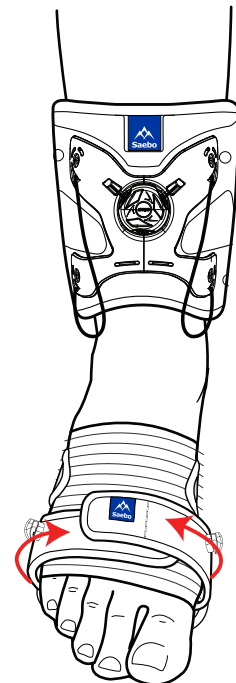

Secure straps across foot.

4


## APPLY CORDS TO HOOKS

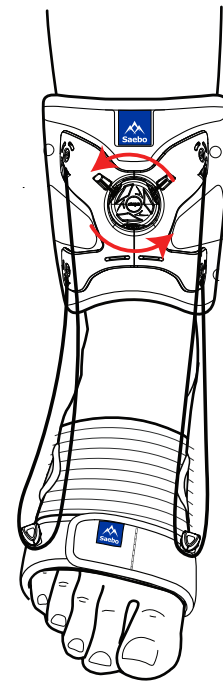

Turn Saebo Step dial counterclockwise to loosen cords. Pull left and right cords at the same time and loop onto hooks.

5

## ADJUST POSITION

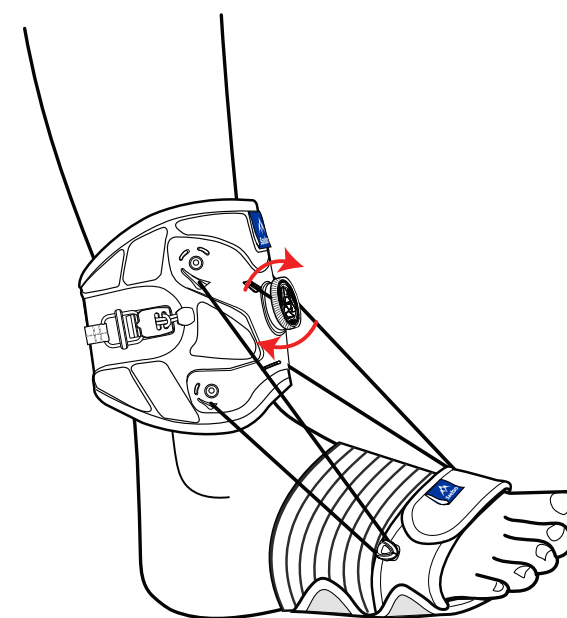

Tighten cords by turning dial clockwise to raise foot to desired position.

## SEE THE VIDEOS

Visit our website to see instructional videos including how to replace eyelet hook hardware.

[www.saebo.com/saebobarefoot](http://www.saebo.com/saebobarefoot)



## Saebo, Inc.

2459 Wilkinson Blvd., Suite 120-B  
Charlotte, NC 28208

888-942-5751  
sales@saebo.com  
www.saebo.com



European Healthcare & Device Solutions (Ireland) Ltd.  
info@europeandevicesolutions.co.uk  
www.europeandevicesolutions.co.uk

Australian Sponsor:  
Ausmedic Australia Pty Ltd  
Unit 3, 3 Basalt Road Pemulwuy NSW  
2145 1300 473 422  
Performhealth.com.au

