USER MANUAL

Rotating bathtub chair

Product code: FS 793S

Tibnago

1. Characteristics

The bathtub chair is made of a stainless steel frame and a PVC seat that includes openings for water drainage. It features a contoured seat and backrest, centrally mounted on a swivel base. Thanks to the clamps that stabilize the chair and the lever mechanism, the seat can be locked in the desired position. Additionally, the chair has anti-slip tips that prevent it from sliding in the bathtub, ensuring safety and stability during use, and side handles to assist patients who have difficulty standing up or sitting down unassisted. The product is constructed from high-strength, moisture-resistant materials.

The set includes: upper frame with seat, lower frame with clamps, mounting screw.

Elements of the commode chair:

- A. Side handle
- B. Adjusting lever
- C. Mounting screw
- D. Clamps
- E. Anti-sliding tips

2. Application

The bathtub chair is ancillary equipment which helps the user to perform physiological activities. The chair is recommended to people with reduced mobility which prevents or hinders the unassisted performance of daily living activities. Rotating seat can be locked in desired position what provides safer and more comfortable bathing as well as getting in and out of the bath.

3. Assembly

The width of chair should correspond to the width of the bath; anti-sliding tips should fully touch the edge of the bath.

Step 1. Place the lower frame on the bath.

Step 2. Adjust the clamps using the screws so that the frame is stable and does not move on the bath edges.

Step 3. Unscrew mounting screw located on the upper frame under the seat. Join lower and upper frame and put them on the mounting element. Tighten afterwards.

Step 4. Pull upwards adjusting lever, set upper frame in desired position and lower the lever till you hear “click” sound of the locking pins.

Step 5. Ensure all elements are assembled correctly and frame is set properly.

4. Technical data

Overall width	74 cm
Inner width	50 cm
Depth of frame	13 cm
Seat dimensions	40 x 32 cm
Weight	5 kg
Maximum capacity	100 kg

5. Notes

imago International Group does not bear responsibility of improper use of the the product, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the bathtub chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- At least once every three months check the condition of:
 - mounting screw
 - frame and seat (if cracks)
 - anti-sliding tips

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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