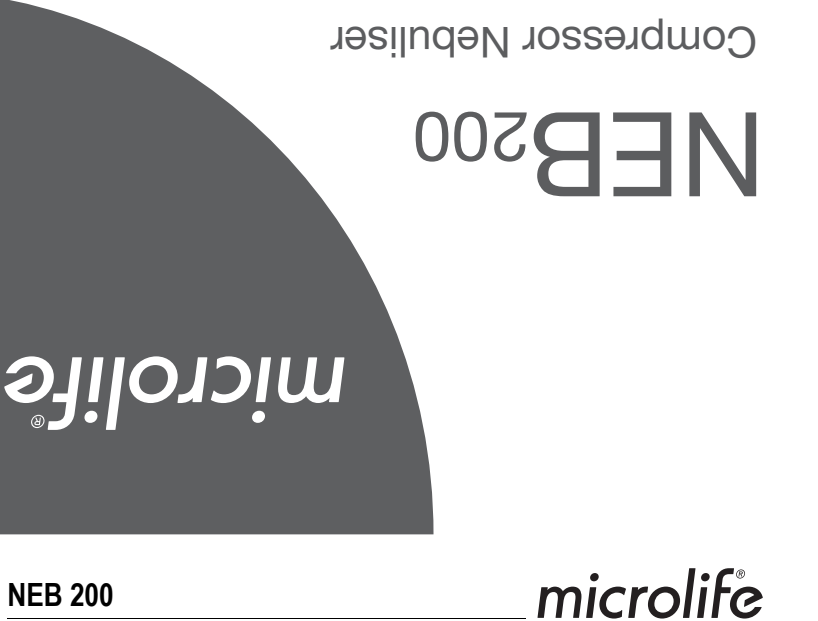
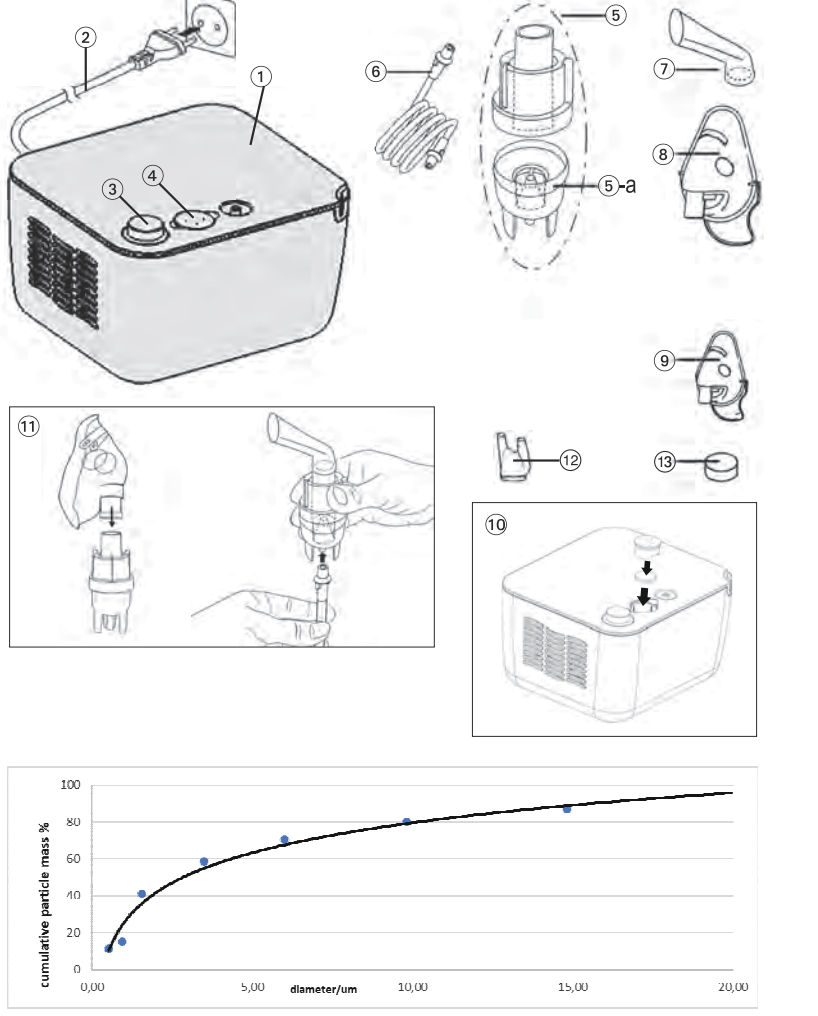
IB NEB 200 N-V-0521 - Revision Date: 2021-01-22

Guarantee Card

Name of Purchaser / Inköpares namn / Ostajin nimi / Forhandlers navn / Kjöpers navn / Pircelja država / Pirkėjo pavardė / Ostja imi / Ф.И.О. покупателя

Serial Number / Serienummer / Sarjanumero / Serienummer / Serienummer / Sérijas numurs / Sérijos numeris / Seerianumber / Серийный номер

Date of Purchase / Inköpsdatum / Ostopäivämäärä / Købsdato / Kijpsdato / Iegades datums / Pärdaivmo date / Ostuukupäev / Дата покупки

Specialist Dealer / Återförskäp / Alkan kauppias / Special-forhandler / Specialisti forhandler / Specialists -pärstavis / Pärdavus įstaiga / Ametlik müügiesind / Спецализированный дилер

Globalcare Medical Technology Co., Ltd
11 Building, 39 Middle Industrial Main Road,
European Industrial Zone, Xixian Town,
528415 Zhongshan City, Guangdong Province, PRC

Compressor Nebuliser

- Piston compressor
- Power lead
- ON/OFF Switch
- Air filter compartment
- Nebuliser
 - a: Vaporiser head
- Air tube
- Mouthpiece
- Adult face mask
- Child face mask
- Assembling nebuliser kit
- Noise piece
- Air filter

Dear Customer,
This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacy will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Stay healthy – MicroLife AG!

1. Explanation of Symbols

- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Read the instructions carefully before using this device.
- Type BF applied part
- Class II equipment
- Serial number
- Reference number
- Manufacturer
- ON
- OFF

IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

CE CE Marking of Conformity

2. Important Safety Instructions

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- Retain instructions in a safe place for future reference.
- Do not operate the unit in presence of any anesthetic mixture inflammable with oxygen or nitrogen protoxide.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Comply with the safety regulations concerning the electrical devices, shown on the rating plate on the bottom of the unit, corresponds to the mains rating. In case the power plug provided with the device does not fit your wall socket, contact qualified personnel for the replacement of a compatible plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, indicated on adapters and extension cables.
- Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not in operation.
- The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, for which the manufacturer cannot be held responsible.
- Do not replace the power lead of this device. In case of a power lead damage, contact a technical service center authorized by the manufacturer for its replacement.
- The power supply cord should always be fully unwound in order to prevent dangerous overheating.
- Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
- Only use the medication prescribed by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Depending on the pathology, only use the treatment that is recommended by your doctor.
- Only use the nose piece if expressly indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.
- Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.
- Do not position the equipment so that it is difficult to operate the disconnection device.
- For greater hygiene safety, do not use the same accessories for more than one person.
- Never bend the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Use of this device is not intended as a substitute for a consultation with your doctor.

3. Preparation and Usage of This Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nebuliser kit (1). Ensure that all parts are complete.
- Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- Connect the nebuliser with the air tube (2) to the compressor (1) and plug the power lead (2) into the socket (230V 50 Hz AC).

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the compressor.

The device requires no calibration.

No modification to the device is permitted.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact MicroLife-Service (see forward).

Replacement of the nebuliser

Replace the nebuliser (5) after a long period of inactivity, in cases where it shows deformations, breakage, or when the vaporiser head (5-a) is obstructed by dry medicine, dust, etc. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original nebulisers!

Replacement of the air filter

In normal conditions of use, the air filter (3) must be replaced approximately every 500 working hours or after each year. We recommend to periodically check the air filter (10-12 treatments) and if the filter shows a grey or brown colour it is wet, replace it. Extract the filter and replace with a new one.

Do not try to clean the filter for reusing it.

The air filter shall not be serviced or maintained while in use with a patient.

Only use original filters! Do not use the device without filter!

6. Malfunctions and Actions to take

- The nebuliser functions poorly or not at all**
 - Ensure the air tube (2) is correctly connected at both ends.
 - Ensure the air tube is not squashed, bent, dirty or blocked. If necessary, replace with a new one.
 - Ensure the nebuliser (5) is fully assembled and the vaporiser head (5-a) is placed correctly and not obstructed.
 - Ensure the required medication has been added.
- Guarantee**

This device is covered by a **5 year guarantee** from the date of purchase. During this guarantee period, at our discretion, MicroLife will repair or replace the defective product free of charge. Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee:

 - Transport costs and risks of transport.
 - Damage caused by incorrect application or non-compliance with the instructions for use.
 - Damage caused by misuse, accident or misuse.
 - Damage caused by accident or misuse.
 - Regular checks and maintenance (calibration).
 - Accessories and wearing parts: Nebuliser, masks, mouthpiece, nose piece, tube, filters, nasal washer (optional).

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will not replace if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

8. Technical Specifications

Nebulisation rate: 0.4 ml/min. (NaCl 0.9%)
Particle size: 63% < 5 µm
2.83 µm (MMAD)
Max. free air flow: 15 l/min.
Operating air flow: 5.31 l/min.
Acoustic noise level: 52 dBA
Power source: 230V 50 Hz AC
Power lead length: 1.6 m
Nebuliser capacity: min. 2 ml; max. 8 ml
Residual volume: 0.8 ml
Operating limits: 30 min. On / 30 min. Off
Operating conditions: 10 - 40 °C / 50 - 104 °F
30 - 85 % relative maximum humidity
700 - 1060 hPa Atmospheric pressure
Storage and shipping conditions: 20 - +60 °C / -4 - +140 °F
10 - 95 % relative maximum humidity
700 - 1060 hPa Atmospheric pressure
Weight: 1306 g
Dimensions: 103 x 160 x 140 mm
IP Class: IP21
Reference to standards: EN 13544-1; EN60601-1; EN60601-2-1; EN60601-1-6; IEC 60601-1-11
Expected service life: 1000 hours

This device complies with the requirements of the Medical Device Directive 93/42/EEC. Class II device as regards protection against electric shocks. Nebuliser, mouthpiece and masks are type BF applied part.

The technical specifications may change without prior notice.

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