


Smart Toilet Seat Lifter - BA07059

The Smart Toilet Seat Lifter is a clever bathroom mobility aid designed to help you open and close the toilet seat with just your feet. Attach the Smart Toilet Seat Lifter to your toilet by following the steps below.


You should have these parts:

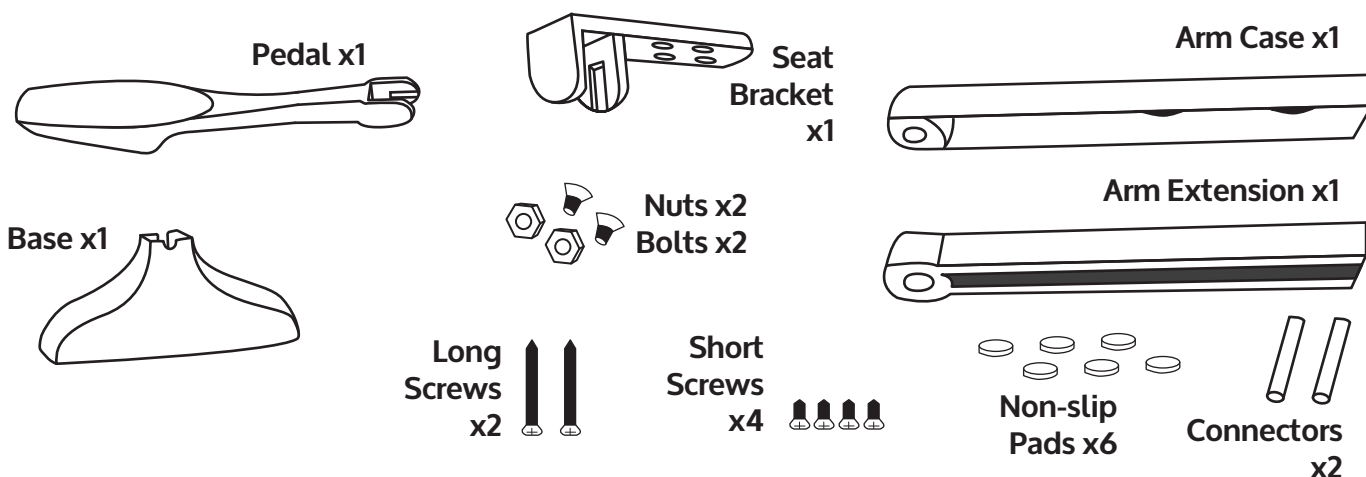

Instructions


1. Connect the arms

Take the two connectors and feed them through the eyes of the arm case and the arm extension. Connect the arm extension to the seat bracket and the arm case to the pedal.

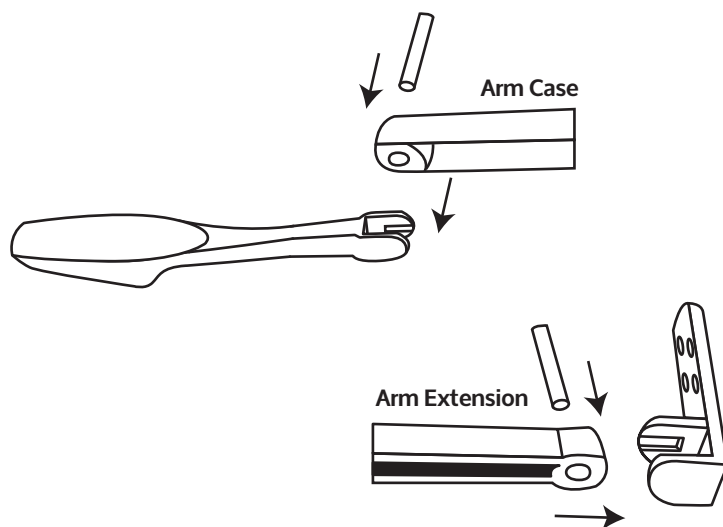

2. Fasten the arm


Fasten the plastic arm extension to the arm case with the nuts and bolts provided.

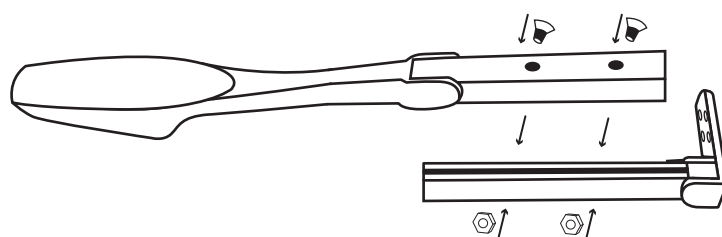

Smart Toilet Seat Lifter - BA07059


3. Install non-slip pads

Optional: Place the non-slip pads on the bottom of the pedal and the bottom of the base, as shown in the diagram.

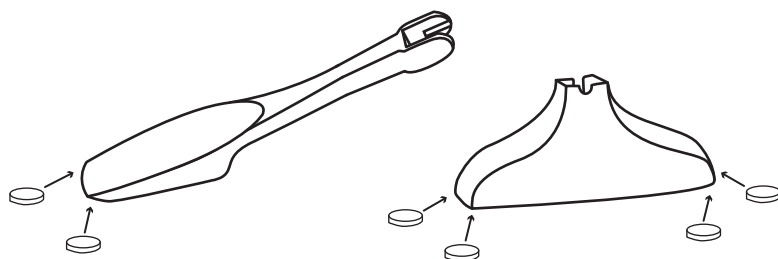


4. Connect the pedal and the base

Line the pedal up with the joint of the base and press firmly to snap into place.

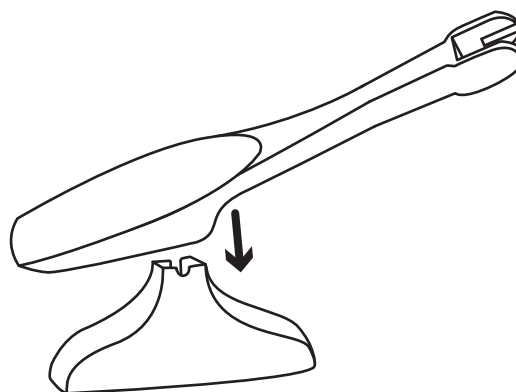

5. Attach to toilet seat

Attach the seat bracket 5 inches from the back of the toilet seat using the four small screws provided. Optional: Use 1/8th inch drill bit to drill pilot holes in the toilet seat for the screws.

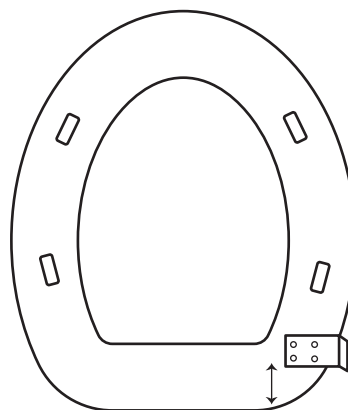

6. Your Smart Toilet Seat Lifter is ready to use!

Set your new Smart Toilet Seat Lifter next to your toilet and adjust the arm to the correct height.

