

Need Help? Email: support@betachek.com

BETACHEK® C50

All-in-One Blood Glucose Meter

MedM HealthApp

12:51

All Readings

Tap to add devices

12:00 pm

No details

4 May 2023

9:59 pm

4.8 mmol/L

No details

9:58 pm

5.1 mmol/L

No details

3 May 2023

10:38 am

5.0 mmol/L

No details

2 May 2023

4:15 pm

6.2 mmol/L

Before meal

4:14 pm

6.6 mmol/L

No details

4:14 pm

6.2 mmol/L

Fasting

1 May 2023

9:38 am

5.3 mmol/L

No details

9:37 am

5.3 mmol/L

No details

9:37 am

5.1 mmol/L

No details

12:46

Details

2 May 2023

4:15 pm

Blood Glucose

6.2

mmol/L

Meals

mmol/L

Before meal

4.4

7.2

Scale: ADA Diabetes Guidelines

Source

Feeling

Note

Betachek C50

Good

Before walk

11:47

All Readings

16 May - 23 May

73%

Normal Glucose

8% Low Glucose

73% Normal Glucose

19% High Glucose

11:50

All Readings

26 Apr 2023 - 3 May 2023

7.0

6.0

5.0

4.0

26

27

28

29

30

01

02

Day

3 Days

Week

12:53

Blood Glucose export

Choose Time Interval

Week

Month

3 Months

Custom

24/4/2023

24/5/2023

Choose Format

Generic CSV

Excel CSV

Choose Share Type

Share

Email

BETACHEK C50

50 tests

S

M




Connecting the BETACHEK C50 meter to the MedM Health-app:


1. Download the MedM Health App from the Apple App Store or Google Play.




2. Open the MedM App and tap on the menu icon (three bars in the top left corner)

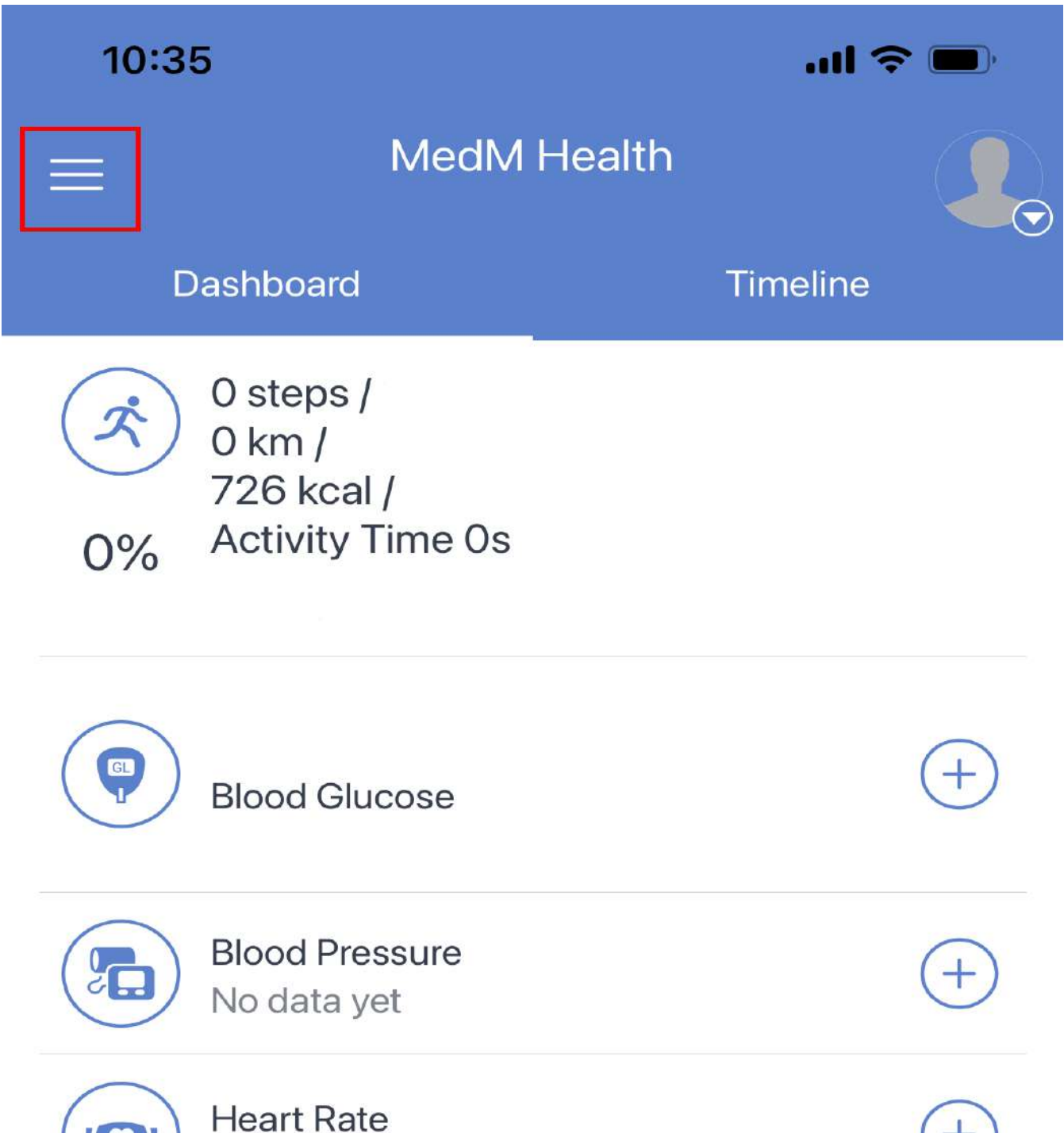


Select “My Devices”

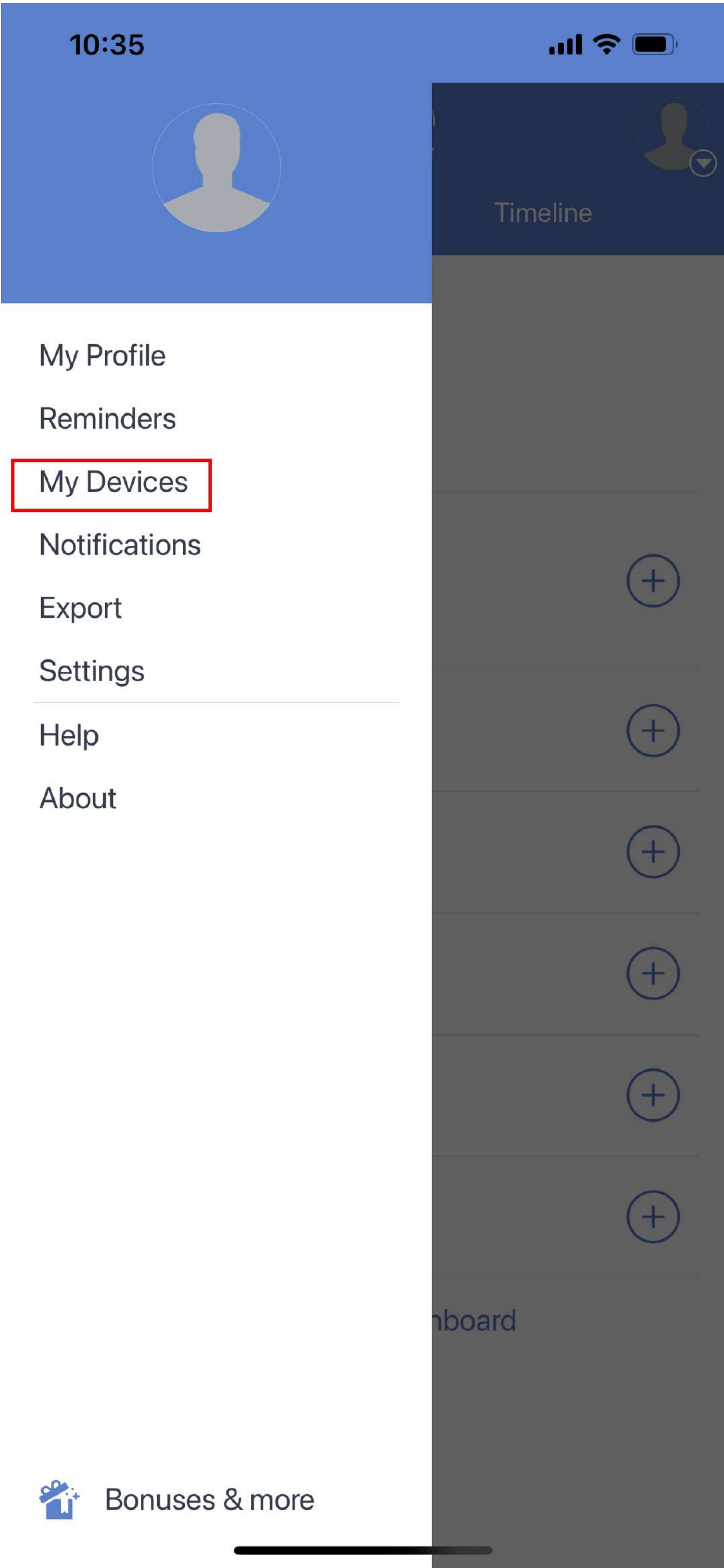

Select “ Add New”

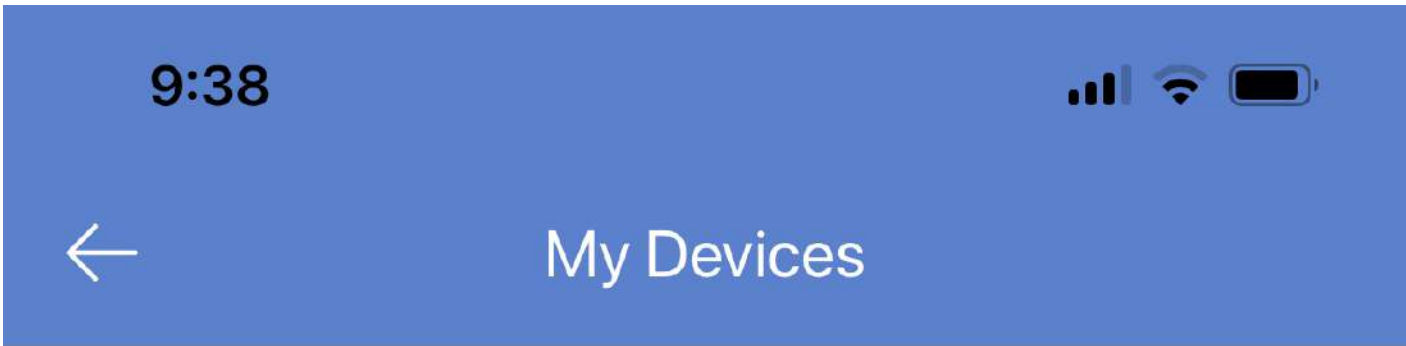

You have no devices.
To add a device tap 'Add New' button.

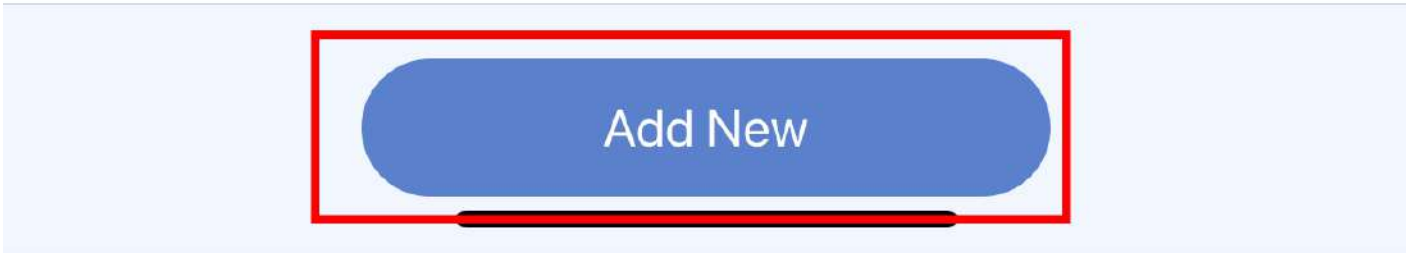

1 Hold down the S and M buttons until the Bluetooth symbol appears.



2 The bluetooth icon flashing shows the meter is ready to pair.



BETACHEK C50 will appear when the app detects the meter. Press on the arrow.

10:01

Add Device

2 devices found

Internal Step Counter

Betachek
C50
BetachekC50
Signal: excellent (-43 dBm)

Manually

Pair MFI

Select “Add to my Devices”

10:01

Betachek

C50

Receive data automatically

App has to be in the foreground with screen unlocked.

Receive data in background

Collect data in background or if screen is locked.

Add to My Devices

10:01



MedM Health



Dashboard

Timeline



Blood Glucose
No data yet



MedM Health



Betachek C50 is successfully
paired with MedM Health.

Make a measurement to upload
data.

Ok



Manage Dashboard

10:01



MedM Health



Dashboard

Timeline



Blood Glucose

No data yet



Blood Pressure

No data yet



Heart Rate

No data yet



Oxygen

No data yet



Temperature

No data yet



Weight

No data yet



Manage Dashboard

Bluetooth Pairing Request

"BetachekC50" would like to pair with your iPhone.

Cancel

Pair