

Need Help? Email: support@betachek.com

BETACHEK® C50

All-in-One Blood Glucose Meter

MedM HealthApp

12:51

All Readings

Tap to add devices

No details

4 May 2023

9:59 pm

4.8 mmol/L

No details

9:58 pm

5.1 mmol/L

No details

3 May 2023

10:38 am

5.0 mmol/L

No details

2 May 2023

4:15 pm

6.2 mmol/L

Before meal

4:14 pm

6.6 mmol/L

No details

4:14 pm

6.2 mmol/L

Fasting

1 May 2023

9:38 am

5.3 mmol/L

No details

9:37 am

5.3 mmol/L

No details

9:37 am

5.1 mmol/L

No details

12:46

Details

2 May 2023

4:15 pm

Blood Glucose

6.2

mmol/L

Meals

mmol/L

Before meal

Scale: ADA Diabetes Guidelines

Source

Feeling

Note

Betachek C50

Good

Before walk

11:47

All Readings

16 May - 23 May

73%

Normal Glucose

8% Low Glucose

73% Normal Glucose

19% High Glucose

11:50

All Readings

26 Apr 2023 - 3 May 2023

70

60

50

40

16

27

28

29

30

01

02

Day

3 Days

Week

12:53

Blood Glucose export

Choose Time Interval

Week

Month

3 Months

Custom

24/4/2023

24/5/2023

Choose Format

Generic CSV

Excel CSV

Choose Share Type

Share

Email

BETACHEK C50

50 tests

S

M

Download on the

App Store

GET IT ON


Google Play

Connecting the BETACHEK C50 meter to the MedM Health-app:


1. Download the MedM Health App from the Apple App Store or Google Play.



2. Open the MedM App and tap on the menu icon (three bars in the top left corner)

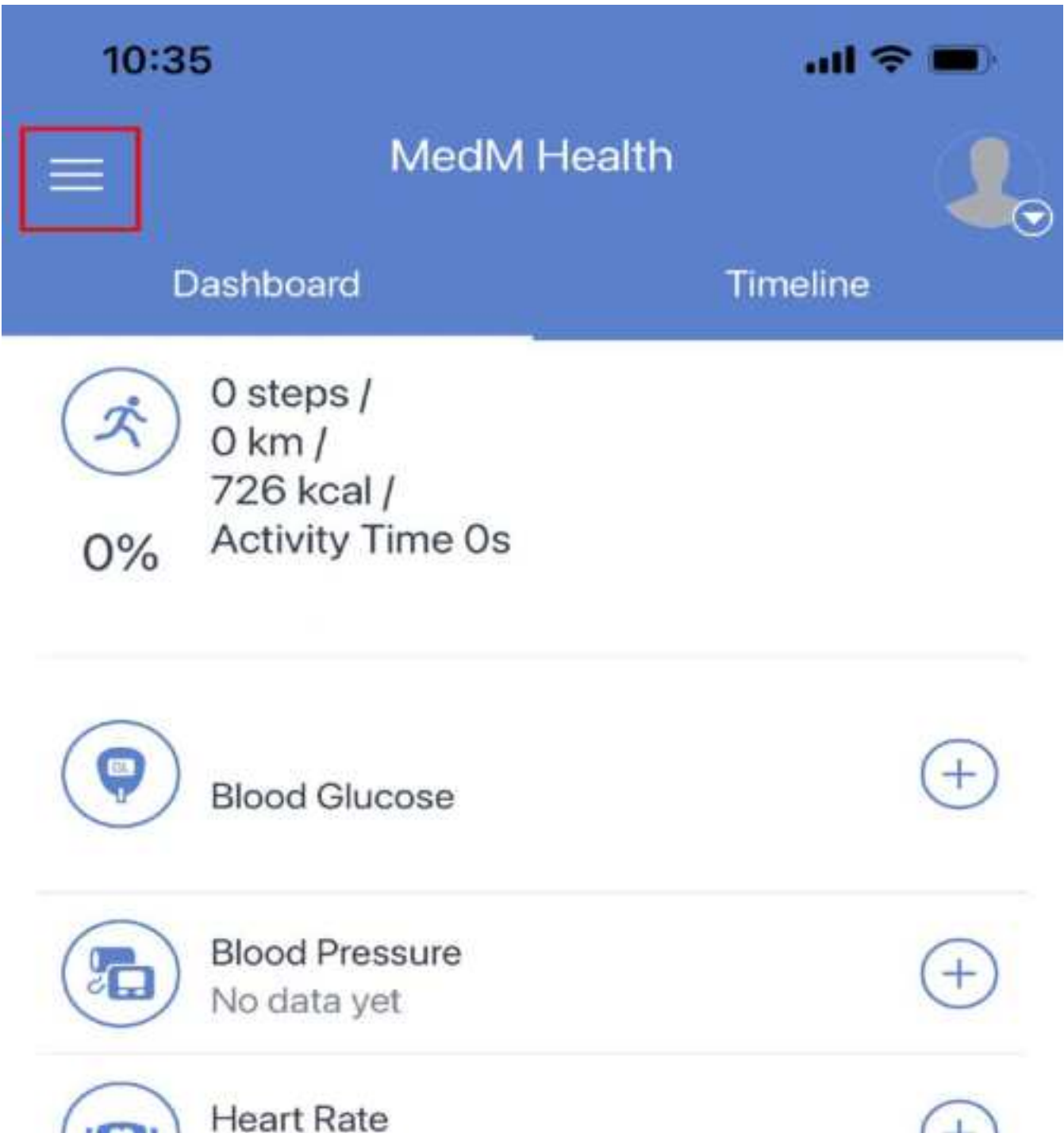

Select “My Devices”

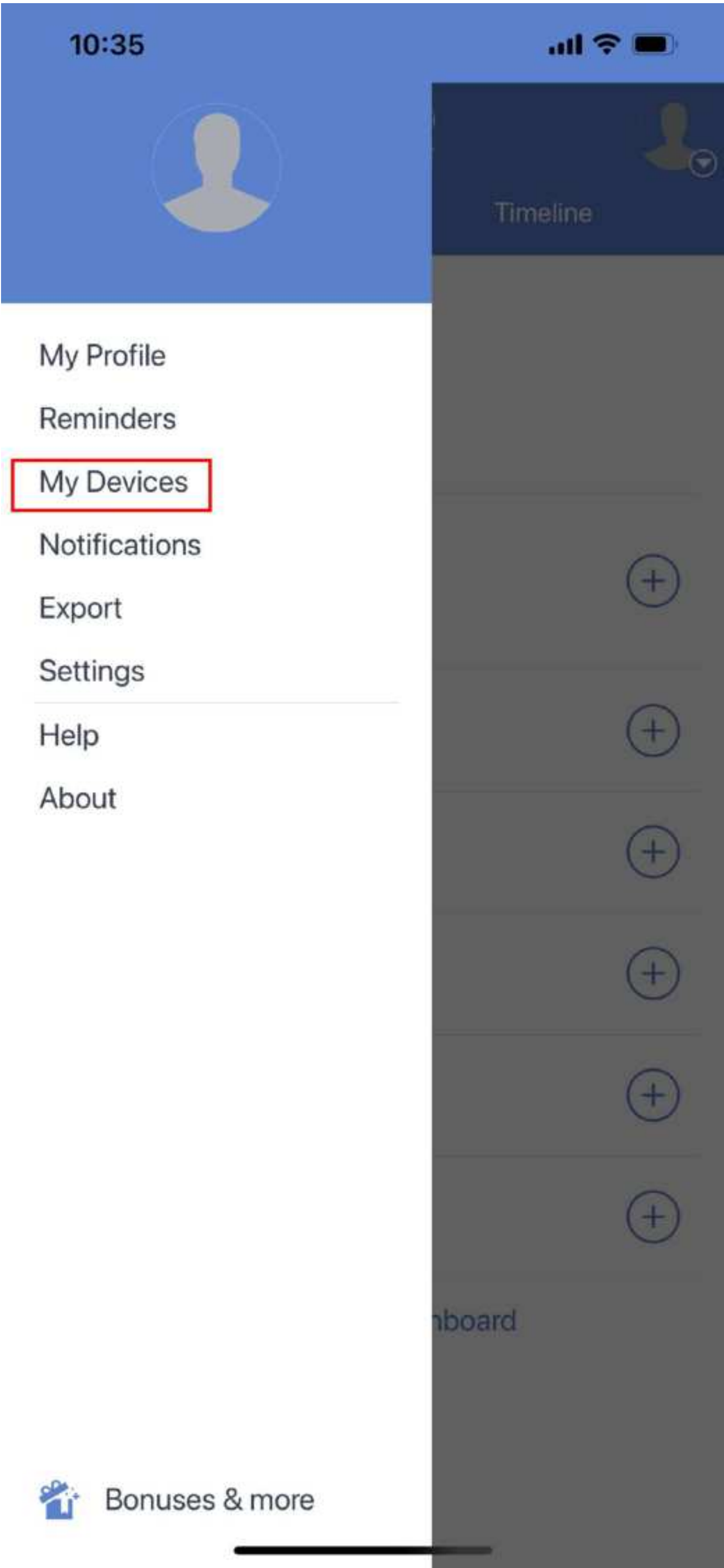

Select “ Add New”




1 Hold down the S and M buttons until the Bluetooth symbol appears.




2 The bluetooth icon flashing shows the meter is ready to pair.



BETACHEK C50 will appear when the app detects the meter. Press on the arrow.

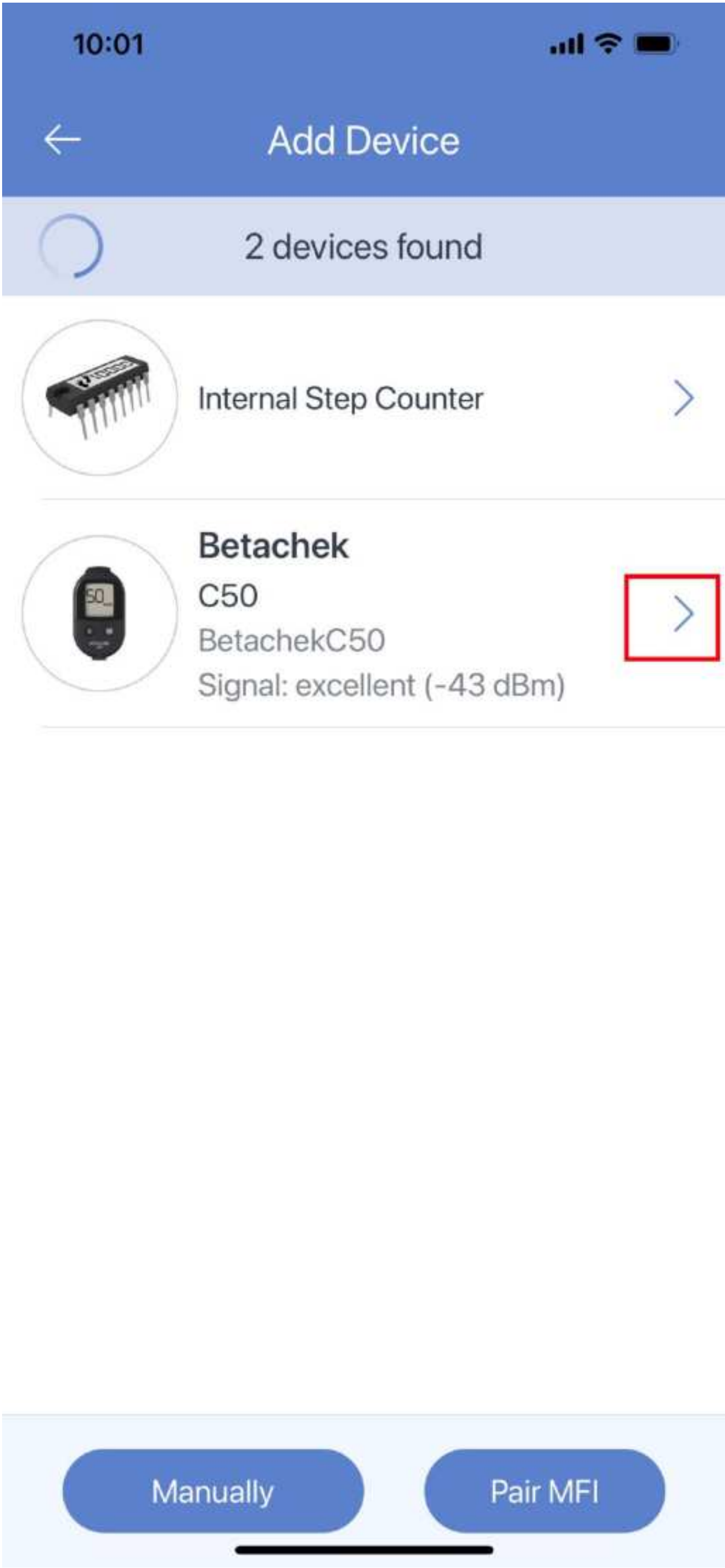

Select “Add to my Devices”

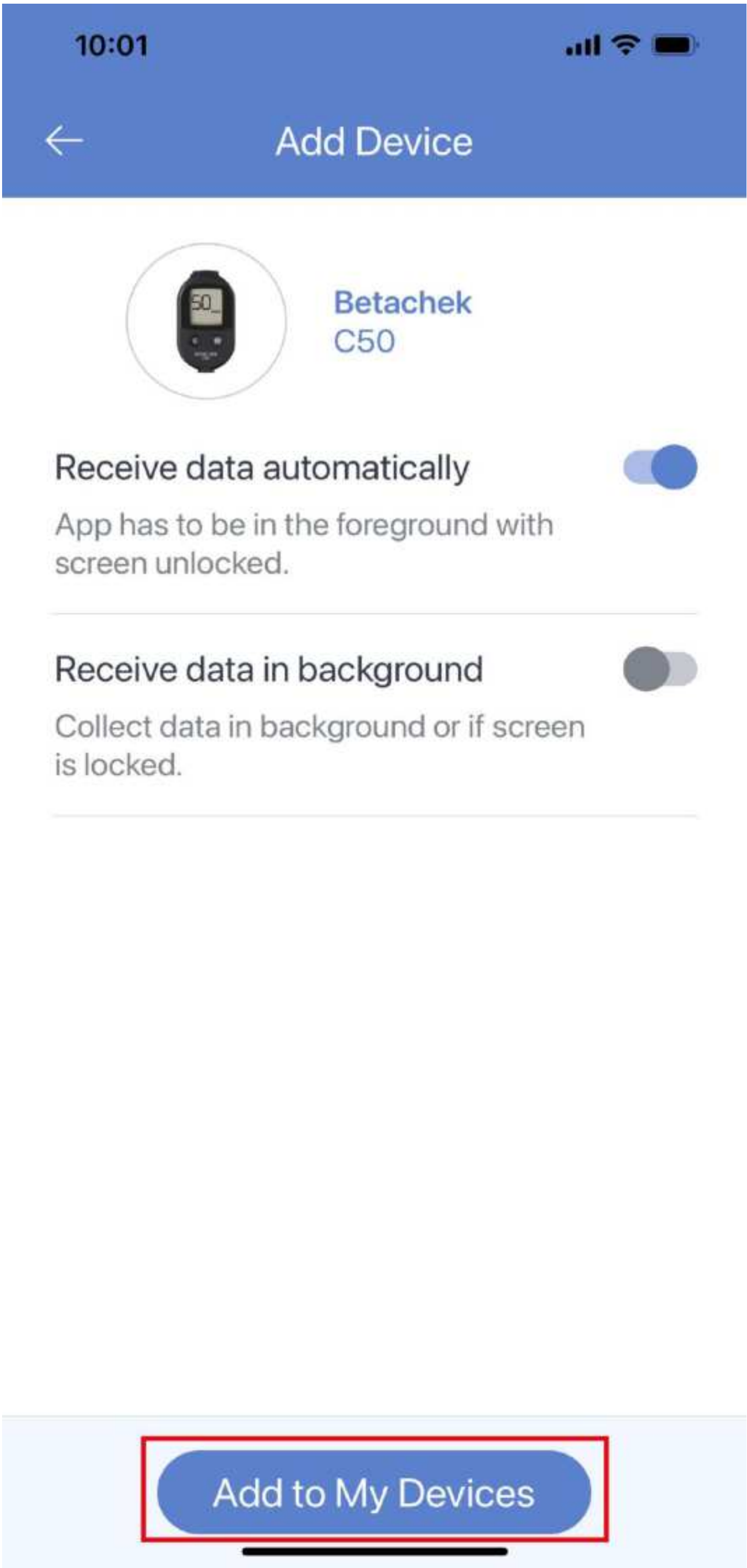

10:01



MedM Health



Dashboard

Timeline



Blood Glucose
No data yet



MedM Health



Betachek C50 is successfully
paired with MedM Health.

Make a measurement to upload
data.

Ok



Manage Dashboard

10:01



MedM Health



Dashboard

Timeline



Blood Glucose

No data yet



Blood Pressure

No data yet



Heart Rate

No data yet



Temperature

No data yet



Weight

No data yet



Manage Dashboard

Bluetooth Pairing Request

"BetachekC50" would like to pair with your iPhone.

Cancel

Pair