
User Guide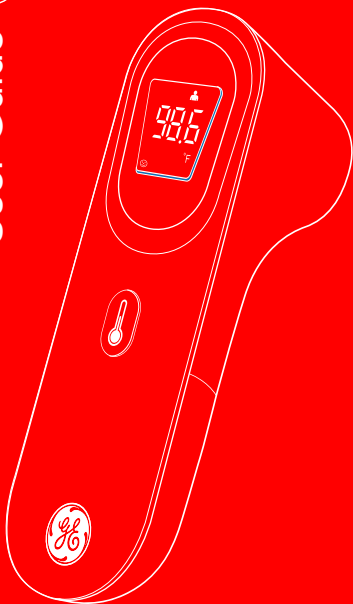

TM3000

TruVitals

Non-Touch 2-in-1 Thermometer

PDF

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Table of Contents

Introduction.....	1
Warnings / Safety Precautions.....	2
Intended Use.....	3
Features.....	3
Package Contents.....	3
Product Overview.....	4
Operation Instructions.....	6
Maintenance and Cleaning.....	12
Error Codes.....	13
Specifications.....	14
Warranty.....	15
Contact Us.....	16

FR

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Non-Touch 2-in-1 Thermometer

User Guide

Thank you for purchasing the Truvitals Non-Touch 2-in-1 Thermometer. This User Guide is intended to provide you with guidelines to ensure that operation of this product is safe and does not pose risk to the user. Any use that does not conform to the guidelines described in this User Guide may void the limited warranty.

Please read all directions before using the product and retain this guide for reference. This product is intended for household use only. It is not intended for commercial use.

This product is covered by a limited one-year warranty. Coverage is subject to limits and exclusions. See warranty for details.

Safety Precautions

- **WARNING:** This device is not intended to be used in children under the age of 2.
- This thermometer is not meant to diagnose illness or take the place of your doctor. If you are experiencing symptoms of illness, contact your physician for further guidance.
- This thermometer is designed to measure your temperature and should not replace consulting your physician.
- Dispose of this thermometer responsibly and in accordance with local regulations.
- Remove the batteries from the thermometer if it will not be used for three (3) months or longer.
- Dispose of old batteries responsibly and in accordance with local regulations.
- When replacing the batteries, change all of the batteries at the same time.
- Do not mix types or brands of batteries.
- Batteries may leak harmful liquids, ignitable materials, or may explode potentially causing personal injury or product damage.
- Do not use this thermometer where flammable gases are present.
- Inspect the thermometer before each use to make sure parts are not broken or missing.
- Do not use the thermometer outside of the proper temperature conditions to ensure accurate readings.
- Handle the thermometer with care and avoid dropping the thermometer to prevent inaccurate readings.
- Clean the thermometer sensor before each use to ensure accurate readings.
- Do not put any protective sleeves on the thermometer as this may impact the accuracy of temperature readings.
- Wait at least 30 seconds between readings to ensure accuracy.
- Keep young children in the same environment for a few minutes before taking their temperature as their bodies react slowly to environment changes.
- For the most accurate reading on children ages 3 and under,

it is recommended to take three (3) temperature readings and use the highest measurement.

- There are no user-serviceable parts inside the thermometer. Do not attempt to repair the thermometer on your own.
- Do not attempt to disassemble, overhaul, or modify this thermometer.
- Keep this thermometer out of reach of children.
- Do not dip or submerge the thermometer in water or other liquids.
- Clean the thermometer using a soft, dry cloth only. Do not use harsh detergents or liquids.
- Keep the thermometer out of direct sunlight in a dry, dust-free environment.

Intended Use

The TruVitals Non-Touch 2-in-1 Thermometer is a battery-operated electronic device with intended use of measuring body temperature through touchless contact by means of an infrared sensor coupled with electronic signal. The device is reusable and intended for home use on people aged 2 years and older.

Features

- Accurate temperature readings in just 1 second
- No contact readings to avoid the risk of contamination
- Distance sensing feature ensures accurate, consistent readings
- Visual and audio fever alerts help you identify potential health risks
- Surface mode provides temperature readings of objects like baby bottles
- Easy-to-read LCD display
- Memory feature records up to 9 measurements
- Pre-calibrated thermometer is ready to use out of the box
- 60-second auto shutoff to preserve battery life


Package Contents

- ① Non-Touch 2-in-1 Thermometer
- ② AAA 1.5 V Batteries (x2)
- ③ User Guide

Product Overview


Thermometer Overview

1. Memory Button
2. Mode Button
3. Battery Cover
4. LCD Screen
5. Measure Button
6. Measurement Sensor
7. Distance Detection Sensor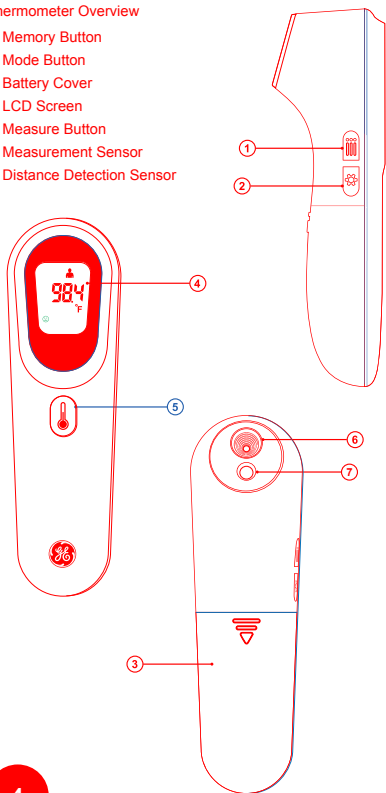

Screen Overview



1. Temperature measurement
2. Measurement in progress
3. Mute mode
4. Body mode
5. Surface mode
6. Recorded temperature in memory mode
7. Fahrenheit or Celsius mode
8. Low battery indicator
9. Temperature is equal to or greater than 100.4 °F (38 °C)
10. Temperature is less than 100.4 °F (38 °C)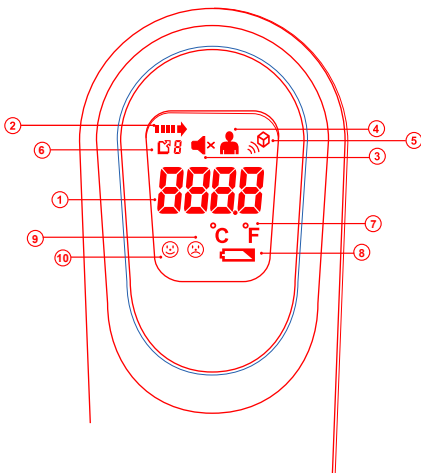

Operation Instructions

Body Mode Temperature Readings

1. Power on the thermometer by pressing the measure button. Check that the body mode indicator is lit on the screen to confirm the thermometer is in body mode. The thermometer is ready and in standby mode when two (2) blinking dashes appear on the LCD screen.

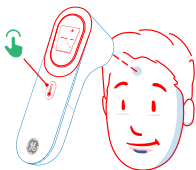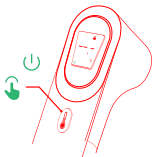
2. Hold the thermometer around 1 inch (2-3 cm) from the forehead and press the measure button.

3. Hold the thermometer still during reading. A beep will indicate that the measurement is complete (unless you are in mute mode).

4. If the thermometer is not close enough, a flashing arrow icon will appear on screen. Move the thermometer closer until the flashing arrow icon disappears, the LCD display lights up, and a temperature result appears on screen. If thermometer is not repositioned, the arrow icon will flash for 20 seconds before the device automatically turns off.

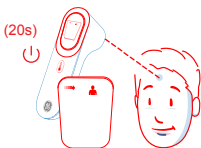5. Once a temperature is taken, the temperature will remain on screen for five (5) seconds before going back to standby mode. Simply press the measure button again to take a new temperature.

6. To power off the thermometer, wait 60 seconds for the auto shutoff feature to power off the thermometer.

NOTES: Environment changes can affect the accuracy of temperature readings. Before use, allow the thermometer to remain in one location without significant air temperature changes for 30 minutes.

Wait at least 30 seconds between readings to ensure accuracy.

In case of error code, refer to the chart on page 13.


Wait at least 30 minutes after strenuous exercise before taking a temperature reading.

Do not wear a hat and make sure there is no hair, sweat, or makeup on your forehead while taking temperature readings. To avoid human error, take multiple measurements and use the average temperature.

The flashing arrow icon lets the user know that the thermometer is currently not in range of the forehead. The flashing arrow will disappear and the thermometer will take temperatures like normal when within optimal range. If the thermometer is out of range and not repositioned, the auto shutoff will activate after 20 seconds.

Fever Alerts

If a temperature reading is above the normal range, the thermometer will give you a fever alert. A frowning face will light up and the LCD display will turn red to indicate that the temperature is equal to or greater than 100.4 °F (38 °C). A smiling face will light up on the LCD display to indicate that the temperature is less than 100.4 °F (38 °C).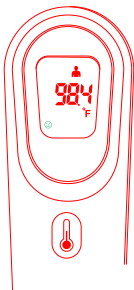

Smiling Face
Less than
100.4 °F (38.0 °C)


Frowning Face
Equal to or greater than
100.4 °F (38.0 °C)

Memory Feature


1. This thermometer automatically stores the last 9 body temperature measurements.
2. To view previous readings, press the memory button. The body or surface indicator will light up to indicate the kind of measurement you are viewing.
3. Press the memory button to scroll to the next stored measurement. While viewing the measurement, a number between 1 and 9 will display in the top-left corner of the screen to indicate which measurement you are viewing.
4. Remove the batteries from the thermometer to clear the memory.

NOTE: If the measure button is pressed while in the memory mode, the thermometer will go back to standby mode instead of immediately taking the temperature.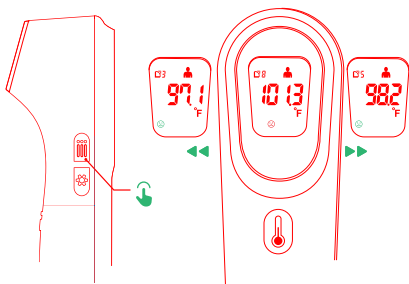

Switching Between Fahrenheit and Celsius


This thermometer can display temperatures in either degrees of Fahrenheit (°F) or degrees of Celsius (°C). To switch the units of temperature for future measurements, press the mode button. A beep will be heard (when not in mute mode) and the desired temperature unit will appear on screen. The new temperature unit will then be applied for temperatures you take in the future.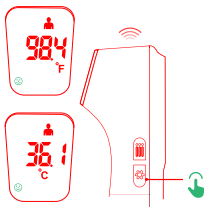

Mute Feature

To activate mute mode, press and hold the mode button for two (2) seconds. This will silence all alerts from the thermometer. To unmute the thermometer, simply repeat the process. An audio alert will indicate that the thermometer is unmuted.


NOTE: When the thermometer is powered on, the thermometer will always default to unmuted in body mode.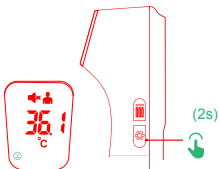

Surface Mode Temperature Readings



1. Power on the thermometer by pressing the measure button. Set the thermometer to surface mode by pressing the memory and mode at the same time. The thermometer is ready when the surface mode indicator is lit and two (2) blinking dashes appear on the LCD screen.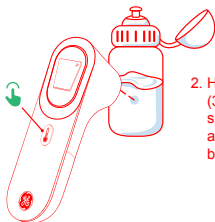

2. Hold the thermometer 1-2 in. (3-5 cm) from the object's surface you wish to measure and press the measure button.

3. Hold the thermometer still during reading. A beep will indicate that the measurement is complete (unless you are in mute mode).

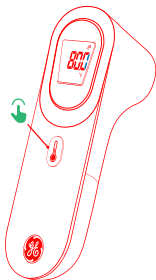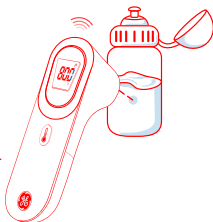4. Once a temperature is taken, the temperature will remain on screen for five (5) seconds before going back to standby mode. Simply press the measure button again to take a new temperature. Make sure to hold the thermometer close to the object you want to measure because in surface mode, the thermometer will always pick up and read a temperature.

5. Hold the thermometer 1-2 in. (3-5 cm) from the object's surface you wish to measure and press the measure button.

NOTE: In case of error code, refer to the chart on page 13.



Maintenance & Cleaning

Battery Replacement

When the batteries need to be replaced, a blinking low battery icon will display on the screen. To change the batteries, follow the steps below.

1. Slide the battery cover down to open the battery compartment.
2. Remove the old batteries and dispose of them properly.
3. Insert two (2) new batteries aligning them with the polarities shown in the battery compartment.
4. Slide the battery cover back into place before powering on the thermometer.

NOTE: Remove the batteries if the thermometer will not be in use for more than three (3) months.

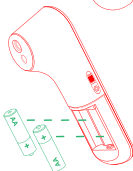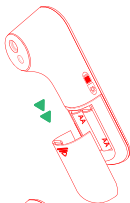Cleaning

To clean the thermometer, wipe it with a dry, soft cloth. If the thermometer may be contaminated, use a cloth moistened with water and a mild detergent or clean it with 75% alcohol.

Take care not to use an excessive amount of liquid when cleaning the thermometer as liquids can seep into the electronic parts and cause damage to the thermometer.

Storage

Store in a safe, dry place free from dust and direct sunlight. Keep this thermometer out of reach of children.

If you are storing the thermometer for more than three (3) months, remove the batteries and keep the thermometer in the original packaging.

Error Codes

Error Code	Code Meaning	Resolution
HI	The temperature measured is higher than the following: Body Mode: 109.4 °F (43 °C) Surface Mode: 212 °F (100 °C)	Operate the thermometer in appropriate conditions. If necessary, clean the sensor tip. Should the error persist, contact customer service.
LO	The temperature measured is lower than the following: Body Mode: 93.2 °F (34 °C) Surface Mode: 32 °F (0 °C)	Operate the thermometer in appropriate conditions. If necessary, clean the sensor tip. Should the error persist, contact customer service.
Err	The ambient temperature is outside of the usable range of 59 °F - 104 °F (15 °C - 40 °C).	Operate the thermometer in appropriate conditions. If necessary, clean the sensor tip. Should the error persist, contact customer service.

Specifications

Operating Specifications

Body Mode Range: 93.2 °F - 109.4 °F (34 °C - 43 °C)

Surface Mode Range: 0 °F – 212 °F (0 °C - 100 °C)

Body Mode Calibration Accuracy: ± 0.4 °F (± 0.2 °C) between 93.2 °F - 109.4 °F (34 °C – 43 °C)

Surface Mode Calibration Accuracy: ≤ 68 °F (20 °C) ± 2 °F (1 °C); > 68 °F (20 °C) $\pm 5\%$

Operating Temperature: 59 °F - 104 °F (15 °C - 40 °C)

Operating Humidity: $< 95\%$ RH

Operating Atmospheric Pressure: 70 kPa ~ 106 kPa

Storage Temperature: -13 °F - 131 °F (-25 °C - 55 °C)

Storage Humidity: $< 95\%$ RH

Storage Atmospheric Pressure: 70 kPa ~ 106 kPa

Memory Capacity: 9 readings

Thermometer Specifications

Dimensions (L x W x D): 5.5" x 1.7" x 1.7" (140 x 42 x 42 mm)

Weight (with batteries): 4.2 oz. (120 g)

Power Supply: 2x AAA 1.5 V batteries

Display Type: LCD

Warranty

LIMITED WARRANTY TO ORIGINAL CONSUMER

This Truvitals Non-Touch 2-in-1 Thermometer ("Product"), including any accessories included in the original packaging, as supplied and distributed new by an authorized retailer is warranted by C&A Marketing, Inc. (the "Company") to the original consumer purchaser only, against certain defects in material and workmanship ("Warranty") as follows:

To receive Warranty service, the original consumer purchaser must contact the Company or its authorized service provider for problem determination and service procedures. Proof of purchase in the form of a bill of sale or receipted invoice, evidencing that the Product is within the applicable Warranty period(s), MUST be presented to the Company or its authorized service provider in order to obtain the requested service.

Service options, parts availability, and response times may vary and may change at any time. In accordance with applicable law, the Company may require that you furnish additional documents and/or comply with registration requirements before receiving warranty service. Please contact our customer service for details on obtaining warranty service:

866-883-0022

info@truvitals.com

Shipping expenses to the Company's Return Facility are not covered by this warranty, and must be paid by the consumer. The consumer likewise bears all risk of loss or further damage to the Product until delivery to said facility.

EXCLUSIONS AND LIMITATIONS The Company warrants the Product against defects in materials and workmanship under normal use for a period of ONE (1) YEAR from the date of retail purchase by the original end-user purchaser ("Warranty Period"). If a hardware defect arises and a valid claim is received within the Warranty Period, the Company, at its sole option and to the extent permitted by law, will either (1) repair the Product defect at no charge, using new or refurbished replacement parts, (2) exchange the Product with a Product that is new or which has been manufactured from new or serviceable used parts and is at least functionally equivalent to the original device, or (3) refund the purchase price of the Product.

A replacement Product or part thereof shall enjoy the warranty of the original Product for the remainder of the Warranty Period, or ninety (90) days from the date of replacement or repair, whichever provides you longer protection. When a Product or part is exchanged, any replacement item becomes your property, while the replaced item becomes the Company's property. Refunds can only be given if the original Product is returned.

This Warranty does not apply to:

- (a) Any non-Truvitals Non-Touch 2-in-1 Thermometer product, hardware or software, even if packaged or sold with the Product;
- (b) Damage caused by use with non-Truvitals Non-Touch 2-in-1 Thermometer products;
- (c) Damage caused by accident, abuse, misuse, flood, fire, earthquake, or other external causes;
- (d) Damage caused by operating the Product outside the permitted or intended uses described by the Company;
- (e) Damage caused by third party services;
- (f) A Product or part that has been modified to alter functionality or capability without the written permission of the Company;
- (g) Consumable parts, such as batteries, fuses, and bulbs;

Truvitals Non-Touch 2-in-1 Thermometer

- (h) Cosmetic damage; or
- (i) If any Truvitals Non-Touch 2-in-1 Thermometer serial number has been removed or defaced.

This Warranty is valid only in the country where the consumer purchased the Product, and only applies to Products purchased and serviced in that country.

The Company does not warrant that the operation of the Product will be uninterrupted or error-free. The Company is not responsible for damage arising from your failure to follow instructions relating to its use.

NOTWITHSTANDING ANYTHING TO THE CONTRARY AND TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW, THE COMPANY PROVIDES THE PRODUCT "AS-IS" AND "AS-AVAILABLE" FOR YOUR CONVENIENCE AND THE COMPANY AND ITS LICENSORS AND SUPPLIERS EXPRESSLY DISCLAIM ALL WARRANTIES AND CONDITIONS, WHETHER EXPRESSED, IMPLIED, OR STATUTORY, INCLUDING THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, TITLE, QUIET ENJOYMENT, ACCURACY, AND NON-INFRINGEMENT OF THIRD-PARTY RIGHTS. THE COMPANY DOES NOT GUARANTEE ANY SPECIFIC RESULTS FROM THE USE OF THE PRODUCT, OR THAT THE COMPANY WILL CONTINUE TO OFFER OR MAKE AVAILABLE THE PRODUCT FOR ANY PARTICULAR LENGTH OF TIME. THE COMPANY FURTHER DISCLAIMS ALL WARRANTIES AFTER THE EXPRESS WARRANTY PERIOD STATED ABOVE.

YOU USE THE PRODUCT AT YOUR OWN DISCRETION AND RISK. YOU WILL BE SOLELY RESPONSIBLE FOR (AND THE COMPANY DISCLAIMS) ANY AND ALL LOSS, LIABILITY, OR DAMAGES RESULTING FROM YOUR USE OF THE PRODUCT.

NO ADVICE OR INFORMATION, WHETHER ORAL OR WRITTEN, OBTAINED BY YOU FROM THE COMPANY OR THROUGH ITS AUTHORIZED SERVICE PROVIDERS SHALL CREATE ANY WARRANTY.

IN NO EVENT WILL THE COMPANY'S TOTAL CUMULATIVE LIABILITY ARISING FROM OR RELATED TO THE PRODUCT, WHETHER IN CONTRACT OR TORT OR OTHERWISE EXCEED THE FEES ACTUALLY PAID BY YOU TO THE COMPANY OR ANY OF ITS AUTHORIZED RESELLERS FOR THE PRODUCT AT ISSUE IN THE LAST YEAR FROM YOUR PURCHASE. THIS LIMITATION IS CUMULATIVE AND WILL NOT BE INCREASED BY THE EXISTENCE OF MORE THAN ONE INCIDENT OR CLAIM. THE COMPANY DISCLAIMS ALL LIABILITY OF ANY KIND OF ITS LICENSORS AND SUPPLIERS. IN NO EVENT WILL THE COMPANY OR ITS LICENSORS, MANUFACTURERS AND SUPPLIERS BE LIABLE FOR ANY INCIDENTAL, DIRECT, INDIRECT, SPECIAL, PUNITIVE, OR CONSEQUENTIAL DAMAGES (SUCH AS, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS, BUSINESS, SAVINGS, DATA, OR RECORDS) CAUSED BY THE USE, MISUSE, OR INABILITY TO USE THE PRODUCT.

Nothing in these terms shall attempt to exclude liability that cannot be excluded under applicable law. Some countries, states, or provinces do not allow the exclusion or limitation of incidental or consequential damages or allow limitations on warranties, so certain limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may have other rights that vary from state to state or province to province. Contact your authorized retailer to determine if another warranty applies.

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AVITA Corporation

Model No.: NT2


Distributed by: AVITA (WuJiang) Co. LTD.

Tel: +86-512-8207-8288

9F, No.78, Sec.1, Kwang-Fu Road, San-Chung District,
New Taipei City, 24158, Taiwan, R.O.C.

Registration Certificate(s): SXZZ20182200808

China Manufacturing License(s): SSYJXSCX20040018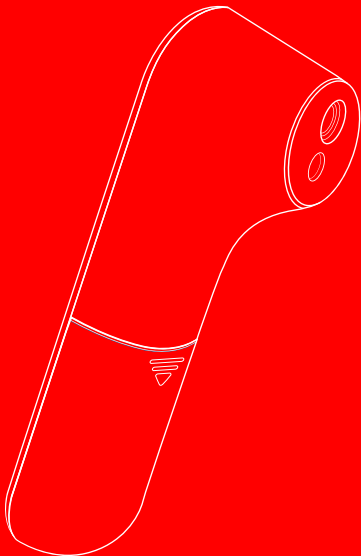

Truvitals is a trademark of C&A IP Holdings LLC in the US.

GE is a trademark of General Electric Company. Used under license.

All other products, brand names, company names, and logos are trademarks of their respective owners, used merely to identify their respective products, and are not meant to connote any sponsorship, endorsement, or approval.

Distributed by C&A Marketing, Inc., 114 Tived Lane East Edison, NJ 08837.

Made in China.

© 2022. C&A IP Holdings LLC. All Rights Reserved.