Etac My-Loo

Rekonditioneringsanvisning

Reconditioning instruction

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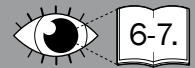
Reconditioning instruction / Re-use

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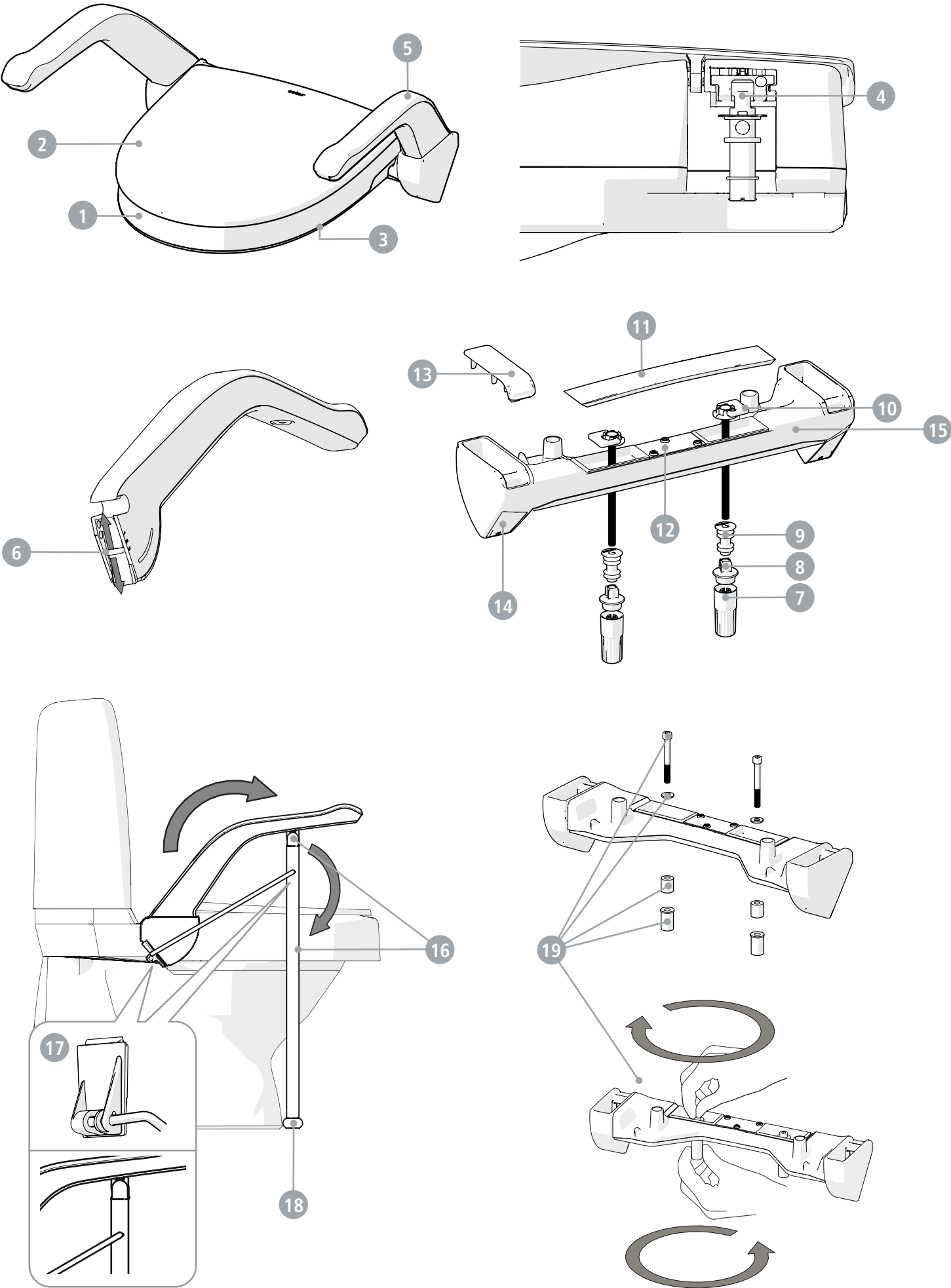
General

- Etac My-Loo may be further used.
- Original spare parts from Etac should always be used.
- Remember that the service life of the product is limited (see


manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

Washing

- Clean the product with normal nonabrasive cleaning agents (pH value of between 5-9). Rinse and dry.
- Disinfect the product with an alcohol-based disinfectant, 70%.
- The product can withstand cleaning in a sterilizer at 85 °C for 3 minutes.

1	Raiser
	Check so that there are no cracks or damages.
2	Lid
	Check so that the lid is properly fixed and can be folded.
	Check so that there are no cracks or damages.
3	Friction pieces
	Check so that the friction pieces are properly fixed and that there are no damages.
4	Raiser fixing
	Check so that the raiser is fixed and can be folded up and down
5	Arm supports
	Check so that there are no cracks or damages.
	Check so that the arm support can be folded up and down
6	Height control
	Check so that there are no cracks or damages.
	Check so that the height control can be set in positions 1-3.
7	Assembly nut
	Check so that there are no cracks or damages.
	Check so that the assembly nut can be tightened.
8	Sleeve
	Check so that there are no cracks or damages.
9	Bushing
	Check so that there are no cracks or damages.
10	Locking handle
	Check so that there are no cracks or damages.
	Check so that the locking handle is fixed
11	Cover
	Check so that there are no cracks or damages.
	Check so that the cover can be clicked into place

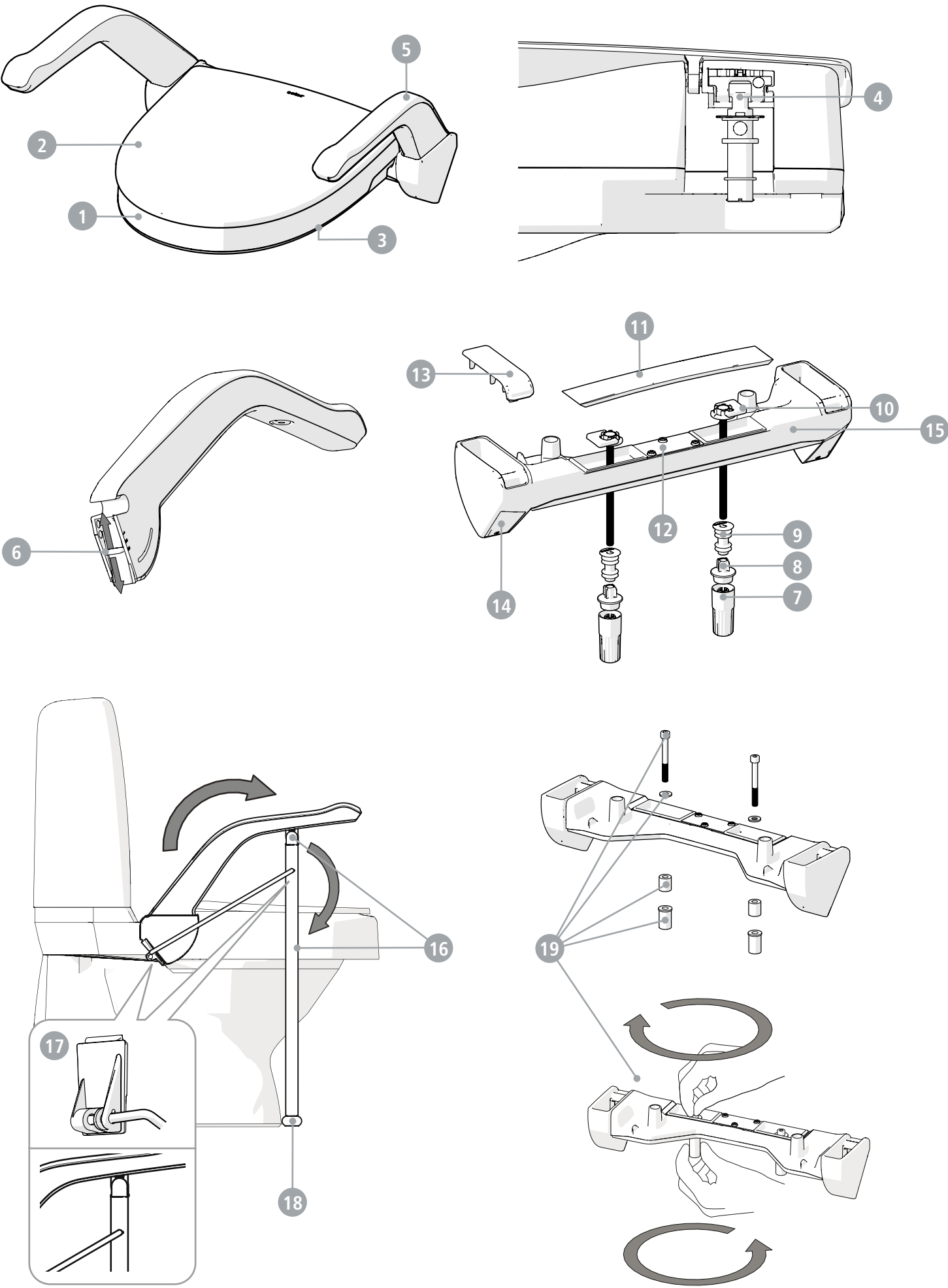12	Cover holder
	Check so that there are no cracks or damages.
	Check so that the cover can be clicked into place
13	Cover, arm support holder
	Check so that there are no cracks or damages.
	Check so that the cover can be clicked into place
14	Holster cover
	Check so that there are no cracks or damages.
	Check so that the cover is fixed
15	Attachment
	Check so that there are no cracks or damages.
16	Support leg
	Check so that the arm support can be folded
	Check so that the support leg attachment is ok and fixed
17	Support leg, parallel rod
	Check so that the parallel rod has no damages and is properly fixed
	Check so that the parallel rod attachment has no damages and is properly fixed
18	Support leg, adjustable ferrule
	Check so that the adjustable ferrule has no damages and can be adjusted
19	Screw kit for top mounting
	Check so that there are no cracks or damages in screws, washers, spacers and plugs.
	Check so that screw and plug can be mounted

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