

Blaid's Hearing Aids User Manual

Blaid's Hearing Aid - BTE

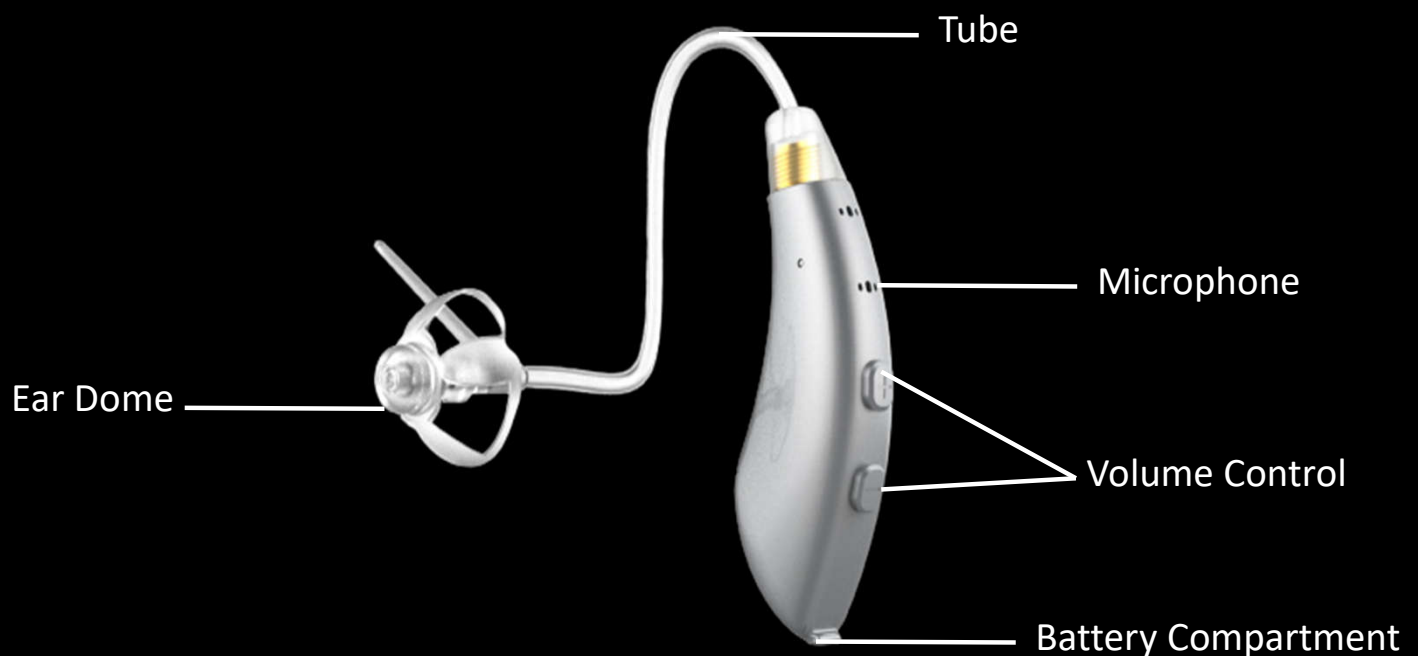


Table of Contents

Battery Usage Guide

- A. Downloading Mobile Application
- B. Connecting Hearing Aids
- C. Basic Controls
- D. Taking the hearing test
- E. Uploading Audiogram
- F. Trouble Shooting

Battery Usage Guide

- Your Blais Hearing aids use standard 312 size batteries. These batteries are commonly available and can be purchased from grocery stores or pharmacies.
- To insert the battery, Open the battery compartment and add the battery. You will hear 2 beeps once the battery is inserted in the battery compartment.




Battery Compartment

A. Downloading the Mobile Application


Blaid's Hearing Aids can be connected and controlled directly from your mobile phone. In order to get started, please download the mobile application from the iOS or Android app store on your mobile phone.

In order to download the app, please follow the following steps:

1. Open your App store on your mobile phone
2. Search for "Acosound" in the search box
3. Download the app

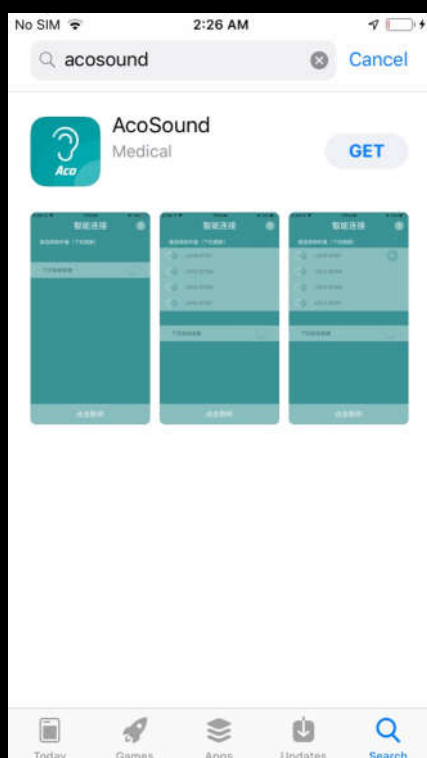

Search & Download
the application

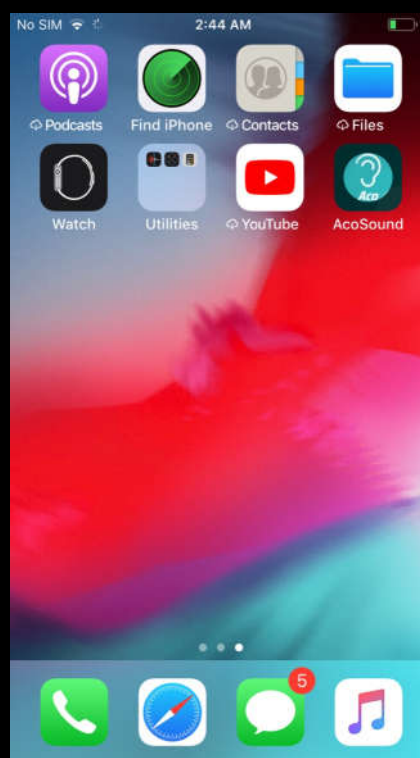


Initialize the app by
clicking the Icon


B. Connecting Hearing Aids with Mobile Application

Your hearing aids will be automatically detected by the mobile application. *Before connecting the hearing aids with mobile application, please ensure that you have inserted the Battery in the hearing aid.*


1. Click on the mobile application to initialise it
2. Your hearing aids will automatically be detected by the mobile app. *In case you do not see your hearing aids in the app, pull down the screen and Release to Refresh*
3. Click on the hearing aid to initiate connection with Mobile phone. Choose the position of the hearing aid as per your ear fitting

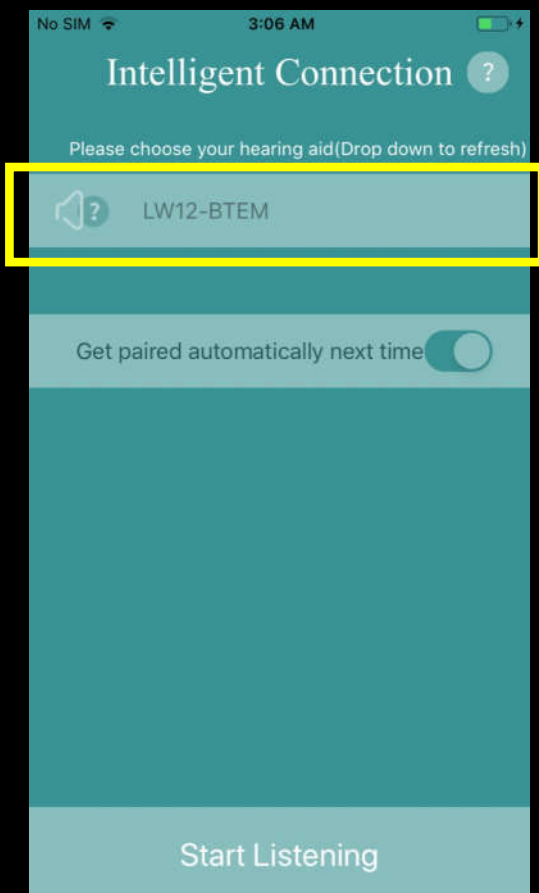


Hearing Aid detected by App

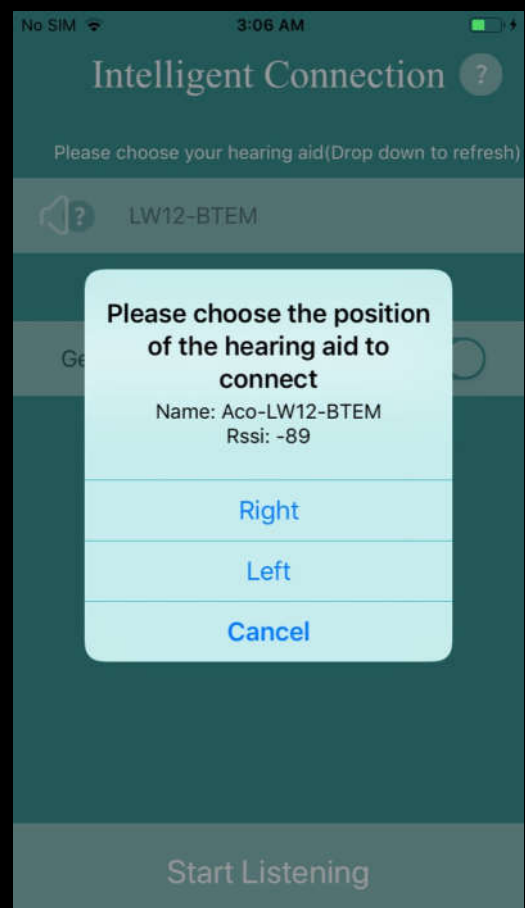

Click on Hearing aid & choose Position

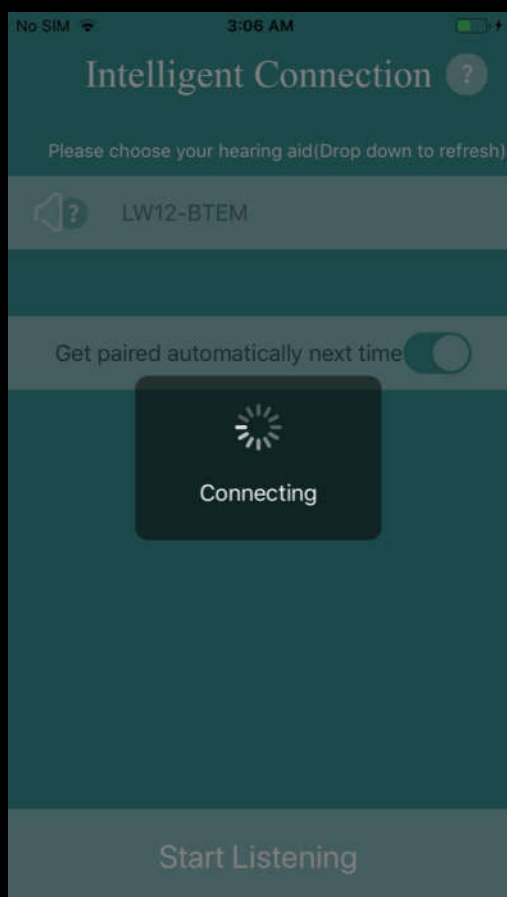

Hearing Aid connecting to app

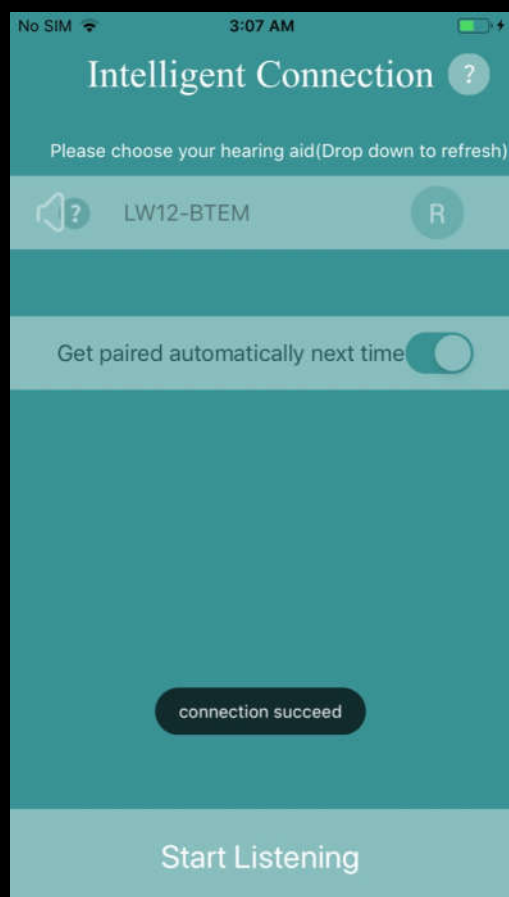


Connection Successful

C. Basic control in the Mobile App

The mobile application lets you control the hearing aids as well as take the hearing test directly from the comfort of your home.

1. After connecting to the hearing aids, click on “Start Listening” button to access the controls
2. Toggle the slider to set the volume
3. Choose from any one of the 4 preset modes to suit your environmental conditions

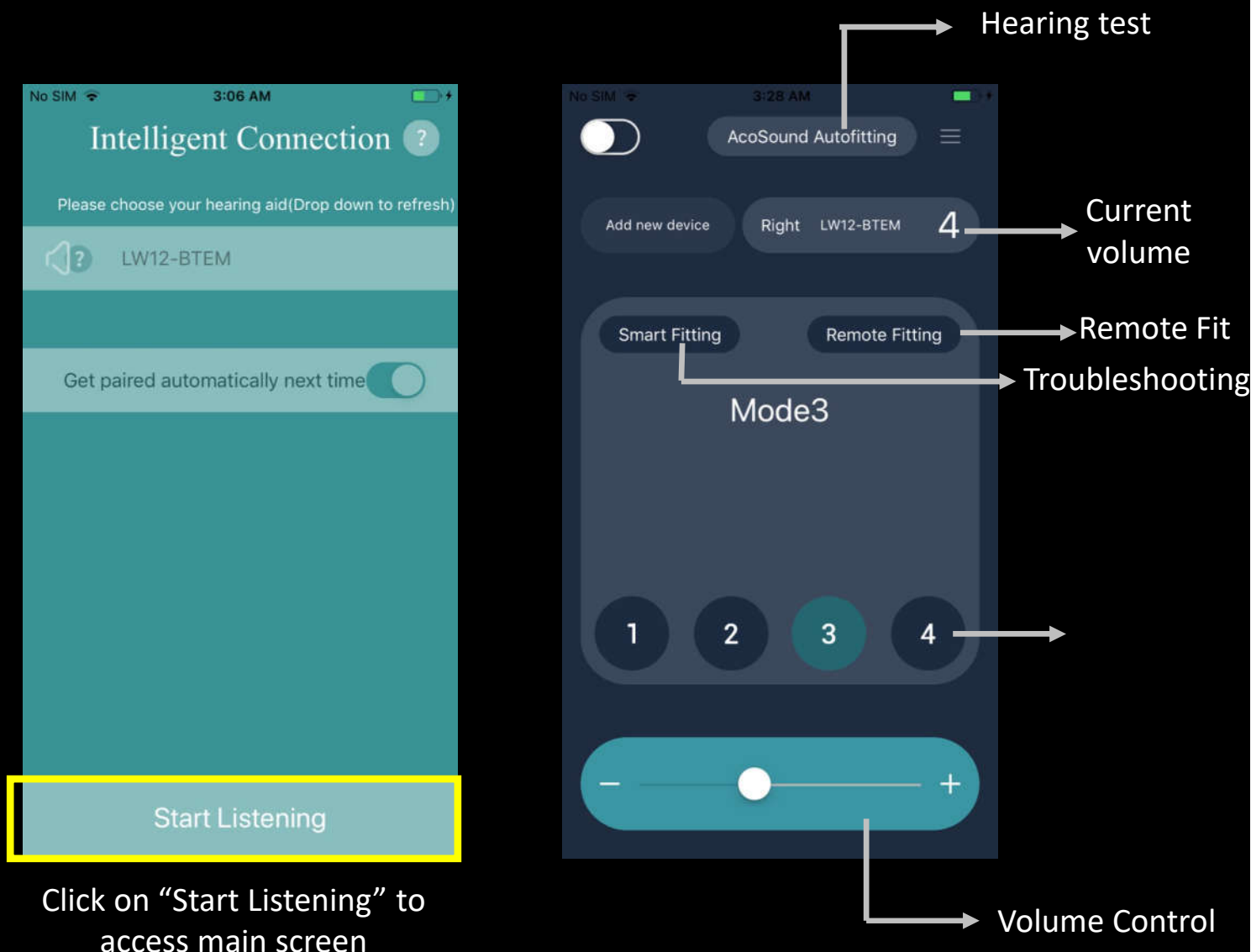


D. Taking the Hearing Test


Blaid's allows you to take a hearing test directly through the mobile app and generate a personalised Audiogram.

1. Click on "Acosound Autofitting" on top centre of the mobile app
2. Select the ear "Left" or "Right" for which you want to take the hearing test
3. Click on "Listening test" to initiate testing.


Please ensure that the hearing aid is inserted in your ear and you select the correct ear before starting the hearing test.

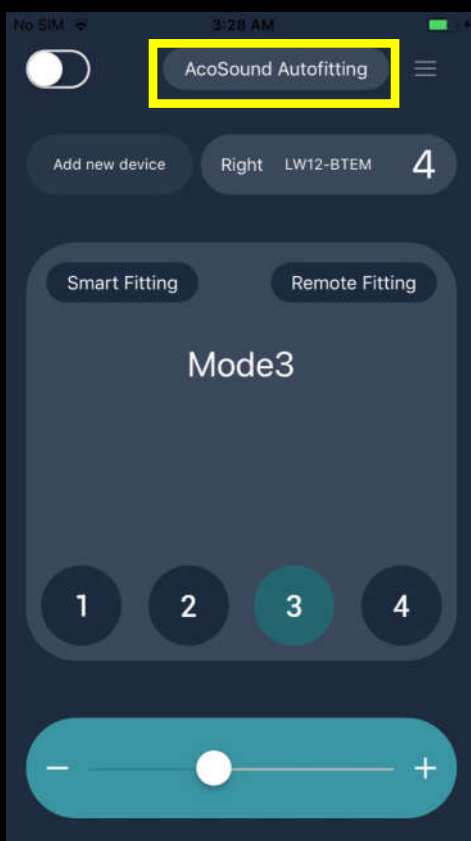

Click "Acosound Autofitting"

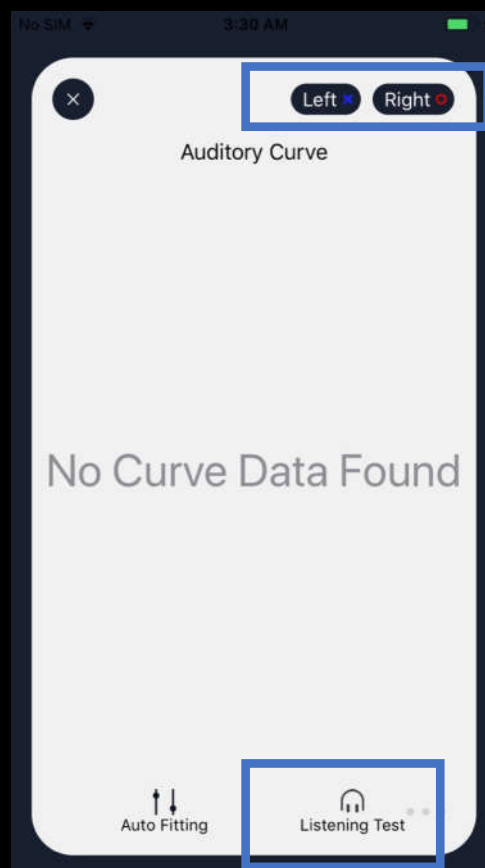

Select ear and click "Listening Test"

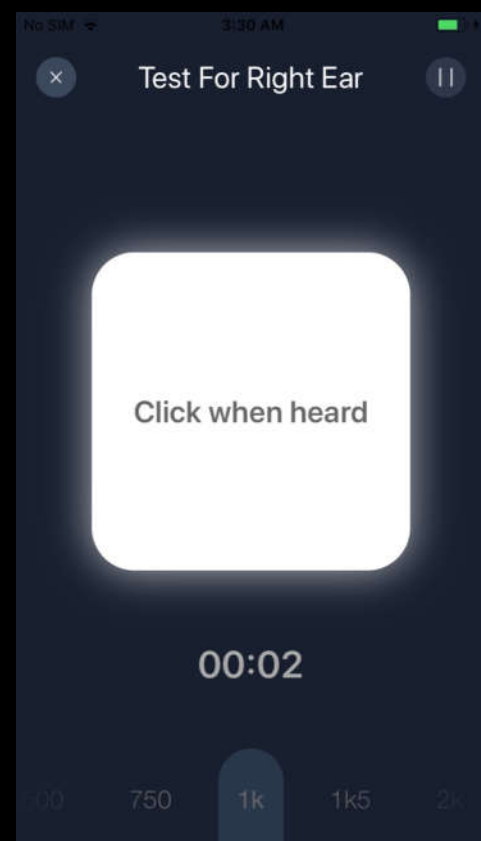


Start Hearing test


E. Uploading Audiogram

You can use the mobile application to upload your existing audiogram, required for remote fitting support

1. Click on “Acosound Autofitting” button on the mobile app
2. Click on the 3 dots on the bottom right corner of the screen
3. Select the “Custom Curve” option to upload your Audiogram

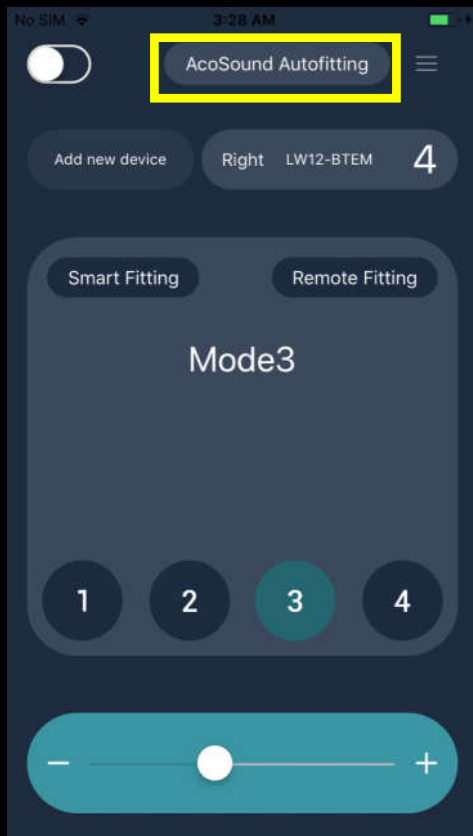

Click “Acosound Autofitting”

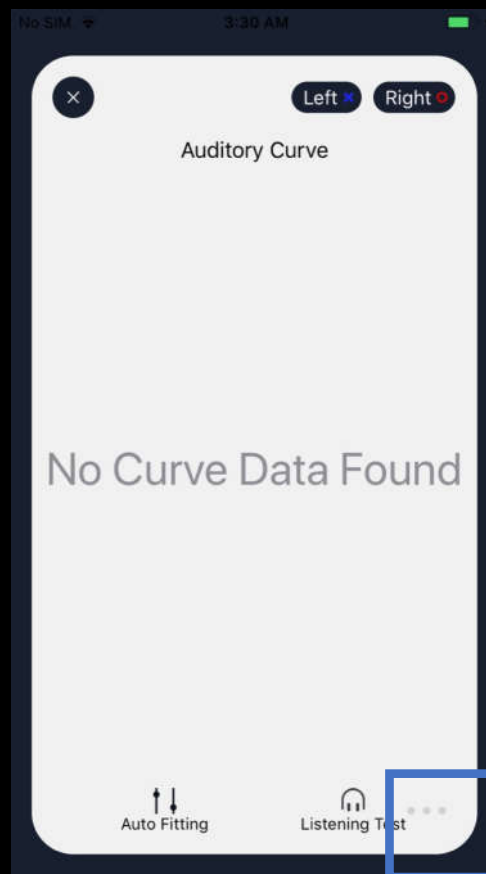

Select ear and click “Listening Test”




Select Custom Curve to Input your Audiogram


F. Trouble Shooting your Hearing Aid

You can use the mobile application to perform basic troubleshooting for your hearing aid


1. Click on “Smart Fitting” button on the mobile app
2. Select the present problem that you are facing in your hearing aids and the app will automatically adjust the settings for your device
3. Click on “Advanced Regulation” to set control advance settings like Treble, Bass, Suppression of Loud Voice or Enhancement of Soft Voice.

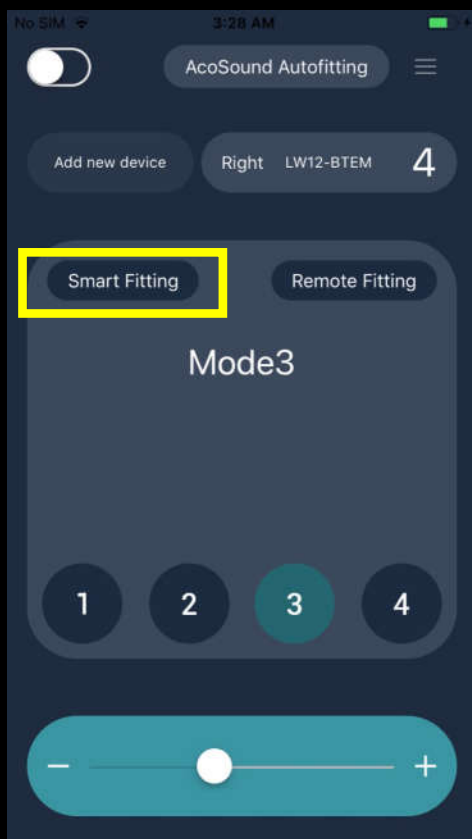

Click “Smart Fitting”

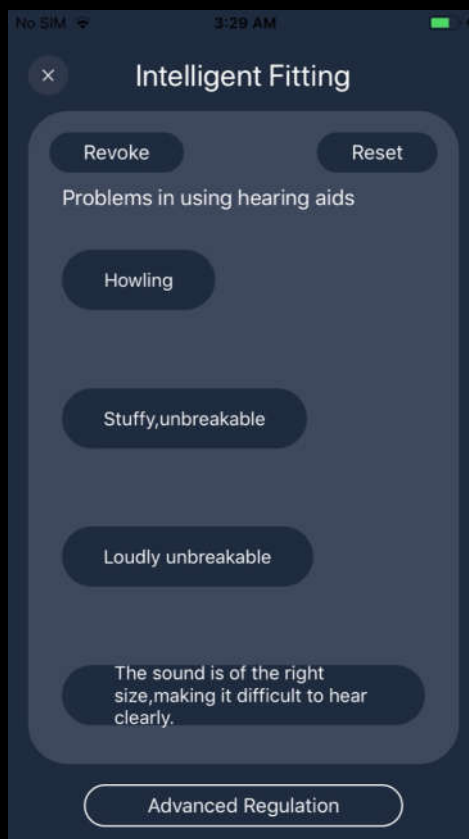

Solve specific issues in
Hearing aids

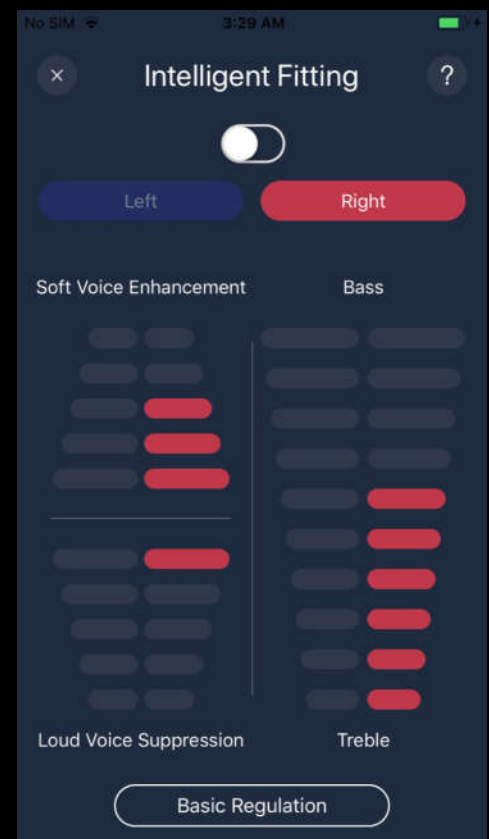

Set advanced features