



MySPM
version 2.0 of 28/09/2020

USER GUIDE

THE FUTURE OF STIMULATION

Welcome to the future and thank you for choosing **Shoulder Pacemaker™** and **MySPM App**.


Shoulder Pacemaker™ is a wearable electrostimulation device used on patients with abnormal scapulohumeral rhythm.

MySPM App is an easy and effective wizard that tracks and records the rehab process with Shoulder Pacemaker™; it should be used only as an accessory of Shoulder Pacemaker™ device, in KIT wireless configuration.

NOTE: This manual is to be considered as an accessory to the device and therefore must always be in support of the same. Please read carefully but also refer to the full manual of the device Shoulder Pacemaker™ in reference to indications, warnings and precautions.

The device must be used by a properly trained healthcare professional.


1

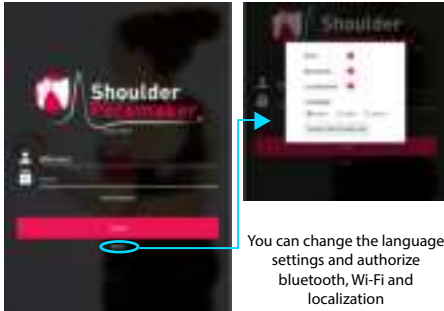

You can change the language settings and authorize bluetooth, Wi-Fi and localization


Log in with the provided credentials

2

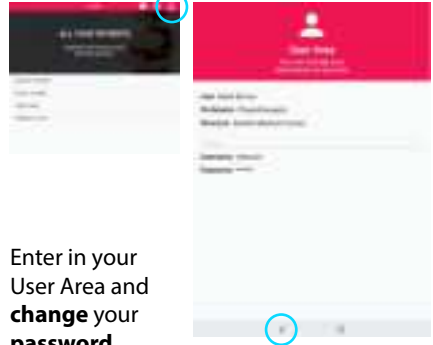

Enter in your User Area and **change** your password

3

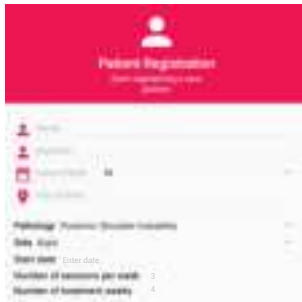

Register your patients

4




Save the code

123456

It will be used to continue the therapy at home

View the patient's **calendar** and the **unique code** associated

5

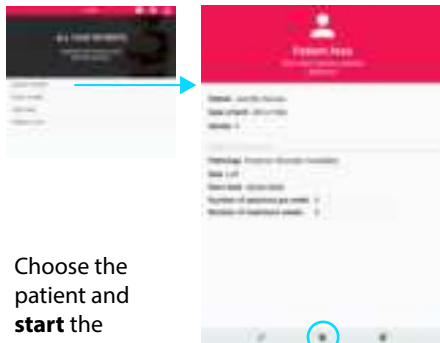

Choose the patient and
start the session

6




Make sure to **correctly place** the device and its accessories

7



Connect the Shoulder Pacemaker™

8

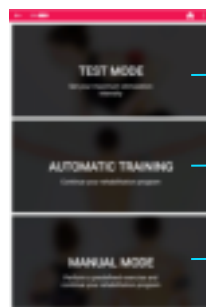

Choose the session mode