



Infant Size 3-18 months.

NOTE: The Mommy's Helper Hush Gear ear muffs **ARE NOT A TOY** and should only be used under adult supervision for noise abatement purposes only.

Instructions for use:

- 1) Use the headband and slide through the clips that are attached to the earmuff.
- 2) This is an adjustable band, please be sure that the band is secured at the closure site.
- 3) Ensure that the Hush Gear Hearing Protectors are adjusted to fit the ear so that they are secure and comfortable.
- 4) The band should be placed on the forehead and not the eyebrow line.
- 5) Be sure to loosen or tighten as needed. For maximum efficiency the hearing protectors should seal around the ear completely.
- 6) When used correctly, the Hush Gear Hearing Protectors are designed to protect ears from potentially harmful noise.
- 7) When cleaning, please remove the acoustic foam and use warm water and mild soap to wipe clean the ear cup.
- 8) Dry thoroughly.

Tested in compliance with CPSC & CPSIA Child Safety Standards.

Noise Reduction Testing: Compliant with ISO 4869-3 testing standards. Insertion loss measured in accordance with EN13819-2, Section 4.1 on acoustic test fixture. Average 28.6 dB noise reduction over a frequency range of 125 – 8000.

Frequency in Hz	125	250	500	1000	2000	4000	8000
Mean Attenuation in dB	9.2	8.0	24.5	39.4	36.1	38.3	41.9

This instruction sheet applies to item #41912; Hush Gear Hearing Protection Infants Pink, Blue and Gray.

Mommy's Helper, Inc
Wichita, KS 67208
USA
800-371-3509
Made in China