MOKA

Gebrauchsanweisung - Instructions for use - Gebruiksaanwijzing
Upute za korištenje - Navodila za uporabo

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Table of contents

Preface	9
Declaration of conformity	9
Service life	9
Disposal	9
Identification plate	9
Reuse	9
Warranty	9
Parts of the product	10
Indication	11
Scope of delivery	11
Safety instructions	11
Unfolding	12
Cane Holder	12
Adjusting the brakes	12
Braking Parking	12
Sitting	12
Curb Climber	13
Folding Transporting Stowing	13
Cleaning Maintenance	13
Technical data	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Accessible instructions for use in easy and/or simple language are not available from the manufacturer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.com

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

Parts of the product

MOKA

- | | | |
|----------------|---|-------------------|
| 1. Handle | 4. Push button to adjust the push handles | 7. Steering wheel |
| 2. Push handle | 5. Seat | 8. Pocket |
| 3. Brake | 6. Rear wheel | 9. Back support |

Indication

INDICATION:

Walking aid for outdoor and indoor use to compensate for or improve limited walking ability. Insured parties are to be provided with walking aids with the intention of compensating for or at least improving the user's limited walking ability (expansion of radius of activity or mobility). Persons with limitations of hand/arm functions require particular adjustments/designs of handles or handrests/armrests. Movable walking aids may not be suitable for persons with balance disorders; in individual cases, this should be tested out. Where appropriate, an attendant must be present. The rollator is to be used only for normal, everyday activities.

CONTRAINDICATION:

The rollator is not suitable for use in the case of: perception disorders, severe loss of balance, inability to sit down.

Scope of delivery

1 x	rollator
1 x	pocket
1 x	back support
1 x	cane holder
1 x	instructions for use


Safety instructions

- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use the tray only for small, lightweight objects which are not sharp or hot. Max. load: 5.0 kg
- Use the basket only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake (pull the brake lever upwards).
- Leaning excessively on the handles can cause the rollator to tip backwards.

- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.
- The braking effect can be impaired by wear and tear on the tires. For this reason, check the braking effect and tires regularly.
- The rollator should NOT be carried by the seat, removable parts or the brake cables.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Assembly


- The rollator is delivered folded (**pic. 1**)
- The handles are lowered to their lowest position
- Place the rollator on a flat surface
- Unscrew the knobs (2 on each side) (**pic. 2**) then undo the Velcro straps

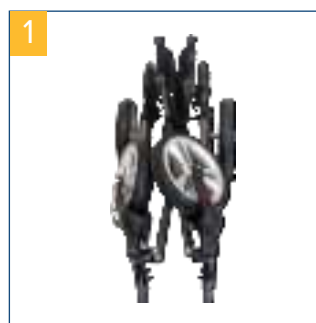- Lower the rear wheels and rotate the castors to unfold the chassis (**pic. 3 + pic. 4**)
- Fix in place by tightening the 2 knobs on each side
- Position the rollator on the ground on its wheels

- Push two half sections apart at the same time and push sideways on the cross-brace to engage the locking hook situated under the seat on the left until the mechanism is fully locked (**pic. 5**)
- Check the general condition, check that all mechanical elements are present, secure, tight and locked in place
- Check that the back support strap is properly fixed in place (**pic. 6**)

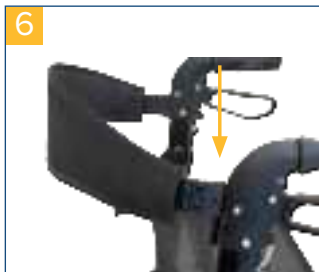Caution

- Make sure that the 4 knobs are completely tight and that the 2 push-buttons are properly engaged
- Keep fingers and objects clear of the folding areas of the rollator
- Check that the transport bag is securely fixed on the front of the rollator
- To fold the rollator, simply pull the folding handle located in the center of the seat

Unfolding

Place rollator on its wheels.
Now press both side rails of the seat section downwards until they click into place.

Assembly accessories

The pocket is hung by its loops from the ends of the frame tubing.

Height adjustment of push handles

Carry out the height adjustment of the handles by pushing the push-button then sliding the tube.

Release once at the required position (at the same level on each side) making sure that it is properly locked in place.

Cane Holder

Put the upper cane holder with the pins into the 2 holes at the push handle pipes below the backrest and close the clip. The lower cane holder part must be fixed with the pin in the fixation whole. Make sure correct assembly. The cane holder can be mounted left or right.

Adjusting the brakes

If braking is poor or too hard, the brake can be adjusted either at the brake housing above or using the adjusting screw on the Bowden cable below (**pic. 7**) to achieve the desired braking effect.


The brake is correctly adjusted when the brake lever as well as the brake wires are free of play and the wheels can be rotated without grinding noises when the brake is released.

Braking | Parking

To brake when walking with the rollator, pull both brake handles upwards with your hands (**pic. 8**). Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place (**pic. 9**)! To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position).

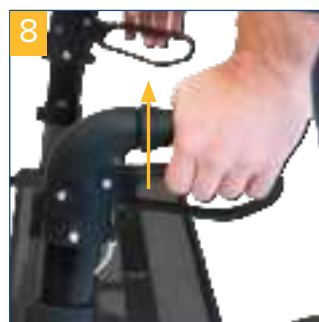The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.

Sitting

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards (**pic. 10**)!

Seated position permitted only as shown and against the direction of movement (**pic. 12**)!

Curb Climber

To negotiate kerbs or similar increases, please use the curb climber provided for this purpose. Make sure that you have a secure footing when using it. Press the tip of your foot on the curb climber so that the front wheels of the rollator lift up and push the rollator over the rise. Remove your foot from the curb climber as soon as you have overcome the obstacle.

Caution: Your balance may be affected when using the curb climber!

Folding | Transporting | Stowing

To fold up the rollator, use the handle to pull the seat section upwards.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.)!
- In the event of defects, contact your specialist dealer from whom you purchased this product!
- Check the condition of the tires regularly. They are “fail-safe”, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

Technical data

Product	MOKA
Article number	700400100
Recommended body size	1,60 - 1,95 m
Turning radius	87 cm
Overall length x Overall width	71 x 67 cm
Handle height	79 - 92 cm
Seat width x Seat depth	45,5 x 22 cm
Seat height	54 cm
Width between handles	47 cm
Folded width	31 cm
Total dimension when double folded LxWxH	69,5 x 31 x 29 cm
Diameter wheels	8" / ca. 203,20 mm
Weight w/o accessories	7,0 kg
Weight with accessories	7,6 kg
Load max.	136 kg
Load max. of the bag	5 kg
Material wheels	PU
Material seat	Nylon
Material handles	PVC plastic
Material frame	Aluminium, corrosion-protected, powder-coated

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Drive Medical GmbH & Co. KG