



A new shortcut to run the software should show up on your desktop, double click it to open the program

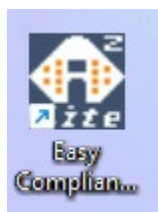


Running a compliance report

This is what it looks like when you open it

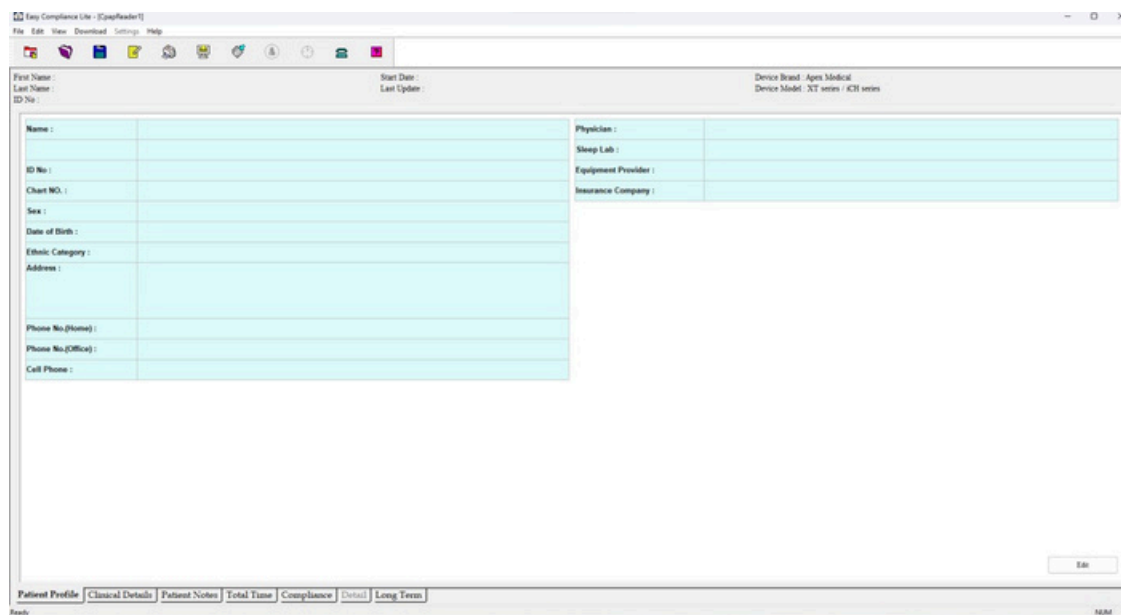

The screenshot shows the 'Easy Compliance User - [EasyReader1]' window. The interface includes a menu bar (File, Edit, View, Download, Settings, Help) and a toolbar. The main area is a form for patient information, divided into two columns. The left column contains fields for Name, ID No., Chart NO., Sex, Date of Birth, Ethnic Category, Address, Phone No.(Home), Phone No.(Office), and Cell Phone. The right column contains fields for Physician, Sleep Lab, Equipment Provider, and Insurance Company. At the bottom, there is a tabbed interface with tabs for Patient Profile, Clinical Details, Patient Notes, Total Time, Compliance, Detail, and Long Term. The 'Patient Profile' tab is currently selected. An 'Edit' button is located in the bottom right corner of the form area.

Click Edit in the bottom right corner to add patient information

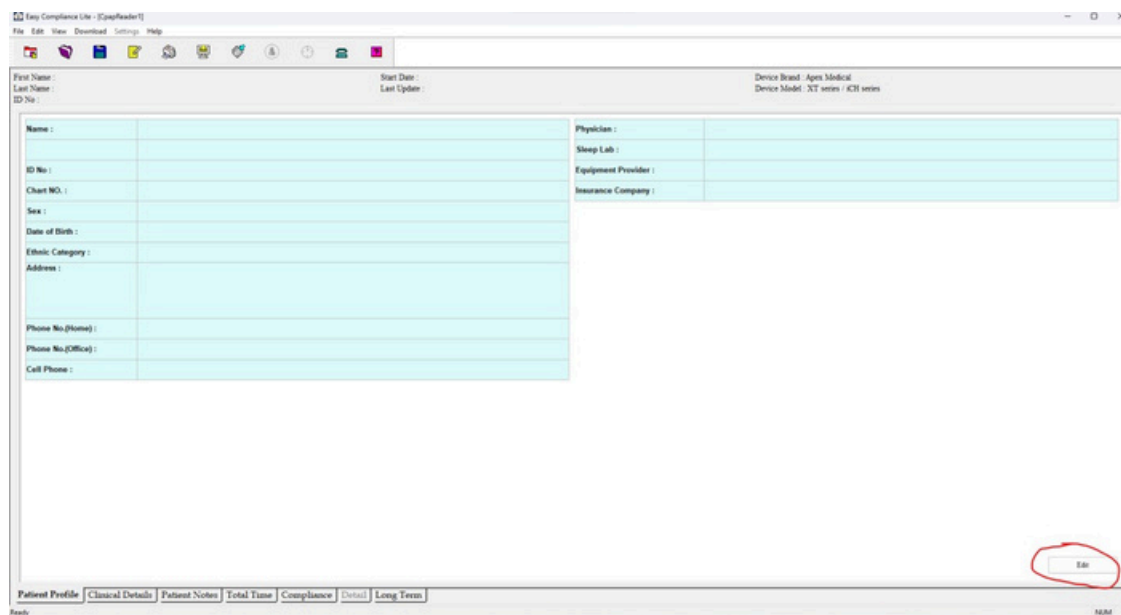

This screenshot is identical to the previous one, showing the 'Easy Compliance User - [EasyReader1]' window. The 'Edit' button in the bottom right corner of the form area is circled in red, indicating where the user should click to add patient information.


No particular information is required to run a report so patients only have to add what they want to

The screenshot shows the 'Easy Compliance Lite' application window. The main window has a menu bar (File, Edit, View, Download, Settings, Help) and a toolbar. Below the toolbar, there are fields for 'First Name', 'Last Name', 'ID No.', 'Start Date', 'Last Update', 'Device Brand', and 'Device Model'. The main area is a large table with columns for 'Name', 'ID No.', 'Chart NO.', 'Sex', 'Date of Birth', 'Ethnic Category', 'Address', 'Phone No.(Home)', 'Phone No.(Office)', 'Cell Phone', 'Physician', and 'Equipment Provider'. A 'Patient Information' dialog box is open, allowing users to enter details for a new patient. The dialog box has tabs for 'Patient Profile' and 'Clinical Details'. The 'Patient Profile' tab is active, showing fields for 'Name and Address' (First Name, Last Name, ID No., Chart No., Street and Number, Town/City, State or Province, Zip or Postal Code, Country), 'Telephone' (Home, Office, Cellular Phone), 'Date of Birth' (Year, Month, Day), 'Sex' (Male, Female), 'Physician', 'Equipment Provider', 'Insurance Company', and 'Ethnic Category'. The 'OK' button is highlighted with a red circle.

Then click Ok when finished

This screenshot is identical to the one above, showing the 'Easy Compliance Lite' application window with the 'Patient Information' dialog box open. The 'OK' button in the dialog box is highlighted with a red circle, indicating the next step in the process.

If they can't see the file they need to make sure they have the correct one listed in this drop down while looking for the file

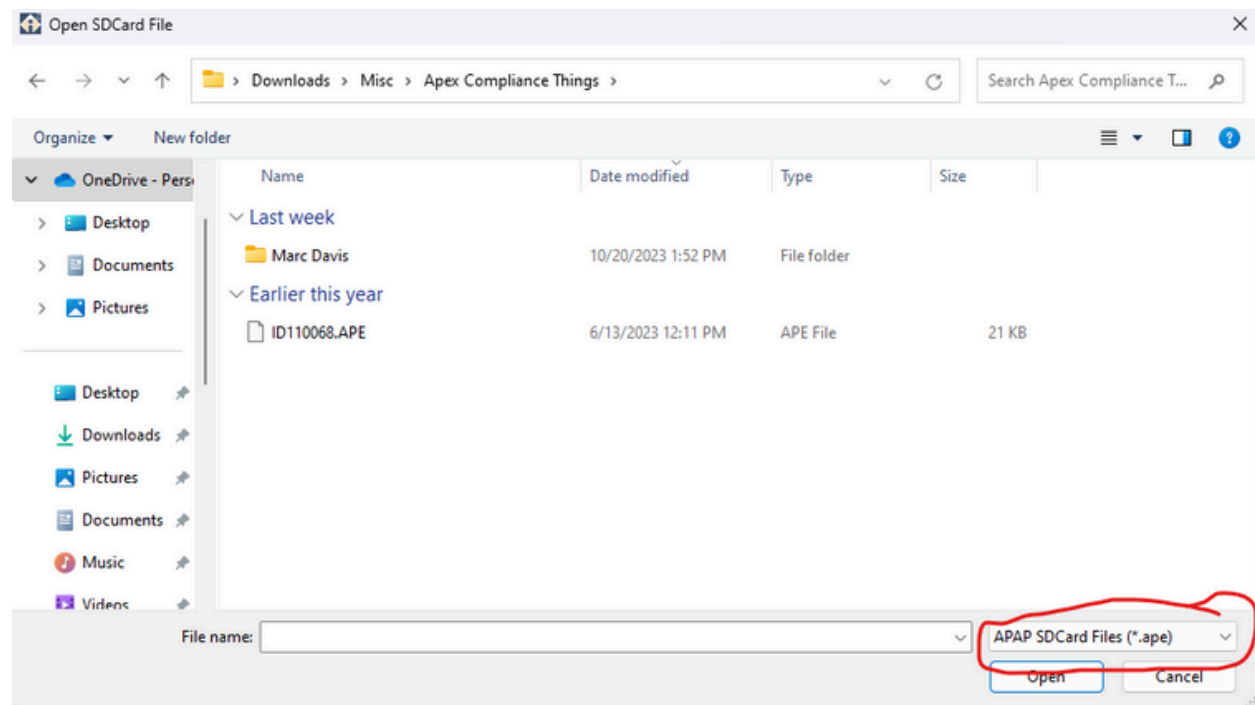

Then click Open and the data will upload

To view the data click on the Compliance tab at the bottom

Easy Compliance Lite - [CapReader1]

File Edit View Download Settings Help

First Name : Last Name : ID No : Start Date : 2021-09-13 Last Update : 2023-10-27 Device Brand : Apex Medical Device Model : SENSE

Name : ID No : Chart NO. : Sex : Date of Birth : Ethnic Category : Address : Phone No.(Home) : Phone No.(Office) : Cell Phone : Physician : Sleep Lab : Equipment Provider : Insurance Company :

Patient Profile Clinical Details Patient Notes Total Time **Compliance** Detail Long Term

Ready NUM

If you want to look at any other details you can explore with the other tabs at the bottom

To print a compliance report click on the Printer icon in the top tool bar, it's the 5th icon

Easy Compliance Lite - [CapReader1]

File Edit View Download Settings Help

First Name : Last Name : ID No : Start Date : 2021-09-13 Last Update : 2023-10-27 Device Brand : Apex Medical Device Model : SENSE

Name : ID No : Chart NO. : Sex : Date of Birth : Ethnic Category : Address : Phone No.(Home) : Phone No.(Office) : Cell Phone : Physician : Sleep Lab : Equipment Provider : Insurance Company :

Patient Profile Clinical Details Patient Notes Total Time Compliance Detail Long Term

Ready NUM

Then depending on what information the patient wants on the report they can select or deselect any of the listed options, it is recommended to just leave all of the options selected, but if the report is too long then they can unselect Long Term Trend

The screenshot shows the 'Easy Compliance Lite' application window. The 'Patient Profile' tab is active, displaying a form with fields for patient information. A 'Print Report' dialog box is open in the center, showing three selected report types: 'Summary', 'Compliance Time', and 'Long Term Trend'. The 'Print preview before printing' option is also checked. The background form includes fields for Name, ID No., Chart NO., Sex, Date of Birth, Ethnic Category, Address, Phone No.(Home), Phone No.(Office), Cell Phone, Physician, Sleep Lab, Equipment Provider, and Insurance Company. The status bar at the bottom shows 'Ready' and 'NUM'.

Then click OK

This screenshot is identical to the previous one, but with a red circle highlighting the 'OK' button in the 'Print Report' dialog box, indicating the next step in the process. The 'Print Report' dialog box remains open, showing the same selected options: 'Summary', 'Compliance Time', 'Long Term Trend', and 'Print preview before printing'.

Then click Print to print the report or to save the report as a PDF you still click print but then use the Microsoft Print to PDF option as the printer

File Edit View Download Settings Help

Print... Next Page Prev Page One Page Zoom In Zoom Out Close

Summary Report

APEX AUTO PAPER-202-10-17 ID No. 1

Date:

Patient Profile

Name:	Address:
DOB:	
Gender:	Sex:
Primary Care:	Primary Care:
Phone No (Home):	Phone No (Office):
Phone No (Cell):	Insurance Provider:
Cell Phone:	Insurance Company:

Clinical Profile

Height:	Weight:
Heart Circumference:	Diagnosis:
Body Mass Index (BMI):	Therapeutic Agent:
Blood Pressure:	Previous Prescriptions:
Bed Time:	Current Pump Type:
Average Sleep Hours:	Wash:
Additional Notes:	Humidity System:

Patient Notes

Date	Event/Time	Notes (20 characters per line)
11/11/17		
11/11/17		
11/11/17		
11/11/17		

Compliance Time

Selected Period:	2017-10-01 - 2017-10-15	Compliance (90% Usage):	1014.6
Total Days:	15 Days	Total Compliance Hours:	8.4
Average Compliance:	55.64%	Average Compliance (Hrs):	8.4
Min Compliance:	1.4	Min Compliance (Hrs):	1.4
Max Compliance:	10.4	Max Compliance (Hrs):	10.4
% of Selected Period in 90% (Hrs):	400 days	% of Selected Period in 90% (Hrs):	88.8%

Efficiency Statistics

Average In-L:	0.4	Device:	CPAP	APAP
Average Pressure:	0.5 cmH2O	Humidity Level:	0.5	0.5
Average Min Pressure:	40.7 mmHg	Leak Pressure Setting:	0.5	0.5
Average Leak:	0.2 ml/s	Leak P/B Level:	OFF	OFF
Average Time with Large Leak:	0.2 min	Average P/B Level:	OFF	OFF

Options: Date:

Compliance Time

APEX AUTO PAPER-202-10-17 ID No. 1

Compliance Time

Usage Profile Range

Duration (H)

Hour

Pages 1-2

NUM