


## 7 Illustrated example of Taking a Picture and Adding Comments

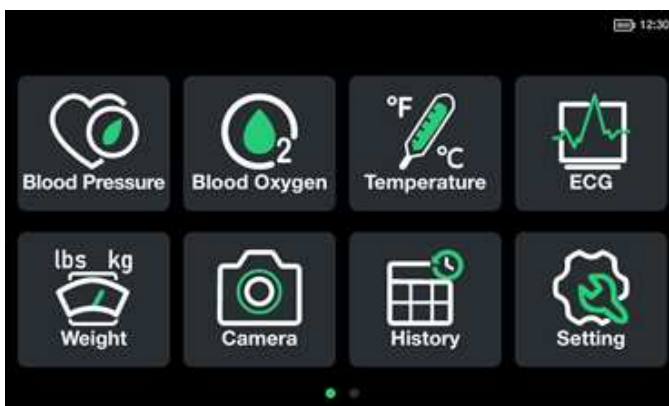

Questions or Comments?  
**Call 1-855-986-2273**

For More Information Contact  
**www.vocare.com**



**1-855-986-2273**

PO Box 207, Zionsville, Indiana 46077, USA

**www.vocare.com**  
**info@vocare.com**

## QUICK REFERENCE GUIDE

VoCare's professional grade mobile medical device




This device was designed to capture and display vital sign results to healthcare providers and lay persons in a clinical or home environment. The device is available only through prescription by a healthcare provider and is intended for spot-checking blood pressure, pulse, blood oxygen saturation, ECG, respiration rate, heart rate, infrared temperature, weight and height. This Quick Guide is a reference that does not substitute for training by a healthcare professional prior to use. These vital sign measurements can be stored on the device for display or sent by way of a third-party Medical Device Data System (MDDS) platform. The device natively does not perform any decision support functions or provide alerts to the user.

1. Illustrated example of Measuring Blood Pressure



2 Illustrated example of Measuring Blood Oxygen

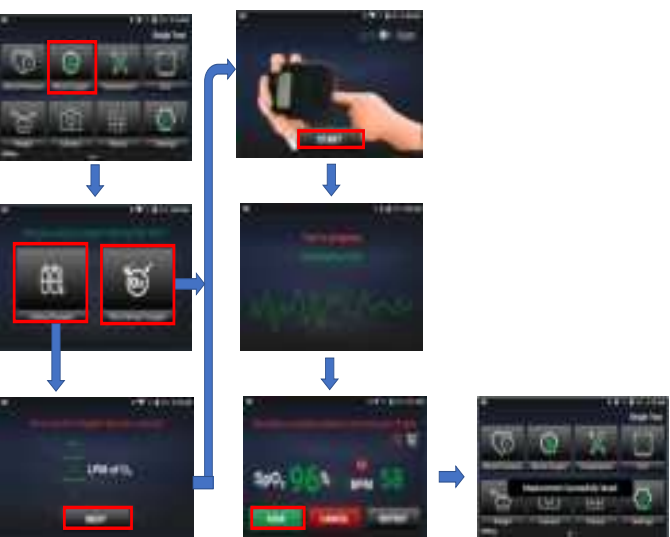

3 Illustrated example of Measuring Forehead Temperature



4 Illustrated example of Measuring ECG



5 Illustrated example of Manually Storing Height



6 Illustrated example of Manually Storing Weight



- SAVE**  
Save the previous measurement in the device locally, then return to the home page.
- CANCEL**  
Cancel the previous measurement and DO NOT save any measuring result and return to the home page.
- RETEST**  
Cancel the previous measurement and restart the measuring process.