Reusable Chamber

REF MYAIRVO**CHAMBER1**








REF 621415 REV D 2022-04 © 2022 Fisher & Paykel Healthcare Limited

Thank you for choosing F&P myAirvo™

Please read this entire user manual before you first use your reusable chamber. Following the steps in this manual, and referring to the images and diagrams will assist you with daily use of your F&P myAirvo device and its consumables.

LANGUAGES

PAGE	LANGUAGE	PAGE	LANGUAGE
1	English (en)	56	Română (ro)
4	Italiano (it)	59	Ελληνικά (el)
7	Deutsch (de)	62	Slovenský (sk)
10	Français (fr)	65	Česky (cs)
13	Español (es)	68	한국어 (ko)
16	ภาษาไทย (th)	71	Русский (ru)
19	Português brasileiro (pt-br)	74	简体中文 (zh)
22	Português (pt)	77	日本語 (ja)
25	Nederlands (nl)	80	Bahasa Indonesia (id)
28	Dansk (da)	84	Hrvatski (hr)
31	Suomi (fi)	87	Slovenščina (sl)
34	Norsk (no)	90	Magyar (hu)
37	Svenska (sv)	93	فارسی (fa)
40	Türkçe (tr)	96	български (bg)
43	العربية (ar)	99	Eesti keel (et)
46	עברית (he)	102	Latviešu valoda (lv)
49	繁體中文 (zht)	105	Lietuvių kalba (lt)
52	Polski (pl)		

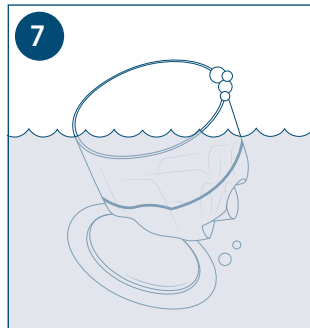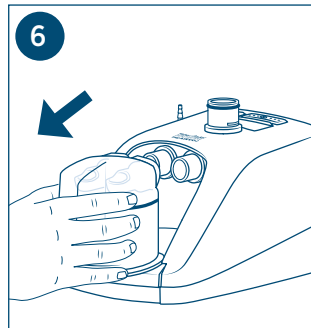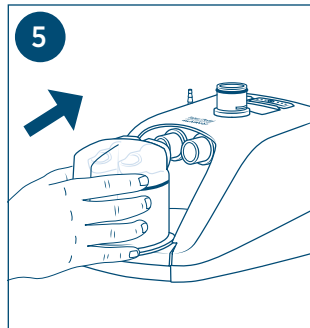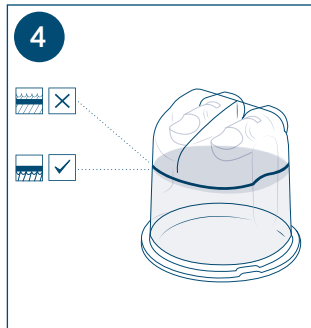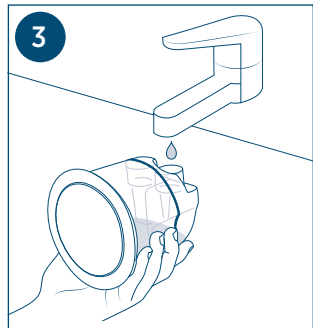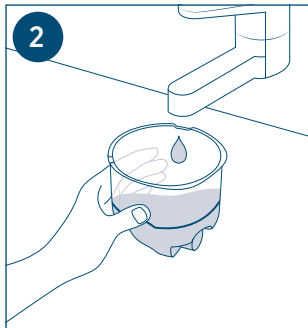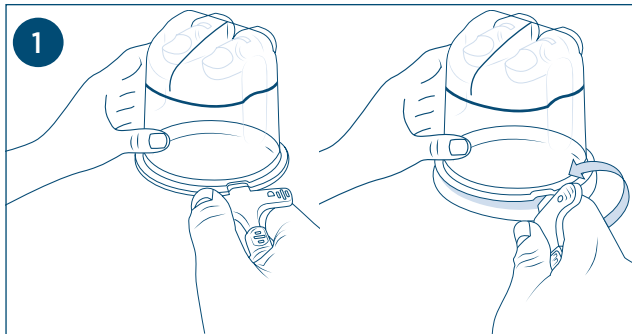Rx only	Prescription only
	Single patient multiple use
CE 0123	Conforms with Regulation (EU) 2017/745
MD	Medical Device
	Contains Hazardous Substances

F&P and myAirvo are trademarks of Fisher & Paykel Healthcare Limited.

For patent information, see www.fphcare.com/ip

Manufacturer
Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013
PO Box 14 348 Panmure,
Auckland 1741,
New Zealand

Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com

Importer/ Distributor
Australia (Sponsor)
Fisher & Paykel Healthcare
Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131.

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Tel: +90 312 354 34 12
Fax: +90 312 354 31 01

UK
Fisher & Paykel Healthcare Ltd,
Unit 16, Cordwallis Park,
Clivernock Road, Maidenhead,
Berkshire SL6 7BU, UK

Tel: 0800 132 189
Fax: +44 1628 626 146

USA/Canada
Tel: 1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

myAirvo Reusable Chamber

 MYAIRVOCHAMBER1

INTENDED USE / INDICATIONS FOR USE

The MYAIRVOCHAMBER1 is a reusable humidification chamber that stores and allows heating of water for the humidification of respiratory gases. For use with the myAirvo Series humidifiers.

SYSTEM SPECIFICATIONS

- Maximum volume: 560ml
- The chamber will last 7 hours when full and set to 25 L/min and 37°C.
- Permissible transport and storage conditions: -10°C to 50°C
- Refer to myAirvo Series Humidifier User Manual for permissible operating conditions.

SETUP

- myAirvo Series Humidifier (“myAirvo”). Refer to the myAirvo Series Humidifier User Manual for information on the myAirvo.

HOME AND LONG-TERM CARE FACILITY USE

The product is intended to be used for a maximum of 2 years for use with your myAirvo, provided that the myAirvo and MYAIRVOCHAMBER1 cleaning and maintenance instructions are followed.

SETUP INSTRUCTIONS

Step 1

Separate the chamber from the base with the removal tool. To separate the chamber from the base, insert the end of the chamber removal tool into the chamber slot and perform a twisting motion.

Step 2

Rinse the chamber in warm water before use.

Step 3

Fill the chamber with water.

Step 4

Place on a flat surface to check that the water level is no higher than the 560ml volume indicated.

Step 5

Fit the chamber to the unit by pressing down the finger guard and

sliding the chamber on, carefully aligning the holes at the top of the chamber with myAirvo ports. Push the chamber on firmly until the finger guard clicks into place.

Contact your Fisher and Paykel Healthcare representative for assistance with setup and use, or to report any unexpected operation.

Warnings:

- Do not fill past the maximum water level line to avoid choking.
- Do not fill the water chamber with hot water to avoid airway burns.
- Use of a non-approved accessory could impair performance or compromise safety.
- Notice to User: If a serious incident has occurred while using this device please inform your local Fisher & Paykel Healthcare representative and Competent Authority in your country.

Caution:

- For inhalation, use only drinkable quality water or distilled water with the reusable chamber.

CLEANING INSTRUCTIONS

DAILY

Step 6

After drying mode is completed, hold the finger guard down and slide the chamber out of the myAirvo device. Discard any remaining water.

Step 7

Separate the chamber from the base with the chamber removal tool. Refer to Step 1 to use removal tool.

Hand-wash the chamber parts in a tub of warm water with a mild dishwashing detergent. Ensure the chamber is visually clean.

Rinse the chamber thoroughly in fresh water. Ensure that all soap residue has been removed.

Leave the chamber parts to dry out of direct sunlight or heat before reassembling. Once dry, refit the base.

DISHWASHING

Once every 7 days, the chamber can be cleaned in a domestic dishwasher. Remove the base from the chamber. Place the chamber top and chamber bottom on the top shelf of the dishwasher and ensure it is placed so that it is able to drain.

REMOVAL OF MINERAL DEPOSITS

Mineral deposits in tap water may cause the chamber base to discolour over time. Should mineral deposits occur, you can reduce these by performing the below step.

Soak the chamber base in a solution of vinegar (1 part) to water (2 parts) for 10 minutes. Clean the chamber by following daily cleaning steps prior to next use.

Cautions:

- Discard the chamber if the maximum water level line is no longer legible.
- For single patient use only. Reuse between multiple patients may result in transmission of infectious substances. Attempting to reprocess will result in degradation of materials and render the product defective.

Note:

- Ensure the rubber O-ring around the base is not dislodged or damaged.

DISPOSAL INFORMATION

Dispose of product and packaging according to local guidelines. Long-term care facilities should discard according to their standard method for disposing of contaminated product.

SYMBOL DEFINITIONS

Catalogue number

Prescription only

Consult instructions for use

Caution

Authorised European Representative

Batch code

Date of manufacture

Shelf life expiry date

Single patient multiple use

CE 0123

Conforms with Regulation (EU)
2017/745

Storage temperature

Manufacturer

Discard as regular waste

Correct water level in the chamber

Incorrect water level in the
chamber

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