TENS and Muscle Stimulator

User Manual

Model: AS8011
V01: 2024/12/11

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Contents

Easy@Home AS8011 TENS Overview	01
Device Components and Features	02
Operating and Charging Instruction	04
Placement of Electrode Pads	05
Troubleshooting	06
Cleaning and Maintenance	07
Warranty Information	07
Safety Warnings	07
Technical Specifications	10

Thank you for purchasing Easy@Home Tens (Transcutaneous Electrical Nerve Stimulation) and Muscle Stimulator. To enable you properly operate this product, please thoroughly read this product introduction first and keep the manual well after reading for future reference.

Easy@Home AS8011 TENS Overview

Easy@Home AS8011 TENS and Muscle Stimulator is an effective and portable TENS and EMS combo unit with a rechargeable battery. All Easy@Home TENS and EMS Units are CE cleared for purchase and can be used at home without a prescription.

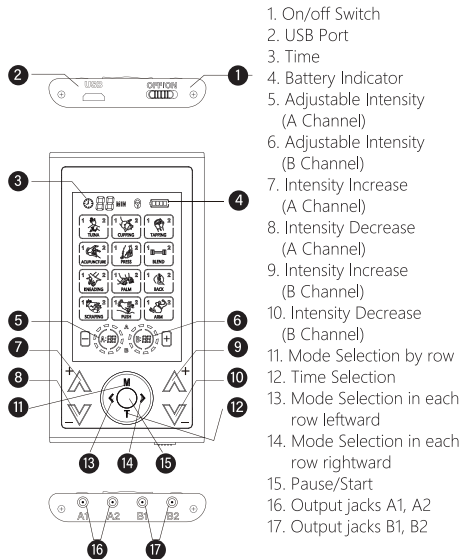
TENS (Transcutaneous Electrical Nerve Stimulation) is one of the most effective forms of drug-free pain relief for various parts of the body; such as the waist, shoulders, joints, hands and feet. TENS delivers micro electric impulses in different frequencies and patterns to the user's skin through adhesive electrode pads such that the underlying nerves are stimulated to trigger the generation of the body's natural painkillers, better known as Endorphins. As the result, the pain and tightness of muscles are relieved. The impulses also trigger stimulation of muscles, which can help strengthen the blood flow and muscle recovery.

EMS (Electrical Muscle Stimulator) Unit sends electronic pulses to muscles to help with muscle toning and firmness, endurance improvement, and to lessen recovery time between workouts.

The AS8011 TENS and Muscle Stimulator is a professional model of the Easy@Home TENS machine product line. It comes with 4 independent output channels, and each channel provides a broader power level of 1-20. This powerful, yet very safe unit is not only suitable for any chronic or occasional pain relief, but also good for muscle recovery and relaxation after physical exercise.

Device Components and Features

There are 24 modes to choose from, they are:

- | | | |
|--------------|-----------------|-----------------|
| 1. Tuina | 2. Hand | 3. Cupping |
| 4. Elbow | 5. Tapping | 6. Weight Loss |
| 7. Stroke | 8. Acupuncture | 9. Shiatsu-Deep |
| 10. Press | 11. Shoulder | 12. Combination |
| 13. Kneading | 14. Oil-Massage | 15. Wrist |
| 16. Palm | 17. Back Pain | 18. Foot |
| 19. Guasha | 20. Scraping | 21. Push |
| 22. Relax | 23. Massage | 24. Pressure |

Pulse width: 100 μ s

Frequency: 0~132Hz

Features

- 4 output channels with 24 different modes
- 20 intensity level setting
- 20 minutes default timer, adjustable from 10-60 minutes
- Rechargeable built-in battery
- Battery Indicator for you to know when to charge
- Large, easy to read LCD display
- Key lock function protection
- 1 year guarantee
- Stimulator pads and pad lead wires are applied part

Operating and Charging Instruction

Operating Procedures

1. Connect the wire with reusable electrode pads.
2. Peel off the transparent film attached to the electrode pads and place the electrode pads on your desired treatment area.
 - 1) For additional relief, use another pair of pads and another jack.
 - 2) Make sure to keep apart each pair of pads minimum half an inch.
 - 3) Make sure the skin where the electrode pads are applied is free from any dirt, oil or lotion.
3. Plug the lead wires into the unit and turn it on. Switch button is at the bottom end of the device, beside the charging port. If the device was turned off automatically, you have to set the switch to 'Off' side first, then push to "ON" side to turn on.
4. Press the left " $\wedge \vee$ " to choose Channel A.
5. Press the "M" to choose different rows of the mode.
6. Press the "< >" to choose different modes in each row.
7. Press the left " $\wedge \vee$ " to increase/ decrease the strength for Channel A. Remember the strength will default to the lowest setting by each mode selection.
8. Press the right " $\wedge \vee$ " to choose Channel B.
9. Press the "M" to choose different rows of the mode.
10. Press the "< >" to choose different modes in each row.
11. Press the right " $\wedge \vee$ " to increase/ decrease the strength for channel B.
12. Press "T" to choose your 10-60 min massage time and enjoy it. Countdown timer is activated when operation starts, and the device will automatically turn off when the countdown ends.
13. Press and hold the middle round button for 3 seconds to lock the control buttons, 3 second again to unlock them.

- Note:**
1. Never connects this product with common headphone.
 2. Please do not touch USB port in the process of using. The USB port should only be used to connect the charger. Do not connect another device.
 3. The charger supplied by the manufacturer must in compliance with IEC/EN 60601-1, the use of unauthorized charger can degrade minimum safety.

Charging the Battery

1. Turn off the controller.
2. Connect the controller and the charger with USB extension cord. Plug charger into any power outlet. The charging process will last approximately 1 hour.
- 3.

Note: Only charge the controller when battery is completely drained the first 2 times. Unplug charger from power outlet when charging is complete. When massage strength decreases, it indicates that the device needs charging. Recharge it and then continue to use the device. Do not use the controller when charging.

Placement of Electrode Pads

You may use TENS stimulate pads wherever you feel pain and fatigue to relax. The TENS provides pain relief for a number of different pain conditions, and this product is designed for the temporary relief of pain associated with:

Aching muscles in the back, waist, shoulders, arms and joints

More treatment points you can refer to the stimulator point chart.

Troubleshooting

Problem	Cause	Solution
One pad feels stronger than the other.	This is normal. Different areas of your body will react differently.	Nothing needs to be done. Make sure the pads are moist and are making good contact.
During the massage, the skin feels a painful burning sensation or the stimulation becomes weakened.	Adhesive gel pads are not adhering firmly to the skin. Or the gel pads are too dry.	Apply a few drops of water to adhesive surface of each electrode pad and make sure the pads are pressed firmly to the skin during application.
Unit is on, but no sensation is felt in the pads.	Both electrode pads are not firmly on the skin. Or there is a loose connection. Or the intensity level needs to be increased.	Make sure both electrode pads are firmly pressed to the skin. Make sure all connections are secure from the unit to the wires and the wires to the electrode pads. Increase the intensity level.
Adhesive gel pads do not stick to skin even after cleaning and moistening the gel pad.	Adhesive gel pads need to be replaced.	Replace the set of electrode pads.
Unit does not turn on or the screen is dim.	Battery is low.	Recharge the battery.
Does not provide adequate pain relief.	Not using it long enough. Not lying down when massaging neck.	Use your Stimulator 20-30 minutes at a time, 3-6 times per day. Lie on your back for neck support when you need relief for headaches or neck pain.

Cleaning and Maintenance

Please use water or neutral detergent to clean the device first, and then wipe it again with a dry cloth. The electrode pads included with the device are disposable, and should be replaced when their adhesiveness becomes worse. If you need new pads, contact Easy@Home customer service via email service@healthcare-manager.com for assistance. Do not let the sticky side of the pad touch anything, including finger tips (to avoid natural oils).

Warranty Information

Easy Healthcare Corporation provides one-year quality warranty from the time of purchase. Should you require repair or replacement during the warranty period, please email service@healthcare-manager.com for assistance. Please retain your sales receipt or invoice, as it will be required to receive service under this warranty. Out-of-warranty service is provided on a case-by-case basis. A fee for repair for replacement may apply on out of warranty devices.

Safety Warnings

1. Never apply the electrode pads to your skin with the power on, which will result in sudden shock. During massaging, if you want to move the electrode pads to another body part, please shut down the device first, and then move the electrode pads to the place that you want to massage.
2. If the electrode is not intact, do not use the device. Degraded sensor and electrodes, or loosened electrodes may cause potential output performance risk.
3. Never use this product in concurrence with the following devices: pacemakers or any other embedded electronic medical devices, heart-lung machine and any other life keeping electronic medical devices, electrocardiograph and

any other medical screening and monitoring devices. Simultaneous use of Easy@Home TENS machine and any of the above devices will cause malfunction and can be very dangerous to the users.

4. Never use this product in concurrence with high frequent surgical equipment, it may result in burns at the site of the stimulator electrodes and possible damage to stimulator.

5. Never use this product near microwave oven, or other high frequent equipment.

6. Please do not operate the products in close proximity (e.g. 1m) to shortwave therapy equipment, it may produce instability in stimulator output.

7. Please do not operate electrode pads near the thorax or heart which may increase the risk of cardiac fibrillation. Do not apply the electrode pads on the head, above the neck, in the pubic region, over Scarred areas, on the throat or over the mouth.

8. Please consult a physician regarding the use of the product under the following circumstances: patients with heart disease, baryodynia disease, suspected or diagnosed epilepsy, acute conditions, malignant tumors, tuberculosis, any contagious disease, high fever, abnormal blood pressure, skin disease, lack of sensation on the skin, wounds on skin; pregnant women, women in menstrual cycle, or puerpera; the one who has wet body after shower or sweat; and patients who are receiving treatment. Use of the device under those circumstances may cause discomfort, accidents and may worsen your condition.

9. Caution should be used in presence of the following.

- when there is a tendency to haemorrhage following acute trauma or fracture;
- following recent surgical procedures when muscle contraction may disrupt the healing process;

- over the menstruating or pregnant uterus;

- over areas of the skin which lack normal sensation.

10. Please do not use the products in bathroom or a moisture environment. Do not apply it while bathing.

11. Do not use this product with any topically applied creams or ointments.

12. Please do not use the products in the condition of air mixed with flammable anaesthetic gas or oxygen or nitrous oxide mixed with flammable anaesthetic gas.

13. Do not disassemble, repair or modify without authorization, you will void any warranty on the product.

14. This Product cannot be repaired until the repairing technical personnel is designated by the manufacturer. The designated repairing technical personnel can require the manufacturer to supply the technical data (e.g. circuit diagram, PCB drawing, component list etc.).

15. Method for disposing the expired equipment and accessories: Do not mix the expired equipment and accessories with the household rubbish, the rubbish should be disposed in accordance with local regulations (Example: put the rubbish into electronics recycling station).

16. Two electrode pads should be used together, and peel off the protective film on the electrode pads before using. Do not connect two electrode pads in the process of using to avoid short cut.

17. Please do not apply electrode pads on same position for over 30 minutes every time.

18. Children and persons with communication disabilities are forbidden to use the device.

19. Do not use other equipment when use the device. Do not use while driving, operating machinery, or during any activity in which involuntary muscle contractions may put the user at undue risk or injury.

20. Do the disposal of waste device regarding your local government instruction.

21. Keep the device out of pets, pests or children reach. They may damage the device and cause incorrect output performance with uncomfortable stimulation.

22. The device may be degraded by lint, dust, light (including sunlight), which can cause incorrect output performance with uncomfortable stimulation.

23. When you exchange new electrode pads, you shall use the same type of the electrode pads.

Adverse Reactions

You may experience skin irritation and burns beneath the stimulation electrodes applied to the skin.

You may experience a headache and other painful sensations during or following the application of electrical stimulation near the eyes and to the head and face.

You should stop using the device and should consult with your physician if you experience adverse reactions from the device.

Technical Specifications

Box Contents

- TENS and Muscle Stimulator x 1
- Large adhesive electrode pad x 8
- Electric lead wire x 4
- USB cable x 1
- Electrode pads sticking board x 2
- Stimulator point chart x 1
- Instruction manual x 1

Technical Information

Name of product	Tens and Muscle Simulator
Model No.	AS8011
Product material	Shell material of controller with ABS, Electrode pads with silica gel
Battery	3.7V, 300mA; Restriction: 4.2V
Consumed current	40mA
Modes	24 kinds of modes
Strength level setting	20 levels
Timer	20 minutes default timer, can adjustable from 10-60 minutes
Weight of main frame	0.45 Kg (including controller, pads, USB extension cord and output cord)
Normal working ambient temperature	5°C ~ 40°C
Normal working ambient humidity and Atmosphere pressure	0% ~ 80% RH; Atmosphere Pressure: 700hPa ~ 1060hPa
Store and transport ambient temperature	-20°C ~ 55°C
Store and transport ambient humidity and Atmosphere pressure	0% ~ 93% RH; Atmosphere Pressure: 700hPa ~ 1060hPa
Safety classifications	Internal powered equipment, Type BF applied part, not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide, continuous operation
Expected service life	The expected service life of the equipment is 5 years

Electromagnetic Compatibility

It has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices to IEC 60601-1-2: 2014. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

CAUTION:

Do not use this device simultaneously with devices having high EMI levels.

Guidance and manufacture's declaration – electromagnetic emission		
It is intended for use in the electromagnetic environment specified below. The customer and the user of the device should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	It use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	It is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacture's declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8kV contact ±15kV air	±8kV contact ±15kV air	Floors should be wood, concrete or ceramic tile. If floor is covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2kV for power supply lines	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1kV line(s) to line(s)	±1kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30A/m	30A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Symbols

	Type BF applied part
	CE marking of conformity
	Ingress protection rating
	Recyclable
	WEEE (Waste Electrical and Electronic Equipment)
	Refer to instruction manual
	Manufacturing date
	Batch code
	Authorized representative in the European Community
	Manufacturer
	Temperature limit of -20°C ~ 55°C
	Humidity limitation of 0% ~ 93%
	Atmospheric pressure limitation of 700hPa ~ 1060hPa

Disposal of Materials

A symbol of Waste Electrical and Electronic Equipment means you should find out and follow local regulations about disposing this kind of product.

- Do not dispose this product as you would other household waste. Dispose this device in accordance with the corresponding local regulations.
- Electrical and electronic devices contain hazardous substances that can have harmful effects on the environment and/or human health and should be recycled properly.

Guarantee: 1 year

Healthcare-Manager.com

Questions or comments?

Please contact us via email:

service@healthcare-manager.com

Distributed by: Easy Healthcare Corporation

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Distribuido por: Easy Healthcare Corporation
Shenzhen Astec Technology Co., Ltd
MedNet EC-REP GmbH