Auria™ Carrying Case

Instructions for Use

English, French, Spanish, German, Dutch, Italian

Auria Carrying Case

The Auria Carrying Case is a multi-functional case that can carry all your basic components for the Auria or Harmony processors. There is a place for the headpiece that contains a plate that attracts the headpiece magnet and several compartments that will hold other accessories and additional earhooks.

The Auria Carrying Case contains a removeable insert (Figure 1) that also serves as a Dri-Aid kit so you can carry your processor with you and remove excess moisture as you store it at night.

Dri-Aid Function

Your Auria Carrying Case acts as a Dri-Aid kit. The insert of the Auria Carrying Case contains silica crystals that can be regenerated.

On the bottom of the Carrying Case insert is a semi-transparent window that will allow you to observe the silica crystals as they absorb moisture. The crystals are a yellow/gold color when dry but turn green (dark) as moisture is absorbed. When the crystals are green (dark), regenerate. See Regenerating the Dri-Aid Function for instructions.

Keep the Auria Carrying Case closed at all times to maintain optimal drying capacity.

*Figure 1. Auria
Carrying Case
Dri-Aid Insert*

Regenerating the Dri-Aid Function

Remove the insert (Figure 1) from the Auria Carrying Case and place on a cookie sheet in a heated oven (200°F/93°C) for approximately 1.5 to 2 hours. If a longer period is required, the crystals can be left in the oven up to 6 hours.

Note: *When opened for the first time, the crystals in the insert may have already absorbed moisture. If this has occurred, regenerate before use.*

Caution: *Do not put the outer case or the Auria and any of its accessories in the oven, only the insert is designed to be heated.*

Caution: *Do not heat over 260°F/126°C or the silica crystals may become damaged.*

Cleaning

If necessary, the Auria Carrying Case can be cleaned with a slightly dampened cloth or tissue. Do not use soap or solvents.

Caution: *Do not immerse the Auria Carrying Case insert in water as it may damage or destroy the silica crystals.*

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©2022 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics AG
Advanced Bionics LLC