

JEGOAT

EARWAX REMOVAL TOOL USER MANUAL



Model: S9

Support: **service@jegoat.online**

Thank You for Purchasing Our Products!
Healthy Ears, Happy Lives!

If you need an extended warranty or have any
questions, please feel free to contact us:

 service@jegoat.online

We also look forward to receiving any advice
from you about our products or service and we
will make efforts to make them better.

Trouble Shooting


Does not turn on	Please charge it for half an hour before using
Screen streaks or flickers	Please charge it for half an hour before using
The picture is not clear	Please wipe the lens gently with your face
Unable to connect to hotspot	Please turn off the product and then restart the connection

Download


Step 1: Find the App Store




Step 2: Search for Earvision and click to download the App

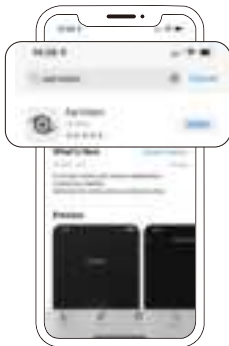


Step 3: Return to the desktop and open the App

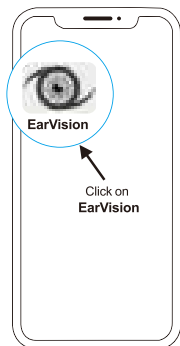

Step 4: App interface display


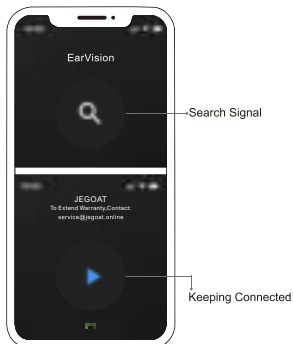


Operating Instructions

1. Before connecting the device, please keep both "Bluetooth" and "Wifi" on your phone turned on.
2. Please turn on the device and press and hold the on/off button of the device for about two seconds to turn on the device (the indicator light will flash when the device is powered on).
3. Open the "EarVision" you have downloaded and click the  in the red box to search for device WIFI.
4. When you use "EarVision" for the first time, the following prompt box will pop up. Please select "Allow" to authorize the use of EarVision.
 - ①. You will be prompted for "Permissions". You need to use "Location", "Bluetooth" and other permissions, please click "OK".
 - ②. When prompted with "Privacy Policy", please select "Agree".
 - ③. If "EarVision" wants to find and connect to devices on your local network, please select "Allow".
 - ④. If "EarVision" wants to use "Bluetooth", please select "OK".
 - ⑤. If "EarVision" wants to use wireless data, please select "WLAN Only".
 - ⑥. If "EarVision" wants to join the WLAN network "softish-xxxx"? Please select "Join".
 - ⑦. If "EarVision" wants to access location information, please select "Precise location" and "Allow only while using the app."

※Your location information will not be disclosed.
Only used to connect to product WIFI.
5. If the WIFI connection cannot be established, please go back or restart the machine and run the application again.

App Screen Display

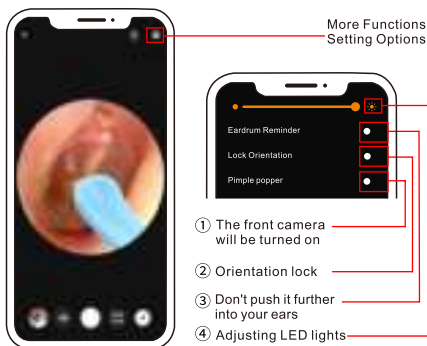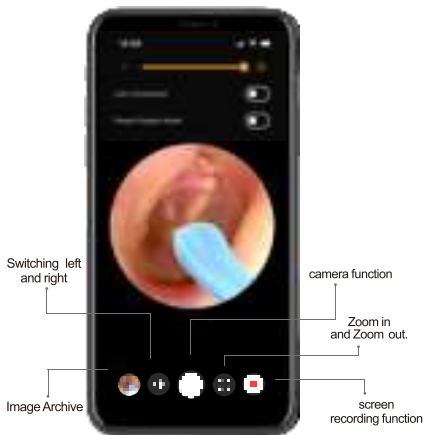

Icons Description



Return



Photo Button



Setting



Mirror Mode



FAQ



Image Archive



Zoom Screen



Anti-Touch



Record Button



Image Archive



Privacy Policy



Search WIFI

Accessories



① Machine*1



② Replaceable Silicone Case



③ Type-C cable*1



④ Silicone ear sleeves storage bottle



⑤ Daily ear care kits



Massage Ear Pick
To provide gentle massage and relaxation in the ear area.



Hard Spiral Head
It can rotate and clean the large earwax in the internal ear canal.



Clean Brush
To clean earwax and debris from the ear pick.



Storage Container



Big-headed Ear Pick



Traditional Ear Pick
Different sizes to suit personal preference or ear canal size



Spiral Ear Pick
360° rotate to gently remove earwax or debris from the ear canal.

Safety Instructions

This product is intended for personal hygiene purposes and is designed for ear and ear canal cleaning. Before using this ear spoon, please carefully read and follow:

1. This product is not a medical tool and should not substitute for medical diagnosis or treatment. If you have any ear or ear canal health concerns, consult with a physician or healthcare professional for guidance.
 2. If the ear canal has been completely blocked by cerumen, please be careful with ear picking and recommend going to the hospital for professional treatment.
 3. If you experience any ear or ear canal pain, swelling, infection, or other abnormal symptoms, cease using this ear spoon immediately and seek advice from a healthcare professional.
 4. Exercise caution and care when using this ear spoon. Avoid over-insertion or applying excessive pressure. Always follow the guidelines provided in the user manual to avoid potential injury or discomfort.
 5. Ensure that the ear spoon is stored out of reach of children and that children do not use this product unsupervised.
 6. Please observe your surroundings before use and do not use it with people running around to avoid impact.
 7. This product is not suitable for children under 3 years old.
 8. Please do not use the camera in liquid to avoid damage caused by water immersion.
 9. Avoid direct exposure of the product to sunlight, especially accessories to avoid softening.
 10. Please do not knock or drop the product, if the product is damaged, please stop using it!
 11. Press and hold the button for 2 seconds to switch off the device.
 12. Please charge the product regularly if it is not used for a long time.
 13. Please do not disassemble, modify or repair this product by yourself.
 14. Using this ear spoon will be considered your voluntary assumption of certain risks.
- The manufacturer and seller are not liable for any damages or injuries resulting from improper or misuse of the ear spoon.

Indicator Light Description

After pressing and holding the power switch button for two seconds to turn on the computer, the indicator light will turn blue. When the light flashes slowly, it means the machine is not connected. When the light is always on, it means the machine is connected. When the light flashes quickly, it means the battery is low and please charge.

Charging Instructions

Please connect the USB cable to the charging port of your device, the indicator light will light up red when charging starts. When fully charged, the indicator light will turn off.

Care and Maintenance

1. After each use, clean and disinfect the ear spoon to ensure hygiene and safety. Do not leave debris or earwax on the ear spoon.
2. After prolonged use, the lens will become dirty, please clean it regularly.
3. The temperature of the product will rise slightly (up to 35°C or less) It's normal.
4. After using, please use a cotton swab or soft cloth to clean the head of the ear spoon and the lens part. When cleaning the camera part, please be careful not to use excessive force.



Product Parameter

Product name	Ear Wax Removal Tool
Executive standard	GB 4706.1-2005
Product weight	16g
Input voltage/ Current	DC 5V/200m A
Charging time	30 min
Usage time	70min
Pixels	High Definition Pixels

Harmful Substance

Part Name	(PB)	(Hg)(Ca)	(Cr)(Vi)	(PBB)	(PBDE)
Printed circuit component					
Plastic shell					
Plastic & Rubber Accessories					

Prepared in accordance with SJ/T11364

: The content of this hazardous substance in all homogenates of this part is under the GB/T26572-2011 specified limit requirements below.