Harmony™ Listening Check

CI-5821

Instructions for Use
Fitting, Care and Maintenance
English, Français, Español, Deutsch, Nederlands, Italiano

Harmony Listening Check

The Harmony Listening Check is a diagnostic tool designed to provide a subjective audio test of equipment. The audio from the sound processors has undergone processing for this purpose and low level background 'white' noise may be heard as a result. This is not indicative of a faulty processor microphone and is not part of the normal processing heard during use by an implant recipient.

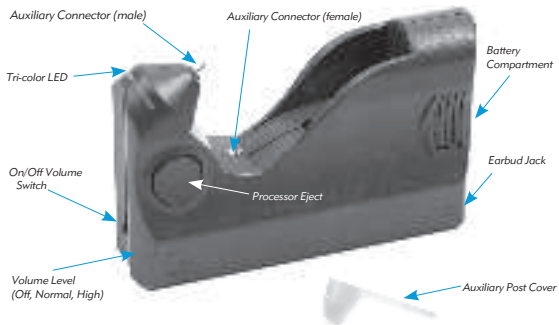
Replacement Harmony Listening Check Earbuds (CI-5822) and Auxiliary Post Covers (CI-7312) can be ordered separately.

Caution: Do not use damaged or leaking batteries as they may burn the skin.

Warning: Small Parts Hazard: The Harmony Listening Check contains small parts that may cause choking if swallowed.

Note:

- *An empty program slot is required to listen to the sound processor internal microphone. The program slot is not empty if a rhythmic beat instead of audio is heard on the Auria. On the Harmony, an empty slot is identified by a constant green LED on the processor.*
- *An FM receiver and transmitter is required to properly check the iConnect Adapter earhook.*
- *A Direct Connect Cable and input source (e.g. MP3 Player) are required to properly check the Direct Connect earhook. Turn down the volume on the input source prior to connection. Do not connect an AC-powered input source without an appropriate patch cable.*


Battery Placement

Requires the use of one AA battery (not included).

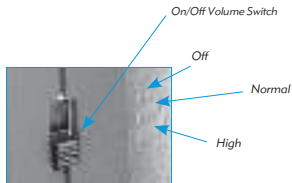Controls





Tri-color LED


LED Color	Green	Orange	Red
Indication	Normal Operation	Low Battery	Replace battery

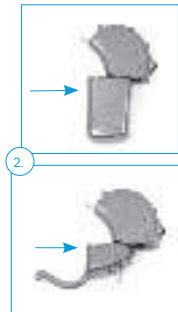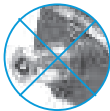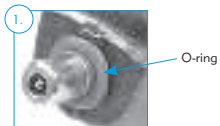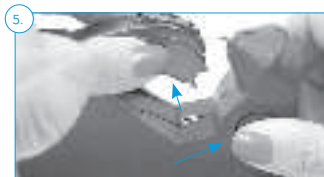Auria/Harmony

Earhooks

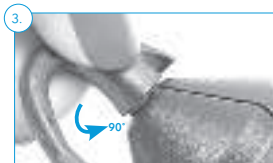Earbuds

- *To avoid any unpleasant sounds, remove earbuds from ears before attaching or detaching any sound processor or earhook or when changing off/volume settings.*
- *Please ensure that the earhook, processor and earbuds are firmly connected prior to testing. If intermittencies are experienced, please check all connections and retest.*

Care and Maintenance

- Always turn off after use.
- Always store in the case provided to protect from dust and moisture.
- Always attach the Auxiliary Post Cover when the Auxiliary Connector is not in use.

Cleaning

- Use a soft cloth to clean. Do not use solvents.

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©2022 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics AG
Advanced Bionics LLC