

## Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.

**Regular  
MHRTS2-MB**



**Elongated  
MHERTS2-MB**



## **2" Raised Toilet Seat w/new Handles**



Weight Capacity 300 lbs


## **MOBB Healthcare Ltd.**

116A Viceroy Rd. Unit 5-6  
Concord, Ontario, Canada  
L4K 2M1

## ASSEMBLY and INSTALLATION

### 1. Remove all parts from the box:

- One Seat Riser, two rest arms, two mounting screws, two plastic washers, two metal washers, and two wing nuts.

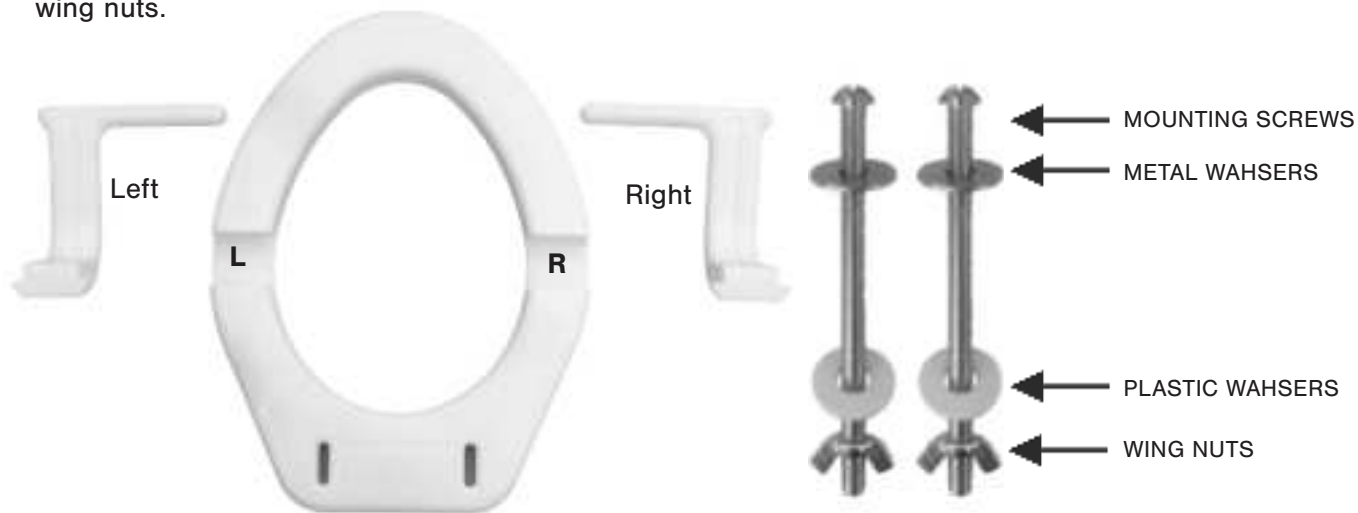

### 2. Insert the arm into the seat riser as illustrated in the diagrams:



## ASSEMBLY and INSTALLATION

3. Remove the existing toilet seat and lid from the toilet frame by unscrewing the seat screws located at the rear of the toilet seat.
4. Place the seat riser on the toilet frame and align the mounting holes.
5. Place the toilet seat on the seat riser and align mounting holes.
6. Place the metal washers on the toilet seat and align mounting holes with toilet frame.
7. Insert the mounting screws passing through the metal washers, toilet seat, seat riser and toilet frame mounting holes.
8. Insert the plastic washers and the wing nuts onto the mounting screws. Slightly tighten the wing nuts, but do not fully tighten them.
9. Be sure the toilet seat is straight and sits flush on the seat riser, then tighten the wing nuts.

## CARE and CLEANING

- REGULAR CLEANING IS RECOMMENDED
- USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY.
- ABRASIVE CLOTHS MAY DAMAGE THE SURFACE
- PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND IT'S PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR