HOW DOES IT WORK?

Human chorionic gonadotropin (HCG) is a hormone produced by the developing placenta shortly after conception and secreted into the urine. The pregnancy test contains antibodies that specifically react with this hormone.

When the strip is immersed in a urine specimen, capillary action carries the specimen to migrate along the membrane. When HCG in the sample reaches the Test Zone of the membrane, it will form a colored line. The absence of this colored line suggests a negative result.

To serve as a procedure control, a colored line will appear at the Control Zone if the test has been performed properly.

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WHEN TO COLLECT URINE FOR THE TEST?

Any urine specimen is appropriate for pregnancy testing. However, the first morning urine specimen is preferred because of its highest concentration of HCG.

HOW TO COLLECT THE URINE SAMPLE?

This test is designed to have urine collected in a clean container just prior to testing.
Urine may be collected in any clean, dry, plastic or glass container.

HOW TO DO THE TEST?

1. Remove the test strip from the sealed pouch.
2. Dip the white absorbent tip of the test strip into the urine up to the black "MAX" line and keep the tip in urine until dye runs almost to the result window (Test Time: 5-10 seconds).
3. Lay the test strip on a clean, dry, non-absorbent surface.

The test strip and urine have to be at room temperature 15-40°C (59-104°F) for testing.

Positive (Pregnant)
Two distinct color lines appear in the Control and Test Zones. One line should be in the control zone (C) and another line should be in the test zone (T).

Negative (Not pregnant)
Only one color line appears in the control zone (C). No appeared line in the test zone (T).

Invalid
No visible lines at all, or there is a visible line only in the test zone (T) but not in the control zone (C). Repeat with a new test strip. If still fails, please contact to be assistance.

NOTE:
The color intensity of the test lines may vary since different stages of pregnancy have different concentrations of hCG hormones. If the color of the test lines is

- 1. As in all very diagnostic situations, a confirmatory pregnancy diagnosis should only be made in a laboratory after evaluating all clinical and laboratory findings.
 - If a serum sample is less than 30 days (two specific gestational weeks) after a suspected reproductive loss or HCG if pregnancy is still suspected, another serum sample should be collected 10-14 days later.
- 2. Low concentration of HCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
- 3. Elevated levels of HCG can be caused by a few conditions other than pregnancy. Therefore, the presence of HCG in serum specimens should not be used to diagnose pregnancy unless these conditions have been ruled out.
- 4. A natural pregnancy can also be distinguished from an ectopic pregnancy based on HCG. However, a systematic decrease may cause confusion in interpreting test results.
- 5. Steroid treatments, based on HCG, may cause false results. Consult your physician.

Sensitivity:
The Pregnancy Test detects hCG in the urine at a concentration of 10 mIU/ml or greater.

Specificity:
The test has been standardized to the WHO International Standard. The addition of 10 (300 mIU/ml) FSH (1,000 mIU/ml) and FSH/LH 600 mIU/ml to negative (0 mIU/ml) hCG and positive (10 mIU/ml) hCG specimens showed no cross-reactivity.

CLINICAL USE AND ANSWERS

Q *How soon after I suspect that I am pregnant can I take the test?*
You can test your urine as early as the first day you miss your period. You can perform the test anytime of the day. However, if you are pregnant, first morning urine contains the most pregnancy hormone.

How accurate is the test?
A clinical evaluation was conducted comparing the results obtained using the pregnancy test to another commercially available urine retestable HCG test. The crossover clinical trial included 238 urine specimens: both assays identified 16 positive and 146 negative results. The results demonstrated a 98% overall accuracy of the pregnancy test when compared to the other urine retestable HCG test.

What factors may affect the test result?
Drugs which contain HCG (such as Pregnyl, Pregval, Pregnyl, AP₁) can give a false positive result. Alcohol, and contraceptives,

penicillins, antibiotics or hormone therapies that do not contain HCG should not affect the test result.

Q What should I do if the result shows that I am pregnant?
 It means that your urine contains hCG and you are probably pregnant. Stop your doctor to confirm that you are pregnant and to discuss the steps you should take.

Q Does the result mean anything other than normal pregnancy if it shows that I am pregnant?
 A number of medical conditions other than pregnancy, including ovarian cyst or ectopic pregnancy (pregnancy outside the uterus) can cause elevated levels of hCG.

Q What should I do if the result shows that I am not pregnant?
 It means that no hCG has been detected in your urine and probably you are not pregnant. If you do not start your period within a week of its due date, repeat the test with a new test. If you receive the same result after repeating the test and you still do not get your period, you should see your doctor.

Q How do I know that the test was run properly?
 The appearance of a red line in the control zone (C) tells you that you followed the test procedure properly and the proper amount of urine was absorbed.

Please take the following steps to increase your chances for a healthy pregnancy and your baby:

1. Start the Pregnancy Test to detect pregnancy when your period is late. You can begin better prenatal care as soon as you learn of your pregnancy.
2. If you get a positive result, it is advisable to visit your doctor immediately.
3. Maintain a well-balanced diet, stop smoking, and reduce your intake of alcohol.

2. Katarin MS. Histological evaluation of early pregnancy arrest. *Soc Sci Med*. 1980; 30(1): 1-13.
3. Call AS, Mah Chiu S. Vaginitis: Appearance of NBT in pregnancy plasma following the initiation of implantation of the blastocyst. *J Clin Endocrinol Metab*. 1975; 60(3): 543-549.
4. Bomanjee SZ, I Razon, H. Damski, J. Adler, M. Wade Serum human chorionic gonadotropin levels throughout normal pregnancy. *Am J Obstet Gynecol*. 1976; 126(2): 879-881
5. Lyden SB, LM Shaw, R. Salazar Plasma concentration of human chorionic gonadotropin from the time of implantation until the second week of pregnancy. *Fertil Steril*. 1982; 37(6): 773-776.
6. Stancovski MR, De Simoni, C Human chorionic gonadotropin in maternal plasma after subcutaneous abortion, spontaneous abortion and threatened miscarriage. *Gynecol Gynaecol*. 1984; 64(2): 351-358
7. Stancovski MR, De Simoni, C Human chorionic gonadotropin and its subunits in hydatidiform mole and choriocarcinoma. *Cancer*. 1977; 39(2): 172-182
8. Brownstein BD, E. Yehlebach, PP Charney, CJT Beta-Estrone synthesis of human chorionic gonadotropin by trophoblasts. *Ann Intern Med*. 1975; 78(5): 85-85

MEANING OF SYMBOLS (ON PACKAGE)

Free Import License	CE marking of conformity
RoHS free	WEEE recycling symbol
REACH compliant	Recycled material
ISO 9001 certified	Drop ship

FDA has been approved by the U.S. Food & Drug Administration
 UL Classified as Class II medical electrical equipment for use

QUESTIONS OR COMMENTS?

Please contact us via email: sales@promom.com
 King Healthcare Group (China) Ltd.
 3061 Shuen Dr., Shan Ridge, IL USA 60277
 Made in China

www.promom.com

Scan the code to get the manual in more languages

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Sewa Thermal Corporation