REF

GIMA 24450 GIMA 24453 GIMA 24455
GIMA 24451 GIMA 24454

Shandong Lianfa Medical Plastic Products Co., Ltd
No.1 Shuangshan Sanjian Road, Zhangqiu,
Jinan City, 250200, Shandong
PEOPLE'S REPUBLIC OF CHINA
Made in China

REF

04-1818, 04-2122, 04-2322, 04-2618, 04-2818

EC REP

Linkfar Healthcare GmbH, Niederrheinstraße 71,
40474 Düsseldorf, Germania

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

CE 0197

MD

STERILE R

STERILE

M24450-M-Rev.0.10.24

ENGLISH

PRESSURE ACTIVATED SAFETY LANCETS

Modello:


PA

Specification

Specification	Depth (mm)
14G-33G	1,2-2,5
14G-33G Lama	1,2-2,5

Product main structure:

The safety lancet is made up of a needle core, an elastic device, a launching device and a casing.

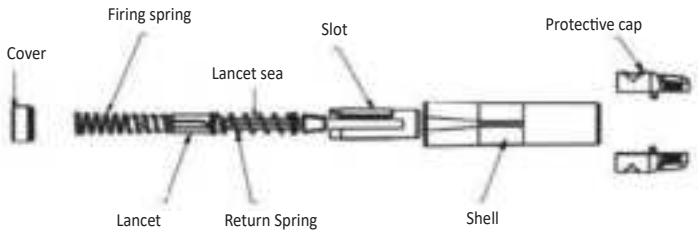Intended Use:

The safety lancets is intended for capillary blood sampling in order to obtain a small blood sample for various tests. It is designed for use by both healthcare professionals and lay users, providing a safe and convenient method for blood specimen collection in clinical and home healthcare settings.

Contraindication:

Use with caution in individuals with coagulation mechanism disorders.

Operating Procedure:

- Please read the instructions before use.
- Wash hands before use. Dry thoroughly.
- Scrub and disinfect the complete skin surface of the blood collection site with medical disinfectants such as iodine or alcohol etc. Healthcare Professionals wear gloves prior to disinfecting the lancing site and completing the lancing procedure as follows.
- Use the safety lancet immediately after opening the primary package.
- Follow the steps instruction to use the safety lancet to draw blood.
- Dispose of any used safety lancet to a biohazard waste container. Wash hands after use.

Instruction:

DOUBLE PROTECTION

1. Wash hands before use. Scrub and disinfect the complete skin surface of the blood collection site with medical disinfectants such as iodine or alcohol etc. Healthcare Professionals wear gloves prior to disinfecting the lancing site and completing the lancing procedure as follows.
2. Twist the protective cap off.
3. Hold the safety lancet against the patient's fingertip and press down gently to draw blood.
4. Push the cap into the used safety lancet reversely until the cap is fully secured and then dispose of it to a proper waste container.
5. Wash hands after use.
6. For the paediatric and neonatal patients, first immobilize the child by asking the parent to:
 - 6.1 Sit on the phlebotomy chair with the child on the parent's lap;
 - 6.2 Immobilize the child's lower extremities by positioning their legs around the child in a cross-leg pattern;
 - 6.3 Extend an arm across the child's chest and secure the child's free arm by firmly tucking it under their own;
 - 6.4 Grasp the child's elbow and hold it securely;
 - 6.5 Use his or her other arm to firmly grasp the child's wrist, holding it palm up.

Indications for use:

- The safety lancet is a single use device indicated for capillary blood sampling.
- For single use only. Not use more than one patient. The device is deactivated after single use.

Waring:

- For single use only. Not intended for more than one use. Not use more than one patient. The device is deactivated after single use, can not be used more than once.
- Improper use of safety lancets can increase the risk of inadvertent transmission of blood borne pathogens, particularly in settings where multiple patients are tested.
- Expiration time is 5 years and using the product beyond the expiration is strictly prohibited.
- The product has no therapeutic or diagnostic function.
- If the lancet protective cap is damaged or lost, please do not use it.
- Use it with caution for those with coagulation mechanism disorders.
- For obese users or individuals who require larger blood volumes, it is recommended to choose products with small "G" value or large puncture depth for blood collection.

Storage and transportation:

1. The device shall be stored at an environment where the temperature is between -20 ° C and 40 ° C and the relative humidity of less than 80%. The storage environment shall be free of corrosive gases, dry, away from sunlight, well-ventilated conditions, and clean.
2. The device shall be taken care during transportation and handling.

Shelf life:

Valid period of product is 5 years.

Clinical benefit:

Help medical staff and patients complete capillary blood sampling in order to obtain a small blood sample for various tests.

Adverse events:

We conducted a search for adverse events and state of the art literature on equivalent and similar devices in the market. No adverse events found for equivalent and similar

products, indicating compliance with the current technical standards.

If any adverse events occur during the use of the product, please contact us and we will resolve the relevant issues as soon as possible.

Additionally, please note that any serious incidents related to the device should be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is established.

Disposal:

- The product must be disposed of after use.
- Products contaminated with blood must be disposed of in accordance with the valid regulations for hazardous materials.
- The product must not be used after the expiry date.
- Safe disposal is carried out in accordance with current national regulations

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos- índice de símbolo
 - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Sym-
 boli-indeksi - Indeks simbolov - Index symbolov - Index de simbol - Symbool index - Indeks sim-
 bola - Szimbólum index - Symbolindeks - Indeks over symboler - Индекс на символа - Simbolių
 rodyklė - Simbolu rādītājs - Sümbolite indeks - Указатель символов - تالوييوس تارشؤم -
 الشركة المصنعة - SA

	IT - Fabbricante GB - Manufacturer FR - Fabricant ES - Fabricante PT - Fabricante DE - Hersteller GR - Παραγωγός PL - Producent CZ - Výrobce SE - Tillverkare FI - Valmistaja SI - Proizvajalec SK - Výrobca RO - Producător NL - Fabrikant HR - Proizvođač HU - Gyártó DK - Fabrikant NO - Produsent BG - Производител LT - Gamintojas LV - Ražotājs EE - Tootja RU - Производитель
	IT - Data di fabbricazione GB - Date of manufacture FR - Date de fabrication ES - Fecha de fabricación PT - Data de fabrico DE - Herstellungsdatum GR - Ημερομηνία παραγωγής PL - Data produkcyj CZ - Datum výroby SE - Tillverkningsdatum FI - Valmistuspäivämäärä SI - Datum proizvodnje SK - Dátum výroby RO - Data fabricației NL - Productiedatum HR - Datum proizvodnje HU - Gyártás dátuma DK - Fabrikationsdato NO - Fabrikasjonsdato BG - Дата на производство LT - Pagaminimo data LV - Izgatavošanas datums EE - Valmistamise kuupäev RU - Дата выпуска
1 2 3 4 5 6 STERILE R	IT - Sterilizzato per irradiazione GB - Sterilized using irradiation FR - Stérilisé par irradiation ES - Esterilizado por irradiación PT - Esterilizado por irradiação DE - Durch Bestrahlung sterilisiert GR - ποστεριώνεται με ακτινοβολή PL - Sterylizowane napromienianiem CZ - Sterilizováno ozařováním SE - Steriliserad genom bestrålning FI - Steriloitu säteilyttämällä SI - Sterilizirano z obsevanjem SK - Sterilizované ožiarením RO - Sterilizat prin iradiere NL - Gesteriliseerd door bestraling HR - Sterilizirano zračenjem HU - Besugárzással sterilizált DK - Steriliseret ved bestråling NO - Sterilisert ved bestråling BG - Стерилизиран чрез облъчване LT - Sterilizuojamas švitinant LV - Sterilizēts ar apstarošanu EE - Steriliseeritud kiiritusega RU - Стерилизовано с использованием облучения

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