



THE EXTRA-POWERFUL CRYOSURGERY DEVICE

CryoPen sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness – as well as an outstanding patient experience.


This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

As the CryoPen B+ works with our largest cartridges, it delivers more power and is suitable for treating larger or deeper lesions.



Key features

- Ergonomic design for superior control
- Leading-edge microtechnology ensures controlled gas flow
- Long cartridge life with easy-to-load 23.5g N₂O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 3 autoclavable applicators
- Suitable for regular use
- Certified **CE1639** **FDA** **MDSAP**

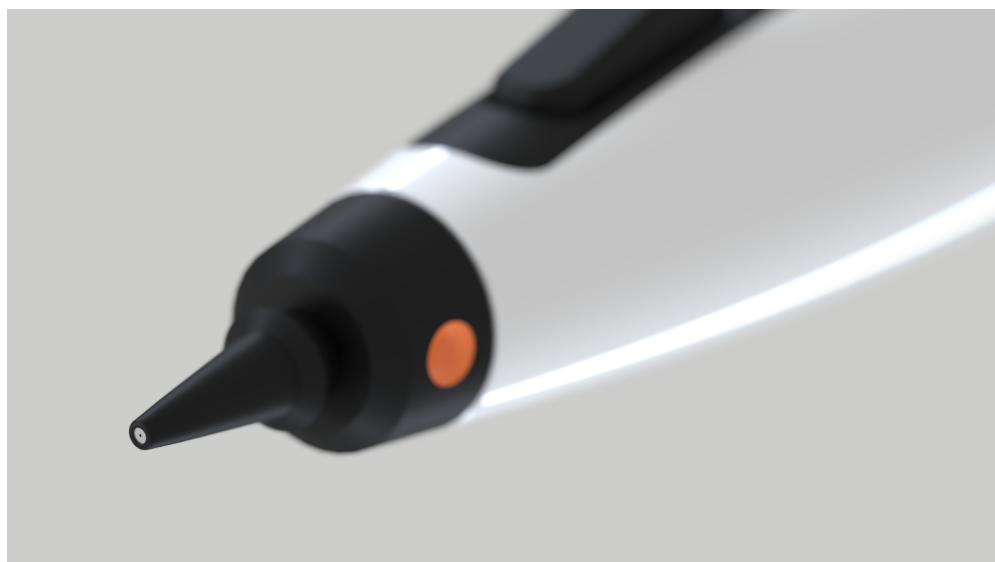



APPLICATORS



Ø 1-4mm

Power: ■



Ø 2-6mm

Power: ■ ■



Ø 4-8mm

Power: ■ ■ ■

Continuous flow time with a 23.5g cartridge



PRODUCT INFORMATION

Device

The CryoPen B+ is suitable for all treatment situations where stronger freezing power is required.

Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, cutaneous leishmaniasis, discoid lupus erythematosus, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, rosacea, lichen sclerosus, granuloma annulare, molluscum contagiosum, seborrheic keratosis and viral lesions.



CONTENTS OF THE STARTER SET

Device

CryoPen B+

Applicators

1-4mm applicator

2-6mm applicator

4-8mm applicator

Cartridges

4x 23.5g cartridges

Accessories

Metal carrying case

Forceps

Device stand

Documentation

Treatment overview

User manual

BASICS

Dimensions

Device: 21.3 x 3 x 3 cm

Carrying case: 28.8 x 22.8 x 6.7 cm

Starter set: 38 x 30 x 7.5 cm

HS codes

90189084 (device)

28112930 (cartridges)

Weight

Device with a full cartridge: 280g

Starter set: 2,467g

