


# Getting Started Guide

## TD I-110




## What's in the Box?

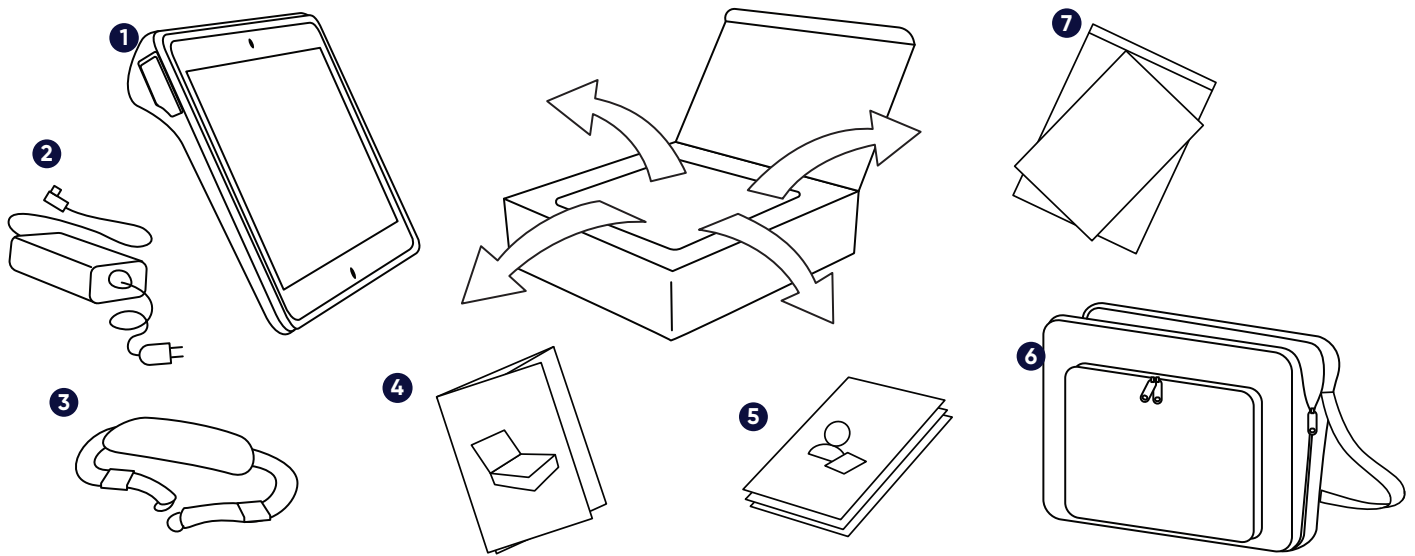

- |                         |                                                             |
|-------------------------|-------------------------------------------------------------|
| 1 Device                | 5 TD Snap Training Cards                                    |
| 2 Power Cable           | 6 Carry Case                                                |
| 3 Shoulder Strap        | 7 Safety & Compliance, License, and Configuration Documents |
| 4 Getting Started Guide |                                                             |

## Getting to Know Your Device

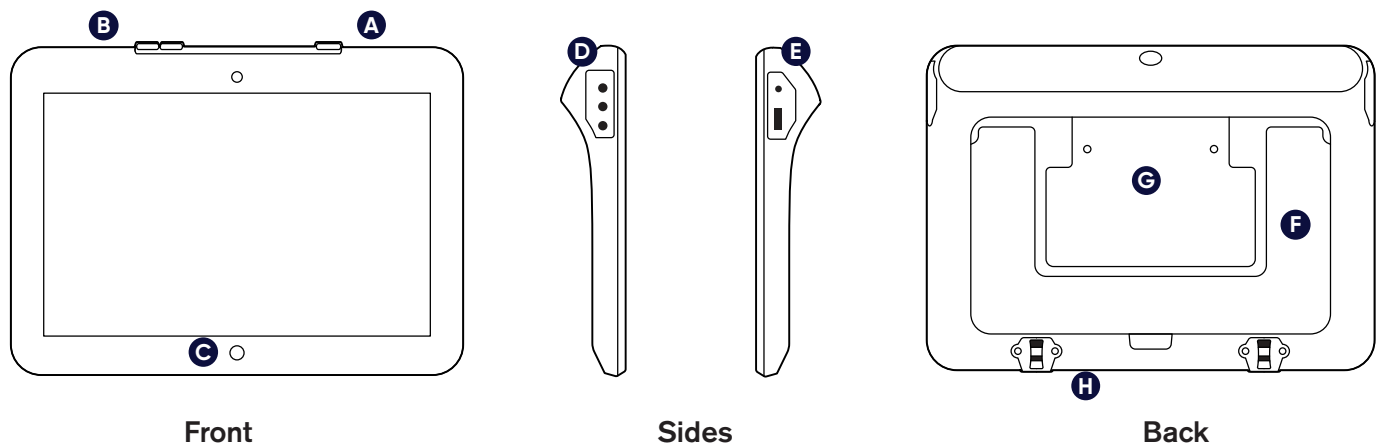



- |                  |                        |                     |
|------------------|------------------------|---------------------|
| A Power Button   | D Audio & Switch Ports | F Stand             |
| B Volume Buttons | Power Connector        | G Mounting Location |
| C Home Button    | E & USB Ports          | H Strap Connectors  |

Follow the steps in this guide to set up and start using your device.

## Step 1: Initial Setup

### Start the Device


- 1 Connect the power cable to the device and plug in to an outlet
- 2 Press the Power button

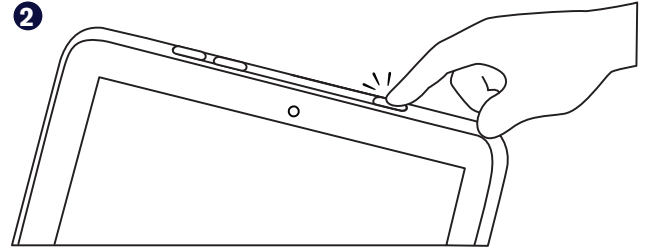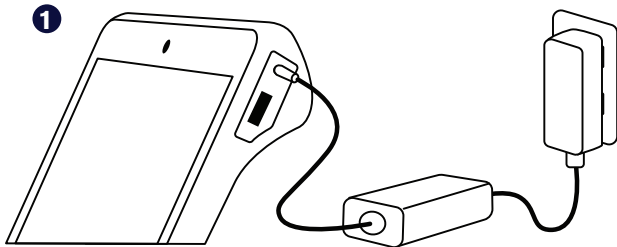

### Windows Setup

If your device has not been pre-configured, you will be prompted to complete the Windows Setup process first.

While this process does not require a Microsoft account, we recommend that you connect to or create the user's account and not a caregiver's account.

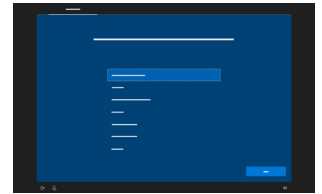

## Step 2: Set Up TD Snap



Double tap the icon to launch the TD Snap app. Follow the prompts to create a new user or restore a saved user file.

## Step 3: Mount & Position

The device may be positioned using a mounting system, laying it flat on a surface, or propped on the integrated kickstand. Start by situating the user comfortably, then find the device position that gives them clear screen visibility and easy access to their selection method of choice. Always position the device to suit the user, not the other way around. It is expected that the device may need to be repositioned throughout the day. Attach the shoulder strap for secure transport.

## Step 4: TD Snap Training Cards

Continue your set up journey with the TD Snap Training Cards that came in the box with your device. The training cards walk you through the features of TD Snap, basic editing, backing up your data, and some activities to help you integrate TD Snap into your daily life.

# Additional Resources

Scan the QR codes with your phone or use the links.



**Tobii Dynavox**

[uk.tobiidynavox.com](http://uk.tobiidynavox.com)



**TD Snap Help**

[qrco.de/bclHVF](http://qrco.de/bclHVF)



**TD Facebook  
Community**

[qrco.de/TDFB](http://qrco.de/TDFB)



**Tobii Dynavox  
Learning Hub (English  
only)**

[learn.tobiidynavox.com](http://learn.tobiidynavox.com)



**TD I -110 Device  
Documentation**

[qrco.de/I-110manual](http://qrco.de/I-110manual)



**myTobiiDynavox**

[mytobiidynavox.com](http://mytobiidynavox.com)



**Learn about access  
methods**

[qrco.de/bbA7US](http://qrco.de/bbA7US)



**UK Technical Support**

0114 481 0011

[support.uk@tobiidynavox.com](mailto:support.uk@tobiidynavox.com)