

-  Do not immerse the device in liquid.
-  It is recommended that the device should be kept in a dry environment. Humidity may reduce the useful life of the device, or even damage it.

9 Troubleshooting		
Trouble	Possible Reason	Solution
The SpO ₂ and Pulse Rate can not be displayed normally	1. The finger is not properly positioned. 2. The user's SpO ₂ is too low to be detected.	1. Place the finger properly and try again. 2. Try again; Go to a hospital for a diagnosis if you are sure the device works all right.
The SpO ₂ and Pulse Rate are not displayed stably	1. The finger is not placed inside deep enough. 2. The finger is shaking or the user is moving.	1. Place the finger properly and try again. 2. Insure user is still with minimal movement.
The device can not be turned on	1. The batteries are drained or almost drained. 2. The batteries are not inserted properly. 3. Device malfunction.	1. Change batteries. 2. Reinstall batteries. 3. Please contact the local service center.
The display is off suddenly	1. The device will power off automatically when it gets no signal within 5 seconds. 2. The batteries are almost drained.	1. Normal. 2. Change batteries.

10 Key of Symbols	
Symbol	Description
	Type BF
	Refer to instruction manual/booklet
SpO ₂ %	The pulse oxygen saturation(%)
PRbpm ♥	Pulse rate (bpm)
	The battery voltage indication is deficient (change the battery in time avoiding the inexact measure)
	1. no finger inserted 2. An indicator of signal inadequacy
	battery positive electrode
	battery negative electrode;
	Power switch
SN	Serial number
	Alarm inhibit
	WEEE (2002/96/EC)
IP22	Ingress of liquids rank
	This item is compliant with Medical Device Directive 93/42/EEC of June 14, 1993, a directive of the European Economic Community
	EUROPEAN REPRESENTATIVE
	Manufacturer
	Manufacture Date
	Storage and Transport Temperature limitation
	Storage and Transport Humidity limitation
	Storage and Transport Atmospheric pressure limitation
	This side UP
	Fragile, handle with care
	Keep dry
	Recyclable

11 Function Specification	
Display Information	Display Mode
The Pulse Oxygen Saturation(SpO ₂)	Digital
Pulse Rate (BPM)	Digital
Pulse Intensity (bar-graph)	Digital bar-graph display
SpO ₂ Parameter Specification	
Measuring range	0%~100%, (the resolution is 1%).
Accuracy	70%~100%: ±2%, Below 70% unspecified
Optical Sensor	Red light (wavelength is 660nm) Infrared (wavelength is 880nm)
Pulse Parameter Specification	
Measuring range	30bpm~250bpm (the resolution is 1 bpm)
Accuracy	±2bpm or ±2% select larger
Pulse Intensity	
Range	Continuous bar-graph display, the higher display indicate the stronger pulse.
Battery Requirement	
1.5V (AAA size) alkaline batteries × 2 or rechargeable battery	
Battery Useful Life	
Two batteries can work continually for 24 hours	
Dimensions and Weight	
Dimensions	57(L) × 31(W) × 32(H) mm
Weight	About 50g (with the batteries)

Appendix:

Electromagnetism Compatibility
Guidance and manufacture's declaration – electromagnetic emissions-
for all EQUIPMENT and SYSTEMS

Guidance and Manufacture’s Declaration – Electromagnetic Emission

The PO2BLU/PO2BLK is intended for use in the electromagnetic environment specified below.
The customer of the user of the PO2BLU/PO2BLK should assure that it is used in such and environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The PO2BLU/PO2BLK uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The PO2BLU/PO2BLK is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A	

Guidance and manufacture's declaration – electromagnetic immunity –
for all EQUIPMENT and SYSTEMS

Guidance and Manufacture’s Declaration – Electromagnetic Immunity

The PO2BLU/PO2BLK is intended for use in the electromagnetic environment specified below.
The customer or the user of PO2BLU/PO2BLK should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 KV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	N/A	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode	N/A	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec	N/A	Mains power quality should be that of a typical commercial or hospital environment. If the user of the PO2PNK/PO2BLK requires continued operation during power mains interruptions, it is recommended that the PO2PNK/PO2BLK be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) Magnetic field IEC-61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields could be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage prior to application of the test level.

Guidance and Manufacture's Declaration – Electromagnetic Immunity –
for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING
Guidance and manufacture’s declaration – electromagnetic immunity

The PO2BLU/PO2BLK is intended for use in the electromagnetic environment specified below.
The customer or the user of PO2BLU/PO2BLK should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the PO2PNK/PO2BLK, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol:. 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the PO2BLU/PO2BLK is used exceeds the applicable RF compliance level above, the PO2BLU/PO2BLK should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the PO2BLU/PO2BLK . ^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended Separation Distances between
portable and mobile RF communications equipment and the PO2PNK/PO2BLK

The PO2BLU/PO2BLK is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the PO2BLU/PO2BLK can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the PO2BLU/PO2BLK as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.39	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

