



**Affordable. Reliable. Home Improvement.**

## **RECLINER**

**MODEL: D577-J**

Technical Support and E-Warranty Certificate  
[www.vevor.com/support](http://www.vevor.com/support)

# VEVOR

Affordable. Reliable. Home Improvement.

## RECLINER

**MODEL: D577-J**



This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

## ASSEMBLY INSTRUCTIONS



**Figure 1**


**NOTE:** Excessive force or effort is never necessary to change the seating position of the chair. Do not operate recliner by applying force or pressure to legrest.




**Figure 2**

**NOTE:** After removing all components from the carton, engage the two casterlocks before beginning assembly.


Figure 2 shows location of two clevis pins, two ring cotter clips, one bolt, and one nut. Remove all above hardware from the back brace.

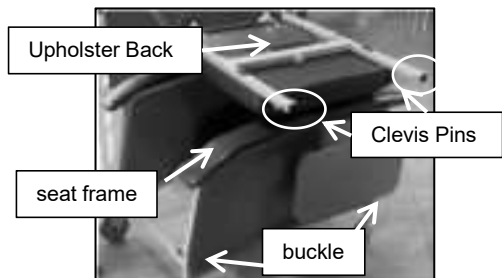

Step 1

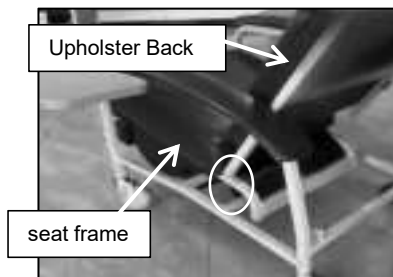

Step 2

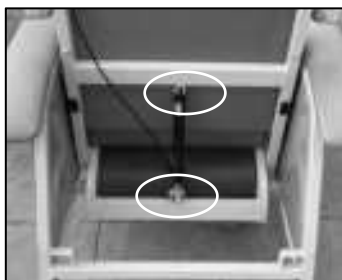

Step 3



Step 4

**Figure 3**

**Step1:** Undo the Clevis Pins on Upholster Back, both sides so mounting holes are exposed. Press the buckles to remove the side plates(both side).

**Step2:** Align the holes on the upright back tubes of the upholstered back with the holes on the seat frame. Replace the clevis pins through the mounting holes. Secure the two clevis pins by replacing the bridge clips.

**Step3:** Remove cardboard shipping retainers from around Gas Spring Cylinder. The holes at both ends of the gas spring are aligned with the holes in the seat frame cross brace and the holes in the upholster back.

**Step4:** Align holes in arm support mount to mounting holes in seat frame. Secure screw with nut.Repeat for other side.

## INSTRUCTIONS FOR USE

### Slide Tables

#### To place either table in raised position:


Raise table all the way up and slide in until tabs engage slots in mount bracket. Press down lightly on table to ensure lock tabs are engaged.

#### To return table to self-storing position:

Lift table up until tabs disengage slots. Slide out and allow table to lower. See Figure 4.

### Articulating Headrest

Articulating headrest may be adjusted to a comfortable position by pulling forward or pushing back.

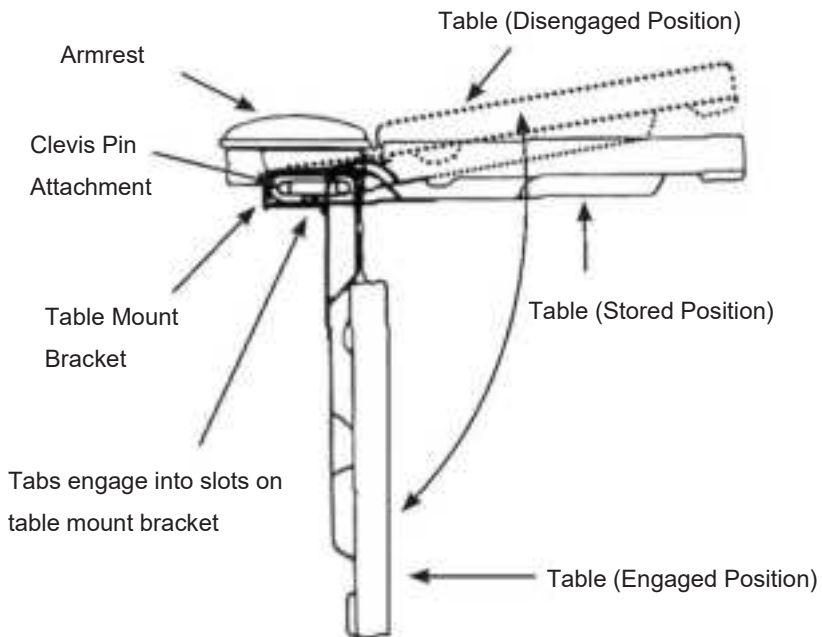

**Figure 4**

## **Seat Positioning**

**Upright:**Initial position.From any position,grasp the gas spring actuating lever, slowly guide the back up fully and release the lever. Return patient to desired position by pushing upward on padded push handle.

**Deep recline:**From Initial position, grasp the gas spring actuating lever with one hand, push back on padded push handle.

**Elevated leg rest:**From Initial position,do not grasp the gas spring actuating lever, push back on padded push handle.

**Ttrendelenburg position:**From any position,grasp the gas spring actuating lever with one hand while actuating the lever with the other, slowly guide the back down fully and release the lever.

**Other:**Articulating headrest may be adjusted to desired position by pulling forward or pushing back.



upright



deep recline



elevated leg rest



trendelenburg position



Upright(back)

Gas Spring  
Actuating  
Lever

**Figure 5**


## CASTER OPERATION

### To lock casters:


To prevent the rear casters from turning, step on the rear pedal of the caster until the pedal engages. (Front of pedal will protrude from wheel cover).

### To unlock casters:

Step on the protruding front part of the pedal to disengage lock to unlatch.

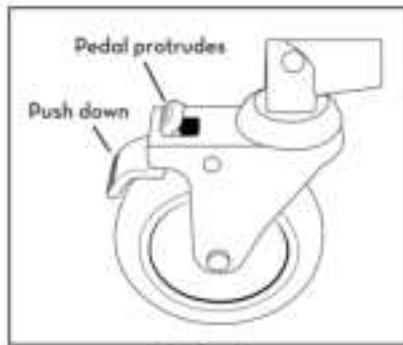

**To Lock**

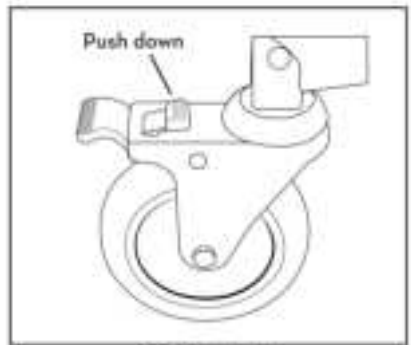

**To Unlock**

**Figure 6**

### CAUTION:

Do not operate recliner by applying force or pressure to the legrest. Avoid placing hands or legs in creases near recliner mechanism.

Casters must be locked except during transport.

## MAINTENANCE&CLEANING

### EVERY THREE MONTHS

Check and clean casters/Tighten nuts/Check brakes on casters/Check and clean lock recline mechanism

For upholstery maintenance and stain removal, the following hints will be helpful:

Ordinary dirt: Wash with warm awater and a mild soap or detergent.



Stubbon dirt: Using a soff bristle brush, follow directions above.

Ground-in dirt: Scrub with powdered cleanser or similar detergent.

Chewing gum: Scrape carefully.

Nail polish and remover: Blot immediately to prevent damage.

Ball point pen: Blot immediately with a white cloth dampened in watch or mineral oil.

Waxing and reffnishing: Improver wearability. Use any quality paste wax.

A diluted solution of 10% (or less) of bleach or clorox may be used when cleaning this chair.

**Manufacturer:** Jiangyin Huashi Medical Equipment Co., Ltd

**Address:** No. 589 Hualu Road, Jiangyin City, Wuxi City, Jiangsu Province,214400,China

**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP	YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX
EC	REP	E-CrossStu GmbH Mainzer Landstr.69, 60329 Frankfurt am Main.



**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**