




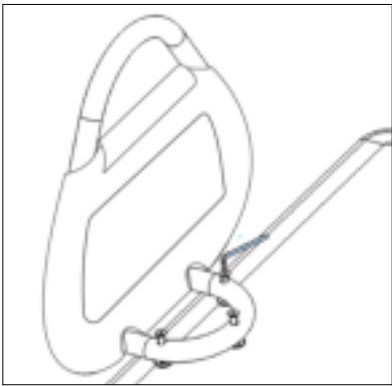


Support Rail Installation Guide

Parts List						
A.		B.		C.		A) Assist Rail
D.		E.				B) Baffle
						C) M8x40 Hex Screw (x3)
						D) M8x65 Hex Screw (x3)
						E) Hex Key

Installation



- 1 Align the assist rail with the mounting holes on the left side of the bed. Secure it in place with the M8x65 hex screws. Use the hex key to tighten.



⚠ WARNING

Only remove the support rail cover to wash. Replace immediately before using. Using the bed without the support rail cover may increase the risk of serious injury.

- 2 **If only installing one support rail, be sure to install the bolster on the opposite side to keep the mattress in its place and prevent entrapment hazards.** To install the support rail, align the baffle with the mounting holes on the opposite side of the bed. Secure it using the M8x40 hex screws. Use the hex key to tighten.

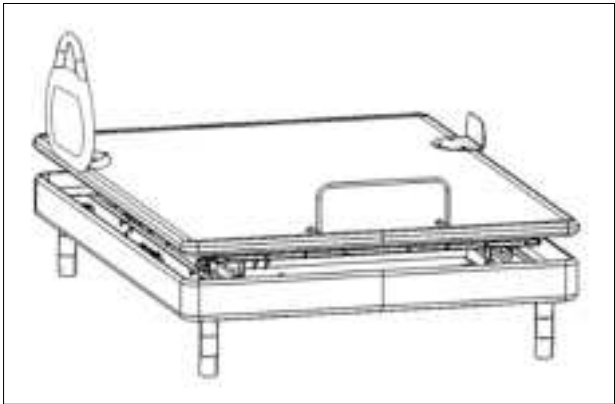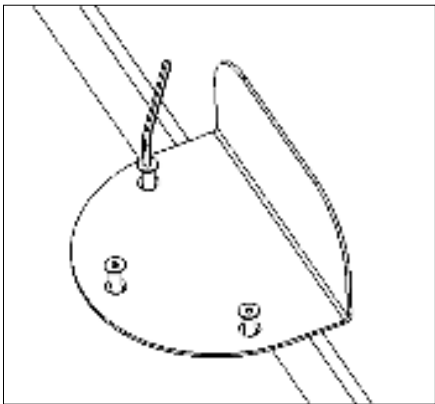

Installation is complete.