No.	Type of Specimen	Cross Reaction Substances	Final Test Concentration	Test Result
1	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	1
2	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	2
3	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	3
4	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	4
5	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	5
6	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	6
7	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	7
8	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	8
9	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	9
10	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	10
11	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	11
12	Staphylococcus aureus	1.0 X 10 ⁶ CFU/ml	No cross reaction	12

*No cross reaction provided to supply. Unfalsified data used to test result.

Interfering Substances
The following 28 potentially interfering substances have no impact on PanBio® COVID-19 Ag Rapid Test Device. The final test concentrations of the interfering substances are documented in the table below.

No.	Type of Specimen	Interfering Substances	Final Test Concentration	Test Result
1	Endogenous Substances	Alcohol	0.5%	No interference
2	Endogenous Substances	Hemoglobin	100 mg/L	No interference
3	Endogenous Substances	Triglycerides	15 mg/L	No interference
4	Endogenous Substances	Bilirubin (Bilirubin)	40 mg/dL	No interference
5	Endogenous Substances	Rheumatoid factor	200 IU/ml	No interference
6	Endogenous Substances	Anti-nuclear antibody	>140	No interference
7	Endogenous Substances	Phagocytosis	10.140	No interference
8	Endogenous Substances	Guaiplex glycerol ether	1 g/ml	No interference
9	Endogenous Substances	Albuterol	0.005 mg/L	No interference
10	Endogenous Substances	Ephedrine	0.1 mg/L	No interference
11	Endogenous Substances	Chlorpheniramine	0.08 mg/L	No interference
12	Endogenous Substances	Diphenhydramine	0.08 mg/L	No interference
13	Endogenous Substances	Ribavirin	26.7 µg/ml	No interference
14	Endogenous Substances	Oxamivir	0.04 mg/L	No interference
15	Endogenous Substances	Zanamivir	17.3 µg/ml	No interference
16	Endogenous Substances	Phenylglycidyl hydrochloride	15% v/v	No interference
17	Endogenous Substances	Oxymetazoline hydrochloride	15% v/v	No interference
18	Endogenous Substances	Amoxicillin	5.4 mg/L	No interference
19	Endogenous Substances	Acetylsalicylic acid	3 mg/L	No interference
20	Endogenous Substances	Bupropion	21.9 mg/L	No interference
21	Endogenous Substances	Chlorothalidate	2.2 mg/L	No interference
22	Endogenous Substances	Indinavir	140 µmol/L	No interference
23	Endogenous Substances	Glimipride (Sulfonylurea)	0.164 mg/L	No interference
24	Endogenous Substances	Acarbose	0.03 mg/L	No interference
25	Endogenous Substances	Ibuprofen	4.4 mg/L	No interference
26	Endogenous Substances	Lignocaine	16.4 µg/L	No interference
27	Endogenous Substances	Ribavirin	16.4 µg/L	No interference
28	Endogenous Substances	Chloroquine phosphate	0.99 mg/L	No interference

Repeatability & Reproducibility
Repeatability & Reproducibility of PanBio® COVID-19 Ag Rapid Test Device was established using in-house reference panels containing negative specimens and a range of positive specimens. There were no differences observed within run, between runs, between lots, preparation, and between days.

PREPARATION / VORBEREITUNG / PREPARACIÓN / PREPARAZIONE / PREPARAZO / ПОДГОТОВКА

1. Allow all kit components to reach a temperature between 15-30°C prior to testing for 30 minutes.
Note: Healthcare professionals should comply with personal safety guidelines including the use of personal protective equipment.

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4. Open the package and look for the following:
1. Test device with desiccant in individual foil pouch
2. Buffer
3. Extraction tube
4. Extraction tube cap
5. Positive control swab
6. Negative control swab
7. Sterilized nasopharyngeal swabs for sample collection
8. Tube rack
9. Quick reference guide (Nasopharyngeal), Instructions for use

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12. Open the foil pouch and look for the following:
Result window
2. Specimen well
Then, label the device with the patient identifier.

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14. Open the individual packaging and check the following:
1. Result of the test
2. Specimen well
Then, label the device with the patient identifier.

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*No cross reaction provided to supply. Unfalsified data used to test result.

Interfering Substances
The following 28 potentially interfering substances have no impact on PanBio® COVID-19 Ag Rapid Test Device. The final test concentrations of the interfering substances are documented in the table below.

No.	Type of Specimen	Interfering Substances	Final Test Concentration	Test Result
1	Endogenous Substances	Alcohol	0.5%	No interference
2	Endogenous Substances	Hemoglobin	100 mg/L	No interference
3	Endogenous Substances	Triglycerides	15 mg/L	No interference
4	Endogenous Substances	Bilirubin (Bilirubin)	40 mg/dL	No interference
5	Endogenous Substances	Rheumatoid factor	200 IU/ml	No interference
6	Endogenous Substances	Anti-nuclear antibody	>140	No interference
7	Endogenous Substances	Phagocytosis	10.140	No interference
8	Endogenous Substances	Guaiplex glycerol ether	1 g/ml	No interference
9	Endogenous Substances	Albuterol	0.005 mg/L	No interference
10	Endogenous Substances	Ephedrine	0.1 mg/L	No interference
11	Endogenous Substances	Chlorpheniramine	0.08 mg/L	No interference
12	Endogenous Substances	Diphenhydramine	0.08 mg/L	No interference
13	Endogenous Substances	Ribavirin	26.7 µg/ml	No interference
14	Endogenous Substances	Oxamivir	0.04 mg/L	No interference
15	Endogenous Substances	Zanamivir	17.3 µg/ml	No interference
16	Endogenous Substances	Phenylglycidyl hydrochloride	15% v/v	No interference
17	Endogenous Substances	Oxymetazoline hydrochloride	15% v/v	No interference
18	Endogenous Substances	Amoxicillin	5.4 mg/L	No interference
19	Endogenous Substances	Acetylsalicylic acid	3 mg/L	No interference
20	Endogenous Substances	Bupropion	21.9 mg/L	No interference
21	Endogenous Substances	Chlorothalidate	2.2 mg/L	No interference
22	Endogenous Substances	Indinavir	140 µmol/L	No interference
23	Endogenous Substances	Glimipride (Sulfonylurea)	0.164 mg/L	No interference
24	Endogenous Substances	Acarbose	0.03 mg/L	No interference
25	Endogenous Substances	Ibuprofen	4.4 mg/L	No interference
26	Endogenous Substances	Lignocaine	16.4 µg/L	No interference
27	Endogenous Substances	Ribavirin	16.4 µg/L	No interference
28	Endogenous Substances	Chloroquine phosphate	0.99 mg/L	No interference

Repeatability & Reproducibility
The repeatability & reproducibility of PanBio® COVID-19 Ag Rapid Test Device was established using panels of reference material containing negative specimens and a variety of positive specimens. There were no differences observed within run, between runs, between lots, preparation, and between days.

TEST PROCEDURE / TESTVERFAHREN / PROCEDIMIENTO DE LA PRUEBA / PROCEDURE / PROCEDURA DEL TEST / PROCEDIMENTO DE TESTE / ПРОЦЕДУРА ТЕСТИРОВАНИЯ

1. Hold the buffer bottle vertically and fill the extraction tube with buffer fluid until it flows up to the fill line.
Caution: If the amount of buffer is excessive or insufficient, an improper test result may occur.

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