READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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CONTENT

1. IMPORTANT SAFETY INSTRUCTIONS.....	2	8. Other Functions	9
2. Introduction	3	9. Care and Maintenance	9
3. Parts and Controls.....	4	10. Troubleshooting Guide.....	9
4. Initial Use	4	11. Specifications	10
5. Entering User Data	5	12. FCC Information	10
6. Taking Measurements	6	13. Warranty	10
7. Evaluating Results.....	7		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,
Your Beurer Team

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

The following signs appear in the Safety Section (page 2) and in this manual on page 2.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	NOTICE	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
		Follow instructions for use
		Type BF applied part
IP 21		Protection against ingress of water

	Disposal in accordance with EC Directive 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer

NOT TO DIAGNOSE OR TREAT ANY MEDICAL CONDITION.

WARNING – To reduce the risk of serious personal injury:

- This scale must not be used by people with medical implants (e.g. pacemakers), as this may affect their functionality.
- Do not use during pregnancy.
- To prevent falls, do not step onto the scale with wet feet or if the surface of the scale is damp.

CAUTION – To reduce the risk of personal injury:

- This device is intended solely for household use and not for medical or commercial purposes.
- Place the scale on a hard, flat, level surface; a hard surface is crucial to accurate measurements.
- Protect the device from impacts, moisture, dust, chemicals, large temperature fluctuations, and nearby sources of heat (ovens, heaters).

NOTICE – To reduce the risk of product and/or property damage:

- This device should be cleaned from time to time. Do not use any abrasive cleaning products and never submerge the device in water.
- Measurement variations are normal, as this scale is not calibrated for use in a professional medical environment.
- Make sure that no liquids come into contact with the scale. Never submerge the scale in water. Never rinse it in running water.
- Do not place any objects on the scale when not in use.
- Protect the device from knocks, damp, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Do not press the buttons violently or with pointed objects.
- Do not expose the scale to high temperatures or strong electromagnetic fields.
- The scale's capacity is 330 lb (149.7 kg/23 st:8.7 lb). The results for weight and bone mass are displayed in 0.2 lb (100g) increments.
- Measurements of body fat, body water and muscle content are displayed in 0.1% increments.
- Calorie requirement is indicated in steps of 1 kcal.
- This scale is set at the factory to weigh and measure in "lb". See instructions to change to kg or stones.
- Repairs must only be performed by Beurer Customer Service.

2. Introduction

This digital body analysis scale is intended for measuring body weight and providing a analysis of your personal fitness data. It is intended for household use only. In addition to measuring body weight, this scale also includes the following functions and features:

- Body fat percentage
- Body water content
- Muscle percentage
- Bone mass
- Basal metabolic rate and active metabolic rate
- Switches between kilograms (kg), pounds (lb), and stones (st)
- Interpretation of body fat percentage
- Automatic shutoff
- Low battery indicator

This scale works on the principle of Bioelectrical Impedance Analysis (BIA). This involves a quick calculation of body content using a harmless electric current that is not even noticeable.

When measurement of the electrical resistance (impedance) is considered alongside other data (age, height, gender, and activity level), body fat percentage and other variables in the body can be calculated. Muscle tissue and water conduct electricity well and therefore have a lower resistance. In contrast, bones and fat tissue have low conductivity due to very high levels of resistance.

Calculated values only approximate the actual values for your body. Only a doctor can precisely establish body fat, body water, muscle percentage, and bone structure using medical methods.

Tips:

- If possible, always weigh yourself at the same time every day (ideally in the morning), after having used the restroom, on an empty stomach and with as little clothing as possible, in order to obtain comparable, consistent results.
- Body fat percentage can only be calculated when you are barefoot; the soles of the feet may be lightly dampened where appropriate. Results may be unsatisfactory if the soles of the feet are completely dry or have rough patches, as this results in low conductivity.
- Stand up straight and still while weighing.
- Wait a few hours after significant physical exertion.
- Wait about 15 minutes after waking so that your body's water content can normalize.
- Remember that only long-term trends are important. Brief deviations in weight within a few days are normally caused by loss of fluids; however, body water plays an important role in terms of well-being.

Limitations


When calculating body fat and additional values, varying and unexpected results may occur in:

- Children under 10 years of age
- Performance athletes and bodybuilders
- Pregnant women
- Persons with a fever, persons undergoing dialysis, persons demonstrating edema symptoms, and persons suffering from osteoporosis
- Persons taking cardiovascular medicine (affecting the heart and vascular system)
- Persons taking vasodilating or vasoconstricting medication
- Persons with significant anatomical deviations in their legs compared with their overall body height (leg length significantly shorter or longer).

3. Parts and Controls

1. Electrodes
2. LCD Display
3. Up button (▲)
4. SET button
5. Down button (▼)
6. Battery Compartment (on rear)
7. Rear of the Scale
8. Unit selector button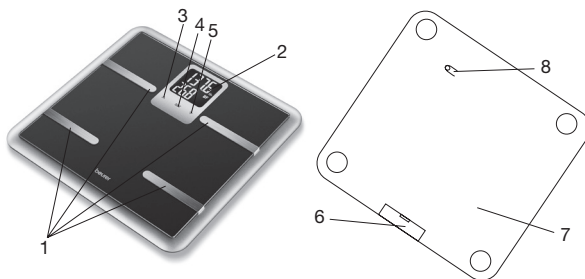

4. Initial Use


Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.

- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

Inserting the batteries

This product requires 3 x AAA 1.5V batteries (included). Remove the plastic battery isolator strip by pulling it from the Battery Compartment Cover. If the scale does not function, remove the batteries completely and reinsert them, ensuring proper polarity.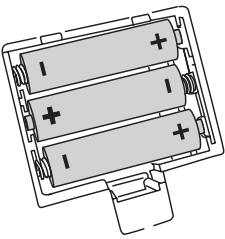

Changing the weight unit

The scale is set to weigh in pounds (lb) from the factory. Switch the scale on by briefly applying pressure to the top surface with your foot for 1-2 seconds. Wait until "lb" appears in the display. Press the Unit Selector Button on the rear of the scale until the desired weight unit appears in the LCD Display ("kg" for kilograms or "st" for stones).

Positioning the scale

Place the scale on a hard, flat surface with no carpet or rugs; a hard surface is necessary for accurate measurements.

5. Entering User Data

To calculate your body fat percentage and other values, you must save your personal user data. The scale can store 10 separate users, which allows you and members of your family to save and access each of your personal settings.

1. With the scale on a hard, flat surface, switch the scale on by briefly applying pressure to the surface with your foot (approx. 1-2 seconds). Wait until "0.0" and the selected measurement unit appears in the LCD Display.
2. Press the SET Button. The first memory space flashes in the display.
3. Press the ▲/▼ button to select the desired memory space and confirm by pressing the SET Button.
4. You can now make the following settings:

User data	Set values
Users/persons	1 to 10
Height	3' 3.5" to 7' 2.5" (100 to 220 cm)
Age	10 to 100 years
Gender	Male (♂), Female (♀)
Activity level	1 to 5

Activity levels

Select an activity level that best reflects your lifestyle.

Activity Level	Physical Activity
1	None
2	Low: Small amount of light physical exertion (e.g. going for walks, easy gardening, simple gymnastics)
3	Medium: Physical exertion at least 2 to 4 times a week for 30 minutes each

Activity Level	Physical Activity
4	High: Physical exertion at least 4 to 6 times a week for 30 minutes each
5	Very high: Intense physical exertion, training or hard physical work at least 1 hour every day

1. Change values by pressing the ▲ Button or the ▼ Button, or press and hold to quickly scroll through the options.
2. Confirm the entries by pressing the SET button.
3. Once the values have been set, “0.0 lb” is displayed.
4. Now stand on the scale with bare feet while “0.0 lb” is displayed.
5. After your weight is displayed, body fat (BF), water, muscle percentage, bone mass, BMR and AMR are displayed.

NOTE: Your measured weight has now been assigned to the selected user and saved.

6. Taking Measurements

Weighing and performing diagnostics

NOTE: Do not let your feet, legs, calves or thighs touch each other. Otherwise, the measurement cannot be taken correctly. Measurement results will also be incorrect if you are wearing socks.

1. Switch the scale on and wait until “0.0 lb” (Fig.2) appears.
2. Select the desired memory space by pressing the ▲ or ▼ Buttons.
3. Step onto the scale with bare feet and stand still with your weight distributed evenly on both sets of sensors.

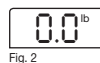
Fig. 2

First, the weight is displayed. If you are barefoot, BIA diagnostics will also display as follows. If a user has been assigned, the user number is briefly displayed (e.g., P-1); afterwards, the following values appear:

1. Weight
2. Body fat % with interpretation (BF)
3. Body water in % 
4. Muscle percentage in % 
5. Bone mass 
6. Basal metabolic rate in kcal (BMR)
7. Active metabolic rate in kcal (AMR) 

Weight measurement only

Tap the scale firmly; “8888” (Fig.1) will initially appear and then “0.0 lb” (Fig.2) will display. Step on the scale wearing shoes. Stand still with your weight distributed equally on both feet until your weight is displayed.

Fig. 1

Fig. 2

NOTE: BIA diagnostics cannot be calculated when you are wearing shoes.

The scale will shut off by itself after approx. 10 seconds.

7. Evaluating Results

Body fat percentage

The following body fat percentages are a guideline only; for further information, ask your doctor.

Male —  +

Age	Low	Normal	High	Very high
10–14	<11%	11–16%	16.1–21%	>21.1%
15–19	<12%	12–17%	17.1–22%	>22.1%
20–29	<13%	13–18%	18.1–23%	>23.1%
30–39	<14%	14–19%	19.1–24%	>24.1%
40–49	<15%	15–20%	20.1–25%	>25.1%
50–59	<16%	16–21%	21.1–26%	>26.1%
60–69	<17%	17–22%	22.1–27%	>27.1%
70–100	<18%	18–23%	23.1–28%	>28.1%

Female —  +

Age	Low	Normal	High	Very high
10–14	<16%	16–21%	21.1–26%	>26.1%
15–19	<17%	17–22%	22.1–27%	>27.1%
20–29	<18%	18–23%	23.1–28%	>28.1%
30–39	<19%	19–24%	24.1–29%	>29.1%
40–49	<20%	20–25%	25.1–30%	>30.1%
50–59	<21%	21–26%	26.1–31%	>31.1%
60–69	<22%	22–27%	27.1–32%	>32.1%
70–100	<23%	23–28%	28.1–33%	>33.1%

Athletes will usually have lower values. Depending on the sport performed, the training intensity and the physical constitution, the results may still be below the stated standard values. Please note, however, that there may be a risk to health in the case of extremely low values.

Body water content

The body water content in % is normally within the following ranges:

Male

Age	Poor	Good	Very good
10–100	<50%	50–65%	>65%

Female

Age	Poor	Good	Very good
10–100	<45%	45–60%	>60%

Body fat contains relatively little water; therefore, body water content may be below the standard values in persons with a high body fat percentage. On the other hand, some athletes may exceed the standard values due to having low levels of fat and a high muscle percentage.

The body water calculation performed using this scale is not suitable for drawing medical conclusions, such as in the case of age-related water retention. Consult your doctor where necessary. A high body water content is generally desirable.

Muscle percentage

Muscle percentage is normally within the following ranges:

Male

Age	Low	Normal	High
10–14	<44%	44–57%	>57%
15–19	<43%	43–56%	>56%
20–29	<42%	42–54%	>54%
30–39	<41%	41–52%	>52%
40–49	<40%	40–50%	>50%
50–59	<39%	39–48%	>48%
60–69	<38%	38–47%	>47%
70–100	<37%	37–46%	>46%

Female

Age	Low	Normal	High
10–14	<36%	36–43%	>43%
15–19	<35%	35–41%	>41%
20–29	<34%	34–39%	>39%
30–39	<33%	33–38%	>38%
40–49	<31%	31–36%	>36%
50–59	<29%	29–34%	>34%
60–69	<28%	28–33%	>33%
70–100	<27%	27–32%	>32%

Bone mass

Like the rest of our body, our bones are subject to natural growth, shrinking and aging processes. Bone mass increases rapidly during childhood and reaches its peak between the ages of 30 and 40. As we age, our bone mass begins to decrease. You can combat this reduction to an extent with the help of a healthy diet (particularly calcium and vitamin D) and regular physical exercise. The stability of the skeletal system may be increased through muscle buildup. Note that this scale does not measure calcium content of bones; instead, it calculates the weight of all the components that make up the bones (organic matter, inorganic matter and water). It is very difficult to influence bone mass, although it does fluctuate within the scope of influencing factors (weight, height, age, gender). There are no recognized guidelines or recommendations.

IMPORTANT:

Do not confuse bone mass with bone density.

Bone density can only be determined by a medical examination, so you cannot draw conclusions on changes to bones and bone hardness (e.g. osteoporosis) using this scale.

BMR

The basal metabolic rate (BMR) is the amount of energy required by the body at complete rest in order to maintain its basic functions. This value primarily depends on weight, height and age. It is displayed in kcal/day and calculated using the scientifically recognized Harris-Benedict equation. This is the amount of energy that is required by your body under all circumstances and must be resupplied to the body in the form of food. If your energy intake is below this level long term, it can affect your health.

AMR

The active metabolic rate (AMR) is the amount of energy that an active body consumes each day. An individual's energy requirement increases when the level of physical activity increases; the scale calculates this by means of the activity level (1– 5) that has been entered.

To maintain the current weight, the energy that the body uses must be replaced in the form of food and drink. If less energy is taken in than is used over a long period of time, the body takes the difference from its fat stores, and weight loss results. However, if the amount of energy taken in exceeds the calculated active metabolic rate (AMR) for a longer period, the body cannot burn off the excess energy. The excess is stored in the body as fat, leading to weight gain.

Context of results over time

NOTE: Only long-term trends are important. Brief deviations in weight within a few days are normally caused by loss of fluids.

Interpretation of these results is based on changes in total body weight, percentage of body fat, body water and muscle content, as well as the length of time over which these changes occur. Rapid changes within the

scope of a few days are considered separately from medium-term changes (in the scope of weeks) and long-term changes (months). As a basic rule, short-term changes in weight are almost entirely changes in water content, whereas medium-term and long-term changes may relate to fat percentage and muscle percentage.

- If you experience short-term weight loss but your body fat percentage increases or stays the same, you have only lost water, such as after a training session, visit to the sauna, or a diet aimed only at fast weight loss.
- On the other hand, if your weight increases in the medium term and your body fat percentage drops or stays the same, you may have built up valuable muscle mass.
- If your weight and body fat percentage both fall at the same time, then your diet is working – you are losing fat mass.
- Ideally, you should support your diet with physical activity, fitness or strength training. This enables you to increase your muscle percentage in the medium term.
- Body fat, body water and muscle percentages must not be added (muscle tissue also contains components made of body water).

8. Other Functions

Replacing batteries

Your scale is equipped with a low battery indicator. If the batteries are weak, “Lo” will appear in the LCD Display and the scale will automatically shut off. In this case, replace the batteries with three new “AAA” size batteries.

9. Care and Maintenance

Clean the scale periodically using a damp cloth and, if necessary, a small amount of mild detergent.

IMPORTANT:

- Do not use aggressive solvents or cleaning agents!
- Never immerse the unit in water!
- Do not wash the unit in a dishwasher!

Disposal:

Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available.

- If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.
- The device is not suitable for use in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
- The device requires no calibration.
- The device contains no user serviceable parts.
- No modification of this equipment is allowed.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used.
- Manufacturer will provide circuit diagrams, component part lists, descriptions, calibration instructions to assist to SERVICE PERSONNEL in parts repair.

10. Troubleshooting Guide

If the scale encounters an error during measurement, the following is displayed:

Display	Cause	Solution
Only the weight is displayed – no diagnostic values are shown	The person is not standing barefoot on the scale.	Repeat the measurement barefoot, or with the soles of the feet lightly dampened if necessary.
Err 1	The maximum load-bearing capacity of 330 lb (149.7 kg/23 st:8.7 lb) was exceeded.	The load must not exceed 330 lb (149.7 kg/23 st:8.7 lb)
	The fat percentage is outside the measurable range (less than 5 % or greater than 50 %).	Repeat the measurement barefoot or, if necessary, moisten the soles of your feet slightly.

Display	Cause	Solution
Lo	The scale batteries are almost flat.	Replace the batteries of scale.
Err 2	Scale zero setting is incorrect.	Wait until the scale switches itself off. Now correctly activate scale, wait for "0.0 lb" to appear and repeat measurement.
8888	Self test.	Place scale on an even, hard surface and wait for "0.0 lb" to appear.

11. Specifications

Batteries	3 x AAA 1.5V batteries
Weight Limits	11 lb to 330 lb (5 kg to 149.7 kg/11 lb to 23 st:8.7 lb)
Working environment	5 °C to 40 °C (41 °F to 104 °F), <= 90% RH, 80 kPa to 106 kPa
Storage environment	-20 °C to + 60 °C (-4 °F to 140 °F), <= 93% RH, 50 kPa to 106 kPa
Protection against ingress of water	IP21

CLASSIFICATION

1. Internally powered equipment;
2. Type BF applied part;
3. IP21;
4. Not intended to be sterilization or disinfection;
5. Not AP / APG equipment;
6. Mode of operation: continuous;

12. FCC Information

IMPORTANT INFORMATION REQUIRED BY THE FCC

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

13. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer glass body analysis scale, Model BG 40, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use

and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer glass body analysis scale, Model BG 40, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www.beurer.com

Manufactured for:
Beurer North America LP
900 N Federal Highway, Suite 300
Hallandale Beach, FL 33009

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