RAKIIIE

WRIST SUPPORT BRACE

User Manual

Meet Rakiie

We're always at your service for your satisfaction! We're all about safe sports and active living. Rakiie braces are here to prevent injuries and enhance your performance. Enjoy your activities more confidently with Rakiie! Beyond products, we're committed to your joy and well-being.

Contact Us Anytime, Anywhere

Visit www.rakiie.com for 24/7 customer service, or email us at support@rakiie.com. This product comes with a 12-month warranty. If there's anything missing, not fitting, or not meeting your expectations, contact us for an exchange or refund.

24/7 Live Chat: www.rakiie.com

E-mail: support@rakiie.com

Guarantee: 12-month warranty


Full Mesh Design:

Full Mesh Design

Breathable for all-day comfort and no heat buildup.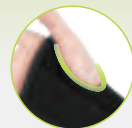


Edge-Free Thumb Hole

Precisely cut for a comfortable, non-restrictive fit.

High-Strength Velcro

Withstands 3000+ uses without losing grip.

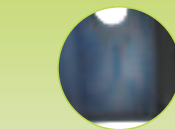Ice-Sense Material

Lightweight, skin-friendly, and cooling for a refreshing feel.

Three Aluminum Support Plates

Stabilizes both palm and back for strong wrist support.

Adjustable Compression Strap

Customizable for optimal wrist support.

Wearing Instructions:

Left Hand Wearing Instructions:

Right Hand Wearing Instructions:

Note: Distinguish the product by the left and right hand icons to avoid incorrect wearing that may affect the product's functionality.

Three Simple Steps to Wear:

Please read these instructions carefully before use.

- 1 Wear the Wrist Support Brace**
Insert hand, placing the thumb through the thumb hole.
- 2 Secure the Elastic Strap**
Adjust the wrist support brace to the desired position, wrap the elastic strap around the wrist, and attach it in place.
- 3 Secure the Velcro**
Attach the top and bottom Velcro to ensure a firm and comfortable fit.

Note:

- For prolonged use, please re-wear periodically to prevent skin issues or circulation problems caused by prolonged pressure.
- Check the directions before use and follow the wearing instructions correctly.
- This product is not intended to be used as a medical device.

Washing Instructions:

- Hand wash only, avoid using bleach, and use a neutral detergent.
- Do not wash with light-colored clothes, as the color may fade or transfer to other fabrics.
- Wash separately from other clothes to prevent wear or fasteners opening.
- Avoid using a dryer or iron, hang to dry in a cool place.

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CET Product Service Ltd.