

Safety Lancet

Cat.No. 640610-640615



Description

The safety lancet for single use is mainly used for clinical peripheral blood sampling.

Application

The sterile safety lancets are mainly used to collect blood samples for hospital clinic tests. They can lighten the sufferers' keen feelings, and have no cross-contamination. They are the first-rank products for hospital clinic tests. It can be used directly without a lancing device, which is safer and convenient.

Feature

Good blood flow: Two springs and a trip bevel tip, sharpened cannula achieve a safe and easy method for blood sampling.

Safely sealed needle tip before and after use --- prevents sharp injuries.

Self-destructing mechanism preventing reuse of the device --- avoid cross-infection.

Easy to handle, intuitive activation in only two steps with no pre-loading --- no need for a lancing device.


Sterilized by gamma irradiation.

Safety Lancet

Cat.No. 640610-640615

Structure

Safety lancet is composed of Needle core, Trigger, Handle, Base, Twister, Spring and Needle.

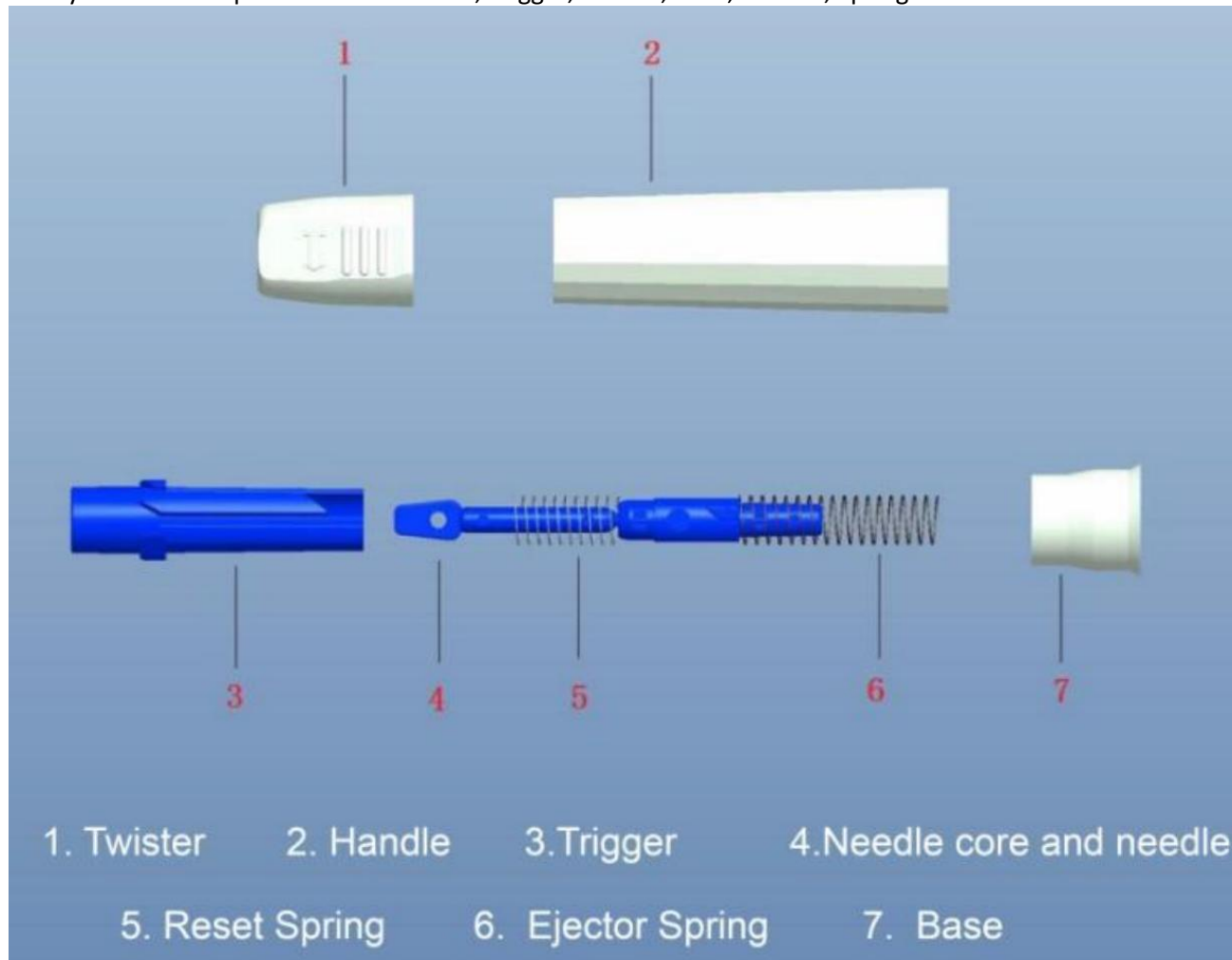

Raw Material Sheet

Serial No.	Components of Products	Material
1	Twister	Acrylonitrile Butadiene Styrene
2	Handle	Acrylonitrile Butadiene Styrene
3	Trigger	Polyformaldehyde
4	Needle core	Polyethylene
	Needle	SUS304 stainless steal
5	Reset Spring	SUS304 stainless steal
6	Ejector Spring	SUS304 stainless steal
7	Base	Acrylonitrile Butadiene Styrene

Safety Lancet

Cat.No. 640610-640615

Specification

Cat No.	Description	Qty/Case (pcs)
640610	Safety Lancet, 30G	2000
640611	Safety Lancet, 28G	2000
640612	Safety Lancet, 26G	2000
640613	Safety Lancet, 23G	2000
640614	Safety Lancet, 21G	2000
640615	Safety Lancet, Blade	2000

User Guide

- (1) Open the package
- (2) Twist and remove the needle cap
- (3) Puncture for a blood sample.
- (4) After use, discard the safety lancets in a sharps container.

Cautions

The product is sterile and for single use.

It should not be used when the cap falls down from the handle.

The used products should be burned or disfigured in line with regulations on the handling of medical waste.

Precautions

- 1) The product is disposable, sterile and without pyrogen. It should not be used when the cap is separated from the container.
- 2) The product is sterilized with Co-60 γ -ray sterilization. Its shelf life is 5 years, so use it within the period.
- 3) The product should not be stored in places with high temperature, humidity and direct sun;
- 4) Before use, for safety purpose, read in detail the instructions for use of the product to know about its usage and functions.

Storage

Store in a dry, ventilated, clean environment with no more than 5 layers.

Shelf Life

5 years