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Pairing your Accu-Chek® Instant meter with mySugr

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Download mySugr from
App Store or Google Play

mySugr - Diabetes Tracker Log 
Master Your Blood Sugar Levels!
[mySugr GmbH](#)
#43 in Medical
4.2 / 5.7k Ratings
Free - Offers In-App Purchases

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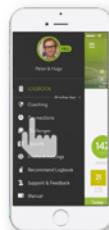
Enable Bluetooth in your
smartphone settings

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
Open mySugr and go to
“Connections” in the side
menu (Android) or at the
bottom of front page (iPhone)

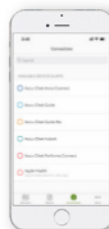4 選擇 “Accu-Chek Instant” Select “Accu-Chek Instant” from the list

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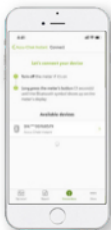
With the meter off, press and hold the
▼ button until the Bluetooth symbol
appears. The pairing symbol and
wireless symbol both appear and flash

Select the meter displayed on mySugr

7 按 “Next: enter PIN code”
Tap “Next: enter PIN code”

8 請輸入逸智血糖機背面的
6位PIN碼
Enter the 6-digit pin number
located on the back of your meter

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References:

1. Hompesch M, Kalcher K, Debono F, Morrow L. Significant Improvement of Blood Glucose Control in a High Risk Population of Type 1 Diabetes Using a Mobile Health App – A Retrospective Observational Study (https://www.mysugr.com/static/assets/downloads/science_and_research/attd-2017-poster.pdf).
2. The mySugr Pro is available in Hong Kong and free for all Accu-Chek Guide and Instant blood glucose meter users
3. Latest data (2021) from mySugr (www.mysugr.com/en/diabetes-app/).
4. Kahn R, Fonseca V. Translating the A1C assay. Diabetes Care, 2008;3(8):1704-1707

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©2021 Roche Diabetes Care