HALO
bassinest®

HOSPITAL SERIES

**INSTRUCTION MANUAL
MANUEL D'INSTRUCTION**

Model/Modèle : 3957

Please read manual carefully before use.
For additional information, please scan or visit
<https://www.halosleep.com/pages/bassinest-hospital>

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ANY PROBLEMS/QUESTIONS? Contact HALO Customer Service at 1-888-999-HALO or hospitalcustomerservice@halodream.com.
For technical support or complaints, please contact hospitalsupport@halodream.com.

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Rev. 2025.06.06

WARNING/CAUTION/NOTE DEFINITION

The words WARNING, CAUTION, and NOTE carry special meanings and should be carefully reviewed.

WARNING

Alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

CAUTION

Alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the product or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

Note: Provides special information to make maintenance easier or important instructions clearer.

SUMMARY OF SAFETY PRECAUTIONS

Carefully read and strictly follow the warnings and cautions listed on this page. Service only by qualified personnel.

WARNING

- Do not add extra padding or bedding to the sleep area to avoid the risk of infant/newborn injury.
- Only use the mattress provided by the manufacturer. Never add a pillow, comforter, or another mattress for padding.
- Do not exceed the safe working load of the BassiNest Tub, Side Storage Caddies, Removable Basin Storage Bin, or Work Surface.
- Always disinfect the mattress between infant/newborns to avoid the risk of cross-contamination and infection.
- Do not immerse the mattress in cleaning or disinfectant solutions.
- Do not allow liquids to pool on the mattress.
- Always completely dry the mattress, basket, and bassinet before storing, adding linens, or placing a infant/newborn in the basket. Drying the product helps to avoid product damage.

CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this manual.
- Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.
- Do not use this product to perform medical procedures.
- Do not apply the caster locks to stop the bassinet while in transport or attempt to transport with caster locks applied.
- Do not operate the product on soft or unstable surfaces, such as sand, grass, or gravel, to avoid instability or difficult mobility.
- Only apply the name card to the outside of the basket to avoid risk of access by infant.
- Do not clean, service, or perform maintenance while the product is in use.
- Do not use cleaners that are ether-based as they may damage the product.
- Do not over expose the product to higher concentration disinfectant solutions as these may degrade the product.

INTRODUCTION

Read this manual before operating or maintaining this product. Set methods and procedures to educate and train your staff on the safe operation or maintenance of this product.

NOTES

- This manual is a permanent part of the product and should remain with the product.
- Halo continually seeks advancements in product design and quality. This manual contains the most current product information available at the time of printing. There may be minor discrepancies between your product and this manual. If you have any questions, contact HALO for information.

PRODUCT DESCRIPTION

The HALO Bassinest® Hospital Series swivel sleeper includes a removable basin storage bin and work surface, two side-storage caddies, locking caster wheels, a clear BassiNest tub, and a mattress.

INTENDED USE


The HALO Hospital Bassinest is designed to be a sleeping surface for infant/newborn patients. The device raises and lowers to allow easier access to the infant/newborn. The bassinet tub allows for easy access to the infant/newborn patient by rotating and swiveling for easy positioning and by its ergonomically-friendly retractable front wall. The HALO Hospital Bassinest was also designed for in-hospital transport.

This product is **NOT INTENDED** for patients that can crawl or reposition themselves independently.

CONTRAINDICATIONS None known.

SETUP

1. Remove all shipping and packaging materials from the product prior to use.

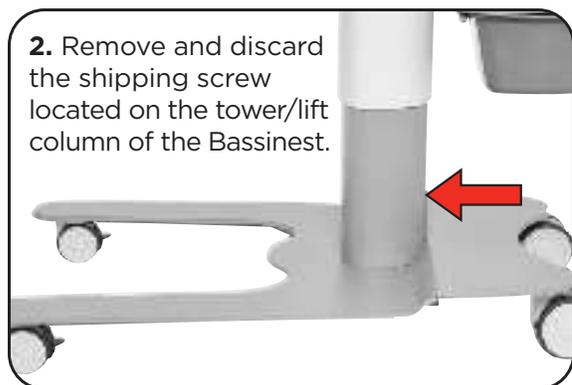
3. Remove wrap from the work surface and removable basin storage bin.
4. Unpack and Install two side storage caddies on each end of clear bassinet tub.
5. Remove mattress from packaging. Clean and disinfect the mattress.
6. Clean and disinfect the entire unit.
7. Place mattress in a tight fitting sheet or pillowcase.
8. Run through operation steps to ensure unit is in working order upon delivery.

SERIAL NUMBER LOCATION

You can find the serial number on the vertical pole below the BassiNest Tub (formatted as “XX-XXXX”).

INTRODUCTION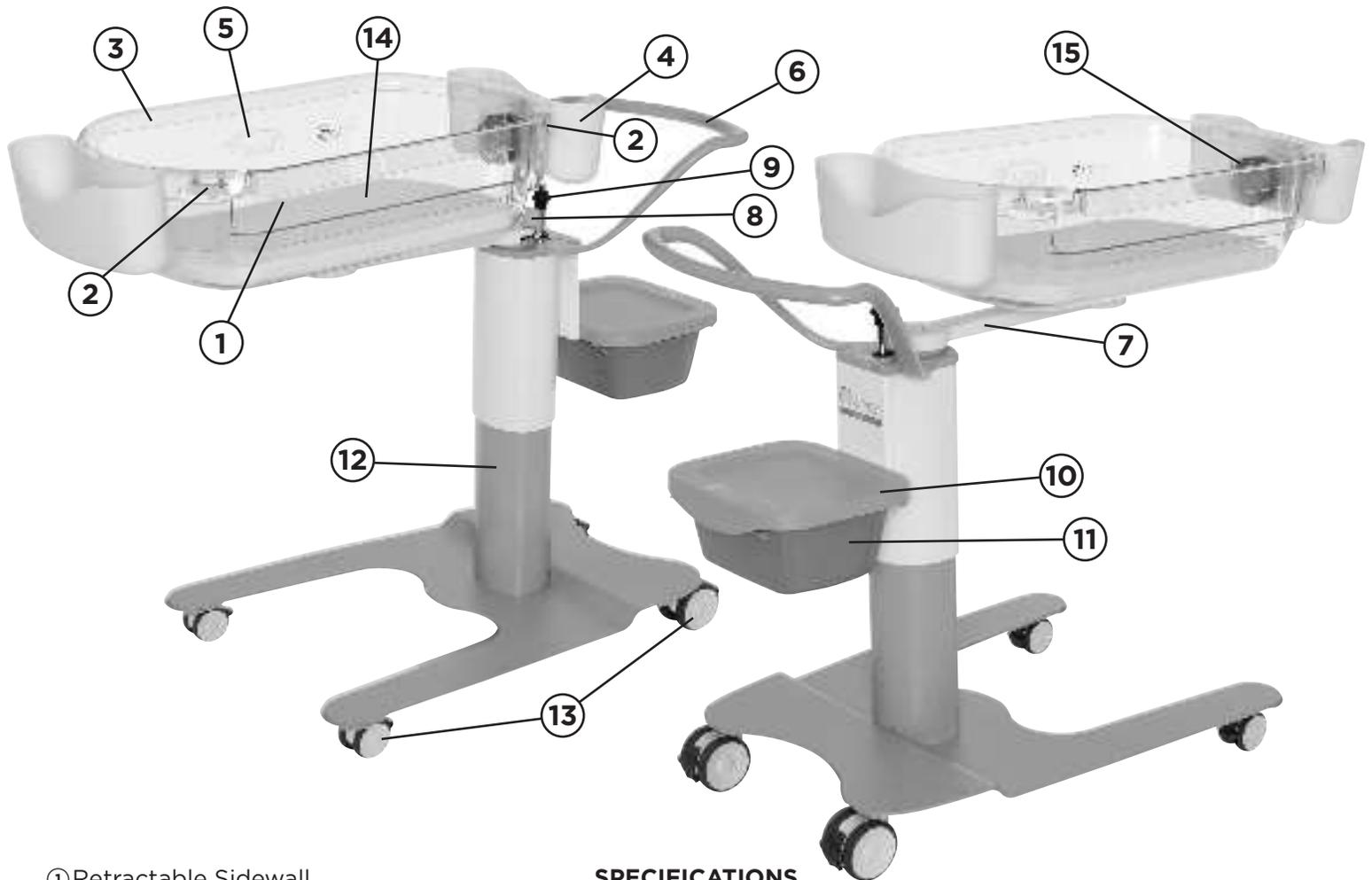

- ① Retractable Sidewall
- ② Sidewall Locks
- ③ BassiNest Tub
- ④ Side Storage Caddy
- ⑤ Information Holder
- ⑥ Mobility Handle
- ⑦ Swivel Arm
- ⑧ Transport Lock
- ⑨ Height Adjustment Knob
- ⑩ Work Surface
- ⑪ Removable Basin Storage Bin
- ⑫ Tower/Lift Column
- ⑬ Locking Caster Wheels
- ⑭ Mattress
- ⑮ Removable Pivot Knobs

ACCESSORIES

Accessories and replacement parts may be available for this product. Call HALO Customer Service: 888-999-HALO (4256).

SPECIFICATIONS


Model Number	3957
Overall Length	49" (124.5 cm)
Overall Width	27.1" (68.8 cm)
BassiNest Tub Dimensions	
Length	34.25" (87 cm)
Width	22" (59.9 cm)
Depth	8.9" (20.1 cm)
Overall Mattress	
Length	25.8" (65.5 cm)
Width	14.2" (36 cm)
Depth	1" (2.5 cm)
Weight Capacity	
BassiNest Tub	20 lb (9.0 kg)
Shelf	25 lb (11.3 kg)
Side Storage Caddy	3.5 lb (1.6 kg) each
Height Range	
High	30.88" (78.4 cm)
Low	20.25" (51.4 cm)
Caster Diameter	4" (10.2 cm) x 2 Rear 3" (7.6 cm) x 2 Front
Bed Clearance Required	4" (10.2 cm)

HALO reserves the right to change specifications without notice.

SYMBOLS

	Do Not Climb/Do Not Exceed 11 kg		Caution
	Proper Positioning for Transport		Serial Number
	Transportation Lock, Rotation Arm Should Be in Locked Position Before Transport		Lot Number
	While squeezing knob, pull up/push down on it to adjust height		Manufacture Date
			Part Number

OPERATION

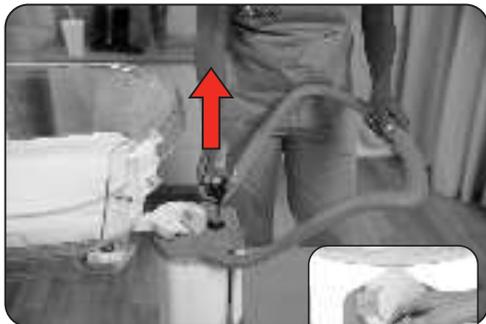1. To raise the BassiNest, pull upwards on the black height adjustment knob to unlock it and then continue to pull upwards until the desired height is reached. Release the knob to automatically lock the height setting. Roll the front wheels of the BassiNest under the mother's bed.

2. To lower the BassiNest, place your thumb on the top of the metal pole, and use your index and middle fingers to gently pull up on the black height adjustment knob to unlock it. While squeezing the knob up in the unlocked position, push down on the pole to lower to the desired height. Never apply more than 30 lb (13,6 kg) of downwards force.

3. Engaging and disengaging the caster locks

- A.** To engage the caster lock, push down on the pedal with your foot.
B. To disengage the caster lock, lift up on the pedal with your foot.

4. To Lock the retractable sidewall in the raised position, slide the two yellow locks such that they fully engage under the top edges of the retractable side wall.

5. To unlock the retractable sidewall, slide the two yellow locks away from the retractable side wall and under the stationary sidewalls.

6. To lower the retractable side wall, first make sure the yellow sidewall locks are in the unlocked position. To lower the wall, put gentle downward pressure from your hand or forearm in the center of the wall. When you release the pressure, the wall will automatically return to the raised position.

OPERATION

7. To remove the retractable sidewall for cleaning, locate the two removable pivot knobs on either end of the tub. Rotate the knobs counter-clockwise to loosen. Pull the knobs off the BassiNest and remove the retractable sidewall. Reverse these steps to reassemble. Only use the product with the sidewall assembled.

8. Two storage caddies can be placed on the ends of the bassinet tub. To remove, simply lift up. Use the storage caddies to store items mother will need easy access to such as diapers, wipes, feeding supplies, etc.

9. There is an additional covered storage located on the tower of the bassinet. Simply slide a standard basin into the track to attach to the bassinet.

TRANSPORTING THE BASSINET

⚠ CAUTION

- Always engage the transport lock, rotate the swivel arm to the forward position, and orient the BassiNest tub parallel to the base before transporting the unit.
- Do not engage the caster locks to stop the bassinet while in transport.
- Do not operate the product on soft or unstable surfaces, such as sand, grass, or gravel, to avoid instability or difficult mobility.
- Do not attempt to transport with caster locks engaged.
- Once positioned, engage the caster locks before unlocking and using the rotational arm.

REPROCESSING

⚠ WARNING

- Always disinfect the mattress between patients to minimize the risk of cross-contamination and infection.
- Do not immerse the mattress in cleaning or disinfectant solutions.
- Do not allow liquids to pool on the mattress.
- Always completely dry the unit before storing, adding linens, or placing a patient in it.

⚠ CAUTION

- Do not clean, service, or perform maintenance while the product is in use.
- Do not use cleaners that are ether-based as they may damage the product.

CLEANING

Tools Required:

- Mild anti-bacterial soap
 - Soft, lint free cloth (2 or more)
1. Apply the caster locks.
 2. Prepare a mild soap and warm water solution.
 3. Wipe the inside and outside of the basket and mattress with a soft, lint-free cloth moistened with soap and water solution.
 4. Wipe all the bassinet surfaces, with a soft, lint-free cloth moistened with soap and water solution.
 5. Wipe off excess moisture from all surfaces with a dry, soft, lint-free cloth.
 6. Allow the surfaces of the basket and bassinet to dry thoroughly.

⚠️ WARNING

- Always disinfect the mattress between infant / newborns to avoid the risk of cross-contamination and infection.
- Do not immerse the mattress in cleaning or disinfectant solutions.
- Do not allow liquids to pool on the mattress.
- Always completely dry the mattress, basket, and bassinet before storing, adding linens, or placing a infant/newborn in the basket. Drying the product helps to prevent the performance of the product from being impaired.

⚠️ CAUTION

- Do not over expose the product to higher concentration disinfectant solutions as these may degrade the product.
- Do not use cleaners that are ether-based as they may damage the product.

DISINFECTING

Recommended disinfectants:

- Quaternary cleaners (ammonium chloride)
- Phenolic cleaners
- Chlorinated bleach solution (1 part bleach to 100 parts water)
- 70% isopropyl alcohol
- Diluted hydrogen peroxide cleaners <10%

1. Apply recommended disinfectant solution by spray or pre-soaked wipes. Do not soak the product.

Note: Make sure that you follow the disinfectant's instructions for appropriate contact time and rinsing requirements.

2. Wipe all the bassinet surfaces, with a dry soft, lint free cloth, to remove any excess liquid or disinfectant.

3. Allow the surfaces to dry thoroughly.

PREVENTIVE MAINTENANCE INSPECTION

Remove product from service before you perform the preventive maintenance inspection, at minimum once per year and check all items listed. You may need to perform checks more often based on your level of product usage. Service only by qualified personnel.

Inspect the following items:

- All fasteners are secure
- All mechanisms operate freely
- All four casters are free of debris and swivel properly
- Swivel arm is securely fastened to lift column
- Height adjustment and mobility handle is securely fastened to lift column
- Bassinet tub is not sloped. Measured vertically from the bottom of tub, the distances of any two points from the floor should vary by no more than 1.2 in. (3 cm)
- Swivel arm pivots freely when unlocked
- Swivel arm locks in the forward position for transport
- Transport lock unlocks properly
- Work surface is level and parallel to the ground
- BassiNest tub is free of cracks
- Retractable sidewall automatically returns to full upward position without pressure
- Retractable sidewall lowers when unlocked and pressure is applied
- Retractable sidewall is free of cracks or other visible damage
- Retractable sidewall is intact and secure, with pivot knobs assembled
- BassiNest raises and lowers
- Mattress has no rips, tear, punctures, or other damage
- Labels are properly adhered and legible
- BassiNest height locks in place when downward force is applied to handle or sleep area
- BassiNest tub rotates freely 360 degrees

Serial number: _____

Completed by: _____

Date: _____

WARRANTY

Halo Dream, Inc. warrants to the original purchaser the HALO Hospital Bassinet, Model 3957, to be free from defects in material and workmanship for a period of two (2) years after date of delivery. HALO's obligation under this warranty is expressly limited to supplying replacement parts for, and/or replacing, at its option, any product which is, at the sole discretion of HALO, found to be defective. If requested by HALO, products or parts for which a warranty claim is made shall be returned prepaid to the factory. Any improper use or any alteration or repair by others in such manner that, in HALO's judgment, affects the product materially and adversely shall void this warranty. Any repair of HALO products using parts not provided or authorized by HALO shall void this warranty. No employee or representative of HALO is authorized to change this warranty in any way.

Warranty exclusion and damage limitations

The express warranty set forth herein is the only warranty applicable to the product. Any and all other warranties, whether express or implied, including any implied warranty of merchantability or fitness for a particular purpose are expressly excluded by HALO. In no event shall HALO be liable for incidental or consequential damages.

To obtain parts and service

Simply call HALO Customer Service a 1-888-999-HALO (4256).

Return authorization

Product cannot be returned without prior approval from the HALO Customer Service Department. HALO reserves the right to charge shipping and restocking fees on returned product. Special, modified, or discontinued products are not subject to return.

Damaged product

ICC Regulations require that claims for damaged product must be made within fifteen (15) days of receipt of the product. Do not accept damaged shipments unless such damage is noted on the delivery receipt at the time of receipt. Upon prompt notification, HALO will file a freight claim with the appropriate carrier for damages incurred. Claims will be limited in amount to the actual replacement cost. In the event that this information is not received by HALO within the fifteen (15) day period following the delivery of the product, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full within thirty (30) days of receipt. Claims for any incomplete shipments must be made within thirty (30) days of invoice.

Expected device life

The HALO BassiNest swivel sleeper has a three year minimum expected service life under normal use conditions and with appropriate periodic maintenance.

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FOR QUESTIONS OR OTHER SERVICE ISSUES: Please contact HALO. Our friendly, specially trained support team will be happy to assist you.

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© 2024 HALO Dream, Inc. All rights reserved.

"HALO," the HALO logo, "BassiNest" and HALO BassiNest® logo design are trademarks or registered trademarks of Halo Dream, Inc. in the U.S. and other countries. HALO products comply with government regulations applicable in the country of sale. Patent: www.halosleep.com/patents. Made in China.

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© Halo Dream Inc, 2024. Tous droits réservés.