

DrFinger



REST HAND SPLINT

USER MANUAL

How to use this rest hand splint?



Step 1

Insert 4 fingers into the corresponding finger holes.



Step 2

Fix the thumb with the thumb strap.



Step 3

Fix 4 fingers with palm strap.



Step 4

Fix the wrist with the middle wrist strap.



Step 5

Bend the lowest wrist support forcefully to make it more suitable for your arm.



Step 6

Fix the arm with the bottom wrist strap.

How to use this rest hand splint?



Step 7

Fix one end of the separate palm strap on the palm obliquely 45 degrees as shown.



Step 8

Pass between the thumb and the palm, and wrap the palm strap around the palm.



Step 9

Fix another end of the separate palm strap.



Step 10

Bend the palm rest to a comfortable angle.



Step 11

Bend the thumb splint to a comfortable angle.



Step 12

Bend the splint at the wrist to a comfortable angle. Finish.

Fits: Men or Women

Material: Foam, Fabric, Aluminum, Nylon

Care: Hand wash and air dry

Color: Black, Blue

This resting hand splint is designed by DrFinger health technology team and orthopedics doctors through continuous research and testing to conform to the ergonomic shape of the resting splint. It is very suitable for patients with stroke fingers, ruptured metacarpal bones, tendinitis, joint sprain treatment, postoperative care, carpal tunnel syndrome, other wrist injuries, and various hand dysfunctions.

If you feel that our Hand Splint are working, we would appreciate your sharing your experience on amazon so other customers can make more informed choices and we will know our efforts pay off.

If you think our Rest Hand Splint can be improved in some way, you also can leave a review on amazon or email us. Your valuable suggestions will be highly appreciated and we will improve our products and services upon your suggestions.

Maybe your suggestions will make a better world.
Wish you a quick recovery in your hand.

Thank you

Warranty

Respected user,

We solemnly promise with the reputation of the company. If you contact us by email, we promise that the product guarantee period will be extended to 12 months for you. Just tell us your order number, and we will register this order number in our 12-month warranty system.

As long as you let us register the order number for you, if you find that any quality problem causes the product to not be used normally within 12 months from the date of purchase, we promise that we will send you a new product for free. If you think this product is not effective for your symptoms, we will refund you the full amount, even though Amazon's 30-day return period has passed.

Email: DrFinger-service@outlook.com

Promiser: *Doris Pan*

CEO of DrFinger Co., Ltd.



Blue



Black



High-density aluminum material inside
the splint, can adjust the shape.

DrFinger