



makseNSE iLogger Case User Manual

[Home](#) » [makseNSE](#) » makseNSE iLogger Case User Manual 

Contents

- 1 makseNSE iLogger Case User Manual
- 2 Patent Information
- 3 Warranty
- 4 Copyright
- 5 FCC Regulatory info:
- 6 RF Exposure Information iLogger Case 4A
- 7 iLogger Case 2
- 8 ISED Canada Regulatory info:
- 9 RF Exposure Information iLogger Case 4A
- 10 iLogger Case 2
- 11 1. Intended Use
- 12 System Components
- 13 2.1. Absorbent pads
- 14 2. Instructions
- 15 3.1. Placing or replacing the batteries
- 16 3.2 Handling Procedures
- 17 5. Troubleshooting
- 18 6. Safety Precautions
- 19 Maintenance
- 20 Environmental Information
- 21 5.Cleaning and disinfection
- 22 8.Compliance with transport standards UN3373
- 23 10. Manufacturer's Declaration
- 24 Read More About This Manual & Download PDF:
- 25 Documents / Resources
 - 25.1 References

makseNSE iLogger Case User Manual



Patent Information

iSens – Electronica Lda. owns the intellectual property rights over the Indexor technology described in this document.

Warranty

All products supplied by iSens – Electronica Lda. are covered by our 2-year-warranty.

Copyright

© This manual shall not be copied (partially or totally), reproduced, in any form, or translated into another language without prior permission from iSens – Electronica Ltd. The manufacturer reserves the right to change the information in this manual without notice. Images on this manual are illustrative only and may not be an accurate representation of the equipment itself.

FCC Regulatory info:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- this device may not cause harmful interference, and
- this device must accept any interference received, including interference that may cause undesired operation.

No changes shall be made to the device without the manufacturer's permission as this may void the user's authority to operate the device.

RF Exposure Information iLogger Case 4A

The iLogger Case 4A complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. The iLogger Case 4A complies with the safety requirements for RF exposure in accordance with FCC Part 2.1093 for portable use conditions.

iLogger Case 2

The iLogger Case 2 complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. The iLogger Case 2 complies with the safety requirements for RF exposure for mobile (>20 cm) use conditions in accordance with FCC rule part 2.1091. Contains FCC ID: 2AC7Z-ESP32WROOM32E and Contains FCC ID: R7T1101102

ISED Canada Regulatory info:

This device complies with ISED's licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- this device may not cause interference, and
- this device must accept any interference, including interference that may cause undesired operation of the device.

RF Exposure Information iLogger Case 4A

The iLogger Case 4A complies with ISED Canada RSS-102 radiation exposure limits set forth for an uncontrolled environment. The iLogger Case 4A complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 section 2.5.1 for portable (≤ 20 cm) use conditions

Le iLogger Case 4A est conforme aux limites d'exposition au rayonnement CNR-102 Canada ISDE établies pour un environnement non contrôlé. Le iLogger Case 4A est conforme aux exigences de sécurité pour l'exposition aux RF conformément à la norme CNR-102 édition 5 section 2.5.1 pour les conditions d'utilisation portables (≤ 20 cm).

iLogger Case 2

The iLogger Case 2 complies with ISED Canada RSS-102 radiation exposure limits set forth for an uncontrolled environment. The iLogger Case 2 complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 section 2.5.2 for mobile (>20 cm) use conditions.


Contains IC: 21098-ESPWROOM32E and Contains IC: 5136A-1101102

1. Intended Use


The iLogger automatically records data on the iRack's RFID to monitor data such as impacts, temperature and time in the case. This data is then accessible in an Indexor System Standard.

System Components

Isothermic Bag with velcro fasteners

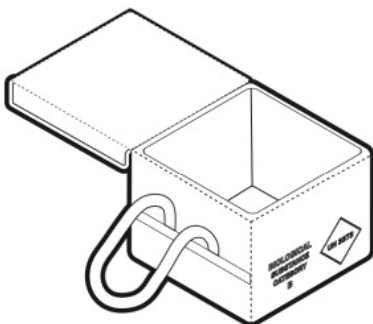

Isothermic EPP Box

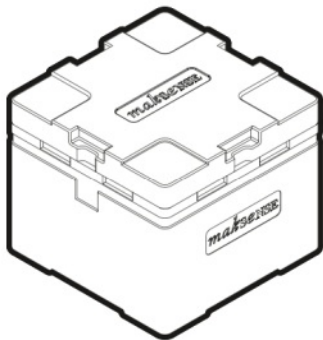

2.1. Absorbent pads

The iLogger is equipped with absorbent pads to contain any spillage that may occur. In the event of a spill, carefully remove the pads and dispose of them in accordance with local regulations and guidelines for potentially biohazardous substances.

2. Instructions



1. Batteries lid
2. Display
3. Status button

3.1. Placing or replacing the batteries

Replacing with the incorrect battery type may result in hazard to the equipment.

Please check the Technical Specifications table for the suitable battery types.

- With the open iLogger, turn the compartment lid anti-clockwise;
- Tilt the case so you can remove the batteries;
- Insert new batteries with the positive (+) pole up as indicated on the compartment
- Turn the lid clockwise to properly secure the batteries in

Do not dispose of batteries in fire. Please follow local environmental applicable regulations.

3.2 Handling Procedures

1. Place the iRacks correctly, in order to keep the samples from moving as much as
2. After placing the iRack, check if it has been correctly detected by pushing the status button to activate the display.
3. Place up to 5 Ice packs using the side slots and on top of the iRacks to decrease the internal temperature.

4. Close the EPP Box
5. Seal the external bag with the Velcro



5. Troubleshooting

This equipment was designed to be reliable. This equipment was designed to be reliable. In case of any issue, please check the product guide accessible at indexorsystems.com. For further assistance, please contact support@maksense.com

6. Safety Precautions

The iLogger Case was designed with the concern for safety. However, to ensure accident-free operations, avoid shaking the iLogger Case or placing it on an unstable surface.

Maintenance

The iLogger System does not require special maintenance.



Do not try to perform any maintenance operations on the iLogger System.



Do not disassemble, modify or repair the iLogger system.

Environmental Information

This symbol on the product indicates that it is covered by Directive 2002/96/EC (Waste Electrical and Electronic Equipment). The crossed-out dustbin indicates that waste electrical and electronic equipment must not be disposed of with unsorted

household waste and that it must be disposed of separately.

Disposal must be carried out in accordance with local environmental regulations on waste disposal. For information about recovery and recycling of this product, please contact the supplier of this equipment.



The correct disposal of the equipment helps avoid dangerous or detrimental effects to human health and the environment.

5.Cleaning and disinfection



Avoid spilling liquids near the display of the iLogger Case system.

Before starting the cleaning process, please remove the absorbent pads. The iLogger can be cleaned with a

soft cloth slightly moistened with ethyl alcohol (70%).

8.Compliance with transport standards UN3373



The procedures for transporting biological samples are in accordance with transport standard UN3373 – Cat. B – Packing instructions P650.

10. Manufacturer's Declaration

The full text of the EU Declaration of Conformity and other relevant documents are available at:

<https://support.maksense.com>

Model	iLogger Case 4A	iLogger Case 2
Dimensions	420 [L] x 420 [W] x 370 [H] mm	460 [L] x 325 [W] x 370 [H] mm
Max. number of Samples carried	240	120
Ice pack slots	5	2
Mechanical resistance	1100 kPa	
Thermal conductivity	0.023 W/mK	
Weight	3.4 Kg	3.2 Kg
Temperature Sensor (TSY S01)	-40 °C to 125 °C (+/-0.1 °C)	
Time interval for temperature recording*	5 minutes	
G-force impact sensor (S Q-ASE-060)	>60 G	
Power Source	3 to 4 x Type C non rechargeable dry cell alkaline batteries	
Outdoor use	the equipment is not intended to be constantly subjected to an outdoor environment	
Operating Temperature Range	-5 to 45C°	
Operating humidity	maximum relative humidity 80 % for temperatures up to 31 °C decreasing linearly to 50 % relative humidity at 40 °C Suitable for use in wet locations	
Operating pollution degree	PD2	
IP Rating	Not rated	
Maximum Altitude	3 000 Meters	
Regulatory Compliance:	WPC, RCM, MIC, TDRA, SASO, FCC, IC, CE, ROHS, IEC 61010-1, cULus	Please check https://maksense.com/maksense/support-page/


Model	iLogger Case 4A	iLogger Case 2
Dimensions	420 [L] x 420 [W] x 370 [H] mm	460 [L] x 325 [W] x 370 [H] mm
Max. number of Samples carried	240	120

Ice pack slots	5	2
Mechanical resistance	1100 kPa	
Thermal conductivity	0.023 W/mK	
Weight	3.4 Kg	3.2 Kg
Temperature Sensor (TSY S01)	-40 °C to 125 °C (+/-0.1 °C)	
Time interval for temperature recording*	5 minutes	
G-force impact sensor (S Q-ASE-060)	>60 G	
Power Source	3 to 4 x Type C non rechargeable dry cell alkaline batteries	
Outdoor use	the equipment is not intended to be constantly subjected to an outdoor environment	
Operating Temperature Range	-5 to 45C°	
Operating humidity	<p>maximum relative humidity 80 % for temperatures up to 31 °C decreasing linearly to 50 % relative humidity at 40 °C</p> <p>Suitable for use in wet locations</p>	
Operating pollution degree	PD2	
IP Rating	Not rated	
Maximum Altitude	3 000 Meters	

Regulatory Compliance:	WPC, RCM, MIC, TDRA, SASO, FCC, IC, CE, ROHS, IEC 61010-1, cULus	Please check https://maksense.com/maksense/support-page/
-------------------------------	--	---

Read More About This Manual & Download PDF:

Documents / Resources

	maksense iLogger Case [pdf] User Manual 2A35O-ILOGGERCASE2, iLogger Case 4A, iLogger Case 2, iLogger Case, iLogger, Case
--	---

References

- [🔗 Page Redirection](#)
- [🔗 Page Redirection](#)
- [🔗 Page Redirection](#)
- [🔗 maksense | Support Page](#)
- [🔗 maksense | Support Page](#)