




maxtec EyeMax2 Micro Instruction Manual

[Home](#) » [maxtec](#) » maxtec EyeMax2 Micro Instruction Manual 

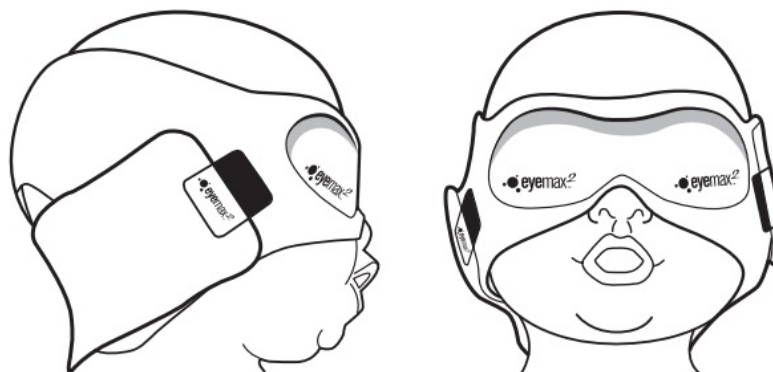

MICRO: OFC 20 – 25 CM (8 – 10 IN)

Contents

- [1 Indications for Use:](#)
- [2 Instructions for use:](#)
- [3 Precautions:](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)
- [5 Related Posts](#)

Indications for Use:

The EyeMax2 provides neonatal eye protection during ultraviolet (UV) phototherapy treatment of jaundice. The EyeMax2 conforms with 170:2002 Clause 5.2 (Scale Number 2-5) for spectral transmittance as appropriate for the indication for use.

Instructions for use:

1. Measure Occipital-Frontal Circumference (OFC) of the patient.
2. Select the appropriate EyeMax2 size based on the OFC. See chart for appropriate sizing.
3. Disconnect the Velcro.
4. Position the eye pads over the eyes.
5. Pass the lower strap under the small of the neck and attach the velcro.
6. Secure the Velcro of both straps and position the EyeMax2 as shown.
7. Make sure the strap completely covers the ears.
8. If the eye patch rides up away from the nose, tighten the lower strap.
9. If the eye patch rides down towards the nose and mouth, repeat the application process beginning with step 1.

Precautions:



CAUTION: Improper EyeMax2 sizing may lead to patient airway obstruction or inadequate eye protection. The Eyemax2 should only be used while the patient is under the continuous, direct supervision of healthcare professionals.



DO NOT tug on the EyeMax2 to reposition it. A premature infant's skin is very delicate and can easily rupture. Avoid obstructing the nose. Check regularly to ensure the EyeMax2 is not blocking the patient's airway. The EyeMax2 is not a bandage. Cover any existing skin abrasions with appropriate bandages before use.



DO NOT over tighten – causing excessive pressure on the eyes. Make sure the strap is covering the ear, not hooked over the ear, and that the ear is not folded over. Replace the EyeMax2 with a new one if it becomes over-stretched. Reuse of this device could cause cross-contamination or product malfunctions.

Catalog # R300P01 **Regular:** occipital-frontal circumference **33-38** cm in packs of 20.

Catalog # R300P02 **Preemie:** occipital-frontal circumference **26-32** cm in packs of 20.

Catalog # R300P03 **Micro:** occipital-frontal circumference **20-25** cm in packs of 20.



Single Patient Use Only
Non-Sterile, Latex-Free



The Eyemax2 is dually noted as a Class I Medical Device and Category 2 PPE device. EC Type-examination for Directive 89/686/EEC by Notified Body No. 0120:




SGS United Kingdom Limited
Unit 202B, Worle Parkway
Weston Super Mare BS22 6WA
United Kingdom



QNET BV: Kantstraat 19
NL-5076 NP Haaren | The Netherlands

Maxtec | 2305 South 1070 West
Salt Lake City UT 84119
801.266.5300 | www.maxtec.com
R111P16-003 Rev. R

Documents / Resources

	<p>maxtec EyeMax2 Micro [pdf] Instruction Manual maxtec, EyeMax2, Micro, OFC, 20 - 25 CM, 8 - 10 IN</p>
---	---

References

-  [Maxtec | The leader in oxygen analysis and delivery products](#)