

MEDTECH
Caring for life
**BP-18 Nebulizer
Kit**



MEDTECH BP-18 Nebulizer Kit User Manual

[Home](#) » [Medtech](#) » MEDTECH BP-18 Nebulizer Kit User Manual 

Contents

- 1 MEDTECH BP-18 Nebulizer Kit
- 2 Product Usage Instructions
- 3 Range of Electromedical Devices
- 4 Features
- 5 INSTRUCTION FOR USE
- 6 FAQs
- 7 Documents / Resources
 - 7.1 References

MEDTECH 
Caring for life

MEDTECH BP-18 Nebulizer Kit



Product Usage Instructions

Nebulizer Usage

1. Connect the nebulizer to a power source.
2. Fill the medication cup with the prescribed medication.
3. Attach the mouthpiece or mask to the nebulizer.
4. Turn on the nebulizer and inhale the medication mist as directed.

BP Monitor Usage

1. Secure the cuff around your upper arm at heart level.
2. Turn on the BP monitor and follow the device's instructions for measurement.
3. Record the blood pressure readings accurately.

Steam Inhaler Usage

1. Fill the steam inhaler with water as per the instructions.
2. Turn on the steam inhaler and inhale the steam for respiratory relief.

Range of Electromedical Devices

Specifications



NEBULIZER - HANDYNEB
CLASSIC



NEBULIZER - HANDYNEB
SMART



NEBULIZER - HANDYNEB
PRO



BP MONITOR - NOVACHECK
BP-18



BP MONITOR - NOVACHECK
BP-11



BP MONITOR - NOVACHECK
BP-11 BL



STEAM INHALER - HANDYVAP
VAP-01



PULSE OXIMETER - OXYGARD
OG-05



DIGITAL THERMOMETER - HANDY
TMP-02



HEATING PAD - HANDYPAD
HP-01



MASSAGER - GUN
GMV-4




MASSAGER - GUN
MPV-1

Nebulizer Kit



Insist on

TWO PARTS BOTTLE

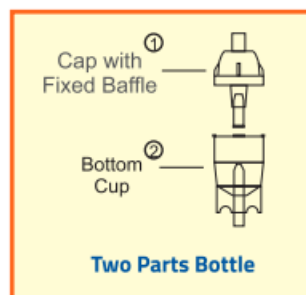

Because of fix baffle, it is Safe for nebulization. No fear of losing baffle

THREE PARTS BOTTLE



It is easy to lose Baffle as it is not fixed So if the raffle falls Nebulization cannot take place and the kit is nonfunctional

RESPIRATORY THERAPY PRODUCT

- Disposable for Single Patient Use

- Non-Intended for Reprocessing

Features

- Non-toxic PVC Tube and mask
- 2 Meter PVC Tube
- BPA-free, 2-Part Medicine Cup

CAUTION:

- Ensure that all connections are secured and air circulates freely through the tubing.
- Do not store at extreme temperatures and humidity. Store in a cool & dry place.
- Do not attach the outlet of the nebulizer directly to an endotracheal tube, tracheostomy tube, or other device that does not offer an alternate pathway for exhalation.

INSTRUCTION FOR USE

These directions are general guidelines intended for use by qualified medical personnel.

1. Place the prescribed solution into the bottom cup of the nebulizer bottle.
2. Attach the cap to the nebulizer cup with a clockwise rotation.
3. Attach the tubing between the gas source and the nebulizer.
4. Connect the nebulizer outlet on top of the nebulizer cap to the mask inlet.
5. Start gas flow. Check for aerosol mist; if necessary, tap the device until it begins to nebulize.
6. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secured. Shape the metal strap on the mask to fit the face.
7. During treatment, periodically tap the nebulizer to minimize residual volume.

Mfg. By

- MEDTECH LIFE PRIVATE LIMITED Regd. Address: B6, Byculla Service Industries, O.K. Marg, Sussex Road, Byculla (East), Mumbai 400027, Maharashtra, INDIA.
- Customer Care No. +917208088720
- E-Mail: support@medtechlife.com
- Website: www.medtechlife.com
- **P02627**



AR01416-00-00

Prepared by  RUPALI	Verified by  LEKHRAJ	Approved by  BHOMRAJ GIRI
---	--	--

REVISION TABLE:

Item Code: P02627		ARTWORK NO.: AR01416-00-00			
Sr. No.	Primary Revision	Secondary Revision	Date	Location	Details
1.	00	00	29-09-2023	-	New Design

FAQs

Q: How do I clean the nebulizer?

A: Disassemble the nebulizer parts after each use and wash them with warm, soapy water. Allow them to air dry completely before reassembling.

Q: Can I use the heating pad overnight?

A: It is not recommended to use the heating pad overnight to prevent overheating or skin irritation. Follow the manufacturer's guidelines for safe usage.

Documents / Resources



[MEDTECH BP-18 Nebulizer Kit](#) [pdf] User Manual

BP-18, BP-11, BP-11 BL, VAP-01, OG-05, TMP-02, HP-01, BP-18 Nebulizer Kit, BP-18, Nebulizer Kit, Kit

References

- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.