

# medtronic TH90 DBS Therapy Patient Programmer User Guide


[Home](#) » [Medtronic](#) » medtronic TH90 DBS Therapy Patient Programmer User Guide 

## Contents

- 1 medtronic TH90 DBS Therapy Patient Programmer
- 2 I have a group(check all that apply)
- 3 To view your therapy
- 4 Turn Therapy Off/On
- 5 How to Check your Battery Status
- 6 Changing Groups
- 7 Adjusting Stimulation
- 8 Documents / Resources
  - 8.1 References
- 9 Related Posts



**medtronic TH90 DBS Therapy Patient Programmer**

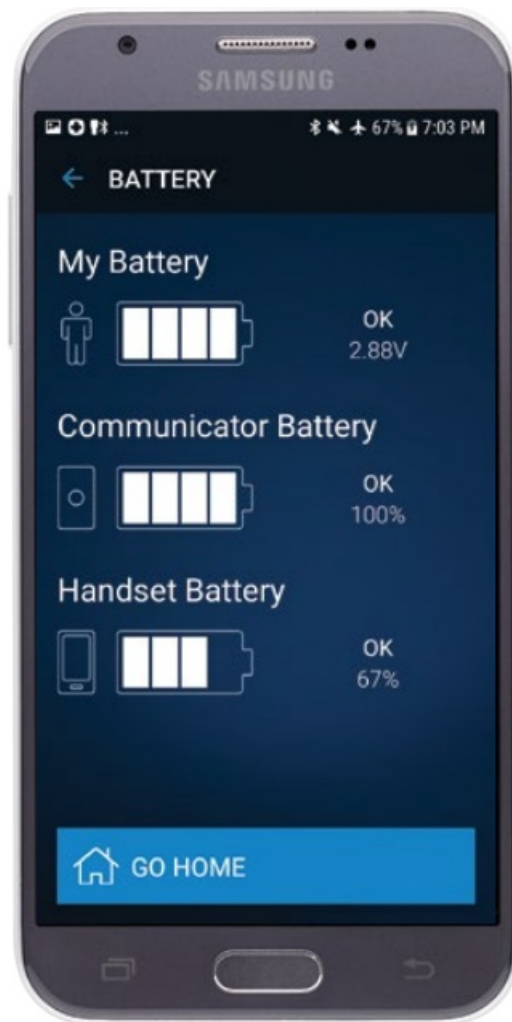


**I have a group(check all that apply)**


**For reference**

- A (for \_\_\_\_\_)
- B (for \_\_\_\_\_)
- C (for \_\_\_\_\_)
- D (for \_\_\_\_\_)

## To view your therapy

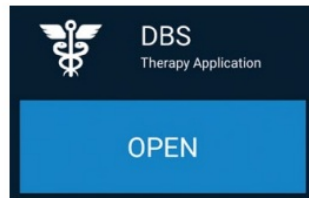

1. Turn on the handset and communicator. Check that they are charged.
2. Tap the OPEN button of the DBS Therapy Application on the handset.

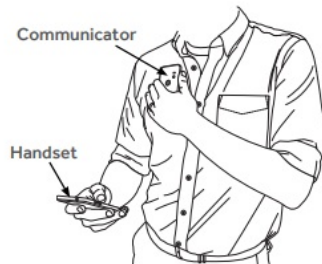


3. Place the communicator directly over the neurostimulator you want to connect to.
4. Keep the communicator over the neurostimulator and tap the CONNECT button on the handset.
5. When the DBS Therapy app is open and connected to the neurostimulator, the HOME screen will be visible.



## Turn Therapy Off/On

**Note:** You will need your communicator held over your neurostimulator for this.

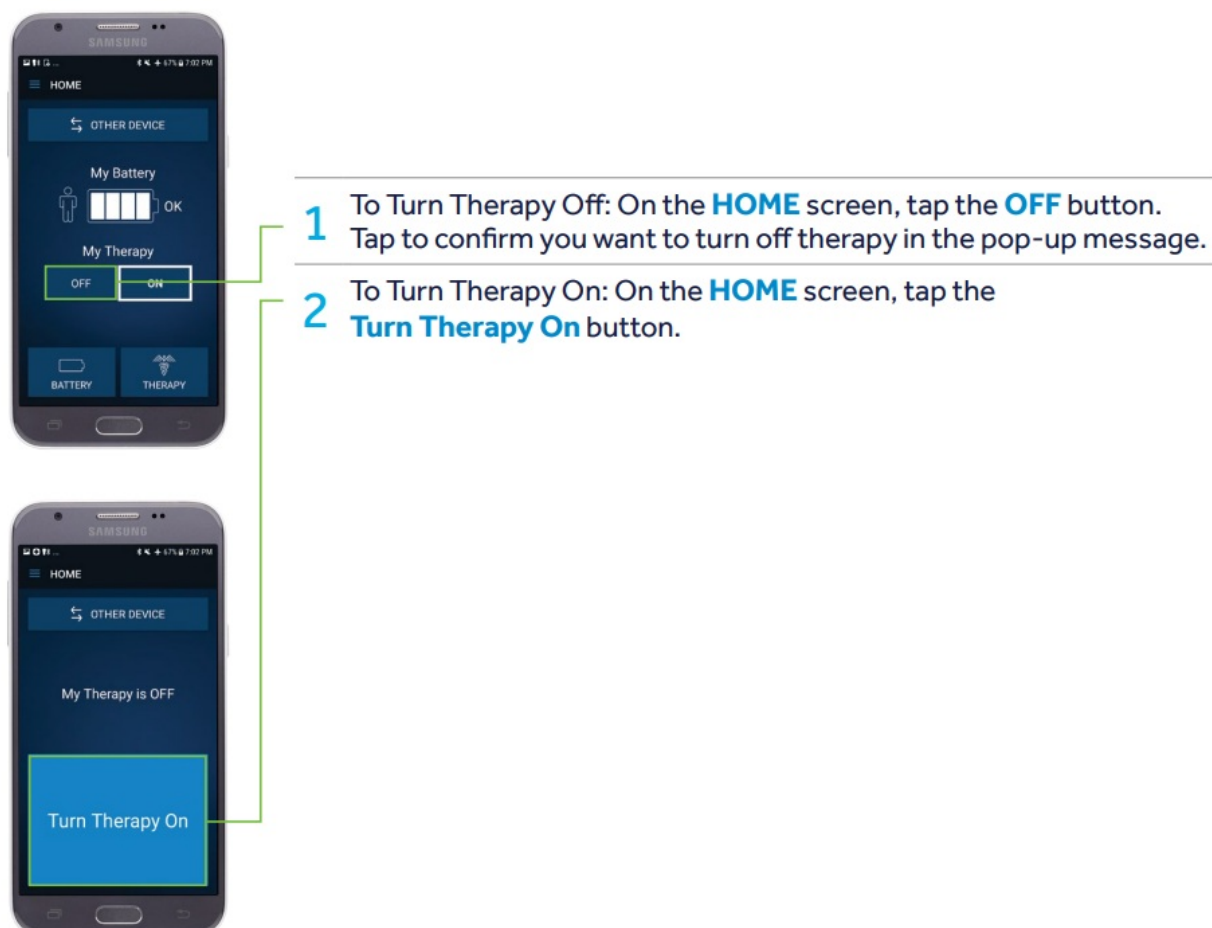

## How to Check your Battery Status



1. On the HOME screen, tap the BATTERY button.
2. The BATTERY screen shows the status of three different batteries:
  1. My Battery – the battery level of the neurostimulator, the device implanted in your body. If you have two neurostimulators implanted, you can view the battery level of the other device by switching to the other device.
  2. Communicator Battery – the battery level of the communicator, the device that you hold over your neurostimulator.
  3. Handset Battery – the battery level of the handset, the device that runs the DBS app.

**Notes:** You can also tap the Menu button and tap Battery.

## Changing Groups

### Notes


- You will need your communicator held over your neurostimulator for this.
- Your clinician may have set up groups, which are preset therapy options that you can choose from. If available, changing groups is a way for you to quickly adjust your therapy.



- 1 On the **HOME** screen, tap **THERAPY**.
- 2 Tap the **GROUP** button.
- 3 Choose the group that you want.

After choosing a group, the THERAPY screen for that group will appear. This means you have successfully changed groups.

## Adjusting Stimulation

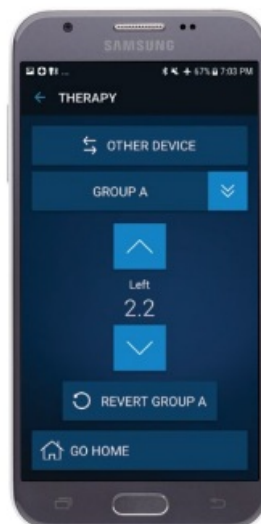

## Notes

- You will need your communicator held over your neurostimulator for this.
- If it is available to you, you can raise and lower the level of your therapy. You may not be able to adjust therapy. Your DBS clinician will decide whether to provide you with this option.
- Your clinician may have used Group Naming if the feature is available (dependent upon the software version of your handset)

1. On the HOME screen, tap THERAPY.
2. Adjust your therapy up and down using the arrow buttons.

- If an arrow turns gray, you cannot adjust therapy any further in that direction.

- Between the arrow buttons, Left is the left side of your body and Right is the right side.


## Medtronic Inc.

- 710 Medtronic Pkwy.
- Minneapolis, MN 55432
- USA
- **Tel:** 1-763-505-5000
- [medtronic.com](https://www.medtronic.com)

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo, and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. UC201913126a EN 02.2020

For video tutorials, visit [Medtronic.com/myDBSProgrammer](https://www.Medtronic.com/myDBSProgrammer)

## Documents / Resources

	<a href="#">medtronic TH90 DBS Therapy Patient Programmer</a> [pdf] User Guide TH90, DBS Therapy Patient Programmer, TH90 DBS Therapy Patient Programmer, Activa Device
---	--

## References

-  [Global Healthcare Technology Leader | Medtronic](#)
-  [DBS Programmer and Recharger Resources | Medtronic](#)

[Manuals+](#).