



midmark Ritter 204 Exam Table User Guide

[Home](#) » [Midmark](#) » midmark Ritter 204 Exam Table User Guide 

Contents

- [1 midmark Ritter 204 Exam Table](#)
- [2 Maintenance](#)
- [3 Cleaning](#)
- [4 Overview](#)
- [5 Documents / Resources](#)
 - [5.1 References](#)
- [6 Related Posts](#)



midmark Ritter 204 Exam Table



This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material. To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface. Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal/Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and a mild cleaner.


Overview

Table Receptacle (optional)

If the maximum load is exceeded, the circuit breakers remove power from the receptacle. Maximum Load: 5 amps

To reset circuit breaker(s)

Push circuit breaker toggle switch.

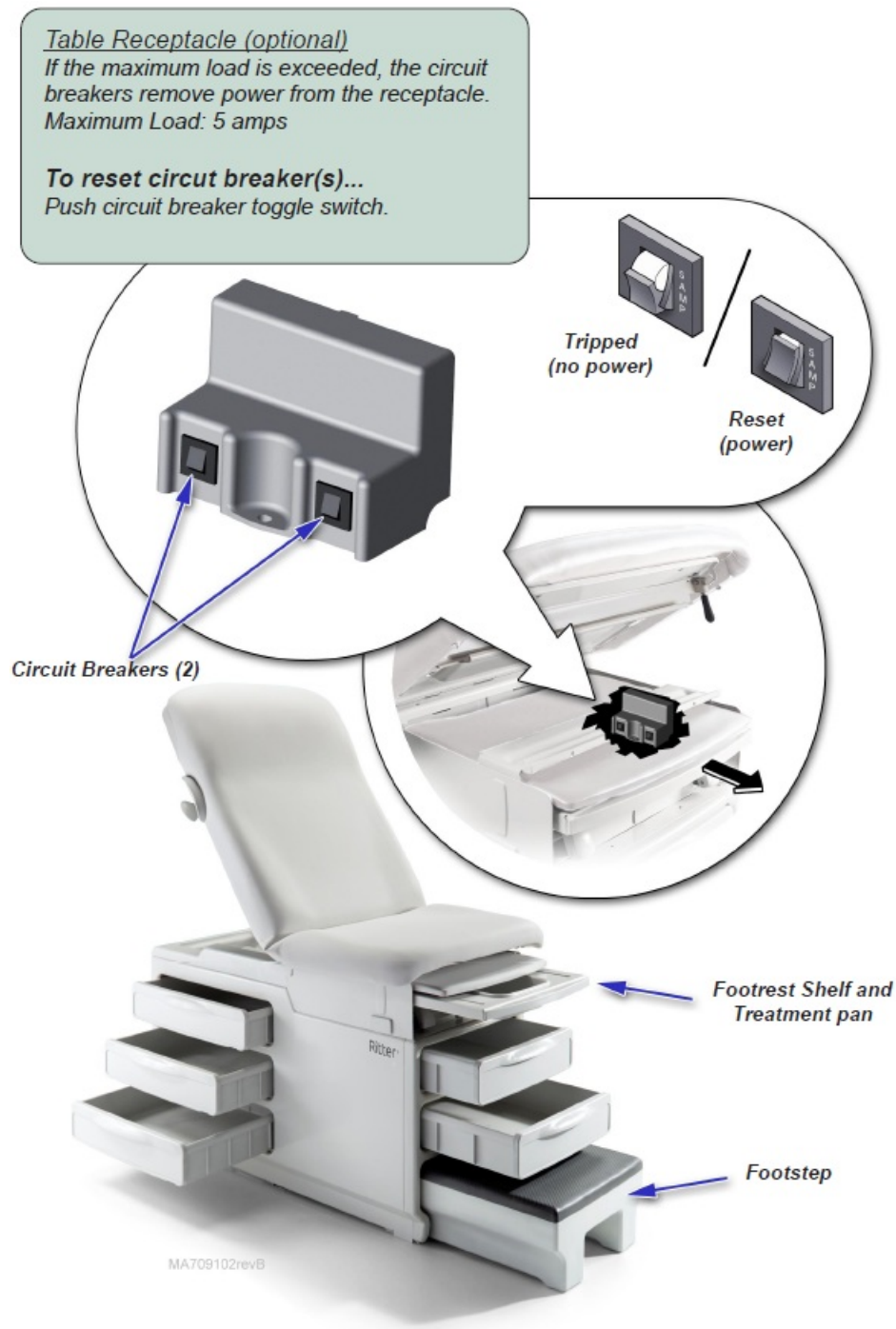

Pelvic Tilt feature (optional)

To elevate seat section


Lift seat section until pelvic support drops into place.

To return to flat position

- Lift seat section slightly.
- Pivot trigger handle backward.

- Lower seat section.



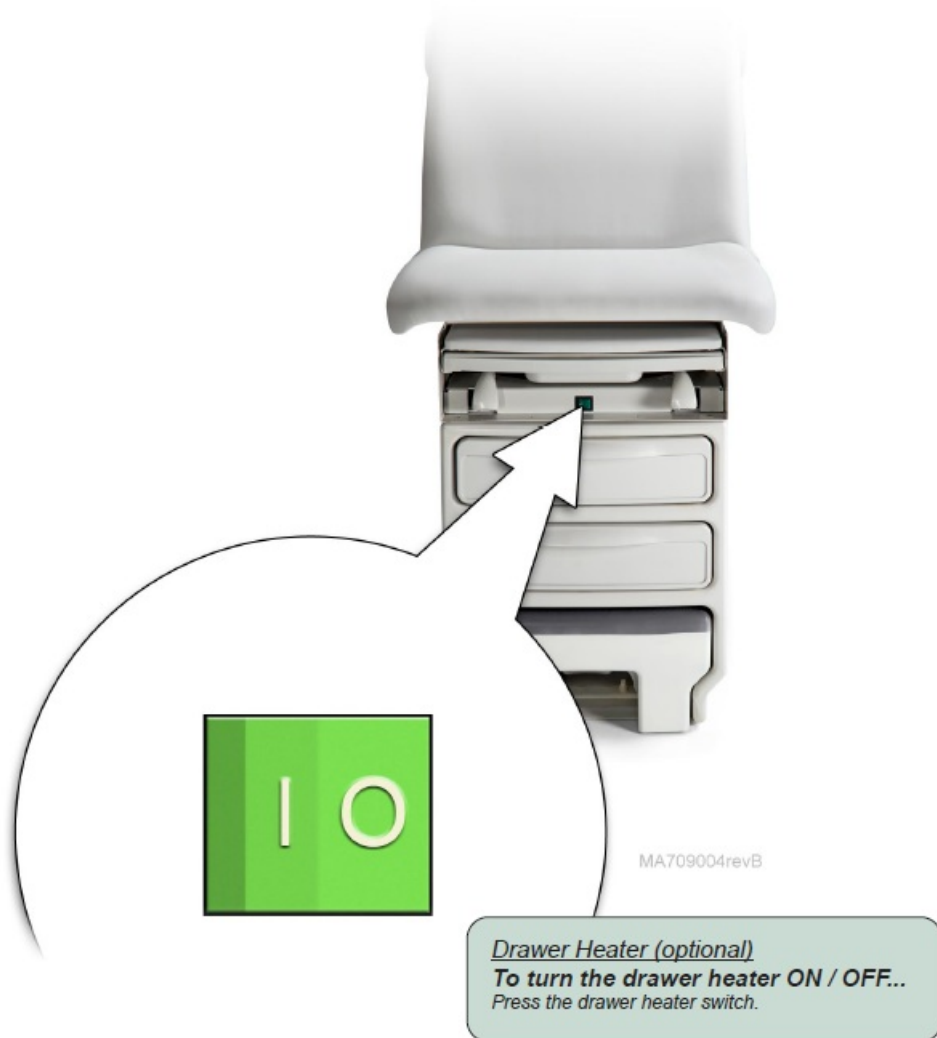

Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:

1.800.Midmark (1.800.643.6275)

8:00 am until 5:00 pm Monday through Friday (EST).

Midmark Corporation

60 Vista Drive Versailles, OH 45380-0286 Phone: 937.526.3662 Fax: 937.526.5542 midmark.com.

Documents / Resources



[midmark Ritter 204 Exam Table](#) [pdf] User Guide

Ritter 204 Exam Table, Ritter, Ritter Exam Table, 204 Exam Table, Exam Table

References

- [Midmark Medical, Dental, Vet Equipment Manufacturer & Supplier](#)