



MIR Spirodoc Stand Alone PFT PC-Based Spirometer Instructions

[Home](#) » [MIR](#) » **MIR Spirodoc Stand Alone PFT PC-Based Spirometer Instructions** 

Contents

- [1 MIR Spirodoc Stand Alone PFT PC-Based Spirometer Instructions](#)
- [2 Optional Oximetry module available](#)
- [3 Enter a New Patient](#)
- [4 Spirometry Testing: MIR Quick Guide Spirodoc™](#)
- [5 Oximetry Testing \(optional feature\):](#)
- [6 Oximetry icons common to all tests:](#)
- [7 6MWT \(walk test\)](#)
- [8 SpO2 Spot test](#)
- [9 Read More About This Manual & Download PDF:](#)
- [10 Documents / Resources](#)
- [11 Related Posts](#)

MIR Spirodoc Stand Alone PFT PC-Based Spirometer Instructions



Optional Oximetry module available



Spirodoc™ MIR Quick Guide

For more information see the Spirodoc™ User's Guide



Enter patient information



Search for a test performed after the selected date



Perform a new test using the selected patient



Review all tests in the database



Modify patient information



Search for a test using patient last name



Access the patient/test database



Print via Bluetooth connection



Search for a test using the date of birth



Insert a special character



Change settings (service menu)



Fast Test without entering patient data

Enter a New Patient

From the main menu press:

1. To access patient data and then to create a new patient


A keyboard will appear on-screen.



2. Type in the patient's first name, then press To delete a character, press
3. Type in the patient's last name (surname) then press
4. Enter the patient's birth date, height, weight and gender.
5. Press or to select the gender and confirm
6. Select the patient's ethnic group from the list and press

Spirometry Testing: MIR Quick Guide Spirodoc™

1. After creating a new patient, the main menu will be displayed again:

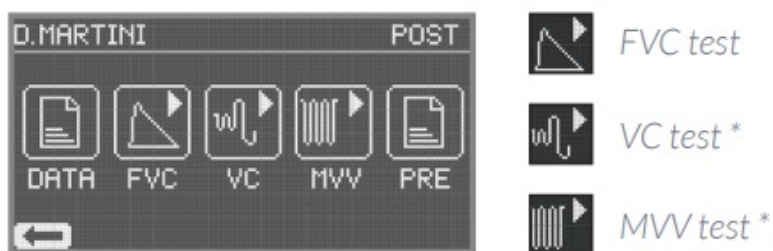

Press to perform POST bronchodilator test. (available after a PRE test has been already performed)

2. The type of turbine in use and the the maneuver instructions will be displayed. Patient should be coached to follow the given instructions.
3. After completing the maneuver, press the Power Key. Results will be displayed.
4. Press and Up or Down arrows to scroll through the results.
5. Press to return to the main menu
6. Press to display the 3 best tests (if available) for the patient session

* VC and MVV tests are not available in case of Fast Test or whenever NHANES III standard has been selected by default.

Oximetry Testing (optional feature):

Press to perform an oximetry test, then select the type of test



Oximetry icons common to all tests:



Display the plethysmograph curve



Check alarm settings



Check alarm settings, if at least one alarm is OFF



Enable alarms



Disable alarms



Display the most recent tests of a patient

6MWT (walk test)

Step 1:



to start the INITIAL REST phase

Step 2:



to move to the WALKING phase

Step 3:



to move to the RECOVERY phase

Step 4:



to insert the distance walked

Step 5:

Press **On/Off** key to end the test
Press **Yes** to insert the following additional data: Oxygen Administration, DYSPNEA, FATIGUE, PULSE

Step 6:

Press **No** to display test result

SpO2 Spot test

Step 1:  to start test

Step 2: Press ON/OFF key to end the test

Sleep test (ODI)

Step 1:  to start Sleep oximetry test (ODI)

Step 2: Automatic energy saving mode after 5 min

Step 3: Press **On/Off** key to end the test

Step 4:  to search for tests >12 hours

MIR USA, Inc. p. 4 5462 S Westridge Dr., New Berlin, WI 53151 P +1 (844) 464-7872 F +1 (262) 364-2030
pousa@spirometry.com www.usa.spirometry.com

Read More About This Manual & Download PDF:

Documents / Resources

	MIR Spirodoc Stand Alone PFT PC-Based Spirometer [pdf] Instructions Spirodoc Stand Alone PFT PC-Based Spirometer, Spirodoc, Stand Alone PFT PC-Based Spirometer, PC-Based Spirometer, Spirometer
---	--