



# Mölnlycke Setopress P.E.C Medium or High Compression Bandage Instruction Manual

[Home](#) » [Mölnlycke](#) » Mölnlycke Setopress P.E.C Medium or High Compression Bandage Instruction Manual 



## Setopress P.E.C Medium or High Compression Bandage Instruction Manual

### GP.E.C MEDIUM / HIGH COMPRESSION BANDAGE

Setopress is a polyamide, elastane and cotton compression (medium/high) extensible ("PECICHE") bandage.

#### Indications for use

Setopress is for the treatment of venous leg ulcers.

#### Contents


- [1 Instructions for use](#)
- [2 Before Extension](#)
- [3 Documents / Resources](#)
  - [3.1 References](#)
- [4 Related Posts](#)

### Instructions for use


To ensure correct application, a simple visual guide is printed on both sides of the bandage. The visual guide consists of colored rectangles which become squares when the correct extension is reached. On one side of the bandage are brown rectangles indicating high compression and on the other side are green rectangles indicating

moderate compression.

### Before Extension

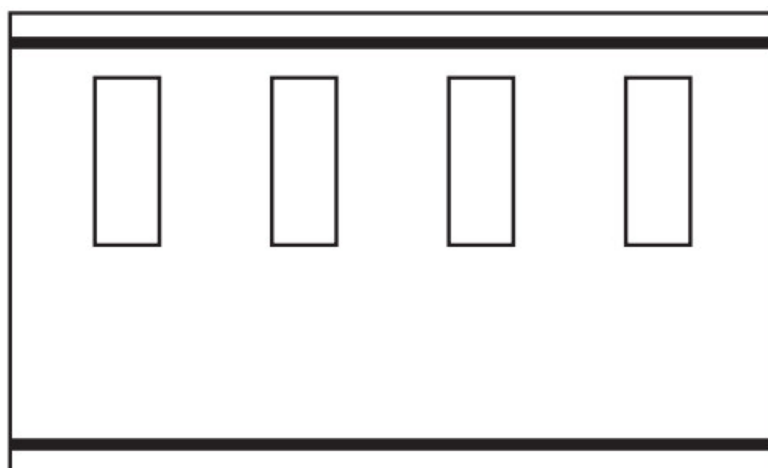


N.B. You have a choice of compression levels.  
Brown rectangles give high compression.

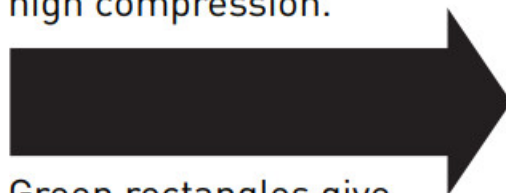


Green rectangles give moderate compression.

### Correctly extended – DO NOT OVERSTRETCH



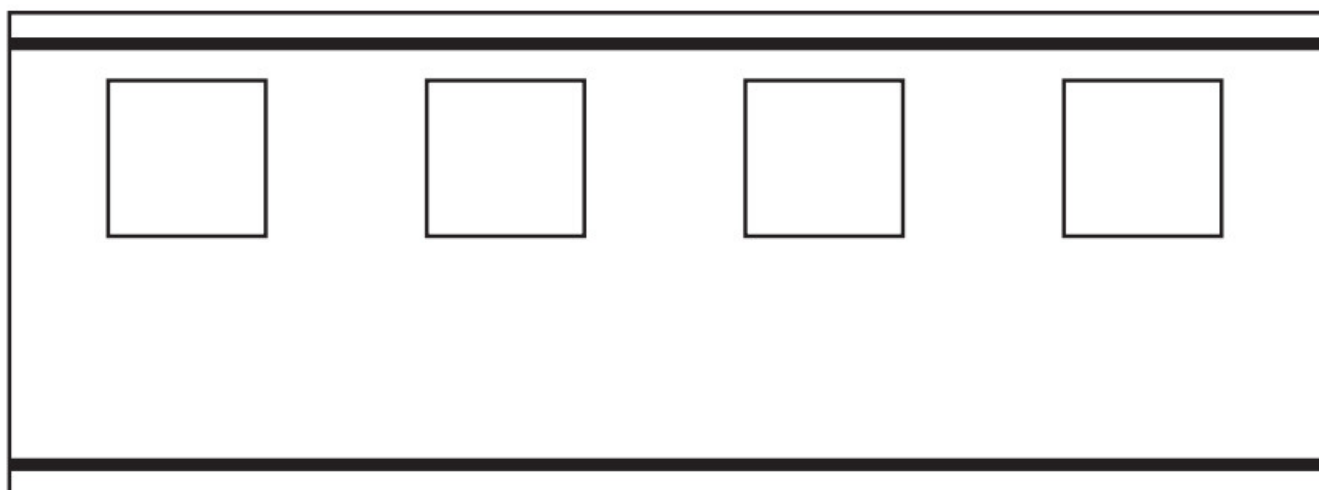


		Applied spirally with 50% overlap			
	Extension +/- 10%	Limb circumference (cm/in.)			
		<18cm	18 – 26cm/ 7 – 10.2in.	27 – 35cm/ 10.6 – 13.8in.	36 – 50cm/ 14.2 – 19.7in.
Approx sub-bandage pressure	70% (green square)	Not suitable	27 -38 mmHg/ 1 – 1.5in.Hg	20 – 26 mmHg/ 0.8- 1.0in.Hg	Not suitable
	100% (brown square)	Not suitable	35 -50 mmHg/ 1.4 – 2in.Hg	27 – 34 mmHg/ 1 – 1.3in.Hg	19 -26 mmHg/ 0.7 – 1 in.Hg

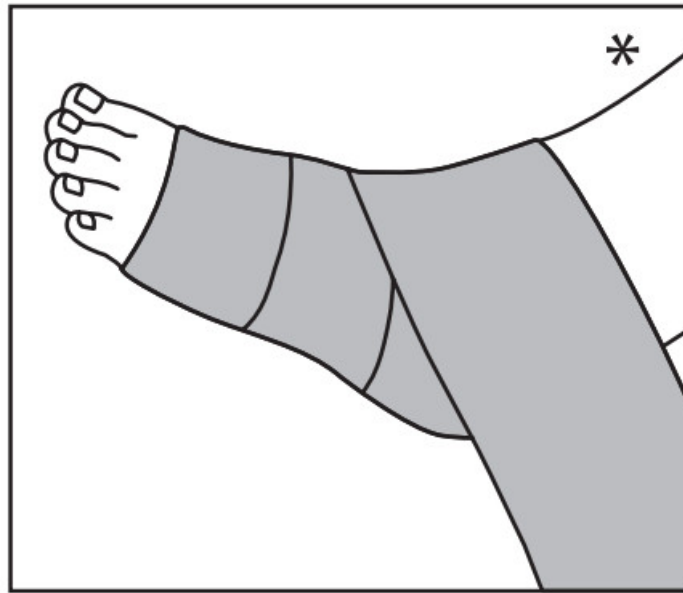




\*A padding layer should be used prior to the application.

**Commence bandaging at the base of the toes and enclose the heel with NO EXTENSION...**

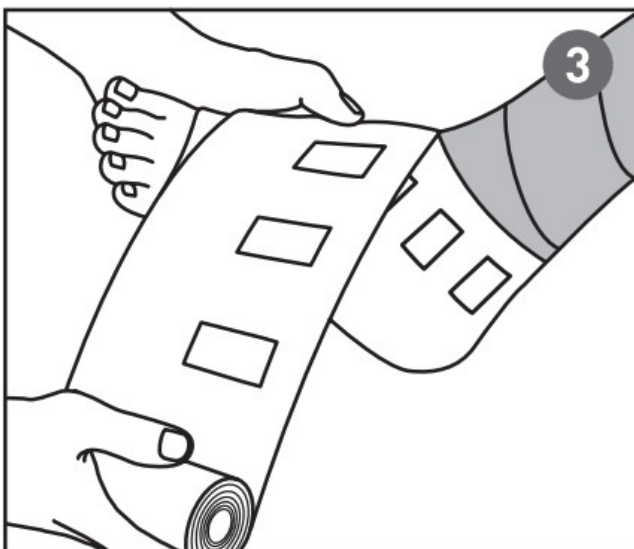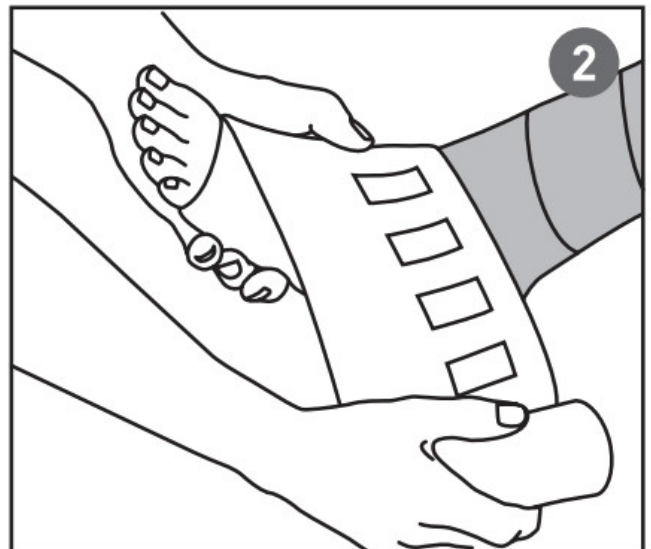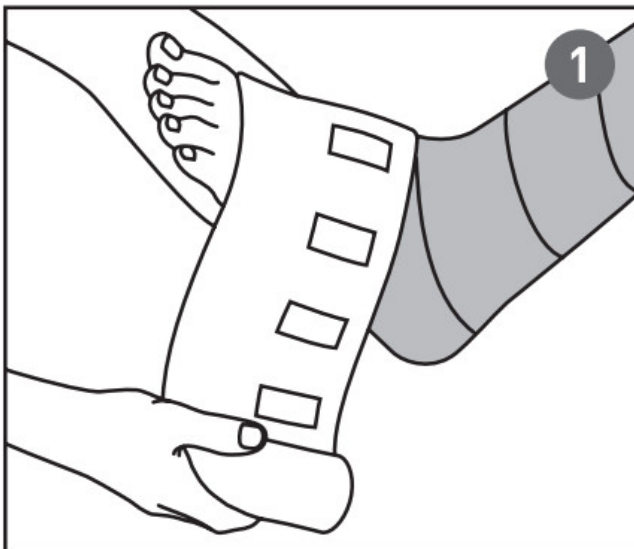

1. Apply around the foot starting at the base of the toes (no extension)
2. Make two turns around the ankle to enclose the heel (no extension)
3. Return to the base of the toes and make one turn around the foot (no extension)



**...NOW WITH EXTENSION continue up the leg in an ascending spiral**

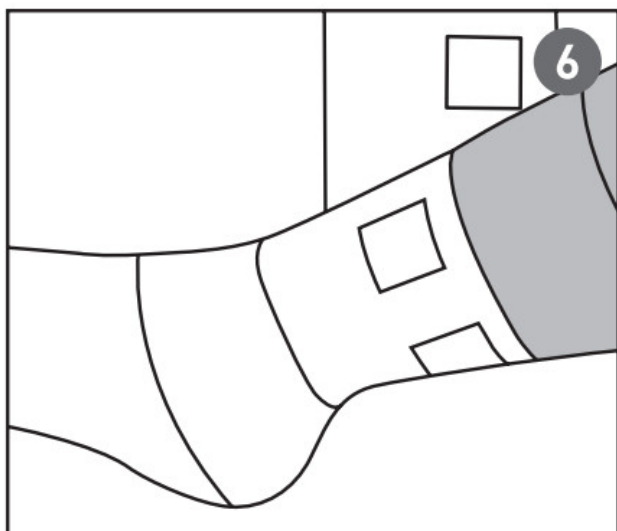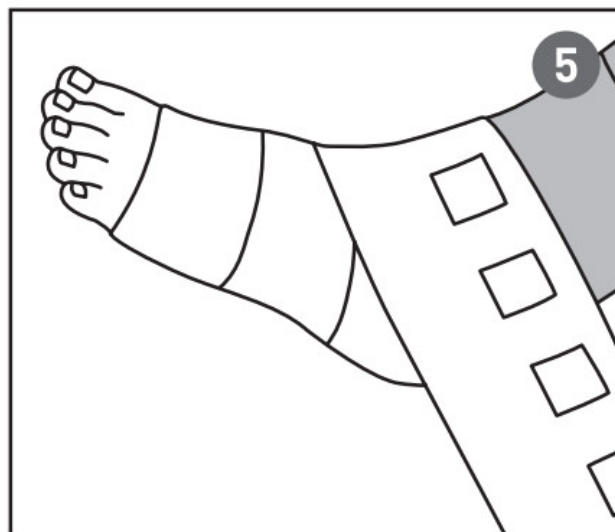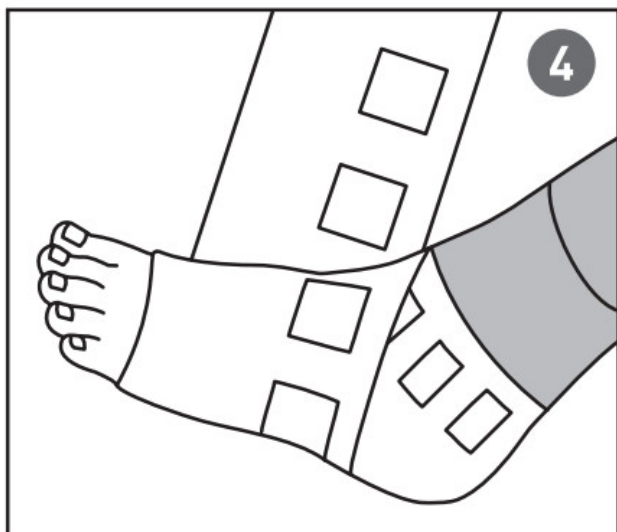

4. Now apply full extension (rectangles become squares)
5. Apply with 50% overlap (full extension)
6. Continue to apply as an ascending spiral (full extension)

**N.B. It is important to use the same indicator for consistent extension throughout the application.**

**...finally secure the bandage just below the knee**

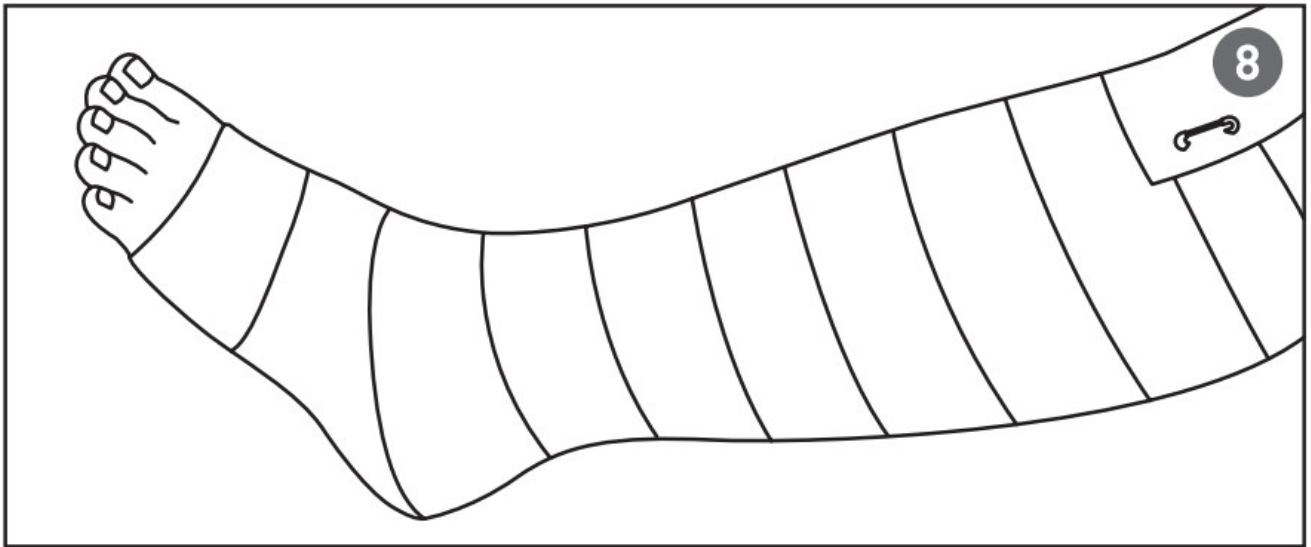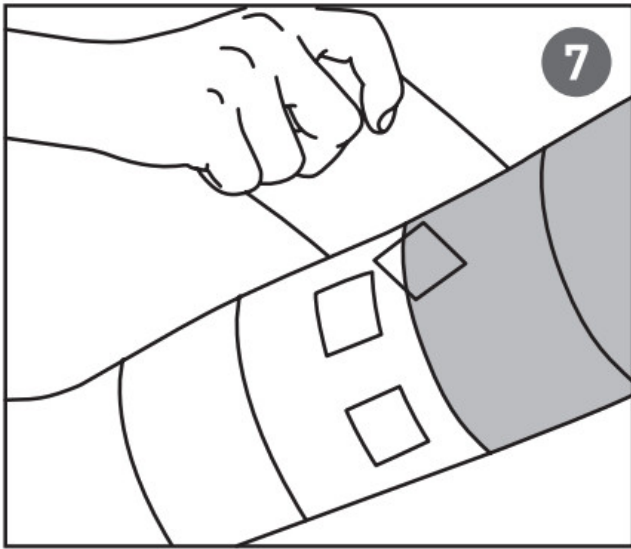

7. Stop bandaging just below the knee. Cut off any excess bandage.

8. Secure.

Setopress may be applied over a primary wound dressing.

### **Contraindication(s)**

Do not use the figure of eight bandaging.

Do not apply if arterial insufficiency is present or suspected.

Do not use on limbs with a circumference of less than 18cm/7in.

### **Precautions**

Do not use Setopress on patients and/or users with known hypersensitivity to the ingoing materials/components of the product.

The effective circumference of the ankle when reading off the sizing table must take into account any padding layer.

Setopress should only be applied by a trained bandage.

Do not use bandage clips.

This product must be used under the direction of a healthcare professional.

The bandage should be changed in line with dressing changes.

Discard and replace if soiled with bodily fluids.

### **Fibre content**

Polyamide, Elastane, Cotton.

Special storage conditions and handling conditions Disposal should be handled according to local environmental

procedures.

#### Other information

If any serious incident has occurred in relation to the use of Setopress, it should be reported to Mölnlycke Health Care and to your local competent authority.

The Mölnlycke and Setopress trademarks, names, and logos are registered globally to one or more of the Mölnlycke Health Care Group of companies.

Size 10cm x 3.5m (relaxed roll)

#### Medical Device



Upper-temperature limit limite

Keep dry

Conserver au sec

**Caution:** Federal (US) Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner)

[www.molnlycke.com](http://www.molnlycke.com)

**Made in UK**

Australian Sponsor Address:  
Mölnlycke Health Care Pty. Ltd  
12 Narabang Way  
Belrose, NSW 2085

**Australia**

USA 1-800-882-4582  
Canada 1-800-494-5134



#### Manufacturer

Mölnlycke Health Care AB

Gamlestadsvägen 3C, Box 13080, SE-402 52 Göteborg, Sweden

**Revised 2021-07**

**327238-09**

Master PD-569688 rev. 02 PD-580466

#### Documents / Resources



[Mölnlycke Setopress P.E.C. Medium or High Compression Bandage](#) [pdf] Instruction Manual

Setopress, P.E.C. Medium or High Compression Bandage, Setopress P.E.C. Medium or High Compression Bandage, Medium or High Compression Bandage, High Compression Bandage, Compression Bandage, Bandage

#### References

-  [A world-leading provider of single-use surgical and wound care products | Mölnlycke](#)

Manuals+.