

Mylab KnowSure Ovulation Testing Kit



# Mylab KnowSure Ovulation Testing Kit User Guide

[Home](#) » [Mylab](#) » Mylab KnowSure Ovulation Testing Kit User Guide 

## Contents

- [1 Mylab KnowSure Ovulation Testing Kit](#)
- [2 What is Ovulation?](#)
- [3 How this test helps?](#)
- [4 How to recognize the LH surge?](#)
- [5 Glossary of Symbols](#)
- [6 Documents / Resources](#)
  - [6.1 References](#)
- [7 Related Posts](#)



## Mylab KnowSure Ovulation Testing Kit



## What is Ovulation?

Ovulation is a process in which your ovary releases an egg once in every menstrual cycle.

## How this test helps?

It is an ovulation test device that helps you to identify the most fertile days in a month when you can try for pregnancy.

## How to recognize the LH surge?

**Calculate the length of the menstrual cycle:**

1. Count the number of days between the first day of your last month's period to the first day of the current month's period.
2. If your cycle is irregular, then take the average of the last 3 month's cycle lengths as the length of your menstrual cycle.

## Calculate the day to start testing

Count ahead this many days starting with the first day of the last period.

Cycle lengths in days	25	26	27	28	29	30	31	32	33
Test day	8	9	10	11	12	13	14	15	16


## How to use ovulation test?

Calculate the date to start testing according to ovulation test Instructions for use.

1. Wash your hands. Make sure they are dry before starting the test procedure.



2. On the calculated day, collect urine specimen in a clean, dry container and put it on a flat surface.
3. Tear the Pouch and keep the individual components ready for the testing. (Check the colour of desiccant, it should be colourless, if it has turned yellowish green, discard the test and use another test. Once opened, test should be used immediately)

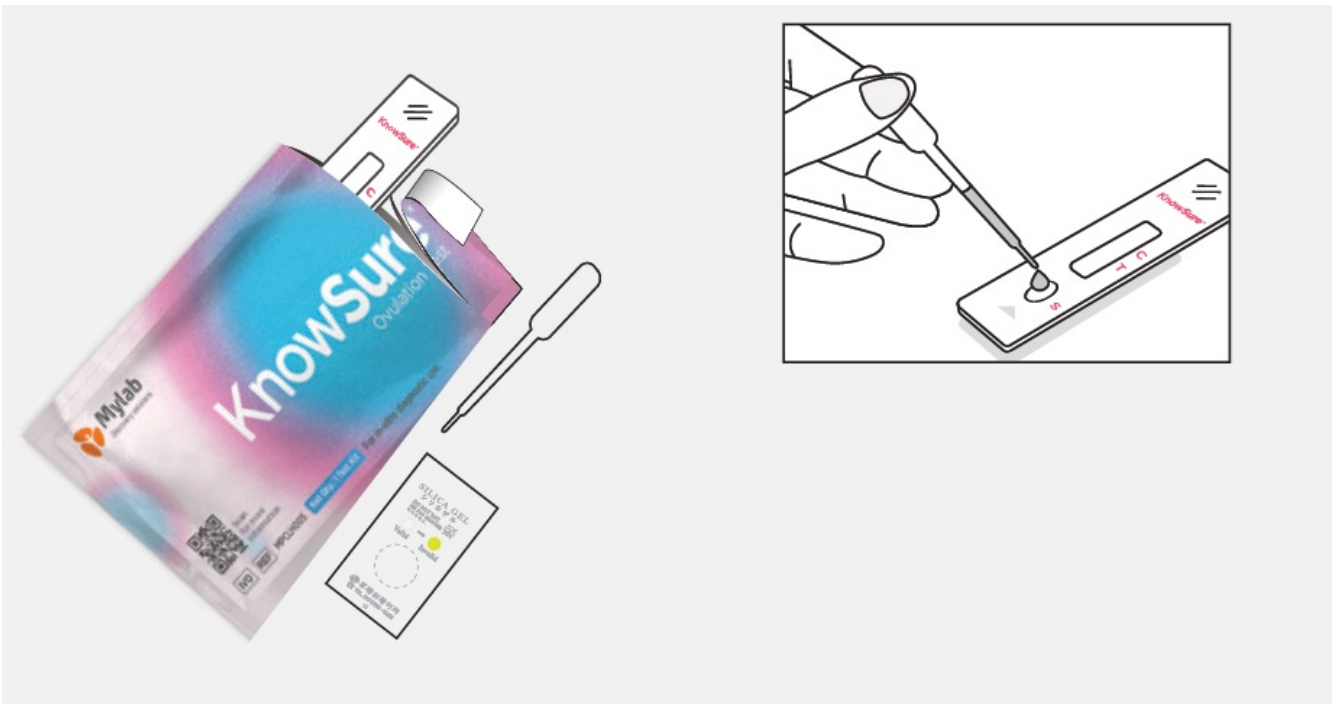

4. Fill the plastic dropper with the specimen. Holding the dropper vertically, dispense 2 drops of urine specimen into the sample well making sure that there are no air bubbles.
5. Start the timer.




## Read the result at 5 mins

Do not read  
After 5 minutes


Note: Note that the test can be done at any time of the day, preferably after 11:00 am and before 10:00 pm. The best times to test are between 11:00 am to 3:00 pm and between 5:00 pm to 10:00 pm.

### How to read the result?


1. Negative: If the coloured line appears at the control side 'C' only.

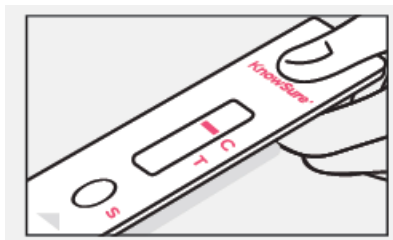


2. Negative: A distinct coloured line appears at the control side 'C' and a light-coloured line at the test side "T".

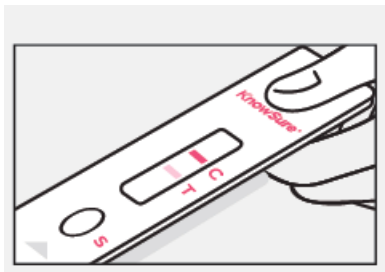


3. Positive: Distinct coloured lines appear at the control side 'C' and at the test side "T"

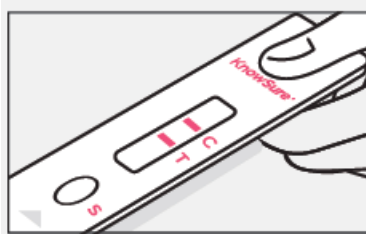


4. Positive: A distinct coloured line appears at the control side 'C' and a dark coloured line at the test side "T".

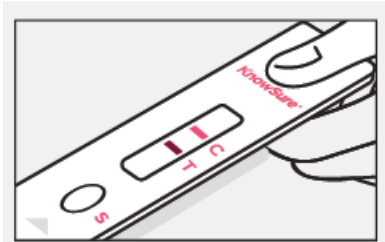

5. Invalid: No line appears at the 'C' and 'T' side

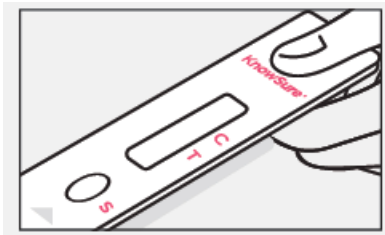

6. Invalid: No coloured line appears at the 'C' side and the line appear only at the 'T' side

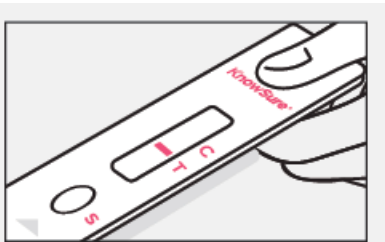















## Limitations

1. For human urine only. False results may occur when the test is used for unintended purposes or with unintended specimens.
2. If the result is invalid, then repeat the test.
3. This test should not be used as a means of birth control.
4. The test is only intended for the detection of LH in hCG-free samples.
5. Women suffering from polycystic ovary syndrome may have elevated LH concentrations.

If you don't get positive result in a month, then please consult with your physician. Made In India. To be sold in India.

## Glossary of Symbols

	Manufacturer		Contains Sufficient for <n> Tests
	Catalogue Number		Batch Code
	Date of Manufacture		Use by date
	Storage Temperature		Keep Dry
	Caution		<i>In-vitro</i> Diagnostic Medical Device
	Do not re-use		Keep Away from Sunlight
	Do not use if pouch or kit damaged		Read package insert before use

IFU-IND-2057-01; Rev.00; 2023-02-10 Manufactured & Marketed by:


Mylab Discovery Solutions Pvt. Ltd. Plot No. 99-B, Lonavala Industrial Copt. Est. Ltd., Nangargaon Lonavala District Pune Maval, Maharashtra (India)-410401

- Email: [info@mylabglobal.com](mailto:info@mylabglobal.com)
- Web: [www.mylabglobal.com](http://www.mylabglobal.com) Customer Service: 1800-121-8684

#### Customer Service:

1800-121-8684 PathoDetect | NATSpert | PathoCatch | OncoScreen DiscoverSeries | AgroDex | VetScreen | Compact XL

## Documents / Resources

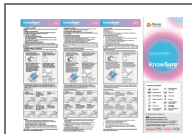

[Mylab KnowSure Ovulation Testing Kit \[pdf\] User Guide](#)  
KnowSure Ovulation Testing Kit, Ovulation Testing Kit, Testing Kit, Kit

## References

- [User Manual](#)

SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.