



[Home](#) » [NAGY](#) » **NAGY Fluor Protector Instruction Manual** 

NAGY Fluor Protector



Contents [[hide](#)]

- 1 [Description](#)
- 2 [Model](#)
- 3 [Specification](#)
- 4 [Performance](#)
- 5 [Composition](#)
- 6 [Dental Bonding Steps](#)
- 7 [Instructions](#)
- 8 [Warning](#)
- 9 [Contraindications](#)
- 10 [Storage](#)
- 11 [Transportation](#)
- 12 [Customer Support](#)
- 13 [Documents / Resources](#)
 - 13.1 [References](#)

Description

Fluor Protector is a protective fluoride varnish for the prevention of dental caries.

Model

Type A (22,600 ppm), Type C (6,000 ppm)

Specification

Type A (0.4g, 10g), Type C (0.4ml, 3ml)

Performance

1. Sensory indicators:
Type A (paste), Type C (liquid)
2. Fluorine ion content (%):
Type A: $2.26 \times (1 \pm 10\%)$ Type C: $0.6 \times (1 \pm 10\%)$
3. **pH:** 4.0-8.0

Composition

Ethanol, natural resin, sodium fluoride, xylitol, thickening agent, food flavor.

Dental Bonding Steps



Instructions

1. Clean the tooth surface.
2. Dry the surface of the teeth (e.g. using a cotton roll).
3. take out no more than 0.4g of this product on the tray provided, use the small brush provided to dip the product and apply it to the teeth, apply repeatedly until a uniform film is formed.
4. Keep the surface of the teeth dry until a non-shiny film is formed.
5. Do not brush teeth and rinse mouth, do not eat or drink within 30 minutes.
6. Use it regularly, 2-4 times a year

Warning

1. Do not swallow.
2. Keep away from ignition sources.
3. Keep out of the reach of children.
4. For use by dental professionals only.
5. Anytime you feel discomfort, brush or rinse your teeth for relief.

Contraindications

1. Contraindicated in patients with ulcerative gingivitis, stomatitis, bronchial asthma.
2. It should not be used during cold and fever, gastrointestinal discomfort.
3. It is contraindicated for those who are allergic to the components of the product.
4. It is contraindicated for pregnant and lactating women.
5. Not suitable for children under 3 years of age.

Storage

Store in a dry place at or below room temperature. Seal immediately after use and finish using as soon as possible after opening.

Transportation

Put it carefully and gently, avoid impact, sunshine and rain.

Customer Support

nagyorol.com



MedNet EC-REP GmbH

Borkstraße 10

48163 Münster Germany



Nagy Oral Tech Ltd.

9 Dushan Road


Qinhuangdao

0086-335-8903835





Documents / Resources

	NAGY Fluor Protector [pdf] Instruction Manual Type A 22, 600 ppm, Type C 6, 000 ppm, Fluor Protector, Fluor, Protector
---	---

References

- [User Manual](#)

000 ppm, 600 ppm, Fluor, Fluor Protector, NAGY, Protector, Type A 22, Type C

■ NAGY 6

Leave a comment

Your email address will not be published. Required fields are marked *

Comment *

Name

Email

Website

☐ Save my name, email, and website in this browser for the next time I comment.

Post Comment

Search:

e.g. whirlpool wrf535swhz

Search

[Manuals+](#) | [Upload](#) | [Deep Search](#) | [Privacy Policy](#) | [@manuals.plus](#) | [YouTube](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.