



# NIPRO DIAGNOSTICS TRUE2go Meter Test Strips and System User Manual

[Home](#) » [NIPRO DIAGNOSTICS](#) » NIPRO DIAGNOSTICS TRUE2go Meter Test Strips and System User Manual 



Meter, Test Strips & System Components



**Checking Your Blood Glucose**  
**Using TRUE2go® A guide for testing your blood glucose**

**Simple Steps to Test Your Blood Glucose**

## Contents






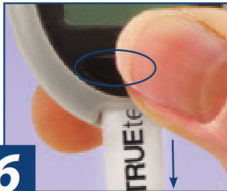
### 1 Blood Glucose Testing

#### 2 Documents / Resources

##### 2.1 References

#### 3 Related Posts

## Blood Glucose Testing

 <b>1</b>	 <b>2</b>	 <b>3</b>
Wash your hands with warm, soapy water.	Remove Test Strip from vial and close vial immediately. Insert Test Strip into Test Port with TRUEtest™ facing up. The Meter turns on.	Lance your finger.
 <b>4</b>	 <b>5</b>	 <b>6</b>
Allow a blood drop to form, touch Tip of Strip to top of blood drop and allow blood to be drawn into Strip. Remove Test Strip Sample Tip from sample drop immediately after dashes appear across Meter Display. <b>CAUTION!</b> Holding the Test Strip Sample Tip to the blood sample too long after the Meter begins testing may cause inaccurate results.	In as little as 4 seconds, glucose result will be displayed.	Hold Meter in a vertical position with Test Strip facing down. Press strip release button to discard the used Test Strip from the Meter.

## WARNING!

NEVER reuse Test Strips. NEVER wipe test Strips with water, alcohol or any cleaner. DO NOT attempt to remove blood or control sample from Test Strips or clean Test Strips and re-use. Reuse of Test Strips WILL cause inaccurate results. NEVER add a second drop of sample to the Strip. Adding more sample gives an error message.

This guide provides a brief overview on testing your blood glucose. For more detailed information on complete blood glucose testing procedure, consult "Testing Your Blood" section in the Owner's Booklet. For assistance using your blood glucose System, call our Customer Care Department at 1-800-803-6025.



© 2011 Nipro Diagnostics, Inc. TRUE2go, TRUEtest and the Nipro Diagnostics logo are trademarks of Nipro Diagnostics, Inc. F4NPD08 Rev. 22


[www.niprodiagnostics.com](http://www.niprodiagnostics.com)



<http://goo.gl/PX5h9>

Scan this code with your smart phone for additional TRUE2go information. Always refer to the Owner's Booklet for detailed product information.

## Documents / Resources

	<p><a href="#">NIPRO DIAGNOSTICS TRUE2go Meter Test Strips and System</a> [pdf] User Manual TRUE2go Meter Test Strips and System, TRUE2go, Meter Test Strips and System, Test Strips and System, Strips and System</p>
---	--

## References

-  [Loading...](#)