



## **Contents** [ [hide](#) ]

- [1 Nordic 1026138 Motion Control Recording Device](#)
- [2 Product Information](#)
- [3 Log-in Procedures](#)
- [4 Opening the software](#)
- [5 Log in with recording mode](#)
- [6 Log in with debriefing mode](#)
- [7 Log in with mobile mode..](#)
- [8 Screen layout](#)
- [9 Camera audio](#)
- [10 Pan-Tilt-Zoom controls](#)
- [11 Pan-Tilt-Zoom camera presets](#)
- [12 Making a recording](#)
- [13 Exporting videos](#)
- [14 Editing recordings](#)
- [15 Software options](#)
- [16 Software options](#)
- [17 FAQs](#)
- [18 Documents / Resources](#)
  - [18.1 References](#)

# NORDIC

## **Nordic 1026138 Motion Control Recording Device**



## **Product Information**

### **Specifications**

- Product Name: CoRecorder
- Application Modes: Recording, Debriefing, Mobile
- Features: Camera screens, Monitor capture, Scene selection,
- Start/Stop recording, PTZ camera controls, Marker buttons, Audio
- control, Screen layout selection, Playback controls, Timeline
- Display, Marker icon/comment, Export to video menu

### **Log-in Procedures**

#### **Recording Mode**

1. Start the software in Recording mode.
2. Role: User
3. Password: Leave empty (default setting)
4. Application: Recording
5. Click Login to access the software in Recording mode.


#### **Debriefing Mode**

1. Start the software in Debriefing mode.

2. Role: User
3. Password: Leave empty (default setting)
4. Application: Debriefing
5. Click Login to enter the software in Debriefing mode.

## live mode

1. Camera screens
2. Monitor capture
3. Scene selection
4. Start/Stop recording
5. Activate PTZ camera controls
6. Recording time
7. Add marker buttons
8. Enable/Disable audio from a camera
9. Screen layout selection buttons
10. Log out
11. Go to PLAYBACK mode

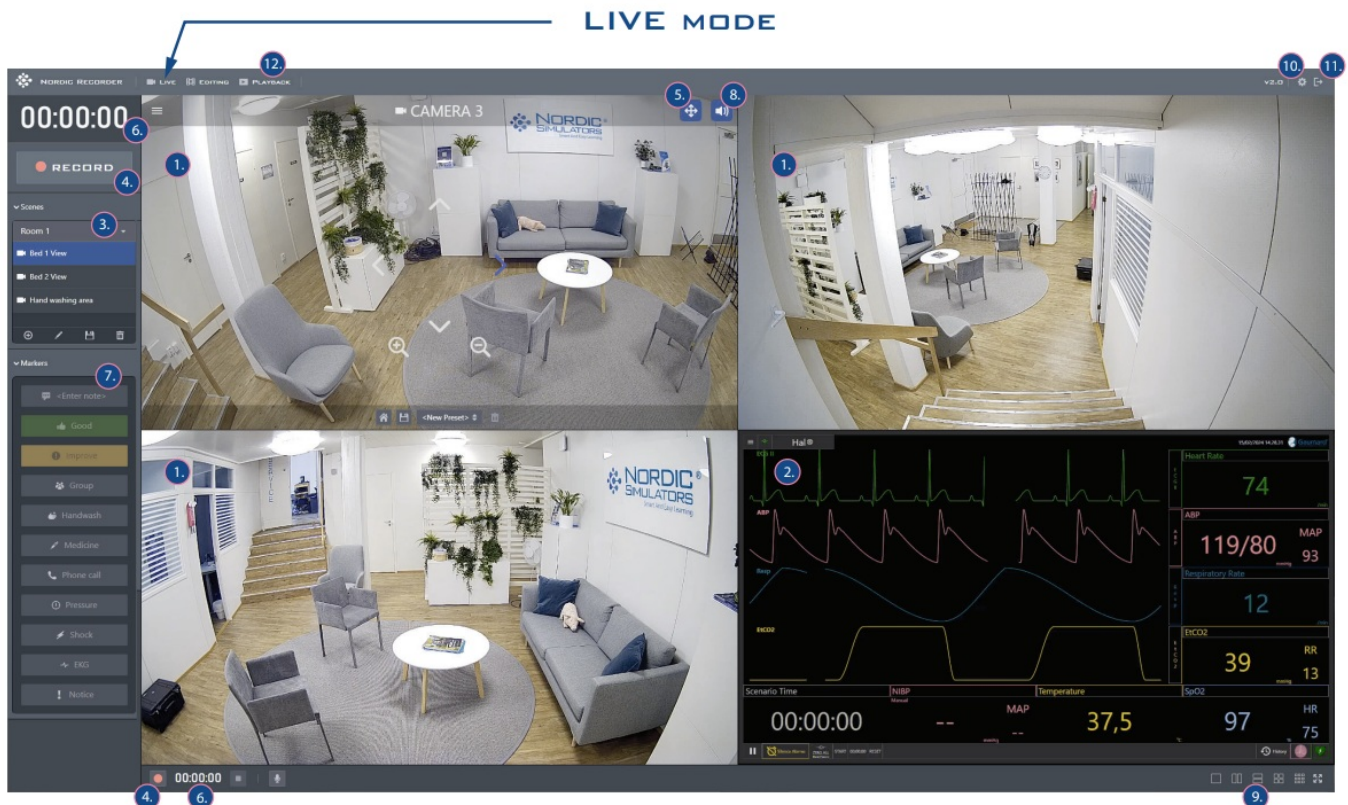

## playback mode


1. Camera screens
2. Monitor capture
3. Screen layout selection buttons
4. Playback controls: Stop, Play/Pause
5. Jump to next/previous marker
6. Timeline of the selected recording
7. Current place on the timeline
8. Marker icon/comment
9. Open the recording list
10. Recording name
11. Current playing time
12. Recording total time
13. Export to video menu
14. Enable/disable audio from a camera
15. Go to LIVE mode



## Opening the software

- Start the computer from the power button.
- The computer starts up to the desktop.

- Start the recording software by double-clicking the icon Nordic CoRecorder
- The web browser-based log-in window opens, which can connect to the recording software.
- In Recording mode, it is possible to watch the cameras, add markers and put the recording on/off.
- In Debriefing mode, it is possible to watch the cameras and recordings.
- In Mobile mode its is possible to put the recording on/off, add markers without the camera pictures or activate the mobile device's camera to the recording.
- The software will remember the last used information and display those when opening the log-in window on a specific computer.
- The software can be opened in three different Application modes: Recording, Debriefing and Mobile..

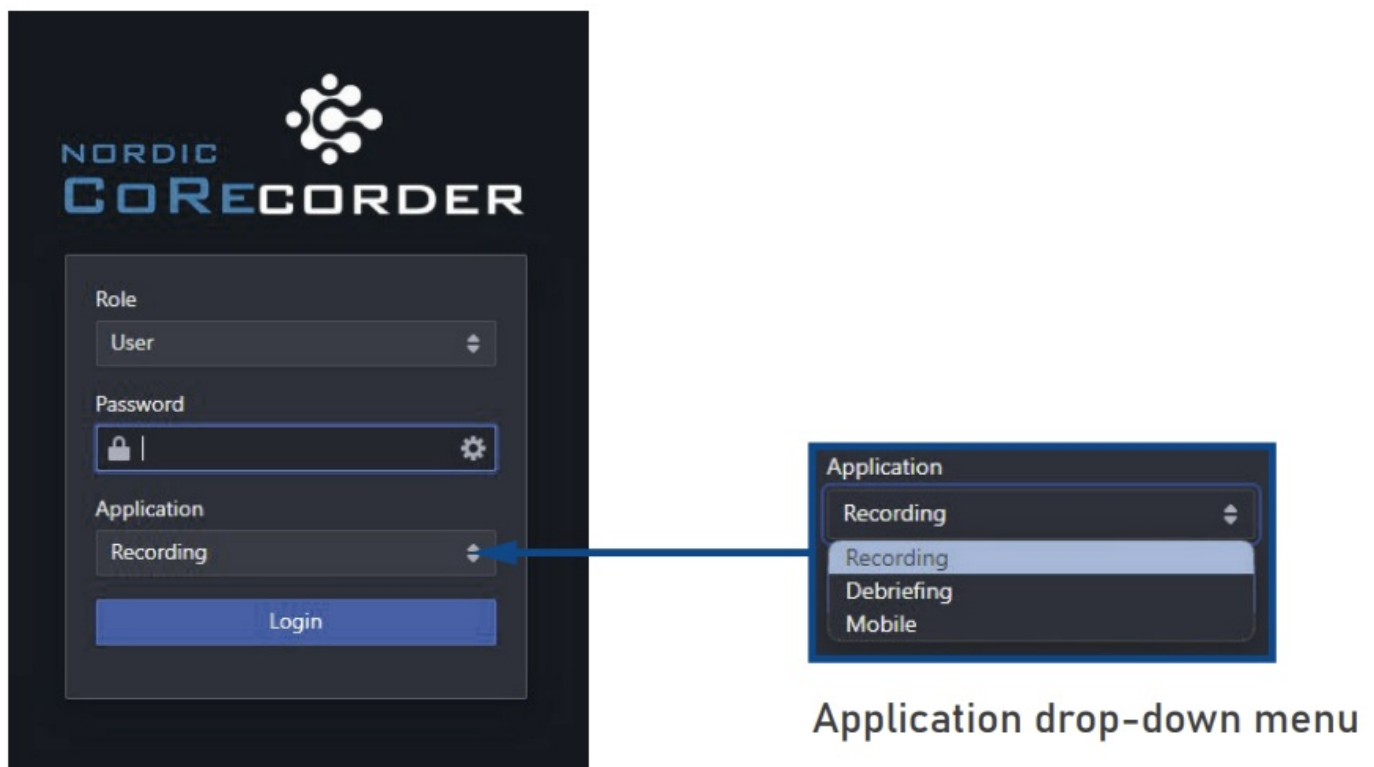


## Log in with recording mode

Start the software in the Recording mode.


- **Role:**
  - User
- **Password:**
  - The default is that there is no User password;d, leave it empty.



- **Application:**
  - Recording

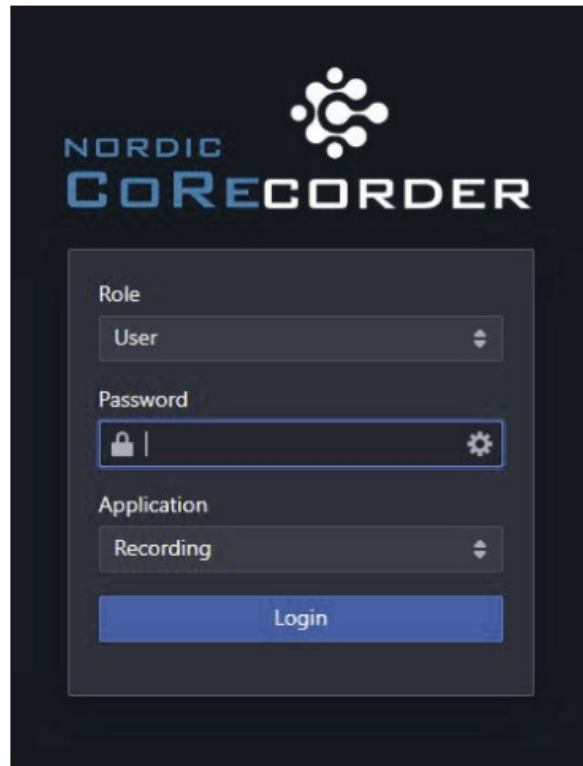

Press Login, and the software opens in Recording mode.

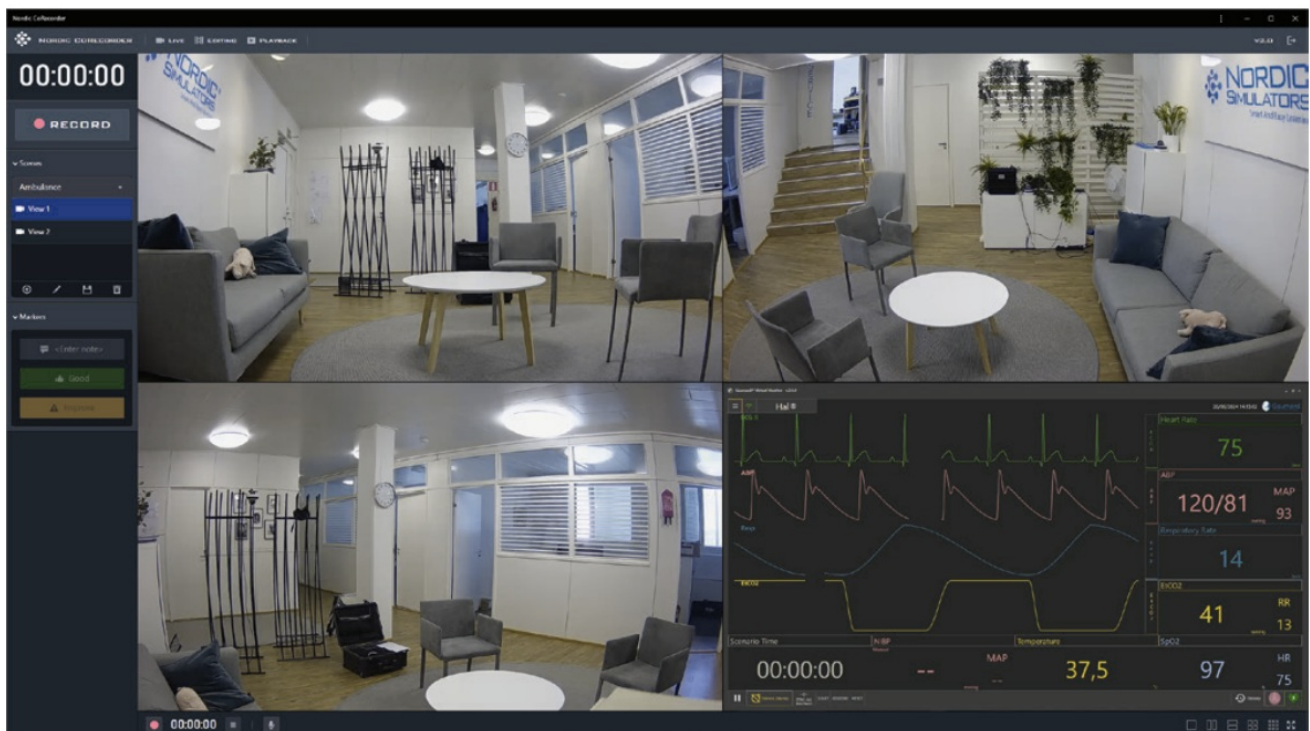

In Recording mode, it is possible to watch the cameras, add markers and put the recording on/off. Recording mode is meant for users in the control room.

**Log in with debriefing mode**

Start the software in the Debriefing mode

**Role:**


- User

**Password:**


- The default is that there is no User password; leave it empty.

**Application:**

- Debriefing

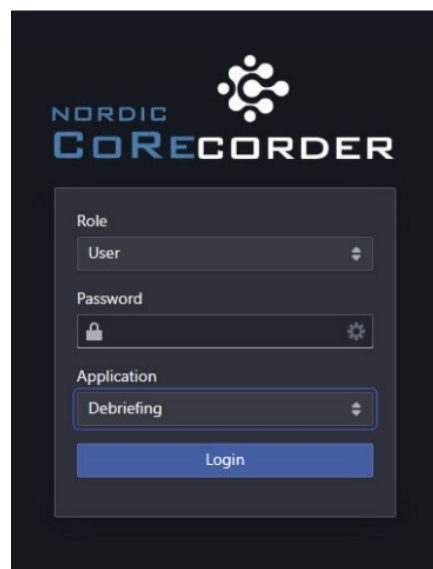

Press Login, and the software opens in Debriefing mode

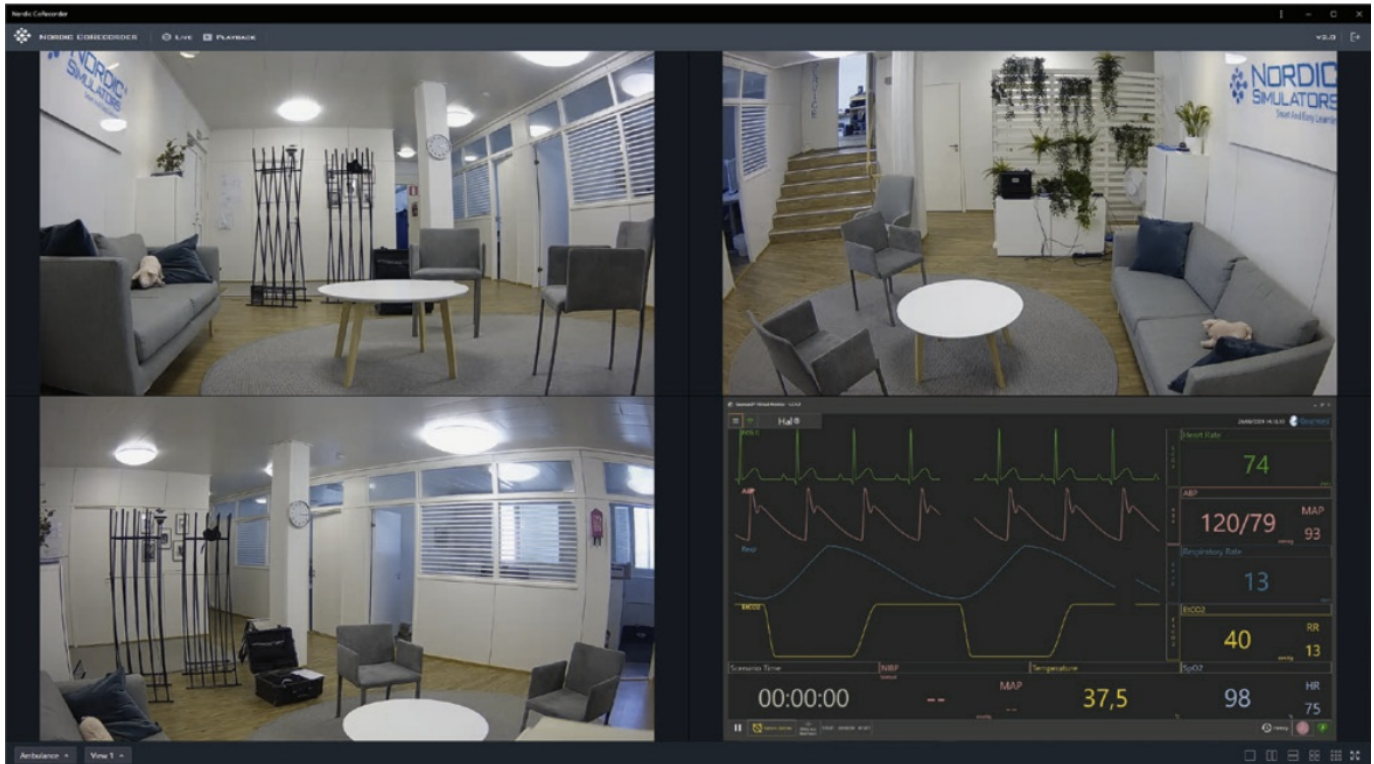

In Debriefing mode, it is possible to watch the cameras and recordings. Debriefing mode can be used for observers in another room.

## Log in with mobile mode..

Start the software in the Mobile mode.

### Role:

- User


### Password:

- The default is that there is no User password;d, leave it empty.


### Application:

- Mobile



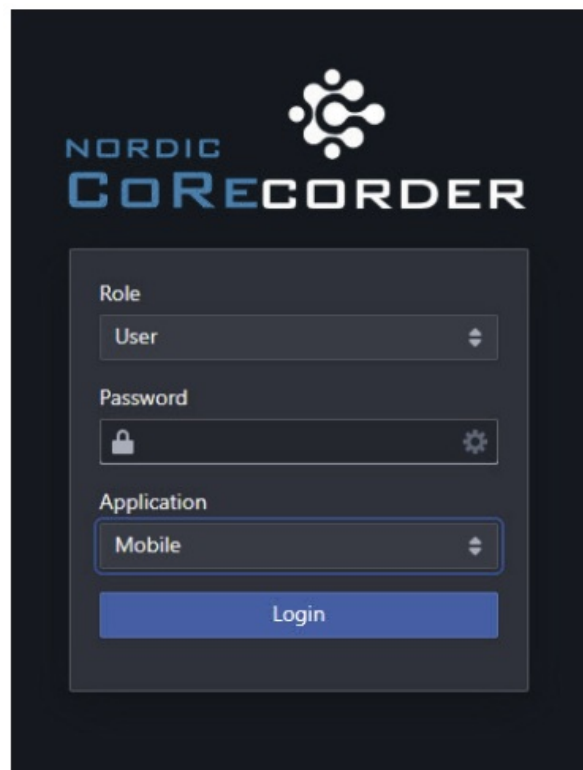

Press Login, and the software opens in Mobile mode.

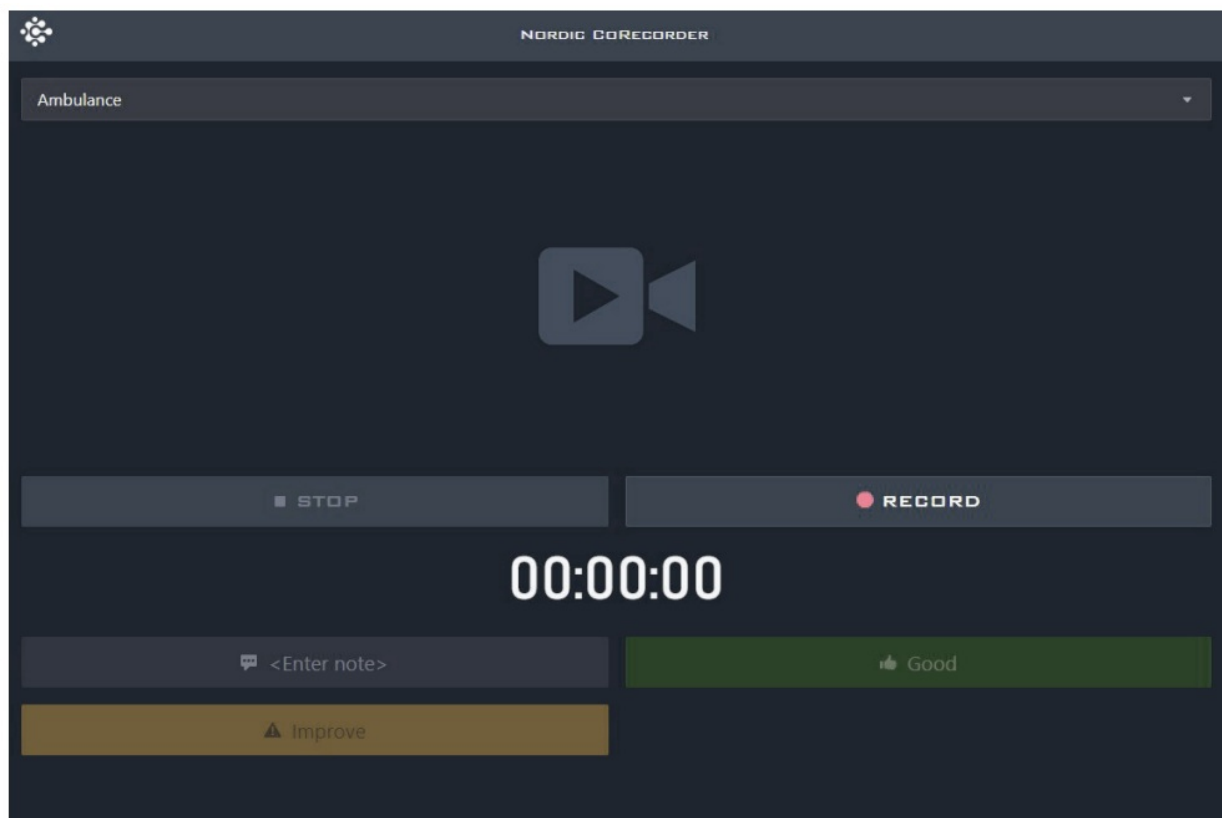

- In Mobile mode, it is possible to put the recording on/off, add markers without the camera pictures or activate the mobile device's camera for the recording.
- Mobile mode is meant for smartphones and tablets.

## Screen layout

With the Screen Layout button, you can choose how many camera windows are displayed.



With the **Screen Layout buttons** you can choose how many camera windows are displayed.

Screen Layout buttons



- The individual camera window can be maximised/minimised by double-clicking on the image.
- You can remove the camera or select another camera for the window from the drop-down menu by clicking on the camera name.
- The current layout can be saved to a scene view.

## Camera selection menu




## Camera audio

- Activate Audio Selection from the upper right corner of the camera window to hear the sounds from the camera's microphone.
- The default setting is that you can listen to one microphone at a time.
- Multiple audio sources can be enabled in the software settings for situations where

the cameras are in different rooms.


- The external microphone connected to the computer can be activated with the Microphone button.
- The microphone must be activated in a scene before starting the recording. The microphone can be listened with the speaker icon that comes visible.

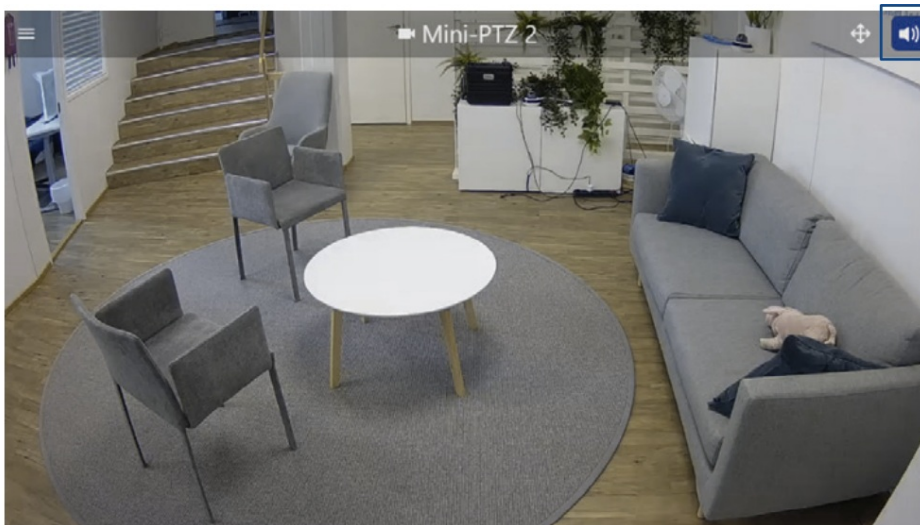

Camera Audio selection

## Pan-Tilt-Zoom controls

- It is possible to control Pan-Tilt or Pan-Tilt-Zoom camera models from the software.
- Activate PTZ controls from the upper right corner of the camera window.
- PTZ controls are then visible on the camera window. Depending on the camera model, you can move and zoom the camera with the on-screen buttons. The camera moves more slowly when pressed inside the arrows.
- Press the PTZ controls button again to disable the controls.

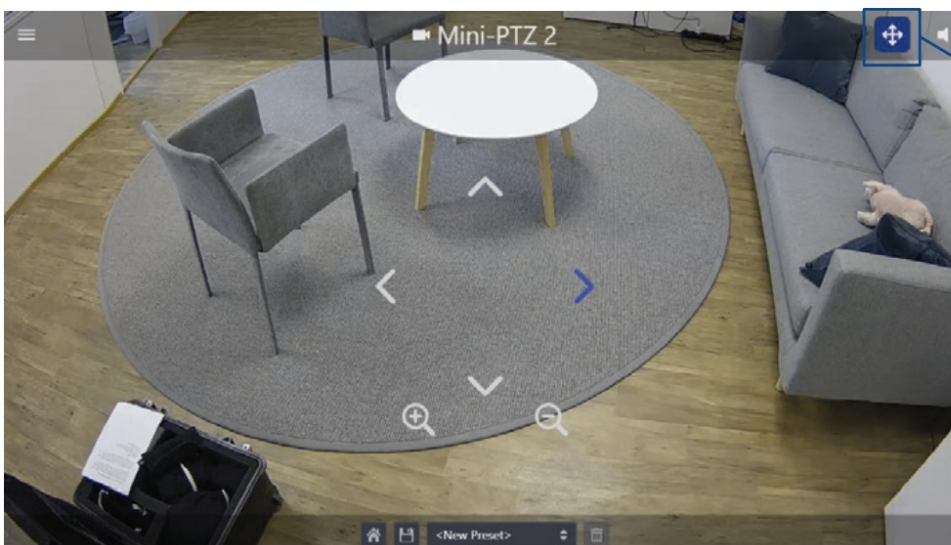


PTZ controls button

## Pan-Tilt-Zoom camera presets

- In some PTZ camera models it is possible to save camera presets. Presets save the

current angle and zoom of the camera.

- By selecting the preset, the camera moves and zooms automatically to that saved location.
- The preset menu shows already made presets even when the PTZ controls are not activated.
- Press the preset name to move the camera to a saved location.

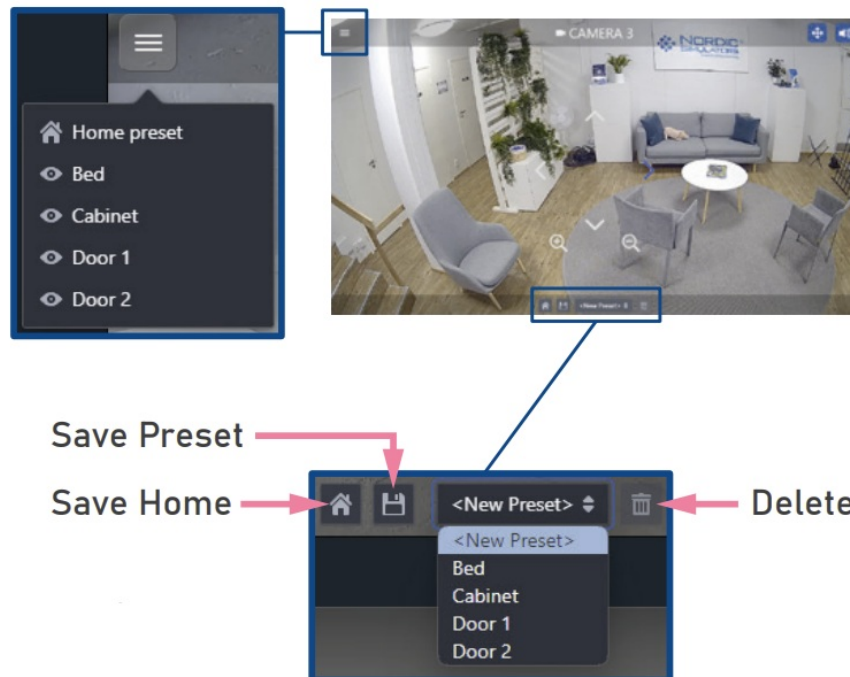


- Activate PTZ controls to modify presets with the menu. The home preset can be saved with its button.
- Make a new one, overwrite an old or delete a preset.


## Making a recording

Log in to the software with recording mode – Start Recording Scene recording OFF..

Scene recording **OFF**

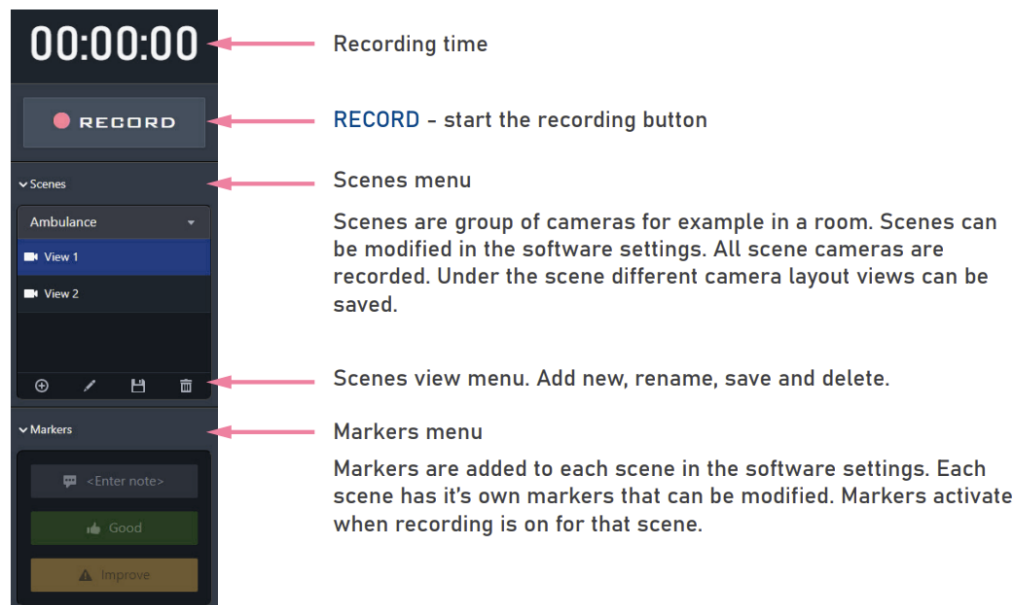


## Scene recording ON

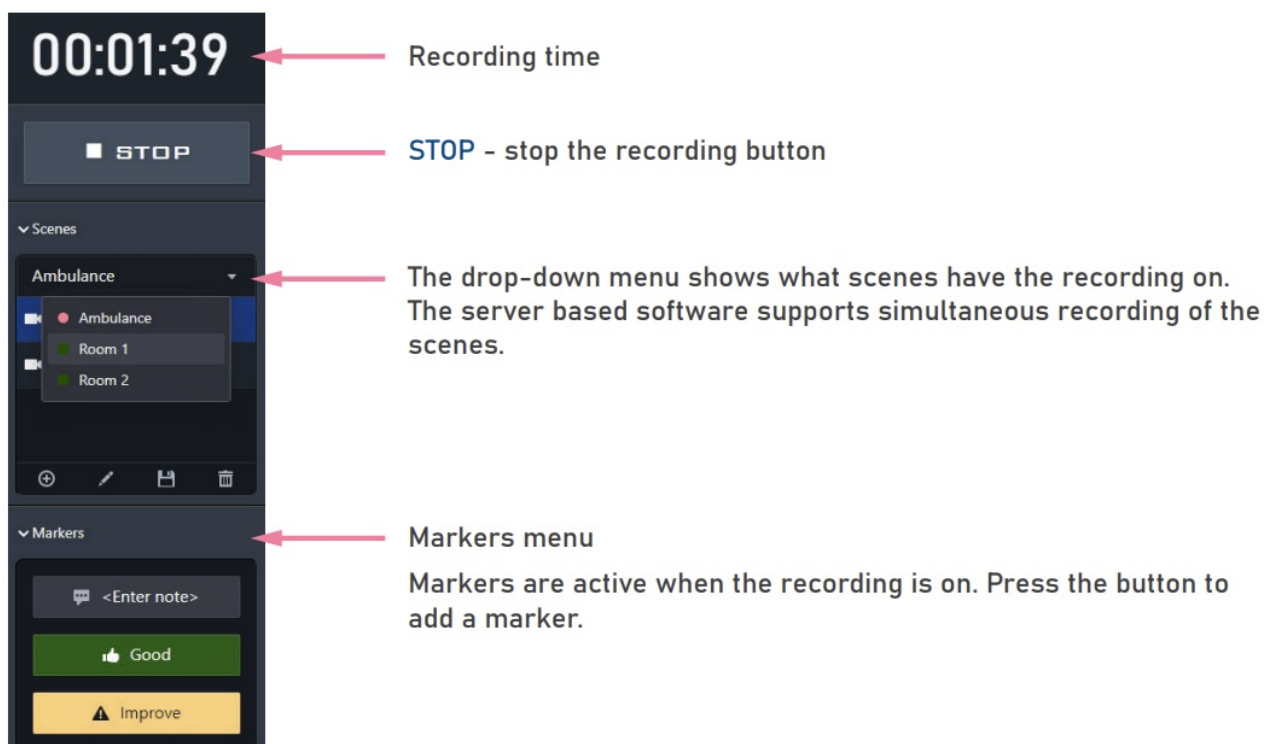


## Making a recording

- Recording mode controls – Stop Recording
- Press the STOP button to stop the recording, and a confirmation window opens.
- By pressing Cancel, the recording will continue.



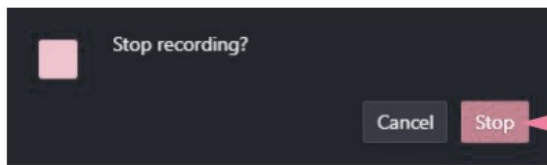

Press **Stop** to confirm

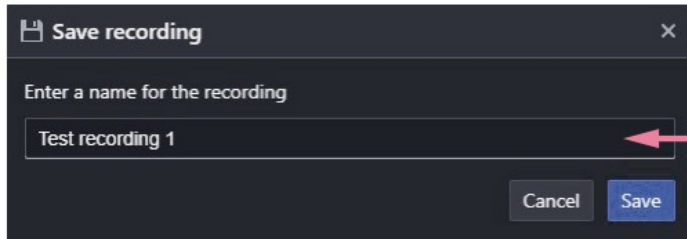

Enter a name for the recording or use the automatically generated name.

Press **Save** to confirm

## Watching recordings




Press the **PLAYBACK** button to open the recording list

Select recording				
Name				
Scene				
Lenght				
Created				
Q Enter query				
<input type="checkbox"/>	Name	Scene	Length	Created
<input type="checkbox"/>	Room 1-26.06.1452	Room 1	00:01:33	26.6.2024 klo 14:52:04
<input type="checkbox"/>	Test recording 1	Ambulance	00:04:49	26.6.2024 klo 14:49:52
<input type="checkbox"/>	Ambulance-18.06.0909	Ambulance	00:01:24	18.6.2024 klo 9:09:33
<input type="checkbox"/>	Room 2-18.06.0909	Room 2	00:01:37	18.6.2024 klo 9:09:29
<input type="checkbox"/>	Room 1-10.06.13:17	Room 1	00:00:16	10.6.2024 klo 13:17:32
<input type="checkbox"/>	Ambulance-04.06.13:28	Ambulance	00:06:46	4.6.2024 klo 13:28:48
<input type="checkbox"/>	Room 2-04.06.13:24	Room 2	00:00:08	4.6.2024 klo 13:24:08
<input type="checkbox"/>	Room 2-04.06.13:16	Room 2	00:03:56	4.6.2024 klo 13:16:09
<input type="checkbox"/>	Ambulance-04.06.13:16	Ambulance	00:01:02	4.6.2024 klo 13:16:02
<input type="checkbox"/>	testi	Room 1	00:03:30	4.6.2024 klo 13:06:48
<input type="checkbox"/>	Room 1-04.06.09:22	Room 1	00:00:27	4.6.2024 klo 9:22:01
<input type="checkbox"/>	Ambulance-04.06.09:21	Ambulance	00:00:20	4.6.2024 klo 9:21:02
Remove Cancel				

## Watching recordings


The selected recording will load to the timeline.

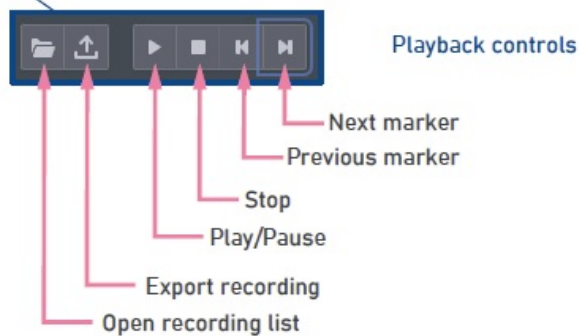

- Press Play to watch the recording. Activate the desired camera audio.
- Use the Next / Previous marker buttons to jump between the recording markers. Marker icons and comments are visible on top of the screen layout.
- Move the recording by clicking the desired place on the timeline.



## Exporting videos

- Recordings can be exported to an .mp4 format video file.
- Select a recording in the recording list, and it will load to the timeline.
- Put the screen layout the way you want it to also be on the exported video. For example a four view layout or just a one view layout.
- Pause the recording and select an active audio channel. This audio channel will be added to the video.

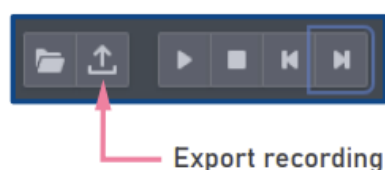

Press the Export recording button, and a menu will open.

- Rename the video if needed and select the resolution.
- Including text enabled will add the markers to the video.
- Press Export to start.
- When finished, the video can be downloaded to the computer by clicking on the download button.
- Please note that the download button will disappear if a new export is made or the software is closed.

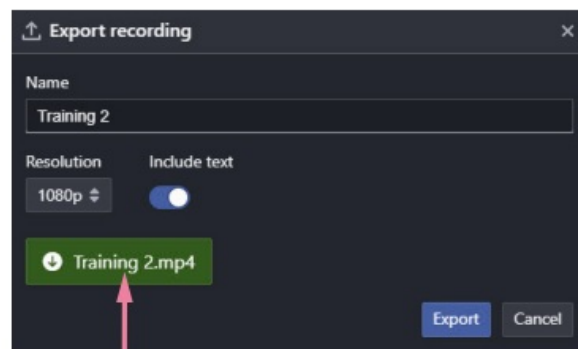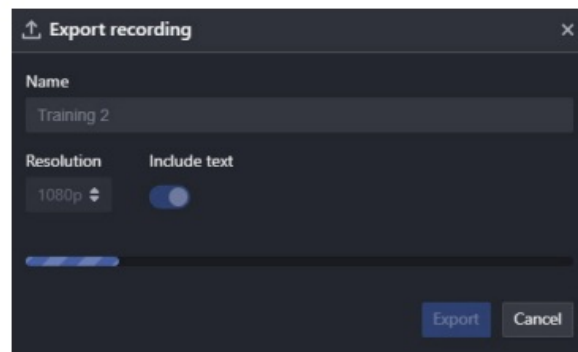

Download button for the exported video file

## Editing recordings



- Recordings can be edited after the recording has finished.
- It is possible to rename the recording, modify/add/delete markers and cut areas of the timeline.



Press the **EDITING** button to open the recording list.  
The recording will open in editing mode



- The Script menu shows all the recording's markers.
- **Edit a marker:** Click the arrow next to the marker's name, and an Edit note pop-up appears. Marker time, colour, icon and text can be edited. Apply the changes.
- **Add a new marker:** Move the timeline where the new marker is wanted. Press the add a new marker button and Apply the changes.
- **Cut an area of the timeline out:** Activate the timeline scissors with the but.

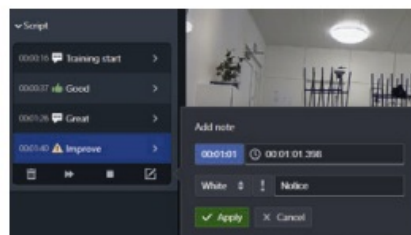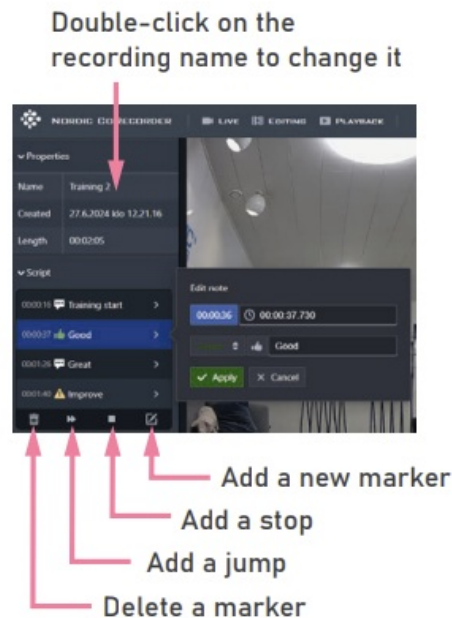


- Click and drag from left to right an area of the timeline. Release the press to apply the cut. The software will add jump or stop functions accordingly.

## Software options

Software options required logging in as Admin. Log in with Admin and click on the Settings icon. System settings open.

## Settings – System

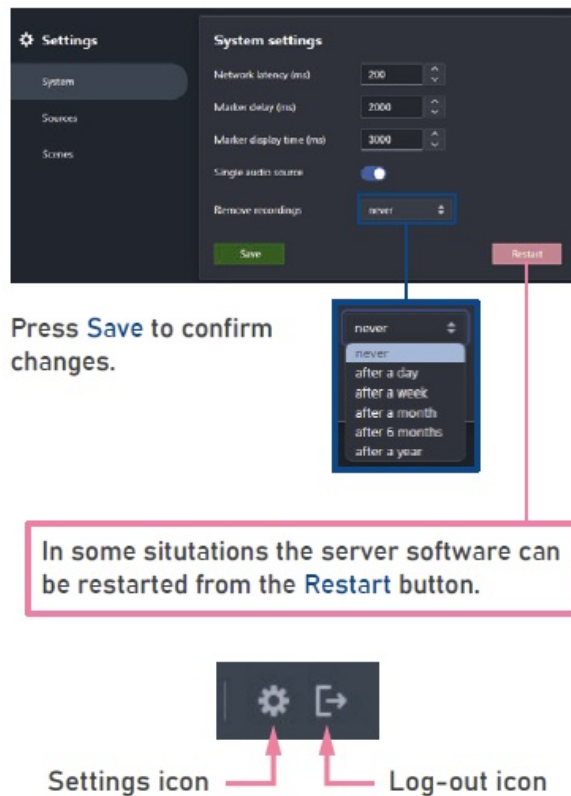


- Network latency (ms) = 200
  - The delay on the cameras compared to real life. Recommended minimum value is 200-500 ms.
- Marker delay (ms) = 2000
  - How much earlier will the markers appear on the recordings?
- Marker display time (ms) = 3000
  - How long will the markers be shown on the recordings?
- Single audio source = enabled.
  - When enabled, only one audio source can be listened at once. If disabled, multiple audio sources can be listened.
- Remove recordings = never.
  - How old d recording will be automatically deleted. The recordings must be deleted manually.

## Settings – Sources

- All the cameras are added to the software on the Sources page.
- New cameras can be added with the Add button.
- Camera's IP Address, Username/Password information are needed. Different camera models are supported.




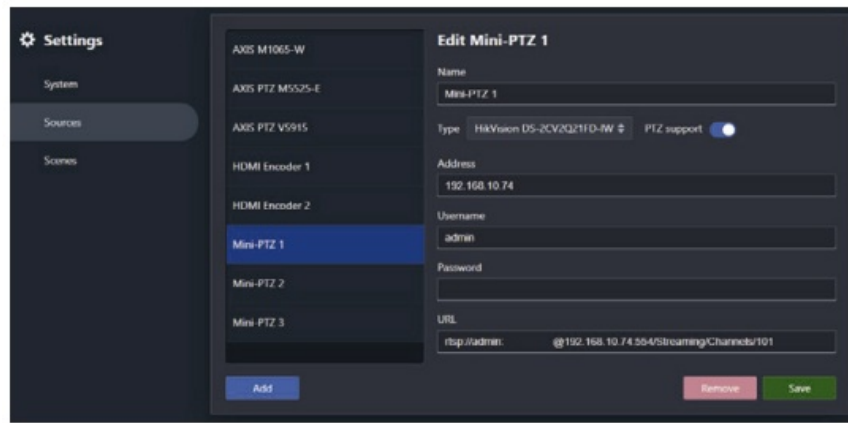

## Software options

### Settings – Scenes

Scenes can be added and modified with the Scenes page. Scenes are visible to the user and consist of a group of cameras and markers. Normally, a scene is, for example, all the training room's cameras. Allscenes's cameras are recorded even if they are not in view.

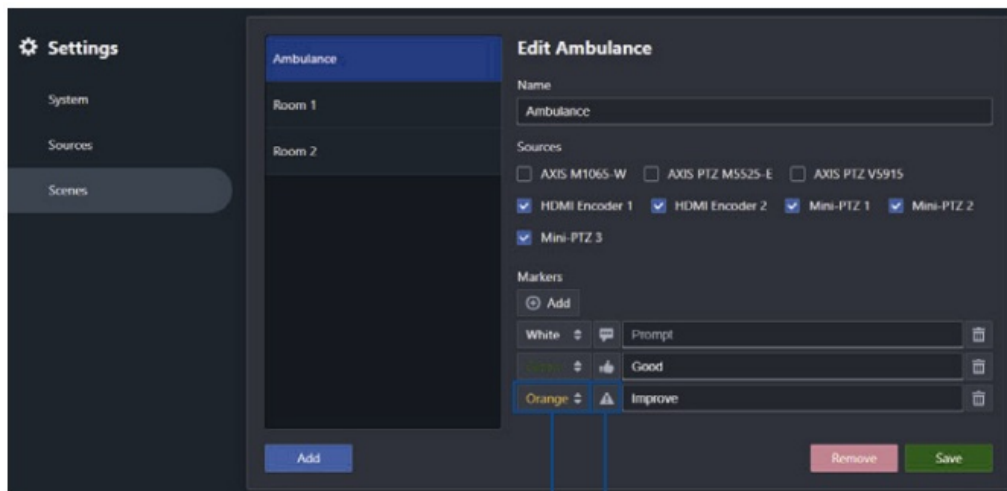

#### Name

Name of the scene


#### Sources

Select cameras


#### Markers

Modify, delete or add new markers

Select the marker color

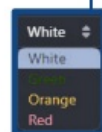

Select the marker icon

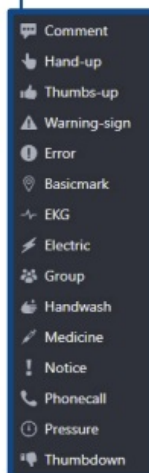

Press **Save** to confirm changes.

## FAQs

**Q: What are the different application modes available in CoRecorder?**

A: CoRecorder offers three application modes – Recording, Debriefing, and Mobile. Each mode serves specific purposes such as monitoring cameras, adding markers, watching recordings, and more.


**Q: How do I start the CoRecorder software?**

A: Simply double-click the Nordic CoRecorder icon on your desktop to launch the software. Choose your desired application mode from the drop-down menu to begin using the software.

**Q: Can I use CoRecorder on mobile devices?**

A: Yes, CoRecorder provides a Mobile mode specifically designed for smartphones and tablets. You can easily access features like recording control, marker addition, and even activate your device’s camera for recordings.

## Documents / Resources

	<a href="#">Nordic 1026138 Motion Control Recording Device [pdf]</a> User Guide 1026138, 1026138 Motion Control Recording Device, 1026138, Motion C ontrol Recording Device, Recording Device, Device
---	---

## References

- [User Manual](#)

■ NORDIC

📎 1026138, 1026138 Motion Control Recording Device, device, Motion Control Recording Device, NORDIC, Recording Device

—Previous Post

**[NORDIC NOSA2STS1AC, NOSA2SHS1AC Bottled Water Cooler Instruction Manual](#)**

Next Post—

**[NORDIC 1026137 Co Rec Mobile User Guide](#)**

---

## Leave a comment

Your email address will not be published. Required fields are marked \*

Comment \*

Name

Email

Website

☐ Save my name, email, and website in this browser for the next time I comment.

**Post Comment**

**Search:**

**Search**

[Manuals+](#) | [Upload](#) | [Deep Search](#) | [Privacy Policy](#) | [@manuals.plus](#) | [YouTube](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.