



numera Libris 2 Demo Fall Detection User Guide

[Home](#) » [Numera](#) » numera Libris 2 Demo Fall Detection User Guide 

numera Libris 2 Demo Fall Detection



Contents

- [1 Demo Fall Detection](#)
- [2 Setting up the Demo Fall Detection – Prerequisites](#)
- [3 Setting up Demo Fall Detection – Dealer Actions](#)
- [4 Testing Demo Fall Detection – Dealer Actions](#)
- [5 Testing Demo Fall Detection – User Actions](#)
- [6 After Testing is Complete](#)
- [7 Documents / Resources](#)
 - [7.1 References](#)
- [8 Related Posts](#)

Demo Fall Detection



- New Demo feature allows a user to test Fall Detection on a Libris 2 device.
- Users can have peace of mind knowing that automatic fall detection is enabled on the device and works.
- Steps to perform the demo are simple, easy to follow and safe.
- Provides a 30-minute window for user to test fall detection.
- Demo feature turns off automatically after 30 minutes – or can be manually disabled – and device returns to normal Fall mode.

Setting up the Demo Fall Detection – Prerequisites

- Realm must be configured to allow for this feature.
- Dealers can contact Numera Technical Support to request this feature **(1.855.546.3399)**.
- Fall Detection must be enabled on the device.
- Device software must be minimum v2.6.1.
- User **must be aware that** Demo mode is being enabled for Fall Detection.

Setting up Demo Fall Detection – Dealer Actions

Edit device settings

change items


GPS Interval ?	10 minutes	usdemo
Inactivity Notifications ?	ON	usdemo
Inactive In Charger ?	2 days	usdemo
Inactive Out of Charger ?	12 hours	usdemo
Fall Detection ?	ON	usdemo.Libris2
Fall Detection - Demo Mode ?	ON	usdemo.Libris2
Audio Prompt Volume ?	0 100	usdemo
Call Volume ?	0 100	usdemo
Primary Call-in Number ?	7604053517	
Periodic Indicator Feedback ?	30 seconds	SYSTEM

OK

Cancel

- To enable Demo Fall Detection (FD)
- Log into the Numera Dealer Portal
- Go to the Device page
- Go to the “Settings” section
- Hit the “Edit” icon
- Select “Demo Mode” – On
- Hit “OK”

Testing Demo Fall Detection – Dealer Actions

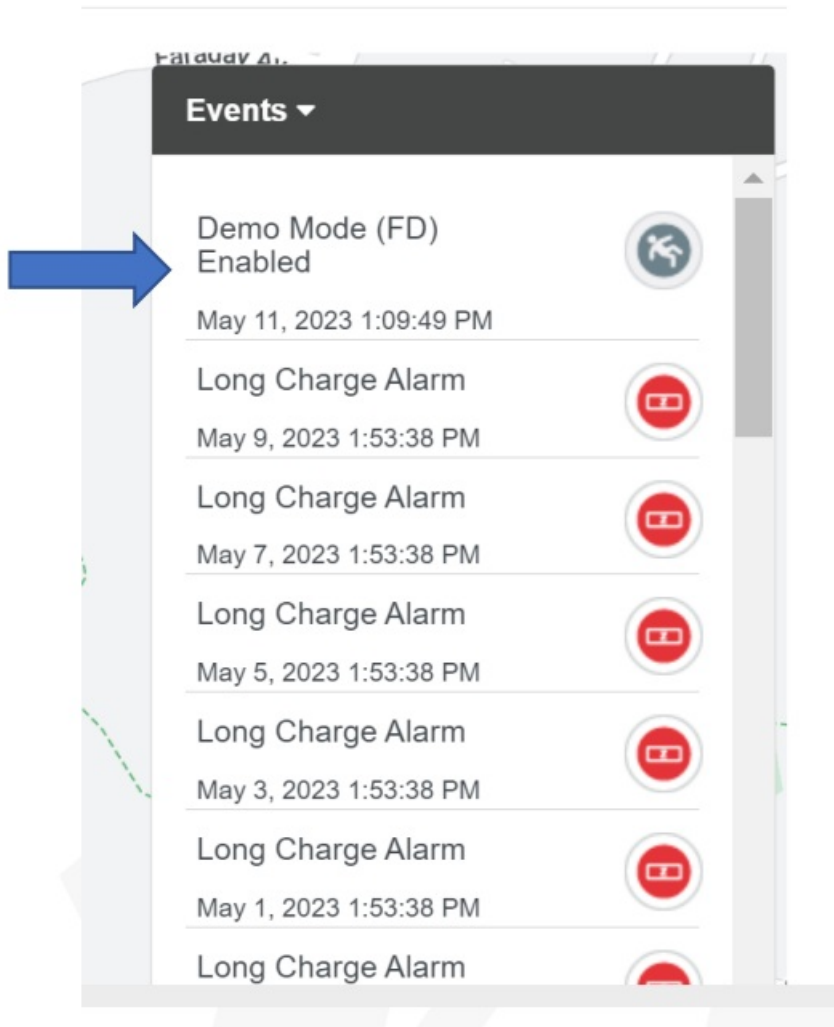

Dealer will confirm they see an event – Demo Fall Detection (FD) Enabled.

- Click on Locations tab to see the event list
- This action sets the timer for 30 minutes.
- Dealer informs the user that Demo FD is enabled.

Testing Demo Fall Detection – User Actions

- User picks up the device.
- User extends their arm out straight, parallel to the ground.
- User drops the device to the ground (it is okay if it bounces).
- Device must be left on the ground for 2-3 seconds and can then be picked up.
- Within a minute, the device will enunciate that it detected a fall and place a call to the station.
- User can tell the operators they are testing Fall Detection

After Testing is Complete

- User has no further action. They can resume wearing the device.
- System will disable the Demo Fall Detection feature automatically after 30 min – or – dealer can manually disable it.
- Device returns to normal Fall Detection mode.


- Dealer can confirm they see a Demo Fall Detection Disabled event.
- Dealer can reach out to the user telling them the FD demo mode is disabled.



COMPANY CONFIDENTIAL – Silent Call
Thank you



Documents / Resources

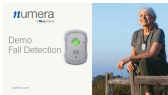	<p>numera Libris 2 Demo Fall Detection [pdf] User Guide Libris 2 Demo Fall Detection, Libris 2, Demo Fall Detection, Fall Detection, Detection</p>
---	--

References

- [!\[\]\(6841ca9b0e023296428e7c9e683b9367_img.jpg\) Medical Alert Systems | Numera](#)

Manuals+.