



PHONAK ActiveVent Receiver Instructions

[Home](#) » [PHONAK](#) » PHONAK ActiveVent Receiver Instructions 

PHONAK

life is on

Contents

- [1 Clean and Care Instructions ActiveVent TM Receiver](#)
- [2 How to clean](#)
- [3 How to Replace the wax filter](#)
- [4 Insert a new wax filter](#)
- [5 Documents / Resources](#)
 - [5.1 References](#)
- [6 Related Posts](#)

Clean and Care Instructions ActiveVent TM Receiver

Congratulations on the purchase of your hearing aids. Your hearing aids have been fitted with ActiveVent speakers. It is important to care and maintain your hearing aids.

Why is cleaning important?

1. Ensures better daily performance.
2. Extends lifespan of the devices.
3. Prevents repair issues that can result from wax build-up and moisture.



How to clean

1. Hearing aid

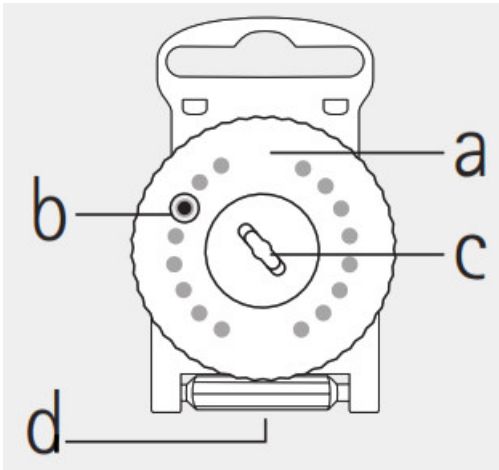

Wipe your hearing aid daily with a damp cloth.

2. Earpiece

Wipe your earpiece daily with a damp cloth.

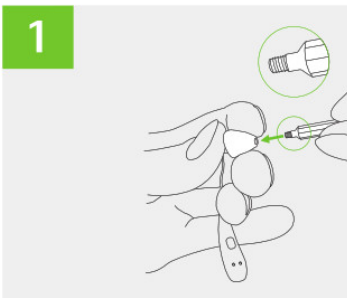
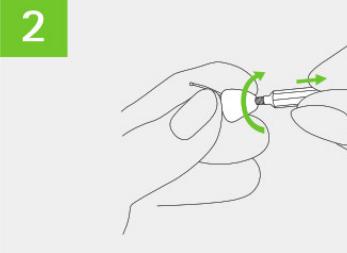
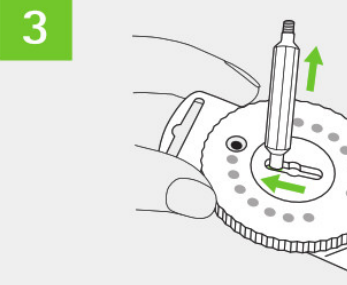
3. Replace wax filter

Weekly wax filter replacement is recommended. However, your hearing care professional will advise the appropriate interval for best performance. Continue reading on the next page on how to replace.

	<p>a) rotating disk b) new filter c) disposal slot for used filters d) changing tool</p>
	<p>a) threaded end for removing used filter b) slotted end for inserting new filter</p>

How to Replace the wax filter

Remove your used wax filter

<p>1. Hold the earpiece firmly and secure the rear of the speaker with your finger to prevent it from dislodging from the earpiece. Insert the threaded end of the changing tool into the opening of the speaker, while supporting the speaker with your finger or thumb.</p>	
<p>2. Rotate the changing tool clockwise until you feel some resistance, then remove the used wax filter by pulling the changing tool away from the speaker.</p>	
<p>3. Dispense the used wax filter by sliding the changing tool along the slot in the middle of the dispenser and lifting the changing tool away.</p>	

Insert a new wax filter


<p>1. Rotate the disk of the dispenser until a new wax filter is visible in the window.</p>	<div data-bbox="1390 109 1473 185">1</div>
<p>2. Pick up the new wax filter by inserting the slotted end of the changing tool.</p>	<div data-bbox="1390 544 1473 620">2</div>
<p>3. Hold the earpiece firmly and support the speaker at its end where the cable is attached. Insert the new wax filter gently into the opening of the speaker.</p>	<div data-bbox="1390 952 1473 1028">3</div>
<p>4. a) Angle the changing tool slightly as you pull it away from the speaker. b) Re-attach the changing tool to the wax filter dispenser.</p>	<div data-bbox="1390 1344 1473 1420">4</div>





Sonova Deutschland GmbH · Max-Eyth-Str. 20
70736 Fellbach-Oeffingen · Germany

Documents / Resources

<div><p>Clean and Care Instructions ActiveVent™ Receiver</p><p><small>Original instructions are the property of your hearing aids. Your hearing aids have been designed and modified to ensure the best possible hearing aid performance.</small></p><p>Why is cleaning important?</p><ul style="list-style-type: none">1. Clean the hearing aid with a soft cloth.2. Remove the hearing aid from the ear.3. Clean the hearing aid with a soft cloth.4. Remove the hearing aid from the ear.<p>© Sonova GmbH</p></div>	<p>PHONAK ActiveVent Receiver [pdf] Instructions</p> <p>PHONAK, ActiveVent Receiver</p>
---	---

References

-  [Hearing Aids | Phonak](#)

[Manuals+.](#)