



PHONAK Roger Upgrader User Guide

[Home](#) » [PHONAK](#) » PHONAK Roger Upgrader User Guide 

PHONAK Roger Upgrader User Guide



A Sonova brand

Phonak Roger Upgrader™

Roger On™ firmware update quick guide

Roger Upgrader™ is a tool designed to update the firmware of your Roger On device, for maximum efficiency.


Pre-requisites

1. Download Roger Upgrader 1.27 or higher




2. Install Roger Upgrader and open the application


1. Connect your Roger microphone to your computer using the cable provided

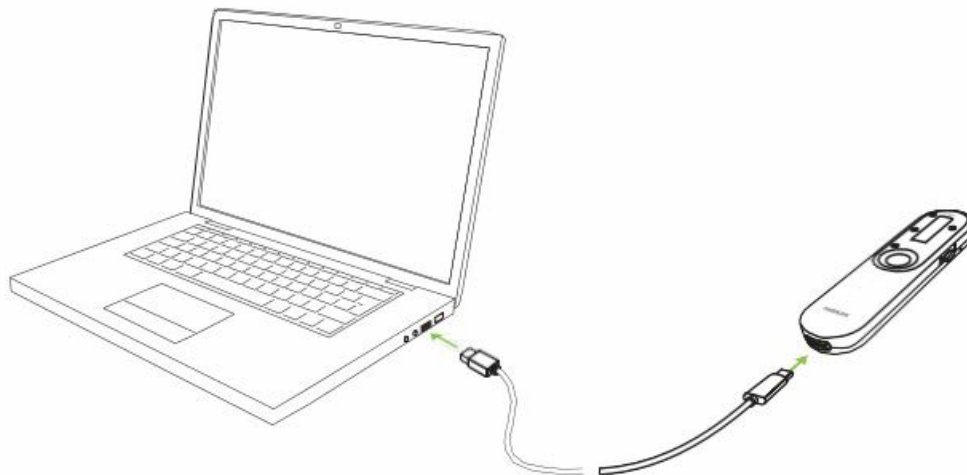


2. Click on “Manual firmware change”

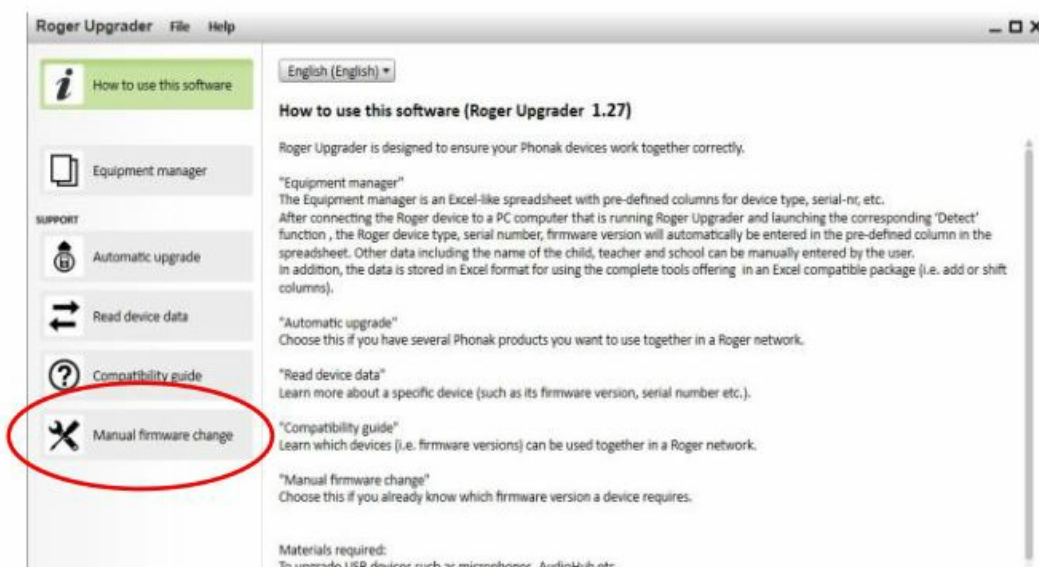


3. Click on “Start assistant”

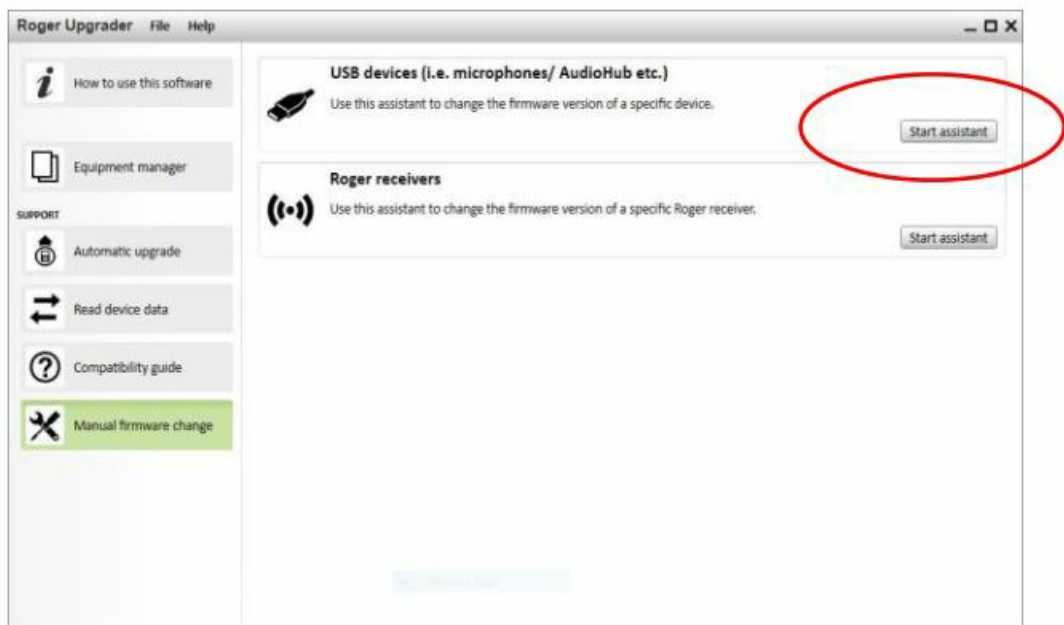

4. Click on "Continue"

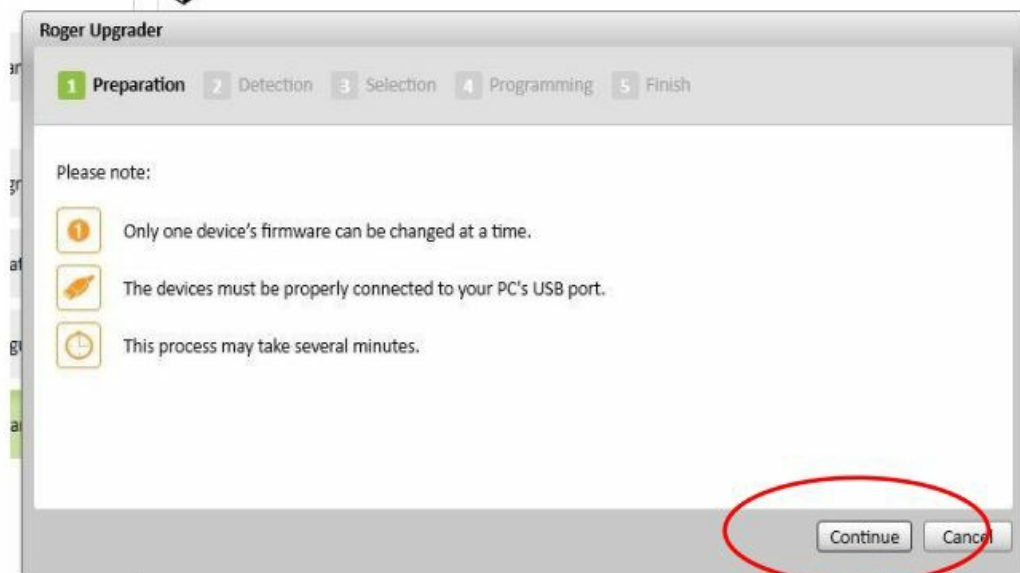

5. Select the firmware version and click on "Continue"

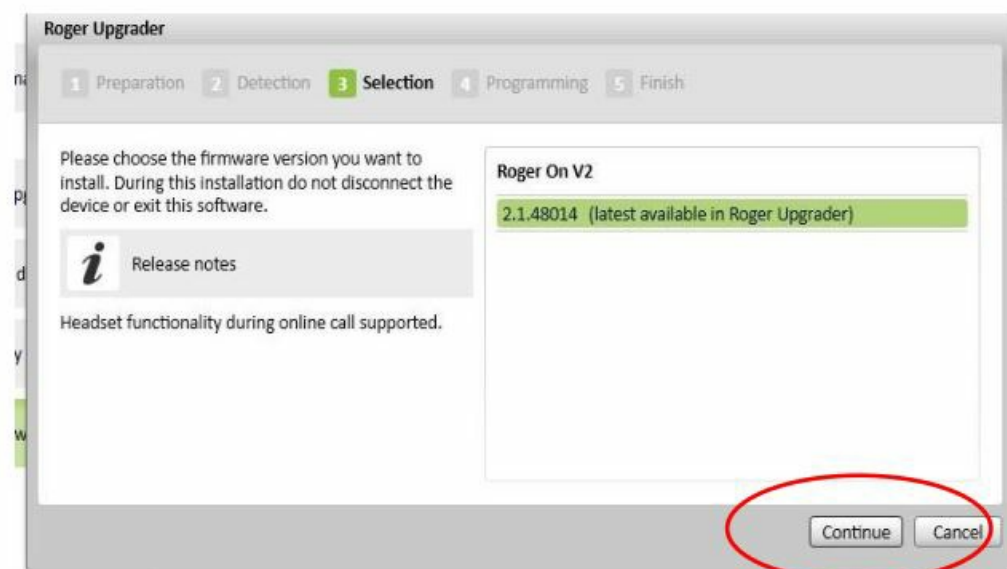



Sonova Communications •
Herrenschwandweg 4 •
3280 Murten • Switzerland
www.phonak.com/roger




Roger firmware update quick guide

Contents

- [1 Documents / Resources](#)
- [1.1 References](#)

Documents / Resources

	PHONAK Roger Upgrader [pdf] User Guide Roger Upgrader, Upgrader
---	--

References

- [Hearing Aid Microphones - Roger | Phonak](#)
- [User Manual](#)

[Manuals+.](#) [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.