



PRODIGY AutoCode Blood Glucose Monitoring System User Guide

[Home](#) » [prodigy](#) » PRODIGY AutoCode Blood Glucose Monitoring System User Guide 

PRODIGY AutoCode Blood Glucose Monitoring System User Guide



Contents

- [1 Simple 3-Step Testing](#)
- [2 Recalling Test Results](#)
- [3 Confirmation Window](#)
- [4 Documents / Resources](#)
- [5 Related Posts](#)

Simple 3-Step Testing

Step 1: Insert the Test Strip


Turn on the meter by inserting the test strip. No Coding required.




“Your Prodigy® meter is on. Please apply blood to the test strip.”



Step 2: Apply Blood


Apply a tiny blood sample when the “” appears on the display. The meter beeps and then counts down from 6 to 1.

 "Now testing."



Step 3: Hear Results

Hear and see your accurate results in 7 seconds.

 "Your blood glucose 6.6 millimoles per liter."



Recalling Test Results



With the meter turned off, press the “M” button twice. The 7-day average will appear, indicating that you are in the memory mode.

If you continue to press the “M” button, the 7, 14 and 28-day averages will appear. After the averages, you can then review the individual test results (450 maximum) stored in the memory.



Confirmation Window

With the test strip inserted, bring the meter close to your finger or other test site and place beside the blood drop. Blood is automatically drawn into the absorbent channel once the test strip is close enough to the blood drop. DO NOT smear blood onto the test strip or place the blood on top of the test strip.

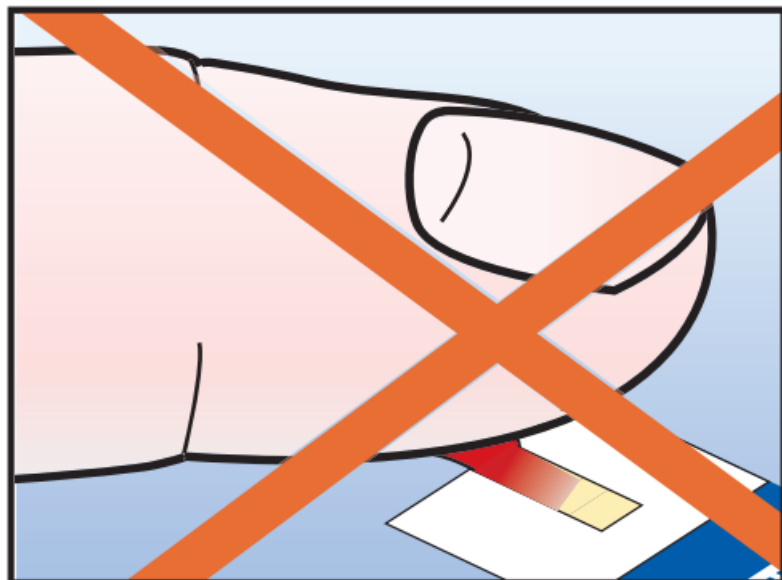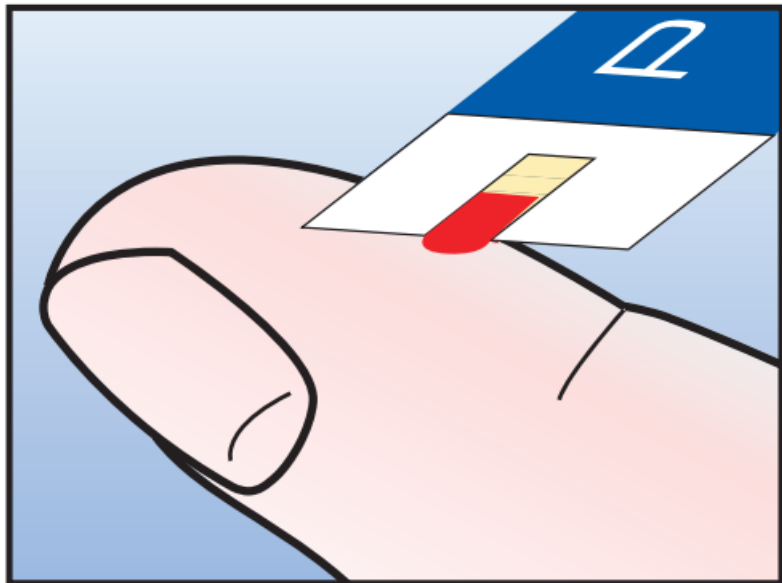

For Customer Care call: 1.800.243.2636

www.prodigymeter.com

Customer Support :
Prodigy Diabetes Care, LLC
P.O. Box 481928
Charlotte, North Carolina 28269 USA

PRODIGY



Documents / Resources



[PRODIGY AutoCode Blood Glucose Monitoring System](#) [pdf] User Guide
AutoCode Blood Glucose Monitoring System, AutoCode, Blood Glucose Monitoring System, Glucose Monitoring System

Manuals+.