



Text Only Readit Wireless Keypad User Guide

[Home](#) » [Readit](#) » Text Only Readit Wireless Keypad User Guide 



Text Only Readit Wireless Keypad Quick Reference Guide

Contents

[1 Text Only Readit Wireless Keypad](#)

[2 Documents / Resources](#)

Text Only Readit Wireless Keypad

This guide will explain each button on the keypad starting at the top left corner and working horizontally across the keypad. The keypad is laid out in three rows of controls. The top row has 4 controls, the next row has 5, and the final row has 7.

Please ensure you have removed the magnetic simplifier to show all controls.

Some of the controls are only useful for low vision users. These controls will be called a ‘Low Vision Control’ at the start of its explanation so it can easily be skipped if not required.

Row 1 – control 1:

The top left button is a triangle pointing downwards. This is the Help button, press it to access the full help menu. This enters an intuitive control describer mode. To exit, press the help button again.

Row 1 – control 2:

Low Vision Control: a circular button. In document mode, quick press to cycle through the text views. Press and hold to see the original image. When in live image mode, quick press this button to rotate the image 90 degrees.

Row 1 – control 3:

Low Vision Control: an octagonal dial that can be rotated to adjust magnification. Quick press to cycle between preferred reading colours, inverse and full colour. To set the preferred reading colours press, hold and rotate.

Row 1 – control 4:

Low Vision Control: a round thumb stick (joystick) on the top right of the keypad. To pan around your document in Live mode, Image view and Overlay view, move the joystick in the direction you want to move your document. Quick press to start reading from the word nearest the pointer.

Row 2 – Control 1:

A horizontal rectangular button with a raised line on its bottom edge located in the far right of the keypad. Quick press to capture and read. Press and hold to force table mode capture.

reddit

Documents / Resources

[illegible]